



spex[®] BY **JAY**[®]

Shaping Better Lives

Stylo

HEAD SUPPORT
TECHNOLOGY

Sleek design for
the ultimate in
head support style



Stylo

The latest head support technology from Spex

The Stylo Head Support has been designed to provide superior head and neck control for individuals with basic and moderate needs, through to complex postural asymmetries.

This next generation head support technology from Spex Seating provides superior adjustability and strength, with its extended links and ball and socket joints ensuring extra strength to endure greater impact forces. In-built adjustment possibilities allows for better positioning of the head and neck in virtually any position.



Stylo 160



Stylo 260



Clinical

Designed for the individual who:

- Presents with increased thoracic or lumbar curvature of the spine and therefore requires the head support to be placed afar from the back support.
- Presents with rotation or lateral flexion of the trunk and may require the head support to be positioned in a specific spot, offset from the back support.

Ultimate and unparalleled adjustability to meet the most complex head positioning needs!

The extended adjustable links, together with ball and socket joints allow for precise head positioning as they permit the head/neck pad to be positioned virtually anywhere within its length. Increased forward reach with 160mm (Stylo 160) or 260mm (Stylo 260) available. Improved ball mount design allows incredible range of angle adjustment, with total adjustment possible up to 270°.





Flip-back option

Available on both the Stylo 160 and 260, the flip-back mechanism option is an excellent choice for individuals who require assistance with transfers in and out of the wheelchair. Simply slide the red collar up to disengage the stem and swing backwards.



One tool for all adjustments

For ease of Stylo adjustment and fine-tuning to the correct position, only one tool is required – a 5mm Hex/Allen Key.



Sturdy positioning

Machined aluminium stem design incorporates two flat edges, ensuring no stem rotation for a fixed position that won't slip. Extra strong design clamps up tightly to prevent lateral movement.



Super quick installation

Head support stem installs simply and easily into the mount, and easily removes when required.

Stylo 160

For the simple, yet adjustable and adaptable head positioning solution

Stylo 160 is crafted to achieve the best of both worlds: simplicity and ease of adjustment, yet highly adjustable and versatile.

Clever use of twin ball joints means the head pad can be orientated to where the optimal support is required, through an available 45° of pitch; coupled with two hinge joints that offer infinite up and down, and forward and backward adjustment. The finished product... an extremely versatile mechanism that allows true adjustment of the pad position, creating a solution that is completely adaptable to the user.

Available in flip-back option for ease of transfers.





Stylo 260

Infinite adjustability
for the most
complex situations!

Stylo 260 is engineered to offer ultimate and unparalleled adjustability to meet the most complex head positioning needs! With simple adjustment and intuitive design, it brings simplicity to the most complex of cases.

Similar to Stylo 160, clever use of twin ball joints means the head pad can be orientated to where the optimal support is required, through an available 45° of pitch; coupled with two hinge joints that offer infinite up and down, and forward and backward adjustment. In addition, it offers incredible reach with length-adjustable links.

Available in flip-back option for ease of transfers.

spex Stylo

	Stylo 160	Stylo 260
CODE	1274-1252-000	1274-1282-000
FLIP-BACK CODE	1274-2252-000	1274-2282-000
HORIZONTAL FORWARD ADJUSTMENT	160 mm	260 mm
VERTICAL ADJUSTMENT IN MECHANISM	135 mm	160 mm
VERTICAL ADJUSTMENT IN STEM	145 mm 80 mm for Flip-back	145 mm 80 mm for Flip-back
SIDEWAYS ADJUSTMENT	200 mm	250 mm
PITCH	45° of head support pad in all directions within the ball joint. The ball joint is also on a hinge which offers an additional 180° in up and down angle!	
WEIGHT OF STYLO STEM	745 grams	830 grams

Stylo 160




HORIZONTAL FORWARD ADJUSTMENT




VERTICAL ADJUSTMENT IN MECHANISM



VERTICAL ADJUSTMENT IN STEM

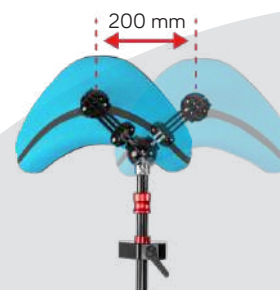


SIDEWAYS ADJUSTMENT

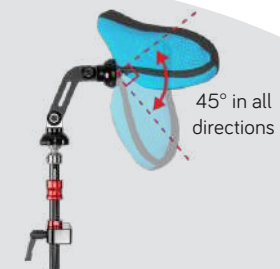

PITCH



Stylo 260

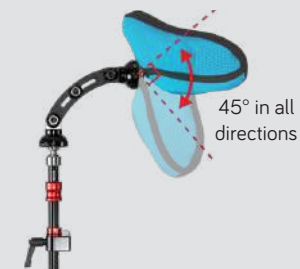

Support pad options



Compact

Basic, compact head support for clients with good head control.

250 x 100 mm 1271-4331-300



Square

Unobtrusive yet provides good contact support.

Small 200 x 160 mm 1271-4311-300
Large 240 x 200 mm 1271-4312-300



Contoured

Offers close lateral and occipital support for clients with reduced head control.

Small 285 x 115 mm 1271-4321-300
Large 340 x 165 mm 1271-4322-300



Adjustable lateral

Lateral wings of the head support can be adjusted and locked at the desired position.

Small 200 x 160 mm 1271-4341-300
Large 240 x 200 mm 1271-4342-300



Comfi

Posterior head pad with extra support on the sub-occipital area to promote head positioning and encourage alignment.

Standard 270 x 150 mm 1271-4351-300
Extended 270 x 165 mm 1271-4352-300



spex

Manufactured by Spex Ltd
237 Romina Drive, Unit 3
Concord, Ontario
Canada L4K 4V3
Phone: (800) 263-3390
Fax: (800) 561-5834
cscanada@sunmed.com
www.sunrisemedical.ca

773900-CAN, Rev. A

DISTRIBUTED BY: