



Wheelchair Tie-Down and Occupant Restraint Systems

Wheelchairs referenced in this supplement have been equipped with a ISO 7176-19:2008 and SAE J2249 compliant Wheelchair Tie-Down and Occupant Restraint System. (WTORS). This supplement is intended to provide transit compliance information and warnings for safe use to the owner of the wheelchair with the transit option installed.

If you have questions about using your wheelchair for seating in a motor vehicle, please contact Sunrise Medical at 1-800-333-4000.

CONTENTS

- A. General Warnings
- B. WTORS and Proper Set-up
- C. Components and Mounted Accessories
- D. WTORS Securement Points
- E. Clear Zones for a Wheelchair in a Vehicle
- F. Securing the Occupant Wheelchair Anchored Belts
- G. Use and Fit Grades for Vehicle Anchored Belts
- H. Seating Systems
- I. Correct Positioning of Belt Restraints
- J. Proper Installation and Fit
- K. Tested Configurations
- L. Testing Compliance Information

Wheelchairs and Their Weight Capacities covered in this Supplement:


Quickie Access — 300 lbs (136kg)

Quickie QS5 X — 300 lbs (136kg)


A WARNING

Refer to the wheelchair Owner's Manual for any weight limiting options that may affect the weight capacity listed above.



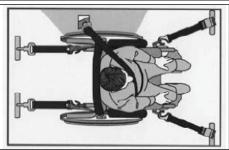



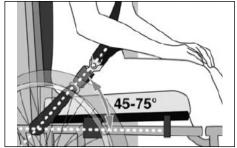

Illustrations from: RIDE SAFE, WC-19
"YOUR TICKET TO RIDE" online resource
www.travelsafer.org / www.rercwts.org

A. GENERAL WARNINGS

A WARNING

When feasible, wheelchair occupants should transfer into the vehicle seat and use the OEM (Original Equipment Manufacturer) vehicle-installed restraint system. The unoccupied wheelchair should be stored in a cargo area or secured in the vehicle during travel.

A WARNING

As of the date of this publication, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Sunrise Medical's position that users of wheelchairs should be transferred into appropriate OEM seating in vehicles used for transportation and that the restraints made available by the auto industry be used. Sunrise Medical cannot and does not recommend any wheelchair transportation systems. Use only Wheelchair Tie-down and Occupant Restraint Systems (WTORS) which meet the requirements of the SAE (Society of Automotive Engineers) J2249.

You may obtain a copy of this document by contacting SAE International at: 400 Commonwealth Drive Warrendale, PA 15096-0001 1.877.606.7323 (U.S. and Canada only) 1.724.776.4970 (Outside U.S. and Canada)

A WARNING

It is important to note that the chairs and transit options included in this Supplement have been designed for use by occupants that weigh more than 51 lbs (23kg). Do not attempt to use this chair in a transit situation for an occupant that weighs less than 51 lbs (23kg), as they will not be restrained properly, and in the event of an accident, there is a risk of serious injury or death.

A WARNING

The buckle of the belt restraint system should not be located near any item that may come in contact with the buckle release button in the event of a vehicle accident or collision.

A WARNING

If a recline angle is required during transport, the manual chair seat/back should be positioned at an angle of no more than 30° to the vertical or the occupant is at risk of injury from sliding under the restraint in the event of an accident or collision while in the motor vehicle.

A WARNING

Visually inspect all WTORS equipment at least once per month, and have worn or broken components replaced immediately.

A WARNING

Interior components that can't be removed from the clear zones should be padded with material that complies with FMVSS201.

A WARNING

Although postural supports and belts may be used in a moving vehicle in addition to the occupant belt restraint system, they should not be relied upon to replace occupant restraints that have been designed and tested for the purpose of securing an occupant during a motor vehicle collision or accident. Any postural supports that can be used during transit must be positioned so that they do not interfere with the clear path for proper transit securement belt restraint.

A WARNING

Backrests of wheelchairs with adjustable seat angles must not be tilted to more than 30°. Do not alter or substitute wheelchair frame parts, components, or seating systems.

A WARNING

A sudden stop and/or collision may structurally damage your wheelchair. Wheelchairs involved in sudden stops should be inspected for possible failures in frame and/or components. Wheelchairs involved in collisions should be replaced.

NOTE- The warranty is void on any wheelchair that is involved in a collision

254925 Rev B

2

B. WTORS AND PROPER SET-UP

- 1. The wheelchairs listed in this supplement have been dynamically tested in a forward-facing mode with a specified crash test dummy restrained by a surrogate vehicle-anchored 3-point belt. The vehicle-anchored 3-point belt should be used to reduce the risk of head and chest impacts with vehicle components.
- 2. Testing was performed for seating in a motor vehicle with the factory installed seating system only. The wheelchairs must be in a forward facing position during travel in a motor vehicle.

C. COMPONENTS AND MOUNTED ACCESSORIES

A WARNING

- 1. To reduce the potential of injury to vehicle occupants, wheel-chair-mounted accessories, including but not limited to IV poles, trays, respiratory equipment, backpacks, and other personal items should be removed and secured separately, before transport in a motor vehicle.
- 2. Postural supports, positioning devices, and/or strap(s) should not be relied on for occupant restraint. These items may be used in addition to the wheelchair anchored or vehicle-anchored belts.
- 3. Wheelchairs with adjustable seat angles should be set to 10°. Do not alter or substitute wheelchair frame parts, components, or seating systems.
- 4. Do not modify your wheelchair, or add accessories that have not been authorized by Sunrise Medical. Do not make alterations or substitutions to wheelchair structural parts or frame components without consulting Sunrise Medical.

D. WTORS SECUREMENT POINTS 1 2


NOTE - The ISO 7176-19:2008 Transit System includes four factory-installed transport brackets.


A WARNING

Only use the transit brackets that have been included with your specific wheelchair, and use them only for the purposes described in this supplement. Depending on your specific wheelchair, the securement brackets may not look the same, and may be in a different location than what is shown in figure 1 and figure 2. These figures are for illustrative purposes only.



Look for this symbol on your wheelchair. It indicates wheelchair securement points.

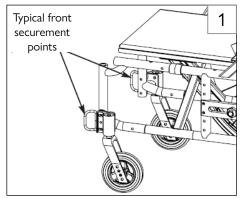

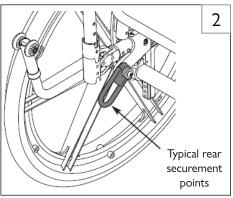



E. CLEAR ZONES FOR WHEELCHAIR IN A VEHICLE 3 4

A WARNING

The wheelchair must be in a forward facing position during travel in a motor vehicle. The recommended clear zones for wheelchair seated occupants restrained by both pelvic and upper-torso belt(s) and/or only by a pelvic belt are shown at right.

- **1. Frontal Clear Zone** (FCZ) (Figures 3, 4). This minimum area should be larger when upper-torso belt(s) are not used.
- **2. Frontal Clear Zone** is measured from the front most point on an occupant's head and is 26-inches (66 cm) with pelvic and upper-torso belt(s) and 37-inches (94 cm) with only a pelvic belt.
- **3.** The Frontal Clear Zone may not be achievable for wheel-chair-seated drivers.
- **4. The Rear Clear Zone** (RCZ) (Figures 3, 4) 16-inches (40.6 cm) from the rearmost point on an occupant's head. There must be no obstruction in this area.
- **5. The estimated seated height** (HHT) (figure 4) from the ground or floor to the top of the wheelchair-seated occupant's head ranges from approximately 47-inches (119.4 cm) for a small adult female to about 61-inches (155 cm) for a tall adult male.

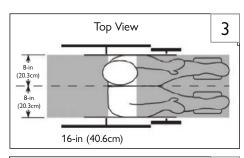

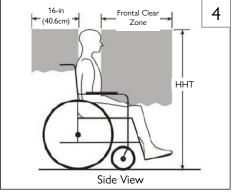


F. SECURING THE OCCUPANT:

WHEELCHAIR ANCHORED BELTS 5

A WARNING

The wheelchairs referenced in this supplement are used by adults and children who weigh more than 51 lbs (23kg). They can be used in conjuntion with a vehicle-anchored shoulder belt and a standard lower-anchorage connector as a crashworthy three-point belt restraint in motor vehicles.

The use of postural pelvic belts attached to the wheelchair base or seat frame is encouraged during travel. These belts should be positioned so that they don't interfere with the proper positioning of crash worthy restraints. The postural belt should NOT be relied on for occupant protection in crash situations.

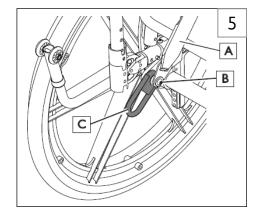

G. WHEELCHAIR'S ACCOMMODATION OF VEHICLE-AN-CHORED BELT RESTRAINTS

A WARNING

The wheelchairs in this supplement have an overall rating of "Good" with regard to the securement of the wheelchair to the test platform and position the ATD used in the frontal impact test of Annex A of ISO 7176-19. A three-point belt was installed and placed on the ATD in accordance with the procedures of D.5, and accommodation scores were assigned per Tables D.1 through D.8 of Annex D. The overall rating is assigned in accordance with D6 of Annex D. This rating is scored as follows: Excellent, Good, Acceptable, or Poor.

- Quickie Access = Overall Rating "Good"
- Quickie QS5 X = Overall Rating "Excellent"

H. SEATING SYSTEMS

A WARNING

This wheelchair has been tested for seating in a motor vehicle with the factory installed seating system only. Ensure that the factory installed seating system is secured to the wheelchair frame before operation. Refer to the seating system owner's manual. If you need to purchase a compatible, crashworthy, wheelchair-anchored pelvicbelt restraint, call Sunrise Customer service at: 1-800-333-4000.


I. CORRECT POSITIONING OF BELT RESTRAINTS 6 7 A WARNING


The angle of the pelvic belt should be within the preferred zone of 30 to 75 degrees to the horizontal (Figure 6).


Steeper side-view pelvic belt angles are important if the pelvic belt is intended to be used for postural support in addition to occupant restraint in a frontal crash. Steeper angles will reduce the tendency for a vertical gap to develop between the user and the belt, thereby reducing the tendency for the user to slip under the belt. Steeper belt angles reduce the tendency of upper-torso belts to pull the pelvic belt onto the abdomen during frontal impact loading.

J. PROPER INSTALLATION AND FIT 6 7 8

- a. The pelvic belt should be worn low across the front of the pelvis.
- b. Position the upper-torso belt(s) over the shoulders.
- c. The belt(s) should not be held away from the body by the wheel-chair components or parts, including but not limited to the armrests or wheels.
- d. Ensure the belt(s) are not twisted.
- e. Adjust the belts as firmly as possible being mindful not to restrict bloodflow, or breathing of the occupant.
- f. If an upper torso belt is in use, the anchor point (A) should be above and behind the occupant to ensure the occupant is properly restrained during transport.
- g. Place the upper torso belt across the middle of the shoulder and the center of the chest, and connect to the pelvic belt near the hip of the occupant.

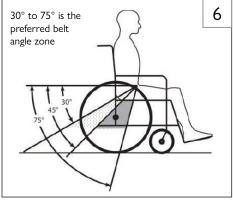

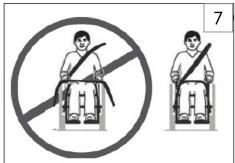

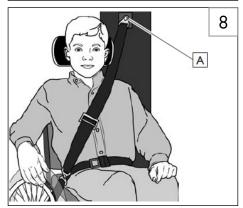

K. TESTED CONFIGURATIONS

Quickie Access was tested in the specific configuration of 16-in x 18-in (40.6 cm x 45.7 cm), 67 lbs (30.4 kg), fixed height back canes, standard back/seat upholstery, 2-in (5 cm) cushion, swing in/out hanger with aluminum angle adjustable footplates and heel loops, polyurethane front casters, standard forks, spoke rear wheels with pneumatic tire and airless insert. Minimum Turning Radius of 64.0 Inches (162.6 cm), and Minimum Turn Around Width of 48.0 Inches (122 cm).

Quickie QS5 X was tested in the specific configuration of 17-in x 19-in (44 cm x 48 cm), 36.6 lbs (16.6 kg), fixed height back canes, standard back/seat upholstery, 2-in (5 cm) cushion, swing in/out hanger with composite angle adjustable footplates and heel loops, polyurethane front casters, standard forks, spoke rear wheels with pneumatic tire. Minimum Turning Radius of 26.4 inches (67.2 cm) and Minimum Turn Around Width of 47.5 inches (120.7 cm).

L. TESTING COMPLIANCE INFORMATION

The Quickie Access [300 lbs (136kg) weight capacity] has been crashtested with a 170 lb (77kg) (Hybrid III Midsize male) crash dummy.

The Quickie QS5 \times [300 lbs (136 kg) weight capacity] has been crash tested with a 172 lb (78 kg) (Hybrid III Midsize male) crash dummy.

All wheelchairs listed have been tested in accordance with ISO 7176-19:2008 Front impact test requirements for wheelchairs.

NOTE:

ISO = International Organization for Standardization

254925 Rev B

6

QUICKIE/ZIPPIE NOTES

254925 Rev A

Sunrise Medical S.r.I. Via Riva, 20 – Montale 29122 Piacenza

Tel.: +39 0523 573111 Fax: +39 0523 570060 www.SunriseMedical.it

Sunrise Medical AG Erlenauweg 17 CH-3110 Münsingen Schweiz/Suisse/Svizzera Fon+41 (0)31 958 3838 Fax+41 (0)31 958 3848 www.SunriseMedical.ch

Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norge
Telefon: +47 66 96 38 00
post@sunrisemedical.no
www.SunriseMedical.no

Sunrise Medical AB Neongatan 5 431 53 Mölndal Sweden Tel.: +46 (0)31 748 37 00 post@sunrisemedical.se www.SunriseMedical.se

MEDICCO s.r.o.
H – Park, Heršpická 1013/11d, 639 00 Brno
Czech Republic
Tel.: (+420) 547 250 955
Fax: (+420) 547 250 956
www.medicco.cz
info@medicco.cz

Bezplatná linka 800 900 809

Sunrise Medical Aps Mårkærvej 5-9 2630 Taastrup Denmark +45 70 22 43 49 info@sunrisemedical.dk SunriseMedical.dk

Sunrise Medical Australia
11 Daniel Street
Wetherill Park NSW 2164
Australia
Ph: +61 2 9678 6600
Email: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

Sunrise Medical (US) LLC North American Headquarters 2842 N. Business Park Avenue Fresno, CA, 93727, USA (800) 333-4000 (800) 300-7502

www.SunriseMedical.com

e

SUNRISE

© 2024 Sunrise Medical (US) LLC 254925 Rev B Sunrise Medical GmbH Kahlbachring 2-4 D-69254 Malsch Deutschland Tel.: +49 (0) 7253/980-0

Tel.: +49 (0) 7253/980-0 Fax: +49 (0) 7253/980-222 www.SunriseMedical.de

Sunrise Medical Ltd. Thorns Road Brierley Hill West Midlands DY5 2LD England

Phone: 0845 605 66 88 Fax: 0845 605 66 89 www.SunriseMedical.co.uk

Sunrise Medical S.L. Polígono Bakiola, 41 48498 Arrankudiaga – Vizcaya España Tel.: +34 (0) 902142434 Fax: +34 (0) 946481575

Sunrise Medical Poland Sp. z o.o. ul. Elektronowa 6, 94-103 Łódź Polska Telefon: + 48 42 275 83 38 Fax: + 48 42 209 35 23

E-mail: pl@sunrisemedical.de

www.Sunrise-Medical.pl

www.SunriseMedical.es

Sunrise Medical B.V.
Defensiedok 20
3433 NE Nieuwegein
The Netherlands
T: +31 (0)30 – 60 82 100
F: +31 (0)30 – 60 55 880
E: info@sunrisemedical.nl
www.SunriseMedical.nl

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

Sunrise Medical S.A.S ZAC de la Vrillonnerie 17 Rue Mickaël Faraday 37170 Chambray-Lès-Tours Tel: + 33 (0) 2 47 55 44 00 Email: info@sunrisemedical.fr www.SunriseMedical.fr

Sunrise Medical Canada Inc. 1000 Creditstone Rd., Unit #2 Concord, ON, L4K 4P8 Canada

Phone: 1-800-263-3390 Fax: 1-800-561-5834 www.SunriseMedical.ca