



- Quickie® IRIS®
- Zippie® IRIS®
- SE
- Folding
- With transit
- Without transit

Supplier: This manual must be given to the user of this wheelchair.

Owner's Manual

User: Before using this wheelchair read this entire manual and save for future reference.

Distribuidor: Este manual debe ser entregado al pasajero de esta silla de ruedas.

Pasajero: Antes de usar esta silla de ruedas, lea este manual en su totalidad y guárdelo para futura referencia.

Manual de Instrucciones

Cada una de las sillas se envía con un manual de instrucciones en inglés. El manual en español o francés está disponible en formato PDF en nuestra página en Internet: www.SunriseMedical.com. Ingrese a la página del producto específico para descargar el manual, o comuníquese con el proveedor autorizado de Sunrise Medical.

Fournisseur: Ce manuel doit être remis à l'utilisateur / utilisatrice de ce fauteuil roulant.

Utilisateur / Utilisatrice: Avant d'utiliser ce fauteuil roulant, lisez entièrement ce manuel et conservez le pour le consulter ultérieurement.

Moded' emploi

Chaque fauteuil est livré avec un manuel d'instructions en anglais. Les versions en espagnol et en français sont à votre disposition en format PDF sur le site: www.SunriseMedical.com. Veuillez vous rendre à la page de votre produit pour télécharger le manuel dans la langue souhaitée, ou contactez un fournisseur agréé Sunrise Medical.

MK-100151 Rev. E

Instructions for Use



SUNRISE MEDICAL LISTENS

Thank you for choosing a Quickie® wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Sunrise Medical authorized dealer. Please feel free to write or call us at the address and telephone number below:

SUNRISE MEDICAL (US) LLC
Customer Service Department
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA
(800) 333-4000



www.SunriseMedical.com/register

Be sure to register your wheelchair and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products, and options to increase your use and enjoyment of the wheelchair.

You can also register your product at:
www.SunriseMedical.com/register

FOR ANSWERS TO YOUR QUESTIONS

Your authorized dealer knows your wheelchair best and can answer most of your questions about chair safety, use, and maintenance.

For future reference, please fill in the following:

Dealer: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this chair was made with Natural Rubber Latex.

**DISPOSAL AND RECYCLING INFORMATION**

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This product is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your product properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations.

II. TABLE OF CONTENTS

I. INTRODUCTION.....	2	G. Cushion Installation.....	16
II. TABLE OF CONTENTS.....	3	H. Wheel Locks.....	16
III. YOUR CHAIR AND ITS PARTS.....	4	I. Hub Lock.....	16
IV. NOTICE — READ BEFORE USE.....	4	J. Anti-Tip Tubes - Rear.....	16
V. GENERAL WARNINGS.....	5	K. Dual-Post Armrests.....	17
A. Weight Limits.....	5	L. Height-Adjustable Armrests (Optional).....	17
B. Intended Use.....	5	M. Adjustable Locking Flip-Up Armrests.....	17
C. Attendants and Caregivers.....	5	N. Cantilever Armrests.....	18
D. Accessories.....	5	O. Swing-Away Hangers/Footrests.....	18
E. Know Your Chair.....	5	P. Heavy-Duty, Lift-Off Footrests (Optional).....	18
F. Reduce the Risk of an Accident.....	5	Q. Articulating Legrest (Optional).....	18
G. Safety Checklist.....	5	R. Elevating Legrest (Optional).....	19
H. Changes & Adjustments.....	5	S. Z-Finity® Footrest System.....	19
I. Environmental Conditions.....	6	T. Z-Finity® Swing-Away Footrest.....	19
J. Terrain.....	6	U. Tilt-In-Space Mechanism.....	20
K. Street Use.....	6	V. Stroller Handle Extension (Option).....	20
L. Motor Vehicle Safety.....	6	W. Reclining Back.....	20
M. When You Need Help.....	7	X. Mono Back Stroller Handle.....	21
N. Choking Hazard.....	7	Y. Folding the Mono Back for Transport.....	21
O. Pinch Point.....	7	Z. Folding / Unfolding Frame (Optional).....	21
VI. SAFETY WARNINGS: FALLS AND TIP-OVERS.....	8	AA. Inspect.....	21
A. Center of Balance.....	8	IX. DEALER SERVICE & ADJUSTMENT.....	22
B. Dressing or Changing Clothes.....	8	A. Dealer Service Introduction.....	22
C. Obstacles.....	8	B. Critical Maintenance Tips.....	22
D. Front Caster Lift.....	8	C. Cleaning.....	22
E. Reaching or Leaning.....	8	D. Rear Axle.....	22
F. Moving Backward.....	8	E. Hub Lock Adjustment.....	23
G. Escalators.....	9	F. Wheel Locks.....	23
H. Ramps, Slopes, and Sidehills.....	9	G. Single-Post Armrest Receiver.....	23
I. Transfers.....	10	H. Adjustable Locking Flip-Up Armrests.....	24
J. Curbs and Single Steps.....	11	I. Cantilever Locking Flip-Back Armrests.....	24
K. Climbing a Curb or Single Step.....	11	J. Swing-Away Height Adjustment.....	25
L. Descending a Curb or Single Step.....	11	K. Angle Adjustable Footplate.....	25
M. Stairs.....	12	L. Heavy Duty Lift-Off Footrest.....	25
N. Climbing Stairs.....	12	M. Articulating Legrest.....	25
O. Descending Stairs.....	12	N. Elevating Legrest.....	25
P. Tilt Use and Back Angle Adjustment (Mono Back).....	12	O. Contracture Footrest.....	26
VII. WARNINGS: COMPONENTS AND OPTIONS.....	13	P. Z-Finity® Footrest System.....	26
A. Anti-Tip Tubes (Optional).....	13	Q. Limb Support.....	27
B. Armrests.....	13	R. Tilt-In-Space Mechanism.....	27
C. Push Handles.....	13	S. Standard Backrest.....	27
D. Stroller Handle.....	13	T. Mono Back.....	28
E. Cushions.....	13	U. Dynamic Mono Back.....	29
F. Fasteners.....	13	V. Dynamic Mono Back (Elastomer Replacement).....	29
G. Footrests.....	13	W. Ventilator Hanger Bracket.....	30
H. Pneumatic Tires.....	13	X. Fixed Stroller Handle.....	30
I. Positioning Belts (Optional).....	14	Y. Adjustable Stroller Handle (Mono Back).....	30
J. Quick-Release Axles.....	14	Z. Caster/Fork Assembly.....	31
K. Rear Axles.....	14	AA. Frame Depth.....	31
L. Rear Wheel Locks.....	14	BB. XLock® Width Adjustment.....	32
M. Modified Seat Systems.....	14	CC. Frame Width.....	32
N. Tray for Vent and Battery.....	14	DD. Seat Pan.....	33
O. Accessory Hook.....	14	EE. Removable Seat Pan.....	33
VIII. USE AND MAINTENANCE.....	15	FF. Carriage.....	34
A. Introduction.....	15	GG. Attendant Wheel Lock Installation.....	35
B. Critical Maintenance Tips.....	15	HH. Footplate Adapter Bracket.....	36
C. Safety Checklist.....	15	II. Lap Belt Instructions.....	37
D. Cleaning.....	15	JJ. Inspect.....	37
E. Troubleshooting Chart.....	15	X. SUNRISE LIMITED WARRANTY.....	38
F. To Mount and Remove Rear Wheels.....	16		



1. Headrest
2. Push Handle/Back Cane
3. Armrest
4. Footrest Hanger
5. Footplate
6. Carriage
7. Caster Fork
8. Caster Tire
9. Wheel Lock
10. 12-in Rear Wheel
11. Anti-tip
12. Adjustable Seat Pan
13. Tilt-In-Space Mechanism Release Lever

NOTE— Not all options are available on both the Quickie® and Zippie® IRIS®. Please refer to the order form for specific option availability.

IV. NOTICE — READ BEFORE USE

A. CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

Sunrise provides a choice of many wheelchair styles to meet your needs. This product is intended for single person use only. Final selection of the type of wheelchair, options, and adjustments rests solely with you and your health care professional. Choosing the best chair and set-up for your safety depends on such things as:

1. Your disability, strength, balance and coordination.
2. The types of hazards you must overcome in daily use (where you live and work, and other places you are likely to use your chair).
3. The need for options that will improve your positioning, safety and comfort (such as anti-tip tubes, positioning belts, or special seating systems).

B. REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

C. WARNINGS

The word **“WARNING”** refers to a hazard or unsafe practice that may cause **severe injury or death** to you or to other persons. The “Warnings” are in Three main sections, as follows:

1. V— GENERAL WARNINGS
Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this chair.
2. VI— SAFETY WARNINGS: FALLS & TIP-OVERS
Here you will learn about practices for the safe use of your chair, and how to avoid a fall or tip-over while you perform daily activities in your chair.
3. VII— WARNINGS— COMPONENTS & OPTIONS
Here you will learn about the components of your chair and options you can select for safety. Consult your Sunrise Medical authorized dealer and your health care adviser to help you choose the best set-up and options for safe use.

NOTE — Where they apply, you will also find “Warnings” in other sections of this manual. Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

V. GENERAL WARNINGS

A. WEIGHT LIMITS

WARNING

NEVER exceed the weight limit specified by Sunrise Medical. The weight capacity provided by your manufacturer is for the combined weight of a rider and items carried using on-board storage. If you do exceed the weight limit, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

Weight Capacities

Quickie IRIS	300 lbs / (136kg)
Quickie IRIS HD	400 lbs / (181kg)
Quickie IRIS with Z-Finity® Footrest	165 lbs / (75kg)
Zippie IRIS	225 lbs / (102kg)
Zippie IRIS with Z-Finity® Footrest	165 lbs / (75kg)

B.-INTENDED USE

The Quickie and Zippie IRIS Series of wheelchair's intended use is to provide mobility to persons limited to a sitting position.

WARNING

DO NOT use this device for purposes other than what is intended by the manufacturer.

1. The wheelchair is not designed for weight training and is unsafe for use as a seat while weight training. Weight training from the wheelchair substantially changes the stability of the chair and may cause tipping.
2. DO NOT stand on the frame of the wheelchair.
3. NEVER allow someone to stand on your chair or use it as a step ladder. This chair is designed for a single rider only.
4. Unauthorized modifications and use of parts or accessories not supplied or approved by Sunrise Medical may change the chair structure. This will void the warranty and may cause a safety hazard. If the warning is ignored, damage to your chair, and the potential severe injury of the person using the chair for unintended purposes can occur.

C. ATTENDANTS AND CAREGIVERS

WARNING

Before you assist a rider, be sure to read all warnings contained in this manual, and follow all instructions that apply. Be aware that after consulting a healthcare adviser, you will need to learn safe and proven body mechanics to use and create assistive methods best suited to your abilities.

D. ACCESSORIES

WARNING

Unauthorized modifications or use of parts or accessories not supplied or approved by Sunrise Medical may change the chair structure. This will void the warranty and may cause a safety hazard.

Some problems that may occur, but are not limited to:

1. Incorrect wheels and/or tires that put the rider at risk of a fall or tip-over.
2. Adding a component to the frame, changing the structural integrity of the chair.
3. Any modification or disassembly can potentially create an unsafe situation where rider and/or attendant are put at risk.

E. KNOW YOUR CHAIR

WARNING

Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding. Start slowly, with easy, smooth strokes. If you are used to a different chair, you may use too much force and tip over. If you use too much force, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

F. REDUCE THE RISK OF AN ACCIDENT

WARNING

1. BEFORE riding, you should be trained in the safe use of this chair by your health care adviser.
2. Practice bending, reaching, and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.
6. Sunrise recommends using anti-tip tubes in every circumstance.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

G. SAFETY CHECKLIST

WARNING

Before Each Use Of Your Chair:

1. Make sure the chair rolls easily and that all parts work smoothly. Check for noise, vibration, or a change in ease of use. (They may indicate low tire pressure, loose fasteners, or damage to your chair).
2. Immediately repair any problem. Your Sunrise Medical authorized dealer can help you find and correct the problem.
3. Check to see that both quick-release rear axles are locked. When locked, the axle button will "pop out" fully. If not locked, the wheel may come off and cause you to fall.
4. If your chair has anti-tip tubes, lock them in place.

If you fail to heed these warnings, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

H. CHANGES & ADJUSTMENTS

WARNING

1. If you modify or adjust this chair, it may increase the risk of a tipover UNLESS you make other adjustments as well.
2. Consult your Sunrise Medical authorized dealer BEFORE you modify or adjust your chair.
3. We recommend that you use anti-tip tubes until you adapt to the change and are sure you are not at risk to tip over.
4. Unauthorized modifications by the end-user, dealer or any other individual, including the use of parts not supplied or approved by Sunrise may change the chair structure. This will void the warranty and may cause a safety hazard.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

I. ENVIRONMENTAL CONDITIONS

⚠ WARNING



1. Use extra care if you must ride your chair on a wet or slick surface. If you are in doubt, ask for help.
2. Contact with water or excess moisture may cause your chair to rust or corrode. Avoid all extreme weather situations if possible.
 - a. Do not use your chair in a shower, pool, or other body of water. The chair tubing and parts are not water-tight and may rust or corrode from the inside.
 - b. Avoid excess moisture (for example, do not leave your chair in a damp bathroom while taking a shower).
 - c. Dry your chair as soon as you can if it gets wet, or if you use water to clean it.
 - d. Avoid contact with salt or salty conditions as this may cause damage to the chair.

J. TERRAIN

⚠ WARNING

1. Based on ANSI/RESNA testing, Sunrise Medical recommends the use of a caster wheel with a minimum diameter of 5-in (127mm), if the wheelchair will be overcoming obstacles up to 1/2-in (12.7mm) on a regular basis.
2. Your chair is designed for use on firm, even surfaces such as concrete, asphalt, indoor flooring, and carpets.
3. Do not operate your chair in sand, loose soil, or over rocky terrain.
4. If you use your chair on terrain that is rougher than described above there is a danger that screws and bolts will loosen prematurely, and that damage to wheels or axles could put the rider at risk of a fall, tip-over, or loss of control.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

K. STREET USE

⚠ WARNING

1. This product is not intended for street use.
2. Avoid streets whenever possible.
3. Obey and follow all legal pedestrian pathways, and laws that apply to pedestrians.
4. Be alert to the danger of motor vehicles in parking lots, or if you must cross a road.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

L. MOTOR VEHICLE SAFETY

1 2

If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the OEM vehicle restraint.



Sunrise Medical does provide a WC-19 Wheelchair Tie-Down and Occupant Restraint System, otherwise called a Transit Option. To identify whether your wheelchair has been manufactured with the Transit Option installed, look for four points of securement: two front points (A), and two rear points (B). These points are recognized by the securement point decal which can be found on or near the securement points. If your wheelchair is equipped with the Transit Option please review the Transit Securement supplement that was provided with the wheelchair for additional information and instructions. You can request a copy from your Sunrise Medical authorized dealer, or Sunrise Medical at 1-800-333-4000 if you don't have it.

⚠ WARNING

If your chair is NOT equipped with the Transit Option:

1. NEVER let anyone sit in this chair while in a moving vehicle.
 - a. ALWAYS move the rider to an approved vehicle seat.
 - b. ALWAYS secure the rider with proper motor vehicle restraints.
2. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.
3. NEVER transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. ALWAYS secure this chair so that it cannot roll or shift.
5. DO NOT use any wheelchair that has been involved in a motor vehicle accident. A sudden stop and/or collision may structurally damage your wheelchair. There may have been a change to the structure of the chair, and/or damaged or broken some of the components. Wheelchairs involved in sudden stops should be inspected for possible failures in frame and/or components. Frame damage may be represented by but not limited to: visual cracks, dents, metal distortion, bends, or damage to the seating mounting. If the chair no longer drives straight, it could be damaged. If the wheelchair has been involved in an accident, discontinue use immediately and contact your Authorized Dealer for a thorough inspection. If damage is questionable or if there is concern regarding the condition of the chair, Sunrise Medical recommends replacement of the chair.

NOTE - The warranty is void on any wheelchair that is involved in a collision.

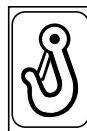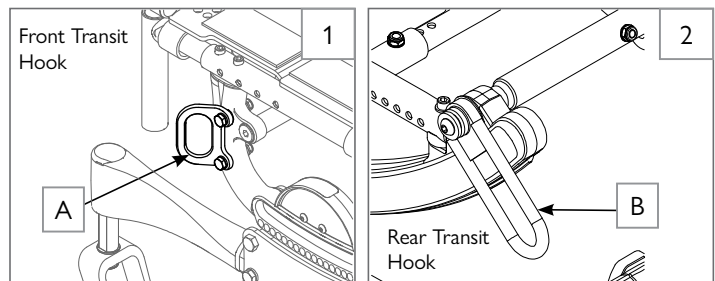

Look for this symbol on your wheelchair. It indicates wheelchair securement points which conform to ANSI/RESNA (1998) V.1 - Section 19 and/or ANSI/RESNA (2012) V.4 WC-19. For more Transit related information, see the "Transit Securement Supplement" included with your chair.

V. GENERAL WARNINGS

M. WHEN YOU NEED HELP

WARNING

For The Rider: Make sure that each person who helps you reads and follows all warnings and instructions that apply.

For Attendants:

1. Work with the rider's doctor, nurse, or therapist to learn safe methods best suited to your abilities and those of the rider.
2. Tell the rider what you plan to do, and explain what you expect the rider to do. This will put the rider at ease and reduce the risk of an accident.
3. Make sure the chair has push handles. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over. Check to make sure push handle grips will not rotate or slip off.
4. To prevent injury to your back, use good posture and proper body mechanics. When you lift or support the rider or tilt the chair, bend your knees slightly and keep your back as upright and straight as you can.
5. Remind the rider to lean back when you tilt the chair backward.
6. When you descend a curb or single step, slowly lower the chair in one easy movement. Do not let the chair drop the last few inches to the ground. This may damage the chair or injure the rider.
7. To avoid tipping, or getting hung up on objects, unlock and rotate anti-tip tubes up, and out of the way.
8. Whenever you aren't attending the wheelchair, ALWAYS use the wheel-lock to secure the rear wheels, and lock anti-tip tubes in place if you must leave the rider alone, even for a moment. This will reduce the risk of a tip over or loss of control of the chair.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

N. CHOKING HAZARD

WARNING

NOTE - Your chair consists of many small parts that could present a choking hazard to young children. Take care to put parts out of reach when doing any work on the chair.

O. PINCH POINT

WARNING

NOTE - Your wheelchair has many moving parts that can create pinch points and possible finger traps. Be aware when making any adjustments, when folding and unfolding, when moving, and any other situation that could cause a pinch point situation.

A. CENTER OF BALANCE**⚠ WARNING**

The point where this chair will tip forward, back, or to the side depends on its center of gravity and stability. How your chair is set up, the options you select and the changes you make may affect the risk of a fall or tip-over.

- 1. The Most Important Adjustment is:**
 - a. The position of the rear wheels. The more you move the rear wheels forward, the more likely your chair will tip over backward.
 - b. The position of the carriage.
- 2. The Center of Balance is also Affected by:**
 - a. A change in the set-up of your chair, including:
 - The seat height and seat angle.
 - Backrest angle.
 - b. A change in your body position, posture, or weight distribution.
 - c. Riding your chair on a ramp or slope.
 - d. The use of a backpack or other options, and the location and amount of added weight can change the balance of your chair.
- 3. To Reduce The Risk Of An Accident:**
 - a. Consult your doctor, nurse, or therapist to find out what axle and caster position is best for you.
 - b. Consult your authorized dealer, BEFORE you modify or adjust this chair. Be aware that you may need to make other changes to correct the center of balance.
 - c. Have someone help you until you know the balance points of your chair and how to avoid a tip-over.
 - d. Use anti-tip tubes.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

B. DRESSING OR CHANGING CLOTHES**⚠ WARNING**

Your weight may shift if you dress or change clothes while seated in chair.

To reduce the risk of a fall or tip-over:

1. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.
2. Lock anti-tip tubes in place. (If your chair does not have anti-tip tubes, back it up against a wall and lock both rear wheels).

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

C. OBSTACLES**⚠ WARNING**

Riding over curbs or obstacles can cause tipping and serious bodily harm. If you have any doubt that you can safely cross any curb or obstacle, ALWAYS ASK FOR HELP. Be aware of your riding skills and personal limitations. Develop new skills only with the help of a companion.

To avoid the risk of traversing obstacles:

1. Keep a lookout for danger – scan the area well ahead of your chair as you ride.
2. Make sure the floor areas where you live and work are level and free of obstacles.
3. Remove or cover threshold strips between rooms.
4. Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.
5. To Help Correct Your Center Of Balance:
 - a. Lean your upper body FORWARD slightly as you go UP over an obstacle.
 - b. Press your upper body BACKWARD as you go DOWN from a higher to a lower level.

6. If your chair has anti-tip tubes, do not go over an obstacle without help.
7. Keep both of your hands on the handrims as you go over an obstacle.
8. Never push or pull on an object (such as furniture or a doorjamb) to propel your chair.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

D. FRONT CASTER LIFT**⚠ WARNING**

Front caster lift can occur when the front wheels, ordinarily in contact with the ground, are either intentionally, or unintentionally caused to lift from the ground while the rear wheels remain in contact. Pitch control (partial or full) should NEVER be attempted without consulting your healthcare adviser and making sure that anti-tips are installed.

If you fail to heed this warning, you are at a high risk of damage to your chair, a fall, tip-over, or loss of control that may occur and cause severe injury to the rider or others.

E. REACHING OR LEANING**⚠ WARNING**

If you reach or lean it will affect the center of balance of your chair. This may cause you to fall or tip over. When in doubt, ask for help or use a device to extend your reach.

1. NEVER reach or lean if you must shift your weight sideways or rise up off the seat.
2. NEVER reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. NEVER reach with both hands (you may not be able to catch yourself to prevent a fall if the chair tips).
4. NEVER reach or lean to the rear unless your chair has anti-tip tubes locked in place.
5. DO NOT reach or lean over the top of the seat back. This may damage one or both backrest tubes and cause you to fall.
6. If You Must Reach Or Lean:
 - a. Do not lock the rear wheels. This creates a tip point and makes a fall or tip-over more likely.
 - b. Do not put pressure on the footrests.
 - c. Move your chair as close as you can to the object you wish to reach, then back up alongside it. Backing up will rotate the casters forward.
 - d. Do not try to pick up an object from the floor by reaching down between your knees. You are less likely to tip if you reach to the side of your chair.
 - e. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.
 - f. Firmly grasp a rear wheel or an armrest with one hand. This will help to prevent a fall if the chair tips.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

F. MOVING BACKWARD**⚠ WARNING**

Use extra care when you move your chair backward. Your chair is most stable when you propel yourself forward. You may lose control or tip over if one of the rear wheels hits an object and stops rolling.

1. Propel your chair slowly and smoothly.
2. If your chair has anti-tip tubes, make sure to lock them in place.
3. Stop often and check to be sure your path is clear.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

VI. SAFETY WARNINGS: FALLS AND TIP-OVERS

G. ESCALATORS

WARNING

NEVER use this chair on an escalator, even with an attendant. If you do, a fall or tip-over is likely.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

H. RAMPS, SLOPES, AND SIDEHILLS 3 4 5 6

WARNING

Whenever possible, avoid riding on a slope, which includes a ramp or sidehill. This will change the center of balance of your chair. Your chair is less stable and more difficult to maneuver when it is at an angle. When moving up a hill, anti-tip tubes may not prevent a fall or tip-over.

1. DO NOT use your chair on a slope steeper than 6°.
2. ALWAYS go as straight up and as straight down as you can. (Do not “cut the corner” on a slope or ramp.)
3. DO NOT turn or change direction on a slope.
4. When you have to use a ramp, always stay in the CENTER of the ramp. Ensure ramp is wide enough that you have no risk of going off the edge.
5. DO NOT stop on a steep slope. If you stop, you may lose control of your chair.
6. NEVER use rear wheel locks to try to slow or stop your chair. This is likely to cause your chair to veer out of control.
7. ALWAYS be aware of:
 - a. Wet or slippery surfaces.
 - b. A change in grade on a slope (or a lip, bump, or depression). These may cause a fall or tip-over.
 - c. A drop-off at the bottom of a slope or ramp. A drop-off as small as 3/4-in (19 mm) can stop a front caster and cause the chair to tip forward.
8. To reduce the risk of a fall or tip-over:
 - a. Lean or press your body UPHILL. This will help adjust for the change in the center of balance caused by the slope or sidehill.
 - b. Keep pressure on the handrims to control your speed on a down slope. If you go too fast you may lose control, and attempting to reduce speed using the handrim while riding at high speeds may cause heat by friction between the hand and the handrim.
 - c. ASK FOR HELP any time you are in doubt.
9. Ramps at home and work: For your safety, ramps at home and work must meet all legal requirements for your area:
 - a. AVOID A DROP-OFF. Ensure there is a section at the top or bottom to smooth out the transition.
 - b. ALWAYS stay in the center of the ramp and control your speed.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and can possibly cause severe injury to the rider or others.

For attendants:

Follow these steps to help the rider go up or down a ramp:

1. Stay behind the chair.
2. Firmly grasp the handles.
3. For going up or down a ramp, the chair should always be facing forward.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.




IF you need to go down a hill:

- a. The Downslope should be less than 6°.
- b. GO SLOW. Control your speed.
- c. ALWAYS go straight down.
- d. NEVER turn on a hill.
- e. ALWAYS lean backward.




IF you need to go up a hill:

- a. The Upslope should be less than 6°.
- b. ALWAYS go straight up.
- c. DO NOT STOP.
- d. NEVER turn on a hill.
- e. ALWAYS lean forward.

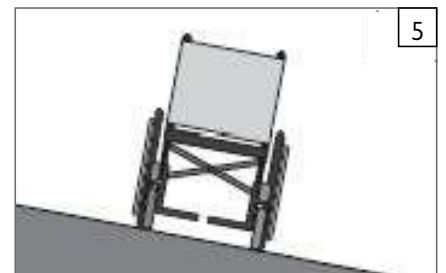

IF you need to traverse a sidehill:

- a. Sidehill must be less than 6° or you are risking a tip over.
- b. DO NOT STOP.
- c. NEVER turn on a hill.

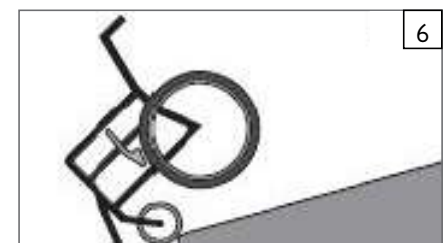

IF you need to traverse a ramp:

- a. Go slowly.
- b. Watch out for drop-offs.
- c. ALWAYS go straight up or straight down.
- d. Stay centered on the ramp.

I. TRANSFERS 7 8

⚠ WARNING

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you.

To avoid a fall:

1. Work with your health care provider to learn safe methods for transfers.
 - a. Learn how to position your body and how to support yourself during a transfer.
 - b. Have someone help you until you learn safe transfer methods.
2. Lock the rear wheels before you transfer.
3. Be aware that the chair can still slide and/or tip. The wheel lock keeps the rear wheels from rolling while you are performing the transfer.
4. Ensure that the pneumatic tires are properly inflated. Low tire pressure may allow the rear wheel locks to slip. (See table in Section VII: H "Pneumatic Tires")
5. Move your chair as close as you can to the seat you are transferring to. If possible, use a transfer board.
6. Rotate the front casters until they are as far forward as possible.
7. If you can, remove the footrests, or swing them out of the way.
 - a. Ensure your feet do not catch in the space between the footrests.
 - b. Avoid putting weight on the footrests as this may cause the chair to tip.
8. Ensure armrests are removed, or out of the way and do not interfere with the transfer.
9. Transfer as far back onto the seat surface as you can. This will reduce the risk that the chair will tip or move away from you.


If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and can possibly cause severe injury to the rider or others.

Optimum Transfer position



Before transferring:

1. Flip-back, or remove the armrests out of the way.
2. Swing, or remove the footrests out of the way (if applicable).
3. Ensure the wheel lock is set.

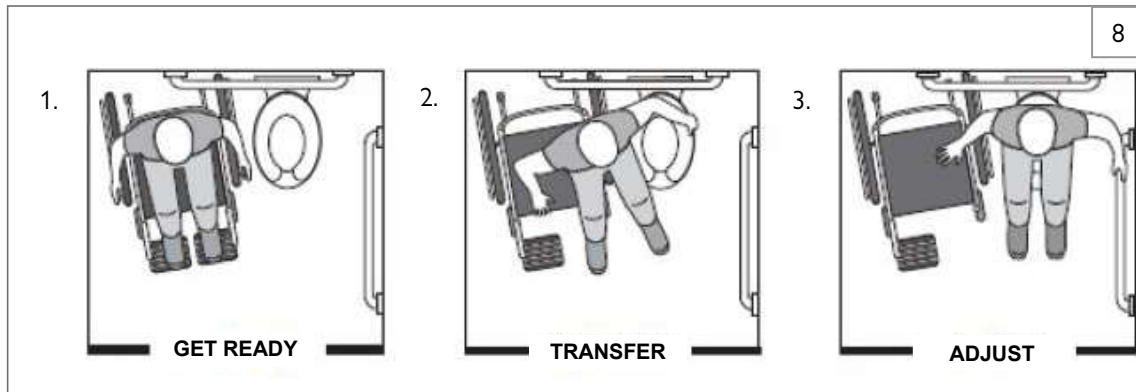

VI. SAFETY WARNINGS: FALLS AND TIP-OVERS

J. CURBS AND SINGLE STEPS 9


WARNING

Before riding over curbs, and negotiating even a single step, ALWAYS ask for assistance first. Curbs and steps can cause tipping and serious bodily harm. When in doubt as to your ability to avoid, or traverse any obstacle, always ask for help. Be aware of your riding skills and personal limitations. Develop new skills only with the help of a companion.

For Attendant: Each person who assists the rider with curbs and steps should read and follow all instructions and warnings pertaining to attendants and caregivers.

1. Do not try to climb a high curb or step (more than 4 inches high (102mm)) UNLESS you have help. Doing so may cause your chair to exceed its balance point and tip over.
2. Go straight up and straight down a curb or step. If you climb or descend at an angle, a fall or tip-over is likely.
3. Be aware that the impact of dropping down from a curb or step can damage your chair or loosen fasteners.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and can possibly cause severe injury to the rider or others.

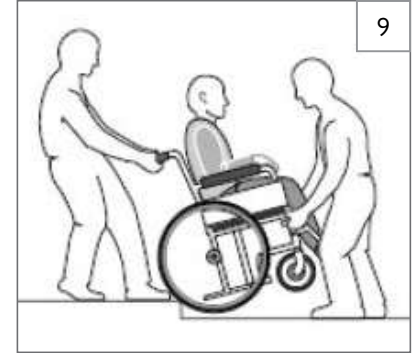

K. CLIMBING A CURB OR SINGLE STEP 9

WARNING

For Attendant: follow these steps to help the rider climb a curb or single step going BACKWARD:

1. Stay behind the chair.
2. Continue backward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the top of the curb.
3. Pull the chair backwards until the caster wheels have cleared the edge of the curb and return the chair to its rolling position.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

L. DESCENDING A CURB OR SINGLE STEP 9

WARNING

For Attendant: Follow these steps to help a rider descend a curb or single step going FORWARD:

1. Stay at the rear of the chair.
2. Several feet before you reach the edge of the curb or step, tip the chair slightly and pull it backward.
3. When the chair is at its balance point, carefully step forward until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
4. Push the chair forward until you are standing on the lower level.
5. When the rear wheels are safely on the lower level, tilt the chair back to its balance point.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

M. STAIRS 10**⚠ WARNING**

1. NEVER use this chair on stairs UNLESS you have someone to help you. Doing so is likely to cause a fall or tip-over.
2. Negotiating stairs with a wheelchair always requires at least two attendants for safety.
3. NEVER lift this chair by its armrests. They may come loose or break.
4. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

N. CLIMBING STAIRS**⚠ WARNING**

1. Use at least two attendants to move a chair and rider up stairs.
2. Move the chair and rider BACKWARD up the stairs.
3. The person at the rear is in control. They tilt the chair back to its balance point.
4. A second attendant at the front firmly grasps a non-detachable part of the front frame and lifts the chair up and over one stair at a time.
5. The attendants move to the next stair up. Repeat for each stair, until you reach the landing.

O. DESCENDING STAIRS**⚠ WARNING**


1. Use at least two attendants to move a chair and rider down stairs.
2. Move the chair and rider FORWARD down the stairs.
3. The person at the rear is in control. They tilt the chair to the balance point of the rear wheels and roll it to the edge of the top step.
4. A second attendant stands on the third step from the top and grasps the chair frame. They lower the chair one step at a time by letting the rear wheels roll over the stair edge.
5. The attendants move to the next stair down. Repeat for each stair, until you reach the landing.

P. TILT USE AND BACK ANGLE ADJUSTMENT (MONO BACK)**⚠ WARNING**

BEFORE you operate the tilt mechanism, or back angle adjustment (MONO back):

1. Always verify that the arms of the rider are stable on the armrests or within the armrests.
2. Always verify that the legs of the rider are stable on the footrest/legrest.
3. Never place hands, feet, or foreign objects into the tilt mechanism.
4. Never push or propel the chair with the seat tilted in a forward position.
5. Never add chair accessories that are not specifically designed for the IRIS.
6. Never exceed a tilted position that aligns the backrest parallel to the ground.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

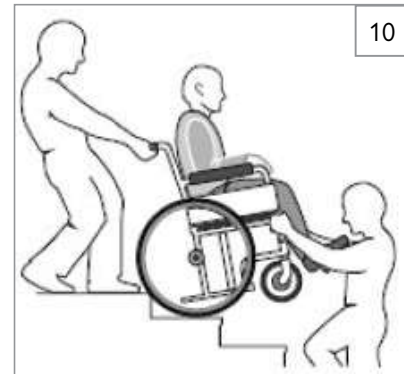

VII. WARNINGS: COMPONENTS AND OPTIONS

A. ANTI-TIP TUBES (OPTIONAL)

WARNING

Anti-tip tubes can help keep your chair from tipping over backward in normal conditions.

1. Sunrise recommends the use of anti-tip tubes:
2. When locked in place (in the “down” position) anti-tip tubes should be BETWEEN 1 1/2 to 2-in (38 to 51mm) off the ground.
 - a. If set higher than 2-in (51mm), they may not prevent a tip-over.
 - b. If set lower than 1 1/2-in (38mm), they may “hang up” on obstacles and cause a fall or tip over.
3. If you have to climb or descend a curb, or overcome an obstacle, it may be necessary to have an attendant make sure the anti-tip tubes are rotated up, and out of the way, so that the chair and rider do not get stuck and/or become unstable.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

B. ARMRESTS

WARNING

Armrests detach and will not bear the weight of this chair.

1. NEVER lift this chair by its armrests. They may come loose or break.
2. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

C. PUSH HANDLES

WARNING

When you have an attendant, make sure that this chair has push handles.

1. Push handles provide secure points for an attendant to hold the rear of this chair, to prevent a fall or tip-over. Make sure to use push handles when you have an attendant.
2. Check to make sure push handle grips will not rotate or slip off.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

D. STROLLER HANDLE

WARNING

1. Always keep hands away from the locking mechanism located at the bottom of the back canes when folding the back down or pulling it back up.
2. Always keep fingers away from the interior of the hinge when readjusting the adjustable stroller handle.

E. CUSHIONS

WARNING

1. Quickie standard foam cushions are not designed for the relief of pressure.
2. If you suffer from pressure sores or if you are at risk that they will occur, you may need a special seat system or a device to control your posture. Consult your doctor, nurse, or therapist to find out if you need such a device for your well-being.
3. Seat pans are not intended to be used as a direct seating surface. A cushion or other seating surface should be placed on the seat pan before use.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

F. FASTENERS

WARNING

Many of the screws, bolts, and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. ONLY use fasteners provided by a Sunrise Medical authorized dealer (or ones of the same type and strength, as indicated by the markings on the heads).
2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can.

NOTE: For safety and optimum performance it is best to not reuse fasteners.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

G. FOOTRESTS

WARNING

1. At the lowest point, footrests should be AT LEAST 2 inches (51 mm) off the ground. If set too LOW, they may “hang up” on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.
2. To Avoid A Trip Or Fall When You Transfer:
 - a. Make sure your feet do not “hang up” or get caught in the space between the footrests.
 - b. Avoid putting weight on the footrests, as the chair may tip forward.
 - c. Footrests should be swung out of the way or removed whenever entering or exiting the wheelchair.
3. NEVER lift this chair by the footrests. Footrests detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

H. PNEUMATIC TIRES

WARNING

Proper inflation extends the life of your tires and makes your chair easier to use.

1. Do not use this chair if any of the tires is under- or over-inflated. Check weekly for proper inflation level, as listed on the tire sidewall.
2. Low pressure in a rear tire may cause the wheel lock on that side to slip and allow the wheel to turn when you do not expect it.
3. Low pressure in any of the tires may cause the chair to veer to one side and result in a loss of control.
4. Over-inflated tires may burst.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

Recommended Tire Pressures			Recommended Pressure or Pressure Range	
Tires	Quickie IRIS	Zippie IRIS	kPa	PSI
Pneumatic (12-in) (305mm)	x	x	276	40
Pneumatic (16-in) (406mm)	x	x	241	35
Pneumatic (20-in) (508mm)	x	x	448	65
Pneumatic (22-in) (559mm)	x	x	448	65
Pneumatic (24-in) (610mm)	x	x	517	75

I. POSITIONING BELTS (OPTIONAL)**⚠ WARNING**

Use positioning belts **ONLY** to help support the rider's posture. Improper use of these belts may cause severe injury to or death to the rider.

1. Make sure the rider does not slide down in the wheelchair seat. If this occurs, the rider may suffer chest compression or suffocate due to pressure from the belts.
2. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open hand, flat, between the belt and the rider.
3. A pelvic wedge or a similar device can help keep the rider from sliding down in the seat. Consult with the rider's doctor, nurse, or therapist to find out if the rider needs such a device.
4. Use positioning belts only with a rider who can cooperate. Make sure the rider can easily remove the belts in an emergency.
5. NEVER Use Positioning Belts:
 - a. As a patient restraint. A restraint requires a doctor's order.
 - b. On a rider who is comatose or agitated.
 - c. As a motor vehicle restraint. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

J. QUICK-RELEASE AXLES**⚠ WARNING**

1. Do not use this chair **UNLESS** you are sure that both quick-release rear axles are locked.
2. An axle is not locked until the quick-release button pops out fully. If the axle is not inserted fully, the wheel may come off during use, endangering the rider.
3. Quick-release axles should be periodically cleaned and inspected for function, and signs of wear or bending. Replace as necessary.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

K REAR AXLES**⚠ WARNING**

A change in set-up of the rear wheels will affect the center of balance of your chair.

1. The farther you move the rear axles **FORWARD**, the more likely it is that your chair will tip over backward.
2. Consult your doctor, nurse, or therapist to find the best rear axle set-up for your chair. Do not change the set-up **UNLESS** you are sure you are not at risk to tip over.
3. Adjust the rear wheel locks after you make any change to the rear axles.
 - a. If you fail to do so, the locks may not work.
 - b. Make sure lock arms embed in tires at least 1/8 inch (3 mm) when locked.

If you fail to heed these warnings, damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

L. REAR WHEEL LOCKS**⚠ WARNING**

Rear wheel locks are **NOT** designed to slow or stop a moving wheelchair. Use them only to keep the rear wheels from rolling when your chair is at a complete stop.

1. NEVER use rear wheel locks to try to slow or stop your chair when it is moving. Doing so may cause a fall or tip-over.
2. To keep the rear wheels from rolling, always set both rear wheel locks when you transfer to or from your chair.
3. Low pressure in a rear tire may cause the wheel lock on that side to slip and may allow the wheel to turn when you do not expect it.
4. Make sure lock arms embed in tires at least 1/8 inch (3 mm) when locked. If you fail to do so, the locks may not work.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

M. MODIFIED SEAT SYSTEMS**⚠ WARNING**

Use of a seat system not approved by Sunrise may alter the center of balance of this chair. This may cause the chair to tip over.

1. Do not change the seat system of your chair **UNLESS** you consult your Sunrise Medical authorized dealer first.
2. Use of a seating system not provided by Sunrise is prohibited for transit use.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

N. TRAY FOR VENT AND BATTERY

NOTE – Vent tray only available on the Quickie IRIS.

NOTE – This option is only compatible with wheelchairs having a minimum floor-to-seat height of 15-in (38 cm).

NOTE – The battery tray components are not compatible with the Attendant Wheel Lock option or Foot Release Tilt.

⚠ WARNING

Never use this wheelchair as a seat in a motor vehicle unless all components are removed from the battery tray, and properly secured separately in the vehicle.

⚠ WARNING

Always contact Technical Service before installing or removing the battery tray components to obtain the correct assembly locations of the strut tubes, roller carriage assembly, and caster brackets.

⚠ WARNING

The combined weight of the user plus the ventilator, battery, and tray components should not exceed 225 lbs (102 kg) on the Zippie IRIS, 250 lbs (114 kg) on the Quickie IRIS, 350 lbs (159 kg) on the heavy duty option, and 165 lbs with Z-Finity footrest option.

O. ACCESSORY HOOK**⚠ WARNING**

The weight limit for the accessory hook is 10 lbs (4.5 kg) (total) for all items.

If you fail to heed these warnings damage to your chair, a fall, tip-over, or loss of control may occur and cause severe injury to the rider or others.

VIII. USE AND MAINTENANCE

WARNING

The owner and/or caregiver for the use of this product, is responsible for making sure that it has been setup and adjusted by a trained service professional under the advice of a healthcare adviser. The chair may require periodic maintenance or certain in-use adjustments that may be performed by the owner or caregiver.

A. INTRODUCTION

- Proper maintenance will improve performance and extend the useful life of your chair.
- Clean your chair regularly. This will help you find loose or worn parts and make your chair easier to use. You will need a mild detergent solution and plenty of cleaning rags.
- If discovered, repair or replace loose, worn, bent, or damaged parts before using the chair.
- To protect your investment, have all major maintenance and repair work done by your Sunrise Medical authorized dealer.
- Inspect and maintain this chair strictly per the maintenance chart.
- If you detect a problem, make sure to order parts, or have service, and repair work done at your Sunrise Medical authorized dealer before use.
- At least once per year, have a complete inspection, safety check, and service of your chair made by a Sunrise Medical authorized dealer.

B. CRITICAL MAINTENANCE TIPS

1. Tire Air Pressure:

Check air pressure in pneumatic tires at least ONCE PER WEEK. The wheel locks will not grip properly if you fail to maintain the air pressure shown on tire sidewall.

2. Axles & Axle Sleeves:

When checking axles and axle sleeves every six months, ensure they are clean and tight.

Loose sleeves will damage the axle plate and will affect performance.

C. SAFETY CHECKLIST

Weekly	Monthly	3 Months	6 Months	Annually	Safety and Function Check
<input checked="" type="checkbox"/>					Tire inflation level
<input checked="" type="checkbox"/>					Wheel locks
		<input checked="" type="checkbox"/>			Visually check for loose hardware
			<input checked="" type="checkbox"/>		Armrests
			<input checked="" type="checkbox"/>		Axle and axle sleeves
		<input checked="" type="checkbox"/>			Quick-release axles
		<input checked="" type="checkbox"/>			Wheels, tires, and spokes
		<input checked="" type="checkbox"/>			Casters
		<input checked="" type="checkbox"/>			Anti-tip tubes
			<input checked="" type="checkbox"/>		Frame, camber tubes, and crossbrace
	<input checked="" type="checkbox"/>				Upholstery
<input checked="" type="checkbox"/>					Rollers
<input checked="" type="checkbox"/>					Tilt cables and plunger
<input checked="" type="checkbox"/>					Rocker arm
				<input checked="" type="checkbox"/>	Service by Sunrise Medical authorized dealer

NOTE – The user or caregiver should perform these weekly and monthly checks to maintain the safety of the chair. If an item is not working properly, please contact your Sunrise Medical authorized dealer.

D. CLEANING

- Paint Finish
 - Clean the painted surfaces with mild soap or detergent at least once a month.
 - Protect the paint with a coat of non-abrasive auto wax every three months.
- Axles and Moving Parts
 - Clean around axles and moving parts WEEKLY with a slightly damp (not wet) cloth.
 - Wipe off or blow away any fluff, dust, or dirt on axles or moving parts.
- Upholstery
 - Hand-wash (machine washing may damage fabric).
 - Drip-dry only. DO NOT machine dry as heat will damage fabric.
- Basic Maintenance Materials available at local store
 - Tire Pump
 - Tire pressure gauge
 - Clean rags or cotton cloth
 - Teflon-based Lubricant
 - Mild Detergent

NOTE– You do not need to grease or oil the chair.

E. TROUBLESHOOTING CHART

Symptoms	Left turn in chair	Right turn in chair	Looseness in chair	Sluggish turning	Squeaks and rattles	Caster flutter	Chair drops fast into tilt and is hard to return.	Chair is hard to tilt and returns too fast.	Solution
	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>					Ensure tire pressure is correct and equal in both rear tires and front caster tires, if pneumatic.
			<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>			Ensure all nuts and bolts are tight.
			<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>				Ensure all spokes and nipples are tight on radial spoke wheels.
									Use Tri-Flow Lubricant (Teflon®-based) between frame connections and parts.
	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>			Check for proper caster fork adjustment. See instructions for caster fork adjustment.
	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>				<input checked="" type="checkbox"/>			Make sure both front casters touch the ground simultaneously. If they do not, add the proper spacers between the bottom bearing of the caster plate and fork stem nut until they do. Make sure you check for this problem on a flat surface.
							<input checked="" type="checkbox"/>		Consult with an authorized dealer to adjust CG/CR forward
								<input checked="" type="checkbox"/>	Consult with an authorized dealer

F. TO MOUNT AND REMOVE REAR WHEELS 11**⚠ WARNING**

Do not use this chair UNLESS you ensure both quick-release axles are locked. An unlocked axle may come off during use and cause a fall.

NOTE – Setting the wheelchair on a flat surface, such as a workbench or table, helps make these procedures easier.


1. To Install Wheel

- Depress quick-release button (A) fully. This will release tension on ball bearings at other end.
- Insert axle (B) through hub of rear wheel.
- Keep button (A) depressed as you slide axle (C) into axle sleeve (B).
- Release button to lock axle in axle sleeve. Adjust axle if it does not lock.
- Repeat steps on the other side.

Note: The axle is not locked until the quick-release button pops out fully (A). Check that the axle is locked by pulling on the wheel in the direction of the axle.

2. To Remove Wheel

- Depress quick-release button (A) fully.
- Remove wheel by sliding axle (C) completely out of axle sleeve (B).
- Repeat steps on the other side.

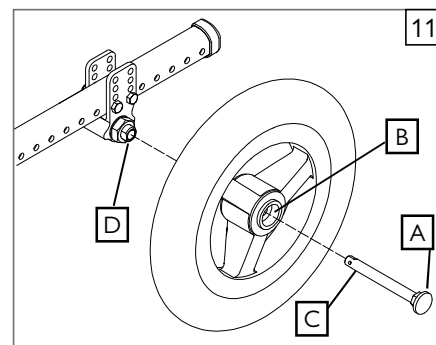**G. CUSHION INSTALLATION**


Place cushion on seat pan with hook material side down. The beveled edge of the cushion should be in front, press it firmly into place.

H. WHEEL LOCKS**⚠ WARNING**

Wheel locks are installed at Sunrise Medical and should be adjusted by your Sunrise Medical authorized dealer. Inspect wheel locks weekly per the maintenance chart. Do not use your chair UNLESS you are sure both wheel locks can fully engage. A wheel lock that is not correctly adjusted may allow your chair to roll, or turn unexpectedly. Wheel locks must be adjusted after making sure the tires have the correct air pressure. When fully engaged, the arm should be imbedded into the tire at least 1/8-in (3mm) to be effective. If you find the wheel locks have slipped or are not working correctly contact your Sunrise Medical authorized dealer for proper adjustment.

I. HUB LOCK 12**1. To operate Hub Lock**

- To lock rear wheels, press button (E). Lever (F) will flip outward. Do not engage hub lock when chair is in motion.
- To unlock wheels, press Lever (F) back down until button (E) pops back out. Each lever operates one side of the chair. To fully lock chair, both buttons must be pushed.

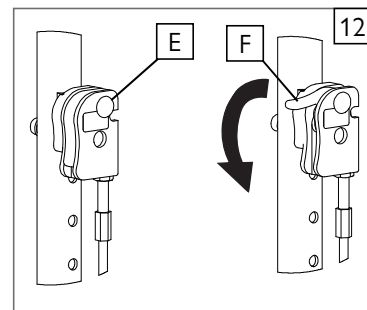**J. ANTI-TIP TUBES - REAR** 13 14

Anti-tip tubes are recommended for all wheelchairs. Do not attempt to overcome obstacles on your own.

1. Inserting Anti-Tip Tubes Into Receivers

- Press in the rear anti-tip button (G) on the anti-tip so that both buttons are drawn inside.
- Insert the anti-tip tube into the receiver (H).
- Turn the anti-tip tube down until front release button pops through the receiver mounting hole.
- Insert second anti-tip tube the same way.

2. Turning Anti-Tip Tubes Up



Turn the anti-tip tubes up when being pushed by attendant, and overcoming obstacles or climbing curbs.

- Press in the rear anti-tip tube release button (G).
- Hold button (G) in and turn anti-tip tube up (I).
- Release the button.
- Repeat with second anti-tip tube.
- Remind attendant to return anti-tips to the down position after completing the maneuver.

3. Adjusting Anti-Tip Tube Wheel

The anti-tip tube wheels may have to be raised or lowered to achieve proper ground clearance (1 1/2-in to 2-in) (38 to 51mm)).

- Press the anti-tip wheel release button (J) so that the release pin is drawn inside.
- Raise or lower to one of the three or six pre-drilled holes.
- Release the button (J).
- Adjust the second anti-tip tube wheel the same way. Both wheels should be at exactly the same height.

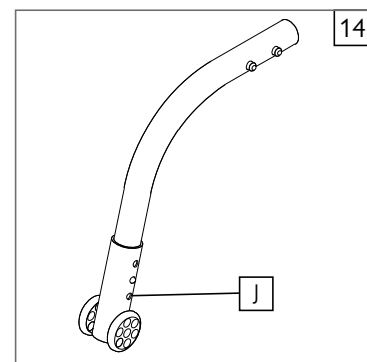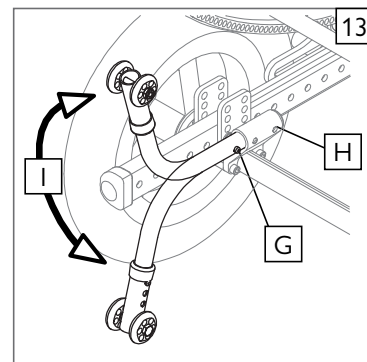

VIII. USE AND MAINTENANCE

K. DUAL-POST ARMRESTS 15 16

1. Installation or Removal

- a. The assembly is held securely in place when the securing lever (A) is facing outside.
- b. Pull front securing lever (A) forward to release.
- c. Insert or remove armrest.
- d. Return securing lever to locked position.


2. Height-Adjustment (optional)

- a. Release the upper securing lever (C).
- b. Set armrest at desired height.
- c. Return securing lever to locked position.
- d. Move armrest up or down to allow armrest to snap into place.

3. Flip Back (optional)


- a. Release the front securing lever only (A).
- b. Flip the armrest back, it remains attached to the rear receiver.
- c. To detach armrest completely pull rear securing lever (B) up.
- d. Move armrest down into place.
- e. Return securing lever to locked position.

Flip-Back receivers

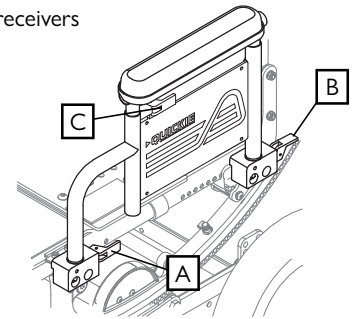

15

Fixed receivers

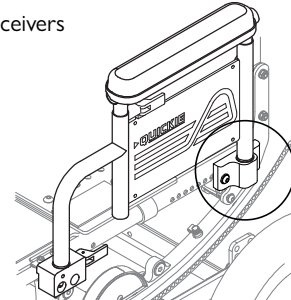

16

L. HEIGHT-ADJUSTABLE ARMRESTS (optional) 17

1. Installation

- a. Slide the outer armpost (1) into the receiver (7) mounted to the wheelchair frame.
- b. The armrest will automatically lock into place.


2. Height Adjustment

- a. Rotate release lever (2) to second stop.
- b. Slide armrest pad (4) up or down to desired height.
- c. Return lever (2) to locked position against armpost.
- d. Push arm pad down until upper armpost locks firmly into place.

3. Removing Armrest

- a. Grip release lever (3) and pull up to remove the armrest.

Height Adjustable Armrests

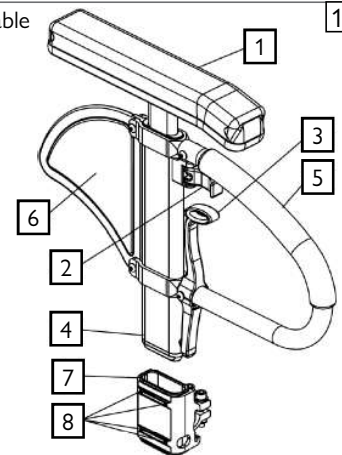

- | | |
|-------------------------|---------------------------------|
| 1. Outer armpost | 6. Side panel |
| 2. Release lever height | 7. Receiver |
| 3. Release lever | 8. Receiver adjustment hardware |
| 4. Armrest pad | |
| 5. Transfer bar | |


17

M. ADJUSTABLE LOCKING FLIP-UP ARMRESTS 18

1. Flip up and down

- a. Release the upper securing lever (2).
- b. Flip the armrest back (1), it remains attached to the back post.
- c. Move armrest down into place.
- d. The lever (2) locks itself in position automatically.

Adjustable Locking Flip-Up

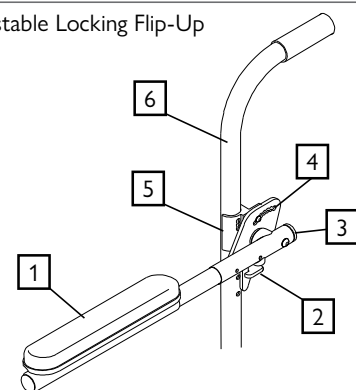

- | | |
|-------------------------|-------------------------|
| 1. Armrest pad | 4. Angle adjust bracket |
| 2. Upper securing lever | 5. Clamp |
| 3. Flip-back tube | 6. Back cane |

18


N. CANTILEVER ARMRESTS 19**1. Flip Back the Armrest for Access and Transfers.**

- Release the armrest (A) by pulling up on the securing lever (this will unlock the armrest).
- Rotate the armrest back.

2. Return The Armrest to The Riding Position

- Rotate the armrest back to riding position.
- Be sure to lock the armrest by returning the securing lever to its locked position.


NOTE – Figure shows lever (A) in locked position.

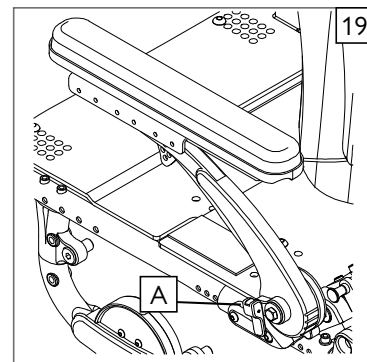**O. SWING-AWAY HANGERS/FOOTRESTS** 20**Swing-in-Swing-out Hangers and Footrests (60°, 70°, 80°)****1. Installation**

- Place swing-in/swing-out pivot saddle into the receiver (B) on front frame tube with the footrest facing inward or outward from the frame.
- Rotate the footrest inward until it locks into place on locking plate.

2. Removal


- To remove footrest, release latch (C) by pulling lever upward or pushing lever downward.
- Rotate footrest inward or outward and lift.

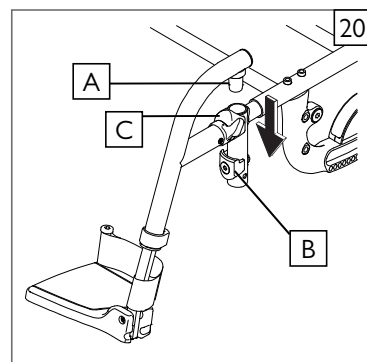**P. HEAVY-DUTY, LIFT-OFF FOOTRESTS (optional)** 21**1. Installation**

- Slide pivot saddle (D) into receiver (E) on front frame tube until it locks into place.

2. Removal

- To remove footrest, push release latch (F) while lifting up footrest from receiver.

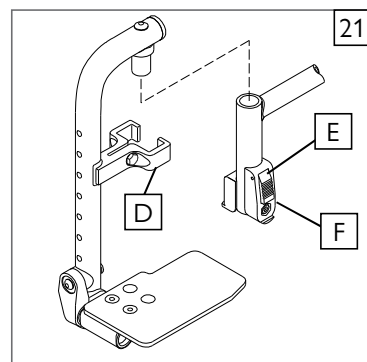**Q. ARTICULATING LEGREST (optional)** 22**1. Installation or Removal**


- To install or remove Articulating Legrest (ALR) see instructions for Swing-away Footrest installation.

2. Elevation Adjustment

- To raise legrest, lift to desired position. Legrest will automatically lock in place.
- To lower legrest, while seated in chair, press release lever (G) down and lower legrest to desired position. Legrest will automatically lock in place.

3. Height Adjustment of the Calf Pad

- Rotate the tube clamp (H) towards the outside.
- Slide calf pad assembly up or down.
- Rotate calf pad assembly (I) back in the desired indented position.

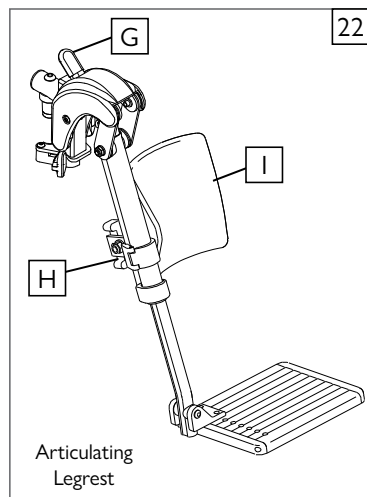


VIII. USE AND MAINTENANCE

R. ELEVATING LEGREST (optional) 23**1. Installation or Removal**

To install or remove Elevating Legrest (ELR) see instructions for Swing-away Footrest installation.


2. Elevation Adjustment

- a. To raise legrest, lift to desired position. Legrest will automatically lock in place.
- b. To lower legrest, while seated in chair, press release lever (A) down and lower legrest to desired position. Legrest will automatically lock in place.

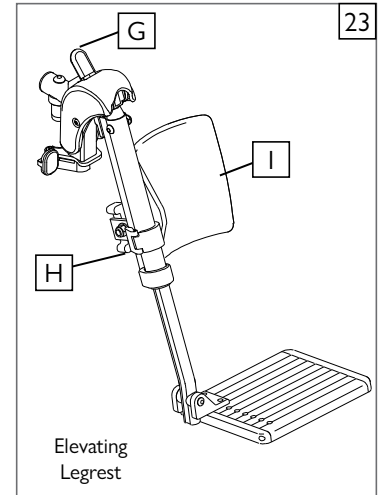**S. Z-FINITY® FOOTREST SYSTEM** 24 25

CAUTION – The Z-finity Footrest system is rated to 165 lbs (75kg)

With the footrest system, Sunrise Medical offers either a single Flip-up Footplate (B), or dual footplates (C) that are able to accommodate all of the possible heights and adjustment angles that the rider may require for their comfort and support.

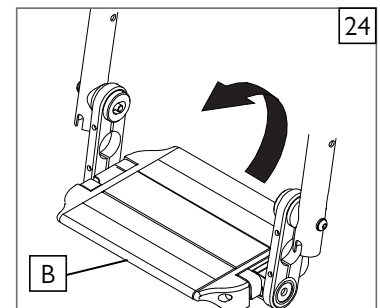**T. Z-FINITY® SWING-AWAY FOOTREST** 25 26

The footrest system can also accommodate a swing-away hanger (D) with dual footplates (C) that are both height adjustable and angle adjustable.



Swing-in-Swing-out Hangers and Footrests (80°)

1. Basic Footrest Installation

- a. From the side, place the pivot (E) into the top of the frame tube which guides the hanger into the correct alignment with the locking plate (F) on front frame tube.
- b. Rotate the footrest towards the front of the chair until it locks into place on locking plate.

2. Removal

- a. To remove footrest, release latch (G) by pulling lever upward or pushing lever downward.
- b. Rotate footrest inward or outward and lift.

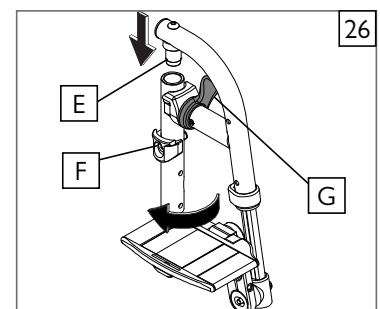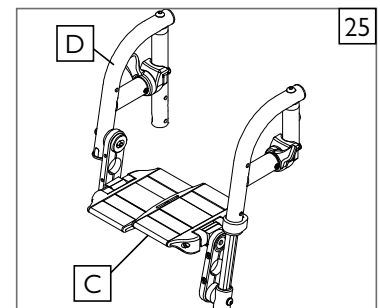


U. TILT-IN-SPACE MECHANISM 27

The tilt-in-space mechanism is installed by Sunrise. Two cable/trigger mechanisms positively lock the chair in place from -5° to 60° depending on the set-up.


The Quickie/Zippie IRIS has the capability to provide from 0° to 5° of forward tilt. DO NOT ATTEMPT TO MAKE THIS ADJUSTMENT. This adjustment must be done by a Sunrise Medical authorized dealer.

1. Trigger Tilt Operation


- Depressing both triggers (A) at the same time disengages the locking mechanism and allows the seat and back to rotate in space. Releasing the triggers positively locks the rockers in place.
- The approximate degree of tilt the seat is at can be seen on the tilt indicator (C).
- To lock out the tilting function, insert the locking pin (B) into the trigger holder.

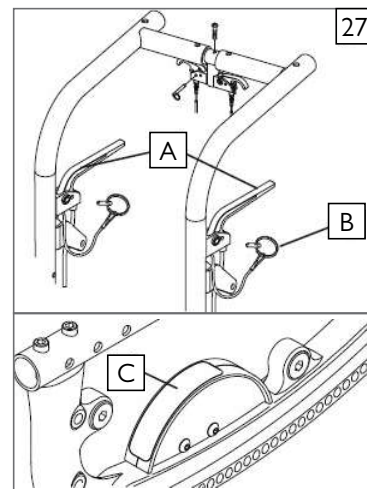**2. Foot Release** 28

- Depressing foot pedal (D) disengages the locking mechanism and allows the seat and back to rotate in space. Releasing the foot pedal positively locks the rockers in place.
- The approximate degree of tilt the seat is at can be seen on the tilt indicator (C).

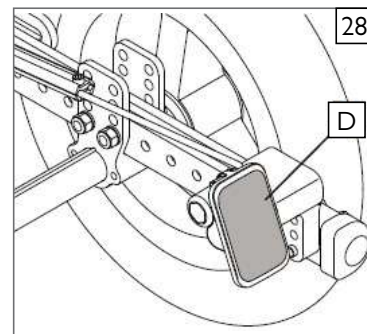**V. STROLLER HANDLE EXTENSION (option)** 29**Installation**

- Press button (E) on the stroller handle so that release pins are drawn inside the tube.
- Insert the tube into the back tube receiver (F) until the release pin protrudes through the receiver mounting hole.

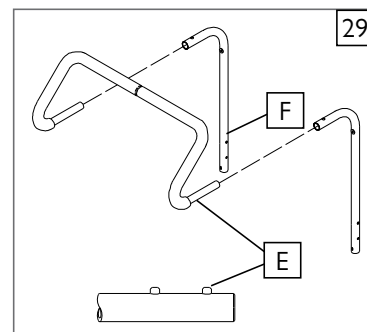
**W. RECLINING BACK** 30

The dual trigger reclining mechanism is installed by Sunrise.

To Recline or Raise Backrest

- Depress both triggers (G) at the same time. Releasing the triggers positively locks the backrest in place within its 60° range.
- To lock out the adjustment system, insert the locking pin (H) into the trigger holder.

NOTE – ALWAYS lock the dual trigger reclining mechanism securely in place after positioning the backrest.

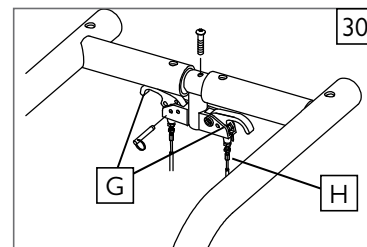


VIII. USE AND MAINTENANCE

X. MONO BACK STROLLER HANDLE 31

Height/Angle Adjustment

- a. Press button (A) on the stroller handle so that it releases the push handle.
- b. Adjust the push handles to a comfortable position.

NOTE- ALWAYS lock push handles by moving them slightly being sure to hear the mechanism click into the locked position.

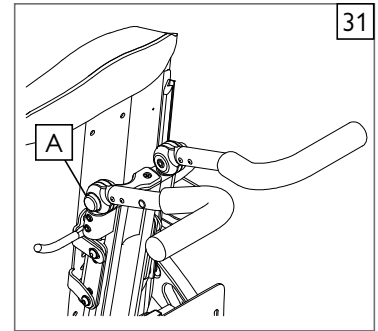


Y. FOLDING THE MONO BACK FOR TRANSPORT 32

Height/Angle Adjustment

- a. Push down on the release lever (B) and fold the backrest forward for easier transport of the chair.
- b. To return the back to the riding position just pull up on the back until it locks into place at the original position.

NOTE - AFTER FOLDING OR UNFOLDING THE MONO BACK, ALWAYS check to make sure it is in the locked position by moving it slightly and listening for the mechanism to click into the locked position.

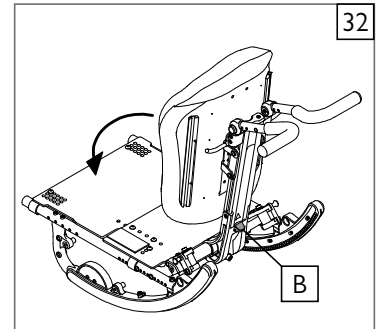



Z. FOLDING / UNFOLDING FRAME (optional) 33 34

1. Unfolding

- a. Attach the wheels and make sure they are locked in the axle sleeves.
- b. Push down on folding struts (C) until they lock. Make sure lock handle (D) is down and double check that the frame is locked by pushing inwards on the frames.
- c. Add the seating system and footrests.

2. Folding

- a. Remove the rider, seating system, and flip-up the footplates. Disengage the wheel-locks.
- b. Unlock the mechanism by pulling up on the lock handle (D), then using both hands, grasp the folding struts while pushing on the locking lever release buttons (E) with your thumbs.
- c. Once the chair starts to fold, push the frames together until the chair is fully folded.
- d. Remove the wheels using the quick release buttons. The chair is ready for transport or storage.

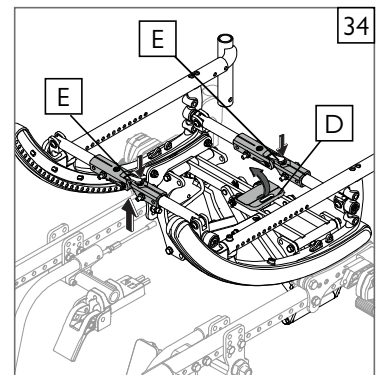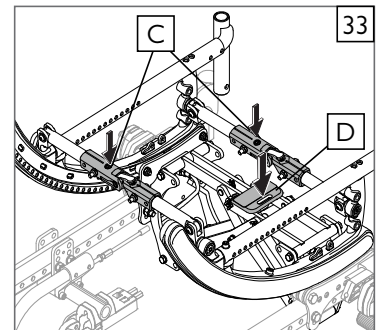

AA. INSPECT

Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures:

1. Review assembly and adjustment sections to make sure the chair was properly prepared.
2. Review the troubleshooting guide.
3. If your problem persists, contact your Sunrise Medical authorized dealer. If you still have a problem after contacting your authorized dealer, contact Sunrise Medical customer service. See the introduction page located in the front of this User's Manual for details on how to contact your authorized dealer or Sunrise Medical customer service.

WARNING

The owner of this chair is responsible for making sure that it has been setup and adjusted by a trained service professional under the advice of a healthcare adviser. Service or adjustments should only be done with the advice of a healthcare professional. Always use parts and accessories that have been recommended and approved by Sunrise Medical when servicing this chair. Service and/or adjustments that require the use of tools should only be done by a Sunrise Medical authorized dealer under the advice of a healthcare professional.

A. DEALER SERVICE INTRODUCTION

1. At least once per year, this chair should have a complete inspection, safety check, and regular service made by an authorized dealer.
2. If you have discovered a worn, bent, or damaged part, repair or replace them with recommended parts **before returning this chair to service.**
3. All major maintenance and repair work should be done by the authorized dealer.

B. CRITICAL MAINTENANCE TIPS

1. Torque settings:

A torque setting is the optimum tightening which should be made on a particular fastener. It is important to use proper torque settings where specified. When not specified, torque settings should be: M5 fasteners = 5 Nm [44in.-lbs], M6 fasteners = 7 Nm [62 in.-lbs], M8 fasteners = 10 Nm [88 in.-lbs]. For all others torque setting should be 6.8 Nm [60 in.-lbs].

2. Dealer Service and Adjustment Reference materials:

Go to www.SunriseMedical.com for parts manuals, instruction sheets, and instructional videos that will aid in the repair of the Quickie/Zippie Family of wheelchairs.

C. CLEANING

1. Paint Finish

- a. Clean the painted surfaces with mild soap or detergent.
- b. Protect the paint with a coat of non-abrasive auto wax.

2. Axles and Moving Parts

- a. Clean around axles and moving parts with a slightly damp (not wet) cloth.
- b. Wipe off or blow away any fluff, dust, or dirt on axles or moving parts.
- c. DO NOT USE 3 in 1 oil, or WD-40 for lubrication. Only use Teflon based lubricant when working on this wheel-1 chair.

D. REAR AXLE

1. Rear Wheel Axle Adjustment


Tight axle sleeves should be maintained for proper performance of the wheelchair.

- a. To adjust the axle (A) you will need a 3/4-in wrench (B) to turn the outside axle nuts (C).
- b. You will also need a 1/2-in wrench to hold the opposite end of the axle (D) and prevent the axle from turning, while you are adjusting the nut.
- c. Turn the outside axle nut clockwise to tighten.
- d. There should only be zero to ten thousandths of an inch (.010-in (.25mm)) of play.

2. Moving the Rear Axle for Weight Distribution

Weight distribution between the front and rear wheels is primarily adjusted by moving the rear axles forward or rearward on the frame. Moving the axles REARWARD provides better forward and rearward tipping stability. Moving the axles FORWARD shifts more of the rider's weight onto the REAR wheels, providing the following benefits: the front end of the wheelchair becomes lighter, thus reducing the turning effort, and with a shorter wheelbase the turning radius is reduced.

CAUTION – Moving the Rear axles can affect the stability of the device, ALWAYS check stability with the rider before putting the chair into service.

IMPORTANT NOTE – The axle adjustment on each side of the chair should be equal. 


NOTE – Use a torque setting of 13.6 Nm [120 in.-lbs] when setting-up the axle plate.

NOTE – The frame must be kept horizontal when height adjustments are made. Refer to matrix in the ordering guide for proper setting.



TOOLS YOU WILL NEED

1. Imperial Hex key set
2. Metric Hex key set
3. Imperial Open-end Wrench set
4. Metric Open-end Wrench set
5. Torque wrench
6. Imperial Hex bit Socket set
7. Metric Hex bit Socket set
8. Phillips and Flat screwdrivers

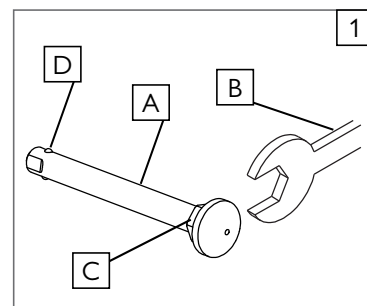


IX. DEALER SERVICE & ADJUSTMENT

3. Axle Position Adjustment 2

- a. Remove both wheels from the chair.
- b. Remove the bolts (A) from the axle plate (B) on both sides of the chair.
- c. Reposition the axle plate in desired position.
- d. Insert and tighten bolts (A).

4. Axle Height Adjustment 2

- a. Loosen and remove bolts (A).
- b. Set axle at desired height using pre-set holes in the axle plate (B) on both sides.
- c. Replace and tighten bolts.

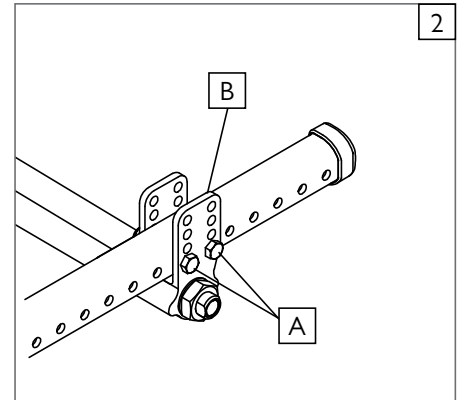


E. HUB LOCK ADJUSTMENT 3

To adjust the cables

- a. Loosen jam nut (C) using a 10mm wrench.
- b. Turn adjustment screw (D) in or out as necessary. Turning screw upward will loosen cable, turning downward will tighten cable.
- c. Re-tighten jam nut (C) against lever mount.
- d. Repeat steps a-c for opposite side of chair as necessary.

To adjust lever position


- a. Remove socket screw (E) and washer (F) from lever and backrest.
- b. Choose desired hole in backrest to mount lever and re-install screw and washer.
- c. Repeat steps a-b for opposite side of chair.

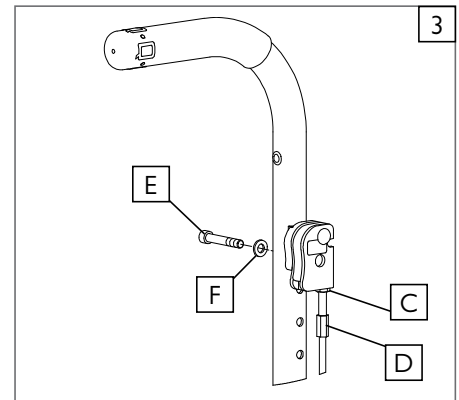

F. WHEEL LOCKS 4

Adjustment

- a. Evenly loosen both screws (G). Do not attempt to remove one screw at a time.
- b. Slide assembly toward rear wheel until clamp embeds into tire to prevent wheel movement, when in locked position.
- c. Tighten screws (G).

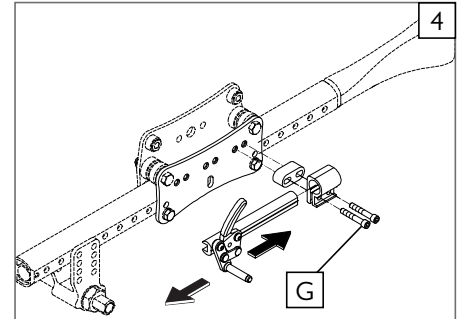

G. SINGLE-POST ARMREST RECEIVER 5


1. Adjusting Armrest Receiver Fit

To tighten or loosen the fit of the outer armrest in the receiver:

- a. Loosen the 4 set screws (J) by using a 5/64-in Hex Key.
- b. With the armrest assembly in the receiver (K) press firmly on the outside of the receiver.
- c. Maintain pressure on the receiver and re-tighten two of the set screws (J).
- d. Remove the armrest assembly and re-tighten the remaining two set screws.
- e. The armrest will fit firmly in the receiver and lessen any noise that may result from vibration.

2. Moving the Receiver Location

- a. Loosen the 3 bolts (I) holding the clamp halves (H) together (do not remove them).
- b. Slide the clamp so that armrest is at the correct position.
- c. Tighten the 3 bolts (I).

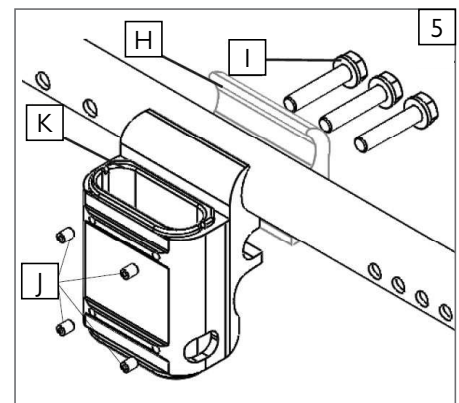

H. ADJUSTABLE LOCKING FLIP-UP ARMRESTS 6

1. Height Adjustment


- a. Loosen bolt and nut (A) and remove flip-back tube.
- b. Loosen bolts (B, C) and remove clamp (D).
- c. Move the clamp (D) up or down to the desired position.
- d. Tighten bolts (B, C).
- e. Install the flip-back tube back in place and tighten bolts (B, C).
- f. Repeat the process for the other armrest.

2. Angle-Adjustment

- a. Loosen bolt (E).
- b. Set armrest at desired angle using preset holes in armrest angle plate.
- c. Tighten bolt (E).

3. Length-Adjustment (optional)


- a. Remove foam grip or armpad (G) from armrest.
- b. Loosen the nut and screw (F) in the adjustable flip-up tube.
- c. Reposition the sleeve (H) at the desired length.
- d. Re-tighten the screw and nut (F).
- e. Put foam grip or armpad (G) back in place.

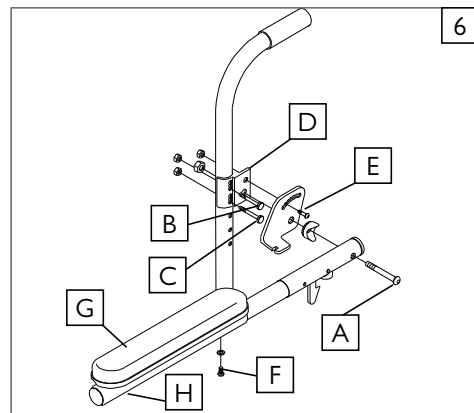

I. CANTILEVER LOCKING FLIP-BACK ARMRESTS 7

1. Height Adjustment


- a. Using a 5mm hex key and 13mm open end wrench loosen button head screw (A) and nut (B) on the inside of the armrest pivot.
- b. Rotate the armrest up or down to a comfortable height and tighten the hardware (A, B).
- c. You should repeat the steps on the opposite armrest, and adjust both sides equally in most cases.

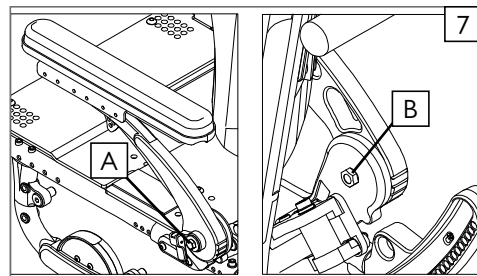

2. Level the Armrest 8

When you adjust the armrest height, you will also need to level the armrest pad.

- a. Using a 5/32-in hex key, loosen the front pivot socket screw (F) and remove the rear socket screw (F) from the bracket (E) so that you can level the armpad (D).
- b. Once you have found the correct adjustment, re-insert the rear socket head screw (F), and tighten the hardware.

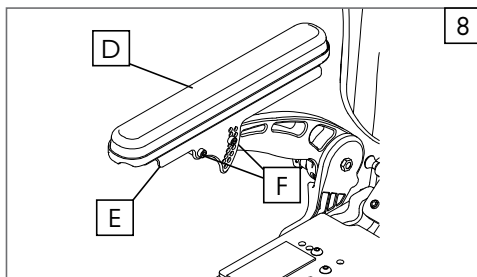



3. Armrest Pad Adjustments 9 10

You can adjust the armrest pad forward or backwards.

- a. Using a 1/8-in hex key loosen the two button head screws (C) holding the armrest pad (D) to the adjustment bracket (E), and move the pad forwards or backwards.
- b. When you have found the right position, re-insert the screws (C) and tighten the hardware.

You can also adjust the armrest pad laterally.


- a. Using a 5/32-in hex key remove the two screws (C) holding the armrest pad (D) to the adjustment bracket (E).
- b. You can achieve an inch of adjustment laterally by flipping the pad 180° as shown.
- c. If you still need more adjustment laterally, you can re-position the armrest pad bracket by removing the 2 socket head screws (F) that hold the bracket to the cantilever arm, and attaching the bracket on the outside of the cantilever arm as shown.
- d. When you have found the right position, re-insert the screws (F) and tighten the hardware.

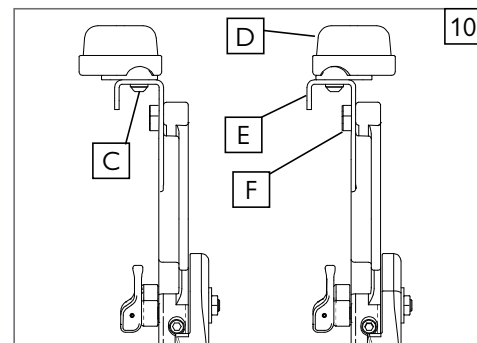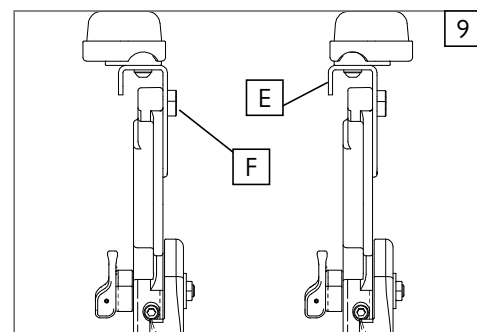


IX. DEALER SERVICE & ADJUSTMENT


J. SWING-AWAY HEIGHT ADJUSTMENT 11**Height Adjustment**

NOTE – It is recommended that you maintain a minimum of 2.0 inches (50 mm) between the lowest point on the footrest and the floor.


- a. Loosen set screw (A) using a 3/16 inch hex key.
- b. Slide footrest extension (B) up or down inside frame tube to desired height.
- c. Tighten set screw (A). Use a torque setting of 16.3 Nm [144 in-lbs]. 

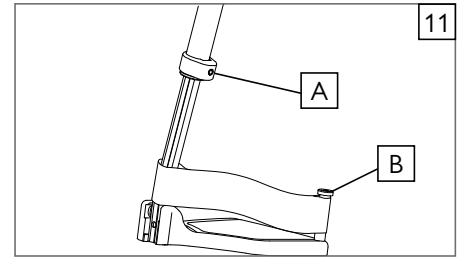K. ANGLE ADJUSTABLE FOOTPLATE 12**1. Composite Angle Adjustable Footplates**

- a. Remove bolt (F).
- b. Pull footplate section (G) away from extension.
- c. Reposition footplate back onto extension in new angular position. Use pre-marked extension coupling (H) as guide.
- d. Re-insert bolt (F) and tighten securely.

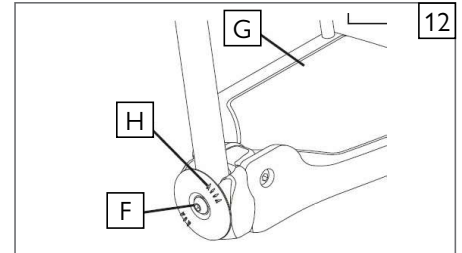L. HEAVY DUTY LIFT-OFF FOOTREST 13**Height-Adjustment, Front Mount**


- a. Loosen and remove the retaining bolt (C) and nut (D) in the footrest mounting bracket.
- b. Position footplate upward or downward in predrilled holes on hanger tube.
- c. Reinsert bolt and nut and tighten.

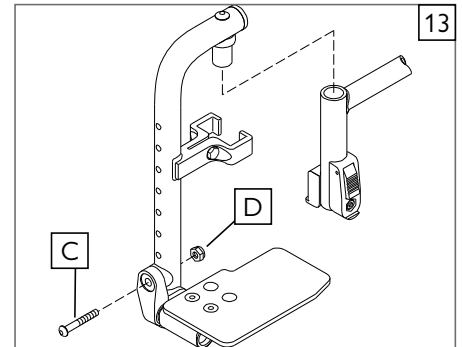M. ARTICULATING LEGREST 14**1. Height Adjustment of the Calf Pad**

- a. Loosen the retaining nut (E) located at the back of the calf pad bracket assembly.
- b. Slide the calf pad bracket assembly up or down.
- c. Tighten the retaining nut (E) once the pad is in the desired position.

Height Adjustment with Extension Tube


- a. (See "Swing-away Height Adjustment" Section J)

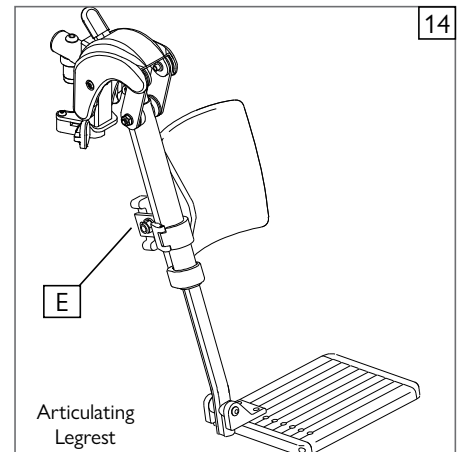N. ELEVATING LEGREST 15**1. Height Adjustment of the Calf Pad**

- a. Loosen the retaining nut (F) located at the back of the calf pad bracket assembly.
- b. Slide the calf pad bracket assembly up or down.
- c. Tighten the retaining nut (F) once the pad is in the desired position.

2. Height Adjustment with Extension Tube

- a. (See "Swing-away Height Adjustment" Section J)

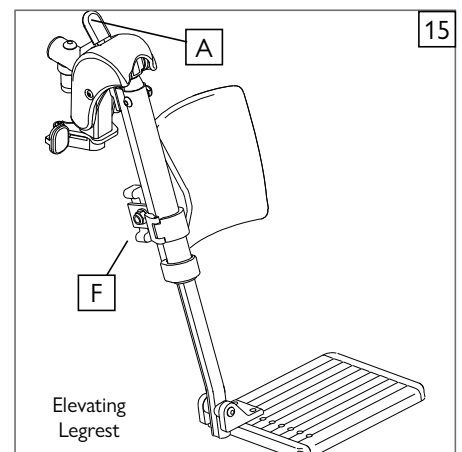

O. CONTRACTURE FOOTREST 16 17 18

1. Outside Mount - Can be mounted laterally. 16

2. Inside Mount - Can be mounted in the center. 17

3. Height Adjustment 18




- Loosen and remove the retaining bolt (A) in the footrest mounting bracket.
- Slide the lower tube upward or downward to the desired position until predrilled holes on hanger tubes are aligned.
- Reinsert bolt (A) and tighten.

4. Footrest Angle Adjustment 18

- Loosen the retaining bolt (B) in the adjustment locking joint.
- Position footrest forward or rearward in desired angle.
- Retighten the bolt (B).

5. Footplate Angle Adjustment 18

- Loosen the retaining bolt (C) in the footplate locking joint.
- Position footplate upward or downward in desired angle.
- Retighten the bolt (C).

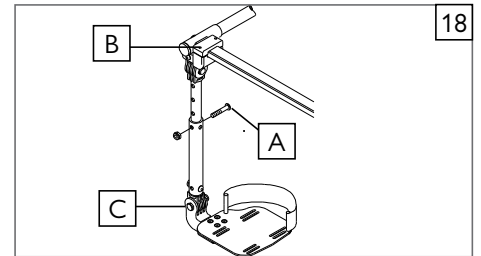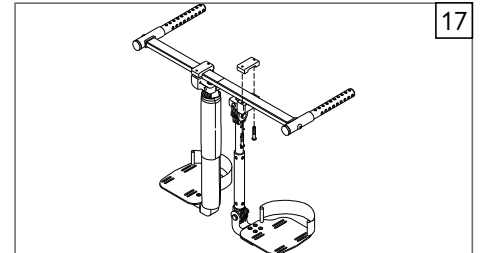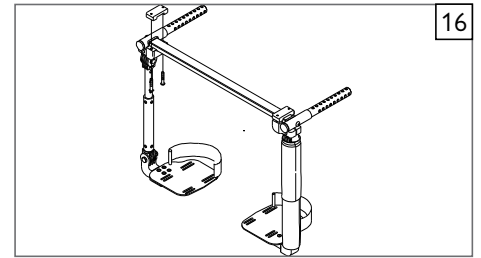


P. Z-FINITY® FOOTREST SYSTEM 19 20 21

CAUTION – The Z-finity Footrest system is rated to 165 lbs (75kg).

The footrest position is key to keeping the body in the proper alignment, and providing a stable support for your legs and torso. Make sure that your Sunrise Medical authorized dealer has correctly adjusted your footrests and/or legrests with the advice of a healthcare professional. The original setup should not be changed without first checking with that adviser. Legrests and footrests should be swung out of the way, or removed for transfers.

NOTE – Footplate must have at least 2 inches (51mm) of clearance between the footplate and the ground to prevent hitting obstacles and risking a fall or tip-over.



1. Footrest/Footplate positioning using the rotational clamp

- Loosen the two rotational/angle clamp screws (D).
- Rotate the footrest/footplate until the desired position and angle is achieved.
- Tighten the clamp hardware to proper torque (DO NOT EXCEED a 13.6 Nm [120 in-lb] maximum torque). 



2. Footrest positioning using footplate spacers




- Remove footplate spacers (E).
- Slide footplate to desired depth position.
- Replace footplate spacers.

3. Height range adjustment (Direct Mount Footrest)

- Adjust the footplate mounting position by removing the button head screw from the adjustment hole and moving the assembly to the desired position. 
- Tighten hardware to proper torque of 13.6 Nm [120 in-lb] maximum. 

4. Height range adjustment (Extension Mount Footrest)

- Loosen set screw (F).
- Slide footrest extension tube up or down inside frame tube to desired height. 
- Tighten set screw to proper torque of 13.6 Nm [120 in-lb] maximum. 

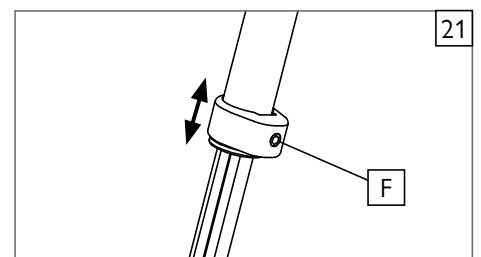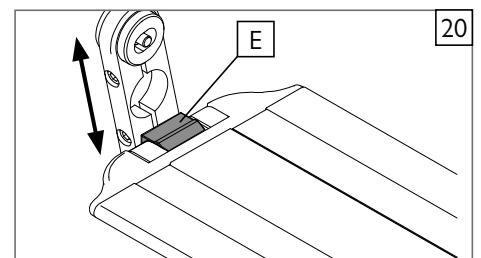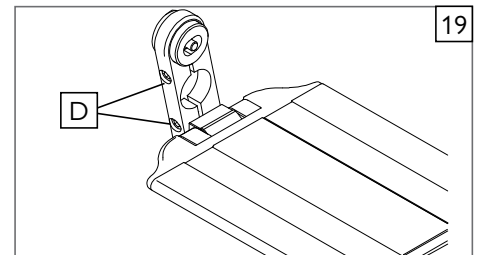

IX. DEALER SERVICE & ADJUSTMENT

Q. LIMB SUPPORT 22

See Swing Away Hangers/Footrests (Section VIII - O) for installation/removal from chair.

1. Height Adjustment 23

- a. Loosen 2 screws (A), do not remove, slide post up or down to desired height.
- b. Re-tighten screws.

2. Angle Adjustment



- a. Press button (B) adjust up or down to desired angle, release button.

3. Length Adjustment

- a. Loosen 2 screws (C), do not remove, slide clamp forward or back for desired length.
- b. Re-tighten screws.

4. Additional Width or Length Adjustment

- a. Remove 2 screws (D), reposition to desired setting and replace screws.
- b. Tighten securely.

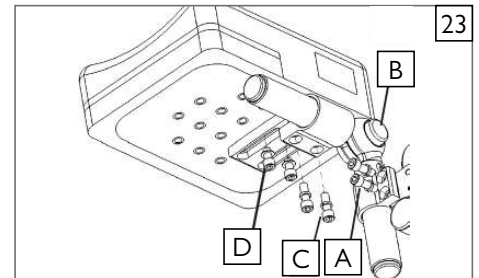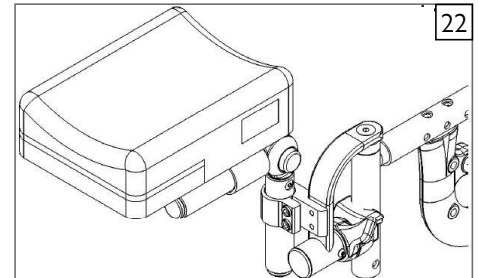

R. TILT-IN-SPACE MECHANISM

The tilt-in-space mechanism is installed by Sunrise Medical. Two cable/trigger mechanisms positively lock the chair in place from -5° to 60° depending on the set-up. The Quickie/Zippie IRIS has the capability to provide from 0° to 5° of forward tilt. **The caregiver or user of this chair should not attempt to make a forward adjustment on their own.** This adjustment must be performed by an Sunrise Medical authorized dealer.

1. Rocker Cover removal 24

NOTE – Rocker arm cover is not available with the Folding Zippie IRIS.



- a. Remove the (4 or 5) screws (A) that hold the cover to the rocker frame (B).
- b. Remove the rocker cover (C) and store with screws in case you need them later.

2. Tilt-stops Adjustment 25

NOTE– When using tilt stops, you must first remove the rocker cover.

Adjustments can be made to limit the amount of tilt (rotation) that the chair provides by using the tilt-stops (D).

- a. Loosen and remove the retaining screw (E).
- b. Locate the tilt-stops to the desired limit either anterior or posterior.
- c. Each half of tilt stop assembly should plug into directly opposing holes on the rocker (F), then secure in place by tightening the screw (E).

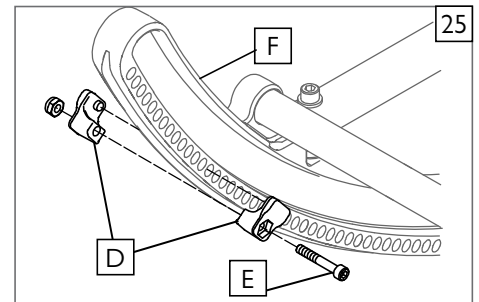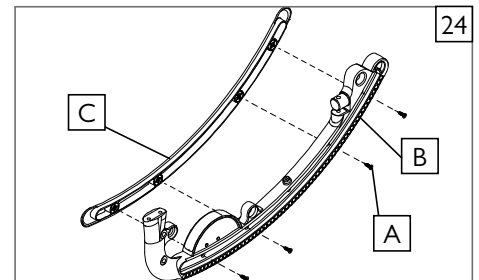

3. Cable Adjustment 26

NOTE – Do not use lubricants on the components or assembly.

- a. Loosen the jam nut (G).
- b. Rotate the cable adjuster (H) until slack is removed from the cable. Make sure the plunger pin can fully extend and retract after adjustment and that pins engage the rocker when triggers are released.
- c. Tighten jam nut (G).

S. STANDARD BACKREST

1. Angle Adjustment 27



- a. Loosen rear bolt (I).
- b. Loosen and remove angle adjustment bolt (J).
- c. Set backrest at desired angle using pre-set holes.
- d. Replace and tighten the front bolt (J).
- e. Tighten rear bolt (I).
- f. Repeat on other side.

IMPORTANT NOTE – The adjustment on each side of the chair should be exactly equal.

NOTE – Use a torque setting of 8.5 Nm [75 in-lbs] when setting-up the angle adjustment of the stroller handle.




NOTE – Backrest position may also need adjusting to maintain optimal CG alignment and stability.

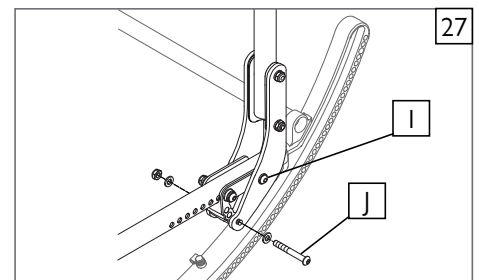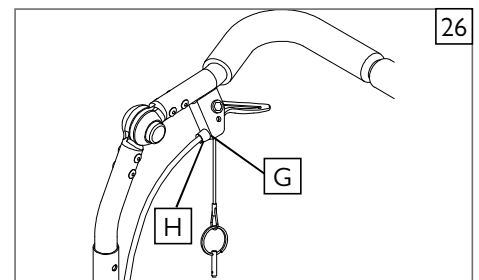


2. Flip-Down Back Function 28

- a. To fold the back down, pull-up the back release pins (A) and push the backrest forward.
- b. To lock it back in position, lift the backrest firmly until it locks in its original position.

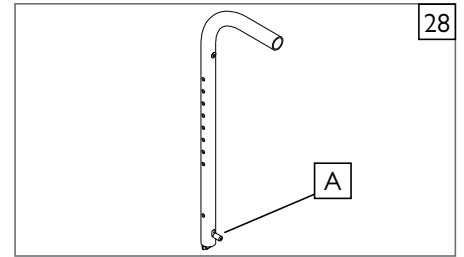

3. Dynamic Backrest Elastomer Replacement 29

- a. Remove socket head screw (B) and lock nut (C).
- b. Using a flat head screwdriver, gently pry out the elastomer (D) as shown.
- c. Firmly press the new desired elastomer into place, and align holes in elastomer with holes in plate (E).
- d. Replace and tighten screws removed above.

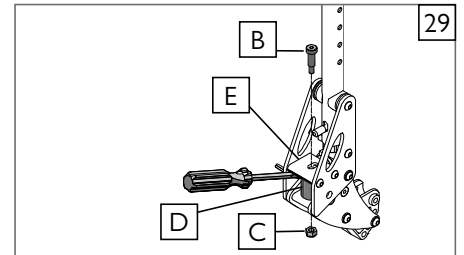


⚠ WARNING

If the chair is equipped with the Dynamic Backrest option, the lockout feature must be in the locked position on both push handles prior to any type of transport of the chair.

4. Dynamic Backrest Lock Engagement 30

- a. Pull key ring (F) and swing lock downward to unlock backrest.
- b. Swing lock upward to lock the backrest.

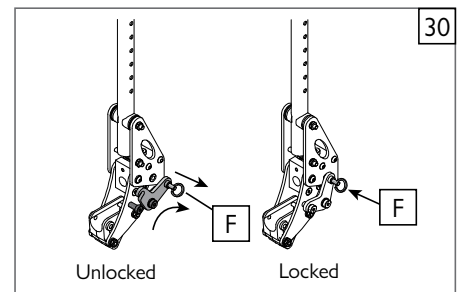


T. MONO BACK

1. Backrest Height Adjustment 31

- a. Using a 5/32-in hex key, loosen the 4 button head screws (G).
- b. Grip the backshell and bracket assembly. It should move freely up or down the Mono back spine.
- c. Set the backrest assembly to the preferred height.
- d. Tighten all 4 of the button head screws (G).

NOTE – Use a torque setting of 6.8 Nm [60 in-lbs] when setting-up the height adjustment of the Mono back.


NOTE – Anytime the backrest position changes, it is a good idea to verify that the optimal CG alignment and stability is maintained.

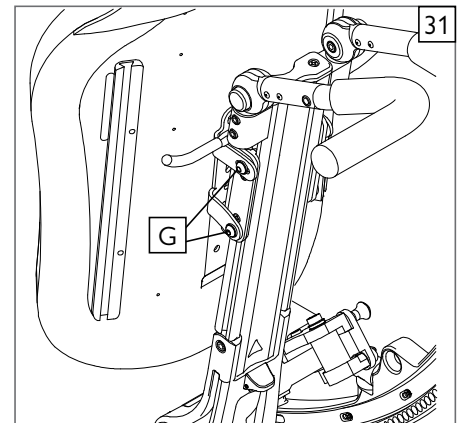

2. Backshell Height Adjustment 32

The backshell has additional height adjustment with its mounting bracket.

- a. Remove the backrest cover.
- b. Using a 10mm open end wrench, remove the 4 nuts (H) that hold the backshell to the Mono back bracket.
- c. Remove the 2 nutplates, and reposition them in available holes (I) up or down on the backshell.
- d. Thread the nuts from the rear of your chair, and tighten them to finish the adjustment.

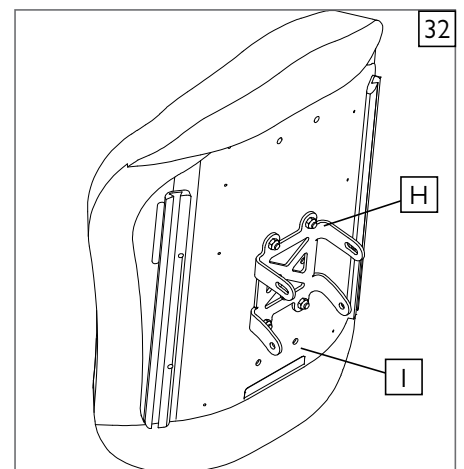


IX. DEALER SERVICE & ADJUSTMENT

3. Backshell Angle Adjustment 33

The backshell has +/- 4° of angle adjustment with its mounting bracket.


- a. Using a 5/32-in hex key, loosen the 2 button head screws (C)
- b. Using a 5/32-in hex key loosen the 2 button head pivot screws (D).
- c. You can now tilt the back shell forward or rearward.
- d. Tighten the screws to finish the adjustment.

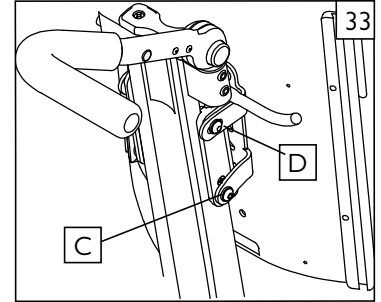

4. Tilt Adjustment with the Mono Back Post and J Bracket 34

The Mono back has additional angle/tilt adjustment through the J bracket.

- a. Using a 3/16-in hex key/socket wrench loosen the 2 outside J bracket screws (E).
- b. Using a 5/16-in hex key, rotate the larger J bracket socket screw (F) clockwise for recline, and counter-clockwise for the upright position.
- c. Set the adjustment using the 3/16-in hex key to tighten the 2 outside screws (E) to 6.8 Nm [60 in-lbs].

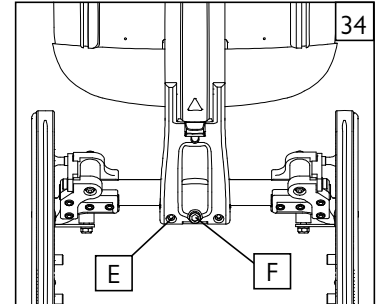

5. Lateral Backrest Adjustment with the Mono Back Post and J Bracket 34

The Mono back adds additional lateral adjustment using the J bracket.

- a. Using a 3/16-in hex key/socket wrench loosen the 2 outside J bracket screws (E).
- b. Grasp the back shell and J bracket from behind and move the assembly left or right to the desired area of support.
- c. Set the adjustment using the 3/16-in hex key to tighten the 2 outside screws (E) to 6.8 Nm [60 in-lbs].




U. DYNAMIC MONO BACK 35

If you have a chair with a Dynamic back function, you can utilize the elastomer inserts by releasing the locking lever.

IMPORTANT NOTE – DO NOT convert to a dynamic back with the rider in the chair.

- a. BEFORE locking or releasing the lockouts (G), make sure there is no pressure on the back support from the rider. Transfer the user to a safe location while this task is performed.
- b. The Dynamic function can be used by pulling on the lockout lever (H) and rotating the lockout bracket (I) down and out of the way.
- c. Repeat the step above for the other side of the lockout system, so that both lockouts are released.
- d. To lockout the dynamic back for a more rigid support, rotate the lockout bracket (I) up while pulling on the lockout lever (H).
- e. Make sure that it "Clicks" into place for a positive lockout.

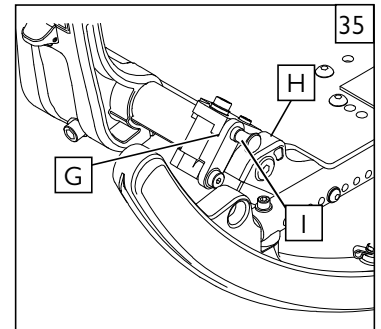


V. DYNAMIC MONO BACK (ELASTOMER REPLACEMENT) 36

There are four Elastomer densities available. Two types are provided with your chair. If your elastomer gets worn out, or you need a different level of support, you can replace the elastomer with a few easy steps.

IMPORTANT NOTE – DO NOT change the elastomer with the rider in the chair.


- a. BEFORE changing out your elastomer, transfer the user to a safe location while this task is performed.
- b. The elastomer inserts (A) can be changed out by using a 5/32-in hex key on the upper socket head screw (B) and a 10mm socket and socket wrench for the lower hex nut (C).
- c. Loosen and remove the nut and hardware on both sides of the chair that hold the elastomer in place on the lockout system.
- d. Remove and replace the old elastomer.
- e. Re-thread the socket head screw from the top, through the bracket and elastomer, and hold the nut with your finger while turning the hex key to get the nut started.
- f. Finish the replacement by using the socket, socket wrench, and hex key to tighten the elastomer hardware to 6.8 Nm [60 in-lbs].

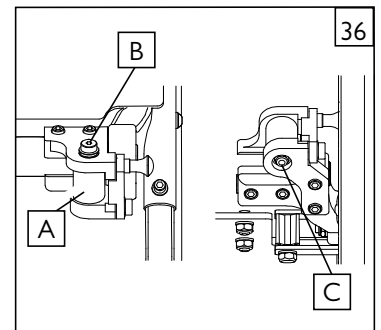

W. VENTILATOR HANGER BRACKET 37

1. Attach the Ventilator to the Ventilator Bracket

- a. Position the ventilator above the hanger bracket (D) and slide it into the bracket.
- b. Using the 2 nylon straps (E, F) secure the Ventilator to the hanger bracket and make sure any cords and tubes are secured and not in danger of pinching in any of the components of the chair.

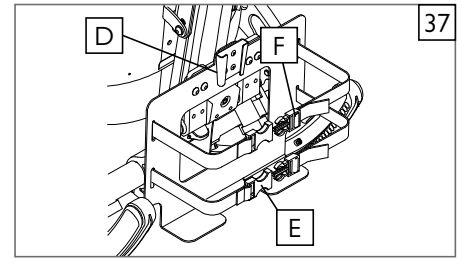


X. FIXED STROLLER HANDLE 38

1. Angle Adjustment of the Stroller Handle

Depress both black push buttons (H) at the same time. This disengages the stroller handle and allows it to rotate freely. Releasing the push buttons positively locks the adjustable stroller handle in the desired position.

2. Height Adjustment


- a. Remove both bolts (G) from the backrest tube (one on each side).
- b. Reposition backrest height.
- c. Re-tighten both bolts (I) in the desired location.

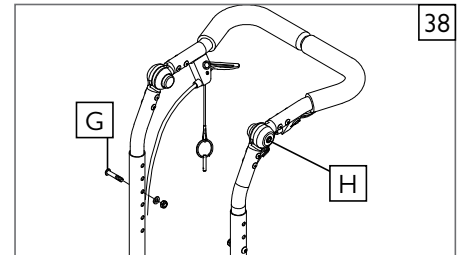

Y. ADJUSTABLE STROLLER HANDLE (MONO BACK)



1. Height Adjustment 39

- a. Using a 5/32-in hex key loosen both screws (I) from the stroller tube attachment clamp (H) (one on each side).
- b. Reposition stroller handle height by sliding the assembly up or down in the channel.
- c. Re-tighten both screws (I) in the desired location.

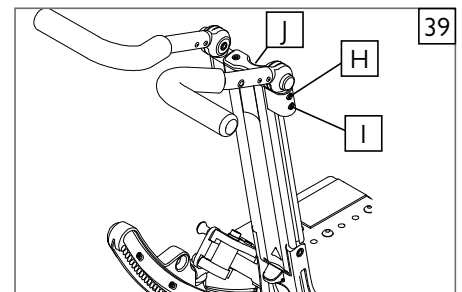

2. Stroller Handle to Push Handle Conversion 39 40 41

- a. Loosen both screws (Do not Remove) (I) from the stroller tube attachment clamp (H),
- b. Remove the screws that hold the cap (J) to the Mono Back, remove the endcap, and slide the brackets up and off of the Mono back rail.
- c. Reposition the stroller handles in the push handle configuration (K) or (L).
- d. Re-tighten screws in both clamps once you have them in the desired location.

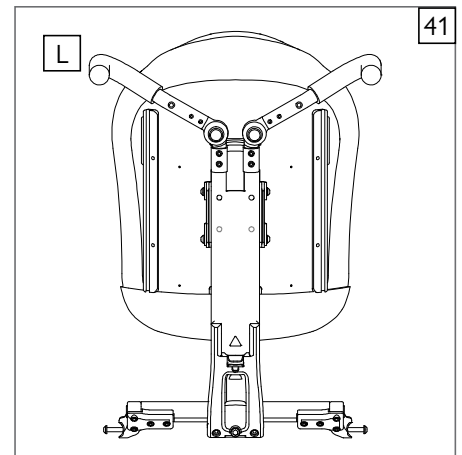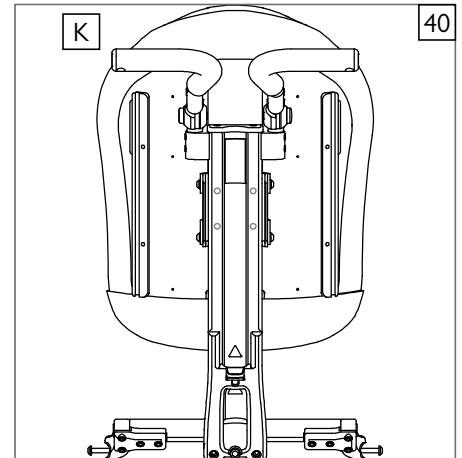

IX. DEALER SERVICE & ADJUSTMENT

Z. CASTER/FORK ASSEMBLY 42


1. Installation

- a. Remove dust cover (A) from caster housing. Pry cover off using a back and forth motion.
- b. Caster assembly holds one nut (B) and one washer (immediately below the nut). Remove nut and washer.
- c. Insert caster stem (D) into caster housing (C) and place washer on stem.
- d. Replace nut (B) on caster stem and tighten until all play is removed; then loosen approximately 1/8 turn, allowing for free rotation.
- e. Replace dust cover to caster housing.

2. Height Adjustment

- a. Loosen and remove bolt (E).
- b. Set axle at desired height using pre-set holes in the fork.
- c. Replace and tighten bolt.
- d. Repeat on other side.

NOTE – The frame must be kept horizontal when height is adjusted. Refer to matrix in the ordering guide for proper setting. Refer to Section IX - D for rear axle height adjustment.

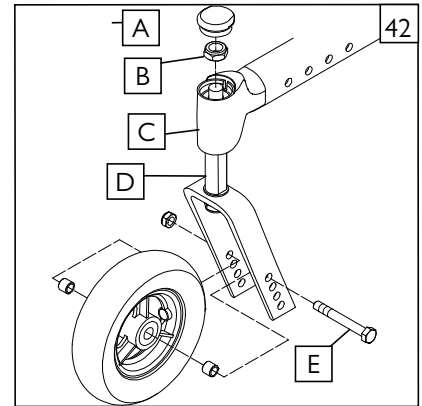

AA. FRAME DEPTH

1. Back Adjustment 43


- a. Remove the screws (F & H) from the back mounting plate (G) on both sides of the chair.
- b. Slide the backrest along the seat rail to the same position on both sides.
- c. Replace and tighten the screws (F & H) to lock in the position on both sides of chair.

IMPORTANT NOTE– The adjustment on each side of the chair should be exactly equal.

NOTE – Use a torque setting of 13.6 Nm [120 in-lbs] when setting-up back depth adjustment.

2. Back Angle Adjustment 44

- a. Remove the screw (F) and loosen (H) from the back mounting plate (G) on both sides of the chair.
- b. Adjust the backrest to the same position on both sides, using one of the three adjustment holes shown.
- c. Tighten the screws (F & H) to lock in the position on both sides of chair.

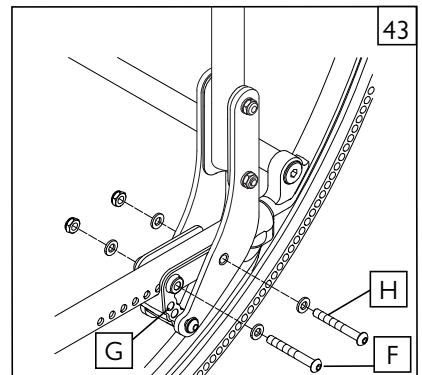



3. Hanger Receiver Adjustment 45

- a. Loosen and remove both socket head screws (I) from the seat rail.
- b. Slide the hanger receiver (J) in the seat rail to the desired position.
- c. Replace and tighten the socket head screws.
- d. Repeat this process on the other side.

IMPORTANT NOTE –These adjustments may also require re-positioning the carriage over the base frame, refer to matrix in the ordering guide for recommended location.


IMPORTANT NOTE – The adjustment on each side should be exactly equal.

NOTE – Use a torque setting of 13.6 Nm [120 in-lbs] when setting-up hanger receiver.

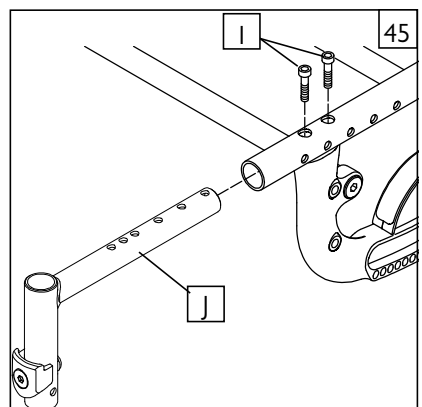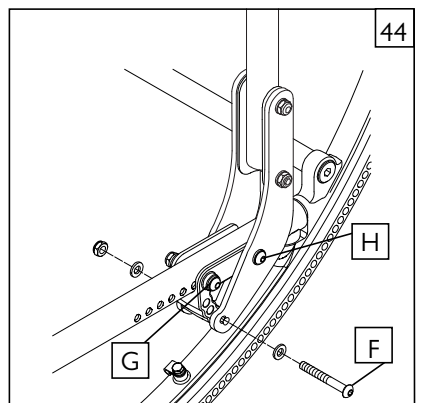

BB. XLOCK® WIDTH ADJUSTMENT 46

1. Frame Width Adjustment either 1-in (25mm) (only adjust 1 side) or 2-in (51mm) (adjust both sides).
 - a. Remove the Hex head screws, and washers (A) from the left and/or right front side frame tabs depending if you want a 1-in or 2-in adjustment.
 - b. Remove the Hex head screws, and washers (B) from the front and rear strut locking tubes.
 - c. Re-align the XLock® assembly with the correct adjustment holes.
 - d. Thread the Hex head screws (A) and washers through the tab holes and into the XLock® brace.
 - e. Thread the Hex head screws (B) into the locking struts in the adjustment holes at the new width.
 - f. Tighten all of the hex head screws to 13.6 Nm [120 in-lbs].

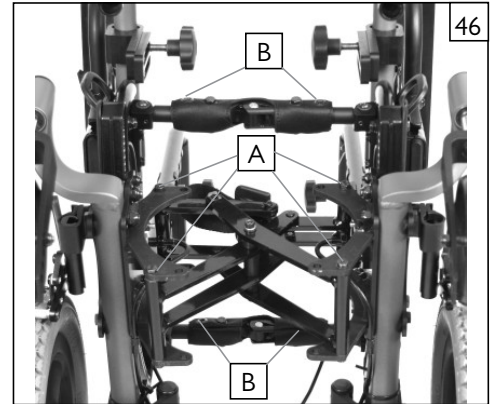

CC. FRAME WIDTH


The frame of the IRIS is built to accommodate growth by using the growth kit components.

1) Fixed Strut Tube Width Adjustment (Rocker assembly) 47

- a. Remove the bolts (C) from both ends of the strut tubes (D) for all strut tubes.
- b. Replace with the appropriate length strut tubes.
- c. Tighten the bolts (C) at each end.
- d. Repeat for each strut tube (D).



NOTE – Use a torque setting of 22.6 Nm [200 in-lbs] when tightening hardware (C).

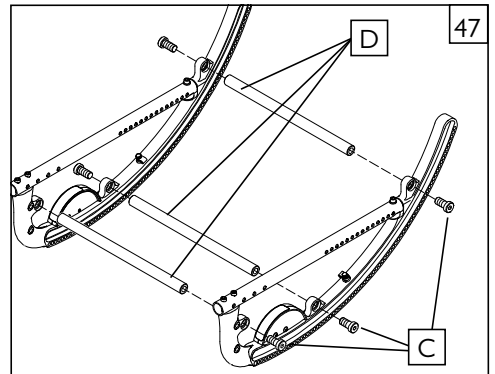


2) Fixed Strut Tube Width Adjustment (Roller assembly) 48

- a. Remove the bolts (E) from both ends of the strut tubes (F) for all strut tubes.
- b. Replace with the appropriate length strut tubes.
- c. Tighten the bolts (E) at each end.
- d. Repeat for each strut tube (F).



NOTE – Use a torque setting of 22.6 Nm [200 in-lbs] when tightening hardware (E).

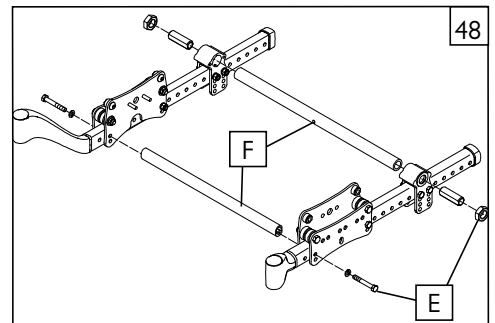


3) Growable Strut Tube Width Adjustment (Rocker assembly) 49

- a. Remove the hardware (G) from all three strut tubes (H) and adjust all three tubes an equal distance.
- b. When you are sure the width is equal for all three adjustable tubes, tighten the hardware to a torque setting of 17 Nm [150 in-lbs] after adjusting the strut tubes.




NOTE – Use a torque setting of 22.6 Nm [200 in-lbs] when tightening hardware (C).

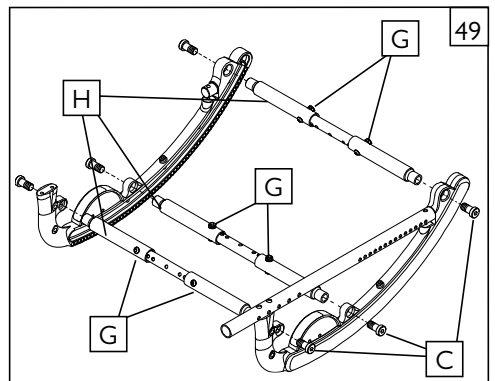


IX. DEALER SERVICE & ADJUSTMENT

4) Growable Strut Tube Width Adjustment (Roller assembly) 50

- Remove the hardware (G, H, & I) from the strut tubes.
- Adjust all three tubes an equal distance, and re-tighten the hardware to a torque setting of 17 Nm [150 in-lbs] after adjusting the strut tubes.

NOTE – Use a torque setting of 22.6 Nm [200 in-lbs] when tightening hardware (J). 

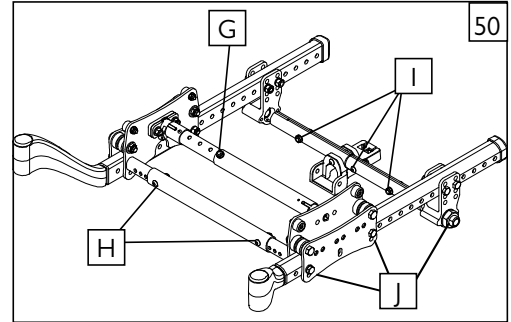

DD. SEAT PAN 51 52

The seat pan of the IRIS is designed to accommodate depth adjustment.


Depth adjustment

- Loosen and remove all bolts (A).
- Adjust the seat pan to the desired depth.
- Replace and tighten all bolts.

NOTE – Use the holes that place the bolts as far away from each other as possible.

Width adjustment

- Loosen and remove all bolts (B).
- Adjust chair to the desired width.
- Replace and tighten all bolts.

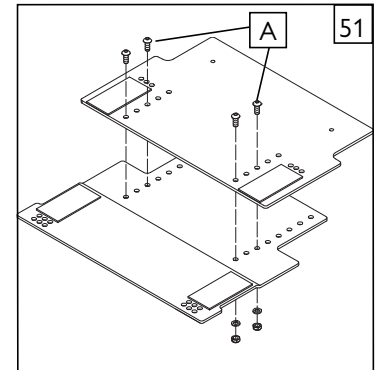


EE. REMOVABLE SEAT PAN 53 54

I. Removing and Attaching the Seat Pan

- To remove, pull out release pins (C) loosen the two knobs (D) on the underside of the seat pan, grasp front and rear of seat pan and firmly pull upward until seat mounting clips disengage from the frame.
- To reattach, position seat pan on frame rail so that the front bracket mounting grooves slide into the knobs on both sides of the frame. Position the rear mounting clips on the frame and push down firmly on seat pan. Be sure to engage all four mounting points on frame rails. Tighten knobs and reinsert release pins.


WARNING

Check for engagement by pulling up lightly on seat pan. It should not move upward. Failure to check for engagement may lead to injury if the seat mounting clips are not fully seated.

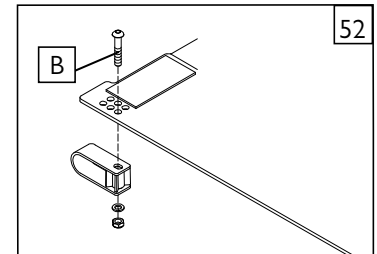


2. Adjusting Seat Pan Width

- Remove the hardware (F) that secure each seat mounting clip (G) to the seat pan.
- Position the seat mounting clips to the desired position, reattach the screws. Each hole is 1/2-in (12.7mm) apart.

NOTE – Tighten screws to 11.3 Nm [100 in-lbs] torque. 


- Repeat steps “a” and “b” for each mounting clip.


NOTE – Both sides must be mounted to same hole position.

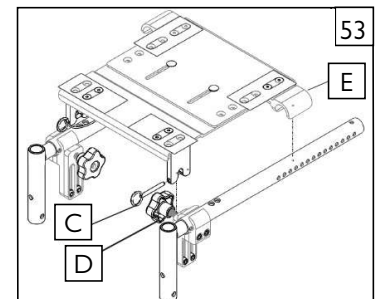

3. Adjusting Seat Pan Depth

- Loosen the two lock nuts (H) on underside of seat pan that attach the front seat pan panel to rear seat pan panel.
- Reposition seat pan to the desired depth, and reattach the lock nuts.

NOTE – Tighten lock nuts to 11.3 Nm [100 in-lbs] torque. 

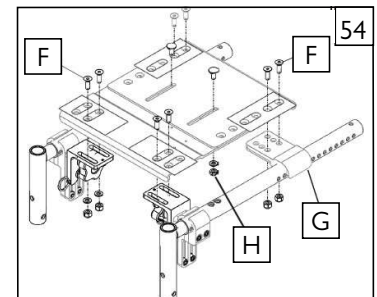


FF. CARRIAGE 55 56

The carriage of the IRIS can be adjusted to preset a range of tilt.

1. Adjustment

- Loosen and remove bolts (C & D).
- Set the carriage at desired range setting using the proper holes.
- Replace and tighten all bolts.

IMPORTANT NOTE – The adjustment on each side of the chair should be exactly equal.

NOTE – Use a torque setting of 13.6 Nm [120 in-lbs] when setting-up carriage. 

2. Range and setting– 55°

- 5° to 50°– use holes E & H
- 0° to 55°– use holes E & G
- 5° to 60°– use holes F & G

3. Range and setting– 40°

- 5° to 35°– use holes E & H
- 0° to 40°– use holes E & G
- 5° to 45°– use holes F & G

NOTE – It is highly recommended to replace both rollers within each carriage assembly at the same time. **Do not use lubricants anywhere on the assembly.**

4. Top Roller Replacement 57


NOTE– For more information about Roller plate assembly on the Folding Zippie IRIS please refer to Sunparts Online located on the Sunrise Medical website.





- Loosen both plunger pin bracket nuts.
- Loosen and remove both lower roller nuts (I), chassis fastener nut (J), and base strut bolts (K) on both sides. Remove base strut.
- Slide the inner carriage plate (L) off the fasteners, towards the center of the chair.
- Loosen and remove both sides of the roller hardware (M).
- Remove old top rollers (N) from the upper roller shafts (O) and replace with new top rollers (P).
- Reinstall the upper roller shafts (O) onto the inner carriage plate (L), installing roller hardware (M).
- Reinstall carriage plate (L) onto the frame assembly. Replace and tighten the nuts (I & J) and strut bolts (K).
- Ensure that the plunger pin is centered and engaged into a rocker hole and pin bracket nuts are tight.
- Repeat on the other side of chair as necessary.

5. Bottom Roller Replacement 58

NOTE – For more information about roller plate assembly on the Folding Zippie IRIS please refer to Sunparts Online located on the Sunrise Medical website.

- Loosen both plunger pin bracket nuts.
- Loosen and remove both lower roller nuts (I), chassis fastener nut (J), and base strut bolts (K).
- Slide the inner carriage plate (L) off the fasteners, towards the center of the chair.
- Remove old bottom rollers (Q) from the roller bushings (R) and replace with new bottom rollers (S).
- Reinstall carriage plate (L) to the frame assembly. Replace and tighten the nuts (I & J) and bolts (K).
- Ensure that the plunger pin is centered into a rocker hole and tighten plunger pin bracket nuts.
- Repeat on the other side of chair as necessary.


NOTE – Use a torque setting of 13.6 Nm [120 in-lbs] when tightening bottom roller hardware (J & K). 

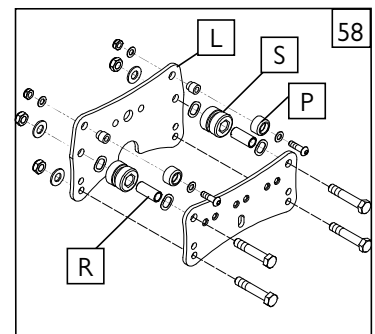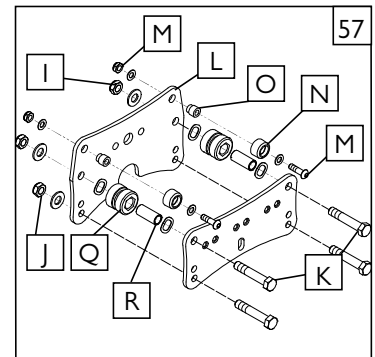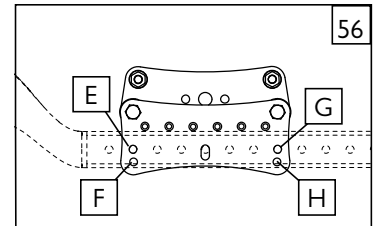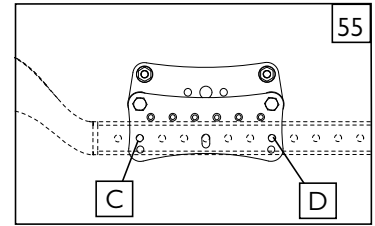

IX. DEALER SERVICE & ADJUSTMENT

GG. ATTENDANT WHEEL LOCK INSTALLATION 59


1. Remove wheels from chair.
2. Attach Mount bracket/cable assembly to axle strut tube using 1/4-in x 1 3/4-in bolt (item 1) with nut and washers.
3. If push-to-lock wheel locks are already installed, remove the push-to-lock assemblies by removing the M6 bolts (item 2) and adjustment nuts (item 3), leaving the slide plates mounted on the chair, and go to step 5. If there are no wheel locks mounted already, proceed with step 4.
4. Attach Wheel Lock Slide Plates to Roller Assembly on each side of the chair using 1/4-in hex bolts (item 4) and Mounting Block (item 5). If axle of rear wheel is above the base frame tube, orient the slide plates upward. If the axle of the rear wheel is below the base frame tube, orient the plates downward. Plate may be flipped forward or reversed as needed for different wheel sizes.
5. Attach the left and right Remote Lock Assemblies (item 6) to the slide plates using the Attendant Wheel Lock Spacer Blocks (item 7), M6 x 60 screws (item 2), lock washers (item 8), and Wheel Lock Adjustment Nuts (item 3). The adjustment nuts marked "R" and "L" are to be mounted to the right and left side of the chair respectively. The smaller protruding oval shape on the nuts should slide inside of the slot in the slide plate. Do not tighten bolts. Slide the Remote Lock Assemblies as far forward as they will go.
6. Re-install wheels and slide the Remote Lock Assemblies rearward until the friction block of the remote lock is approximately 1/4-in (6mm) away from the tire when unlocked. If the lock cannot be adjusted to this position, you will need to adjust the position of the slide plate until it can be adjusted as stated. Once remote locks are properly positioned, tighten bolts.
7. Attach cables to Remote Lock Assemblies by sliding the stud at the end of the cable into the slot in the Remote Lock Assembly. Pull back on the cable sheath and slide the end of the cable sheath into the pocket on the Attendant Wheel Lock Spacer Block (item 3) as shown. Be sure that the cables are routed away from all moving parts. Attach the cables into place on the chair with zip-ties provided.
8. Adjust the cable adjustment screws (item 9) as necessary to ensure that the wheel locks can get to their fully extended position when the pedal is engaged. Position of the Remote Lock Assembly may need to be fine-tuned to optimize the force needed to engage the pedal, depending on tire type. If excessive force is needed to engage the pedal, cable life may be reduced. Once proper adjustment is attained, lock cable adjustment screw in position by tightening jam nuts against mounting block.

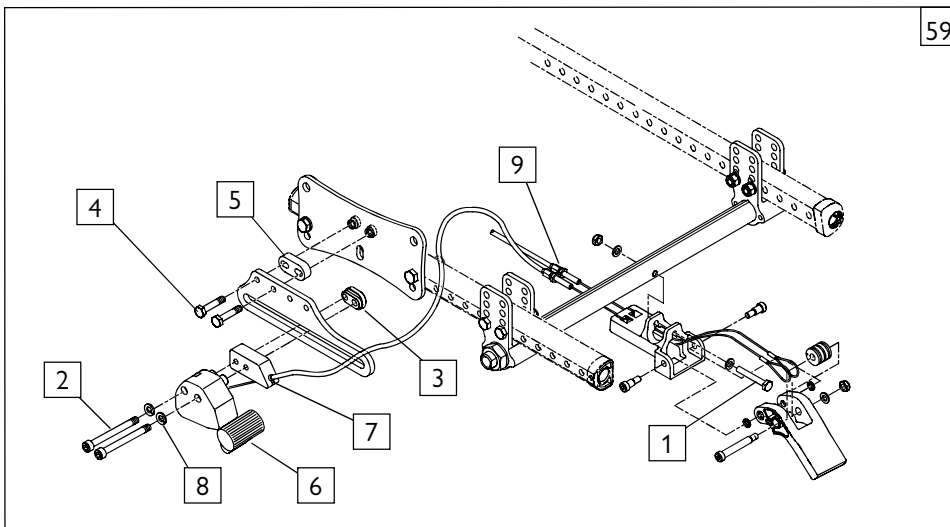




HH. FOOTPLATE ADAPTER BRACKET 60 61 62

The footplate adapter bracket (A) can be installed by the dealer. The main purpose of this part is to allow for height adjustment while using front mount footplates on a quick release hanger. The plate can be reversed which adds to its range of adjustment.

1. Footplate Height Adjustment

- a. Remove the hardware (B, C) and remove the footplate (D) and adapter plate assembly.
- b. Move the footplate (D) and assembly up or down to the desired position, using the "Front mount hanger setup" table and (figure 36) as reference.
- c. Re-insert and tighten the hardware (B, C) to a torque of 10.9 Nm [96 in-lbs]. 
- d. Repeat the process for the other hanger and footplate.

NOTE – In most cases you will want to make sure the footplates are adjusted so that the height is the same on both sides.

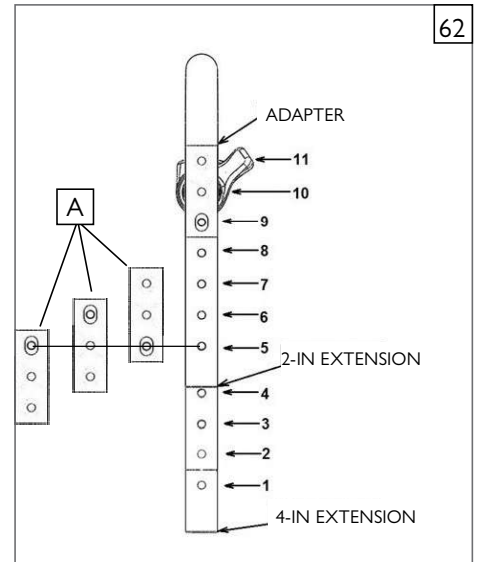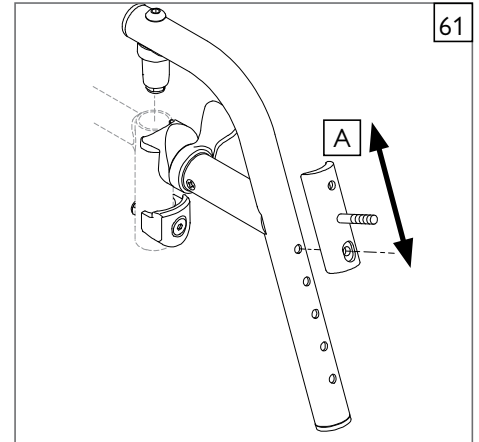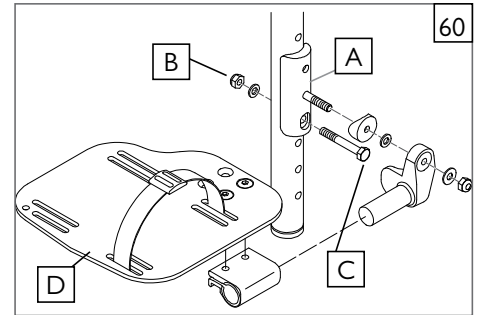

FRONT MOUNT HANGER SETUP											
Hanger Option	4-in Extension				Standard Hanger Holes					Footplate Adapter	
			2-in Extension		5	6	7	8	9	10	11
Hole	1	2	3	4	5	6	7	8	9	10	11
70D	16-in	15-in	14-in	13-in	11 1/2-in	10 1/2-in	9 1/2-in	9-in	8-in	7-in	6-in
80D	16 1/2-in	15 1/2-in	14 1/2-in	13 1/2-in	12-in	11-in	10-in	9-in	8-in	7-in	6-in
90D	—	—	—	—	11 1/2-in	10 1/2-in	9 1/2-in	8 1/2-in	7 1/2-in	6 1/2-in	5 1/2-in

IX. DEALER SERVICE & ADJUSTMENT

II. LAP BELT INSTRUCTIONS 63 64 65 66

NOTE – Lap belts are fitted as positional aids and are not suitable as transportation restraints.

Positioning Belt Kit

The positioning belt kit contains: one positioning belt (A), three tri glide buckles (B), and two cable ties (C).

1. To assemble the positioning belt, take the positioning belt strap and thread through the tri-glide. Repeat on both ends of positioning belt.
2. To fit the positioning belt to the chair, take the positioning belt strap and wrap around the back post.
3. Thread the positioning belt back through the tri-glide to form a loop around the back post. Thread positioning strap through tri-glide as shown.
4. To secure the positioning belt to the chair, take one of the cable ties supplied and thread through the positioning belt strap. Then wrap around the frame and fasten together. **DO NOT** over tighten positioning belt. Repeat steps 2-4 with both ends of positioning belt.
5. To adjust the positioning belt use the tri-glide buckles and the fastening buckle.
6. The positioning belt should be adjusted so the fastening buckle is situated in the middle of the stomach.
7. The positioning belt should be adjusted so a hand can be placed between the user and the belt.

WARNING

The positioning belt must only be fitted by a Sunrise Medical authorized dealer. The positioning belt should only be adjusted by a professional, or a Sunrise Medical authorized dealer. The positioning belt must be checked on a daily basis to ensure it is adjusted correctly (see step 6) and is free from any obstruction or adverse wear. Sunrise Medical does not encourage the transportation of any person in a vehicle using this positioning belt as a method of restraint. Please see Sunrise Medical transit booklet for further advice on transportation.

Lap Belt Maintenance


Check lap belt and securing components at regular intervals for any sign of frays or damage. Replace if necessary.

NOTE – The lap belt should be adjusted to suit the end user as detailed above. Sunrise Medical recommends that the length and fit of the belt be checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length.

JJ. INSPECT

Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures:

1. Review assembly and adjustment sections to make sure the chair was properly prepared.
2. Review troubleshooting guide.
3. If your problem persists, contact your Sunrise Medical authorized dealer. If you still have a problem after contacting your authorized dealer, contact Sunrise Medical customer service. See the introduction page for details on how to contact your authorized dealer or Sunrise Medical customer service.

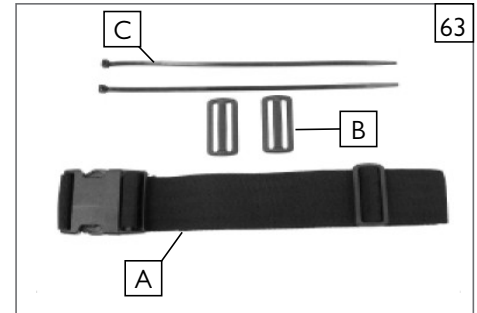

A. FOR LIFETIME

Frame and cross-brace (if applicable) warranty:

1. Although the anticipated useful service time of this wheelchair is five years, Sunrise Medical guarantees the frame and cross brace against defects in material and workmanship for life or for as long as the original purchaser owns the chair.
2. This warranty does not apply if:
 - a. The chair is subject to abuse.
 - b. The chair is not maintained as recommended in the owner's manual.
 - c. The chair is transferred to a different person from the original owner.

B. FOR ONE (1) YEAR

We warrant all Sunrise-made parts and components of this wheelchair against defects in materials and workmanship for one year from the date of first consumer purchase.

C. LIMITATIONS

1. We do not warrant:
 - a. Tires and tubes, upholstery, pads, and push-handle grips.
 - b. Damage from neglect, accident, misuse, or from improper installation or repair.
 - c. Products modified without Sunrise Medical's express written consent.
 - d. Damage from exceeding the weight limit.
2. This warranty is VOID if the original chair serial number tag is removed or altered.
3. This warranty applies in the USA and Canada only. Check with your Sunrise Medical authorized dealer to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this wheelchair through a Sunrise Medical authorized dealer.

D. WHAT WE WILL DO

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. WHAT YOU MUST DO

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Return the wheelchair or part(s), freight pre-paid to the designated Sunrise Medical location.
3. Pay the cost of labor to repair, remove, or install parts.

F. NOTICE TO CONSUMER

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

G. ADDITIONAL WARRANTY INFORMATION

For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights and remedies under a law in relation to the goods to which the warranty relates.

Record your serial number here for future reference:

(Serial number label is located on the bottom of the right base frame tube.)



Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
Tel.: +39 0523 573111
www.SunriseMedical.it

Sunrise Medical AG
Erlenauweg 17
CH-3110 Münsingen
Schweiz/Suisse/Svizzera
Tel +41 (0)31 958 3838
www.SunriseMedical.ch

Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norge
Telefon: +47 66 96 38 00
post@sunrisemedical.no
www.SunriseMedical.no

Sunrise Medical AB
Neongatan 5
431 53 Mölndal
Sweden
Tel.: +46 (0)31 748 37 00
post@sunrisemedical.se
www.SunriseMedical.se

MEDICCO s.r.o.
H – Park, Heršpická 1013/11d,
639 00 Brno
Czech Republic
Tel.: (+420) 547 250 955
Fax: (+420) 547 250 956
www.medicco.cz
info@medicco.cz
Bezplatná linka 800 900 809

Sunrise Medical Aps
Mårkærvej 5-9
2630 Taastrup
Denmark
+45 70 22 43 49
info@sunrisemedical.dk
www.SunriseMedical.dk

Sunrise Medical Australia
11 Daniel Street
Wetherill Park NSW 2164
Australia
Ph: +61 2 9678 6600
Email: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

Sunrise Medical (US) LLC
North American Headquarters
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com



© 2025 Sunrise Medical (US) LLC
MK-100151 Rev E

Sunrise Medical GmbH
Kahlbachring 2-4
D-69254 Malsch
Deutschland
Tel.: +49 (0) 7253/980-0
Fax: +49 (0) 7253/980-222
www.SunriseMedical.de

Sunrise Medical Ltd.
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
England
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

Sunrise Medical S.L.
Polígono Bakiola, 41
48498 Arrankudiaga – Vizcaya
España
Tel.: +34 (0) 902142434
Fax: +34 (0) 946481575
www.SunriseMedical.es

Sunrise Medical Poland
Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Polska
Telefon: + 48 42 275 83 38
Fax: + 48 42 209 35 23
E-mail: pl@sunrisemedical.de
www.Sunrise-Medical.pl

Sunrise Medical B.V.
Defensiedok 20
3433 NE Nieuwegein
The Netherlands
T: +31 (0)30 – 60 82 100
F: +31 (0)30 – 60 55 880
E: info@sunrisemedical.nl
www.SunriseMedical.nl

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

Sunrise Medical S.A.S
ZAC de la Vrillonnerie
17 Rue Mickaël Faraday
37170 Chambray-Lès-Tours
Tel : + 33 (0) 2 47 55 44 00
Email: info@sunrisemedical.fr
www.SunriseMedical.fr

Sunrise Medical Canada Inc.
1000 Creditstone Rd., Unit #2
Concord, ON, L4K 4P8
Canada
Phone: 1-800-263-3390
Fax: 1-800-561-5834
www.SunriseMedical.ca