

# JAY<sup>®</sup>

## Jay<sup>®</sup> Solution

### IMPORTANT CONSUMER INFORMATION

*NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.*

*User*

*Instruction*

*Manual &*

*Warranty*



**CAUTION—**

*Caution* Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

All Jay fluid cushions are designed to help reduce the incidence of pressure sores. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. Jay fluid cushions are not a substitute for good skin care including; proper diet, cleanliness, and regular pressure reliefs. In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check or skin redness. The clinical indicator for tissue break down is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage. Avoid sharp objects or exposure to excessive heat or open flame. Avoid sharp objects or exposure to excessive heat or open flame.

**FITTING A PRESSURE DISTRIBUTING CUSHION**

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (2.5cm - 5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and fluid pad toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered on the Jay Flow™ Fluid pad in the seat well of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the Jay Adjustable Solid seat.

Avoid using overly stretched back rest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may

result. Replace stretched back rest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact a authorized supplier for a free demonstration of these products. The pelvic inserts can be used to increase the contour of the cushion and increase the lateral pelvic stability.

**EASY MAINTENANCE & CLEANING**

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear and inspect the sleeve for punctures or any other abnormalities. Check the foam base to ensure foam consistency.\* Please refer to your specific fluid cushion section for any additional maintenance, cleaning, or assembly instructions.

**To clean cover**

**Remove the cover from the foam base close the zip, and machine wash in warm water and drip or tumble dry on low heat. No ironing required. Note: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach. To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base. Note: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing. Use only Jay-designed covers on Jay fluid cushions Use only Jay-designed covers on Jay fluid cushions. To clean foam base**

**To clean sleeve**

Remove the cover and the sleeve from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth. Let it completely air-dry. Note: Avoid submersion of the foam base in water. Reinstall the cover over the foam base. Ensure the back of the

**To CLEAN FOAM BASE**

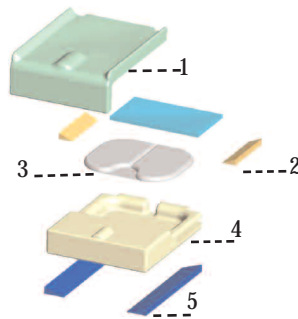
**Remove the cover and the sleeve from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth. Let it completely air-dry. Note: Avoid submersion of the foam base in water. Reinstall the cover over the foam base. Ensure the back of the cover matches up to the back of the foam base.**

**Jay Solution Cushion by Jay**

*The Jay Solution Cushion is designed with the benefits of skin protection in a light-weight, comfortable and low-maintenance design. The Easy incorporates a Jay Flow Fluid pad, a soft foam base, with a moisture resistant outer cover. The Jay Solution helps to provide pressure reduction which may help reduce the incidence of decubitus ulcers. It is comfortable and doesn't restrict movement,*

**JAY SOLUTION COMPONENTS**

- 1-Visco- Elastic Foam
- 2-Modular Pelvic Insert
- 3-Jay Fluid Technology
- 4-Jay Solution Foam Base
- 5-Built-In Lateral Wedges

**CAUTION**

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician

inspects your skin to ensure that red pressure spots are not developing. You should regularly check

for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops

redness, discontinue the use of the cushion immediately and see your doctor or therapist.

**JAY SOLUTION****CUSHION WARRANTY**

The Jay Solution cushion is guaranteed to be free from defects in materials and workmanship for a period of 12 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 12 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover and sleeve. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

Sunrise Medical  
 Home Healthcare Group  
 Mobility Products Division  
 237 Romina Drive, Unit 3  
 Concord, Ontario  
 Canada  
 (905) 660- 6704

© 2006, Sunrise Medical Canada