



QUICKIE QS5 X	3
FRAME	3
FRONT SIDE FRAME	3
FIXED FRONT SIDE FRAME	4
REAR SIDE FRAME	6
CROSS FRAME	7
TRANSIT KIT	8
AXLE PLATES AND AXLES	8
AXLE PLATE	9
QUICK RELEASE AXLES	10
WHEEL LOCKS	10
FOLD DOWN PUSH WHEEL LOCK (Knee Lock)	11
PUSH TO LOCK WHEEL LOCK (Metric Forged)	12
PULL TO LOCK WHEEL LOCK (Metric Forged)	14
ERGO SCISSOR WHEEL LOCK (Metric Forged)	16
COMPACT WHEEL LOCK (Metric Forged)	17
CASTORS AND FORKS	17
QS5 X FORK ASSEMBLY	18
QS5 X SINGLE SIDE FORK ASSEMBLY	19
QS5 X FROG LEG SUSPENSION FORK	20
3 SPOKE CASTERS	21
ALUMINUM SOFT ROLL CASTERS	22
ANTI-TIPS	22
ANTI-TIP	23
SWING-AWAY ANTI-TIP	25
WHEELS, TIRES & HANDRIMS	25
5 SPOKE MAG	26
LITE SPOKE	27
SPINERGY SPOX (18 SPOKE)	28
SPINERGY LX (12 SPOKE)	29
PROTON (24 SPOKE)	30
PNEUMATIC & PNEU. AIRLESS TIRE	31
FULL POLY TIRE	32
SHOX TIRE	33
PRIMO ORION TIRE	34
BLACK HIGH PRESSURE CLINCHER	35
SCHWALBE MARATHON PLUS TIRE	36
ALUMINUM ANODIZED (SCREW MOUNT)	37
ALUMINUM ANODIZED (TAB MOUNT)	38
PLASTIC COATED (SCREW MOUNT)	39
PLASTIC COATED (REG TAB MOUNT)	40
NATURAL FIT 5 SPOKE MAG (SCREW MOUNT)	41
NATURAL FIT (SCREW MOUNT)	42
NATURAL FIT (TAB MOUNT)	44
Q-GRIP (SCREW MOUNT)	46
Q-GRIP (TAB MOUNT)	47
CURVE L TETRA (SCREW MOUNT)	48
CURVE L TETRA (TAB MOUNT)	49
ELLIPSE 3R (SCREW MOUNT)	50
ELLIPSE 3R (TAB MOUNT)	51
ELLIPSE 3R 5 SPOKE MAG (SCREW MOUNT)	52
CARBO LIFE GEKKO (SCREW MOUNT)	53
CARBO LIFE GEKKO (TAB MOUNT)	54
TUBES, AIRLESS INSERTS AND ETPU INSERT	55
SEAT	55
SEAT SLING	56
BACKREST	56
BACKREST PUSH HANDLES	57
HALF FOLD BACK	58
ANGLE ADJUSTABLE BACKREST	59
EXO BACK UPHOLSTERY STD (Folding Chairs)	60
EXO TENSION ADJ UPHOLSTERY	64
EXO PRO TENSION ADJ UPHOLSTERY	67
TENSION ADJUSTABLE BALLISTIC NYLON UPHOLSTERY	70
TELESCOPING PUSH HANDLES	73
TELESCOPING PUSH HANDLES FOR JAY BACK	74
FOOTREST	74
60, 70 & 80 DEG SWING IN/OUT EXT MOUNT HANGERS	75
SWING IN-OUT EXT MOUNT ELR & ALR RETROFIT	76
SWING IN-OUT EXT MOUNT ELR REPLACEMENT PARTS	77

SWING IN-OUT ALR EXT MNT REPLACEMENT PARTS	79
ELR & ALR CALF PADS	81
COMPOSITE FOOTPLATE.....	83
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	85
MULTI ANGLE ADJ FOOTPLATE	88
ADULT ALUMINUM ANGLE-ADJ FOOTPLATE	90
ELR/ALR ALUMINUM FOOTPLATE (CLAMP)	92
FOLDING PLATFORM FOOTPLATE.....	94
PLATFORM ULTRA LITE FOOTPLATE	96
PLATFORM CARBON FIBER FOOTPLATE.....	97
AUTO-FOLD PLATFORM CARBON FOOTPLATE	98
ARMREST & SIDE GUARDS	99
PADDED SWING-AWAY ARMREST	100
SINGLE POST HEIGHT ADJ ARMREST & RECEIVER.....	101
FLIP BACK HEIGHT ADJ LITE ARMREST.....	103
PLASTIC SIDE GUARDS.....	105
ALU SIDE GUARDS FIXED	106
CARBON FIBER SIDE GUARDS SLIM & LARGE	107
ACCESSORIES	107
BACKPACKS AND SEAT POUCHES.....	108
PELVIC POSITIONING BELTS.....	109
CRUTCH HOLDER WITH VELCRO STRAP	110
TIP ASSIST	111
TRAVEL WHEEL.....	112
LED LIGHTS	113
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	114
ADJUSTABLE LEG STRAP	115
SPOKE GUARDS QUICKIE (Black & Clear).....	116
WATER BOTTLE AND CAGE.....	118
QUICKIE CADDY	119
MOBILE PHONE POCKET	120
TOOL KIT	121

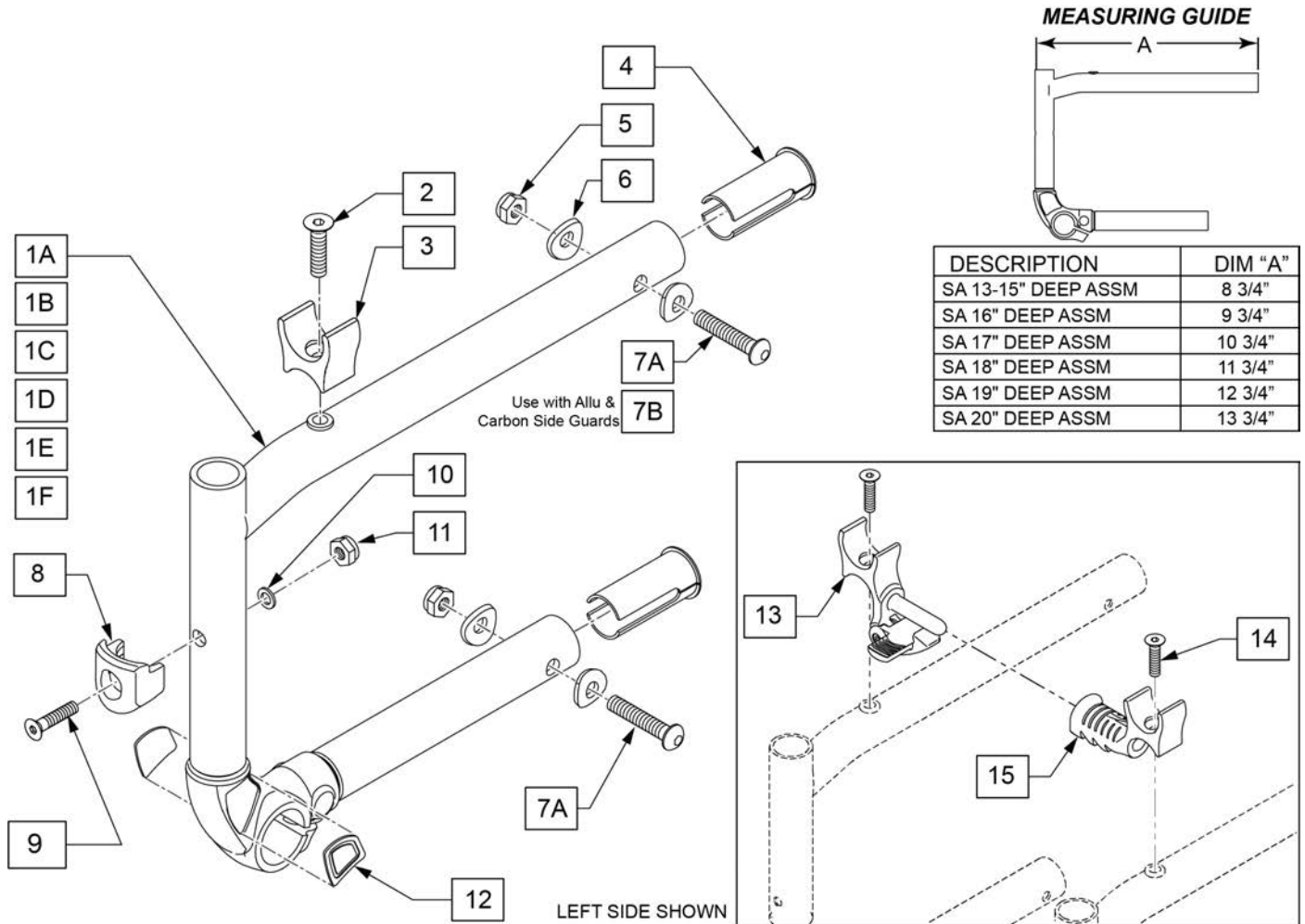


Pos.	Item Number	Description	Remarks
1A-7A,8-11	169741S	FRONT FRM SA ASSM 13-15 D QS5X	
1B-7A,8-11	169742S	FRONT FRAME SA ASSM 16 D QS5X	
1C-7A,8-11	169743S	FRONT FRAME SA ASSM 17 D QS5X	
1D-7A,8-11	169744S	FRONT FRAME SA ASSM 18 D QS5X	
1E-7A,8-11	169745S	FRONT FRAME SA ASSM 19 D QS5X	
1F-7A,8-11	169746S	FRONT FRAME SA ASSM 20 D QS5X	
2,3	169747	SADDLE KIT FOLDING	
4	170134	FRAME BUSHING KIT	
5,6,7A,7B	169748	FRONT FRAME FASTENER KIT QS5X	
7B	102262-006	BHCS,M6X1X40,FULL,NP	
8-11	119514S	LATCH RECEIVER KIT	
12	170133	TOUCH POINT REPLACEMENT KIT	1 inside piece & 1 outside piece
13-15	170204	FRAME FOLD LOCK KIT STD	

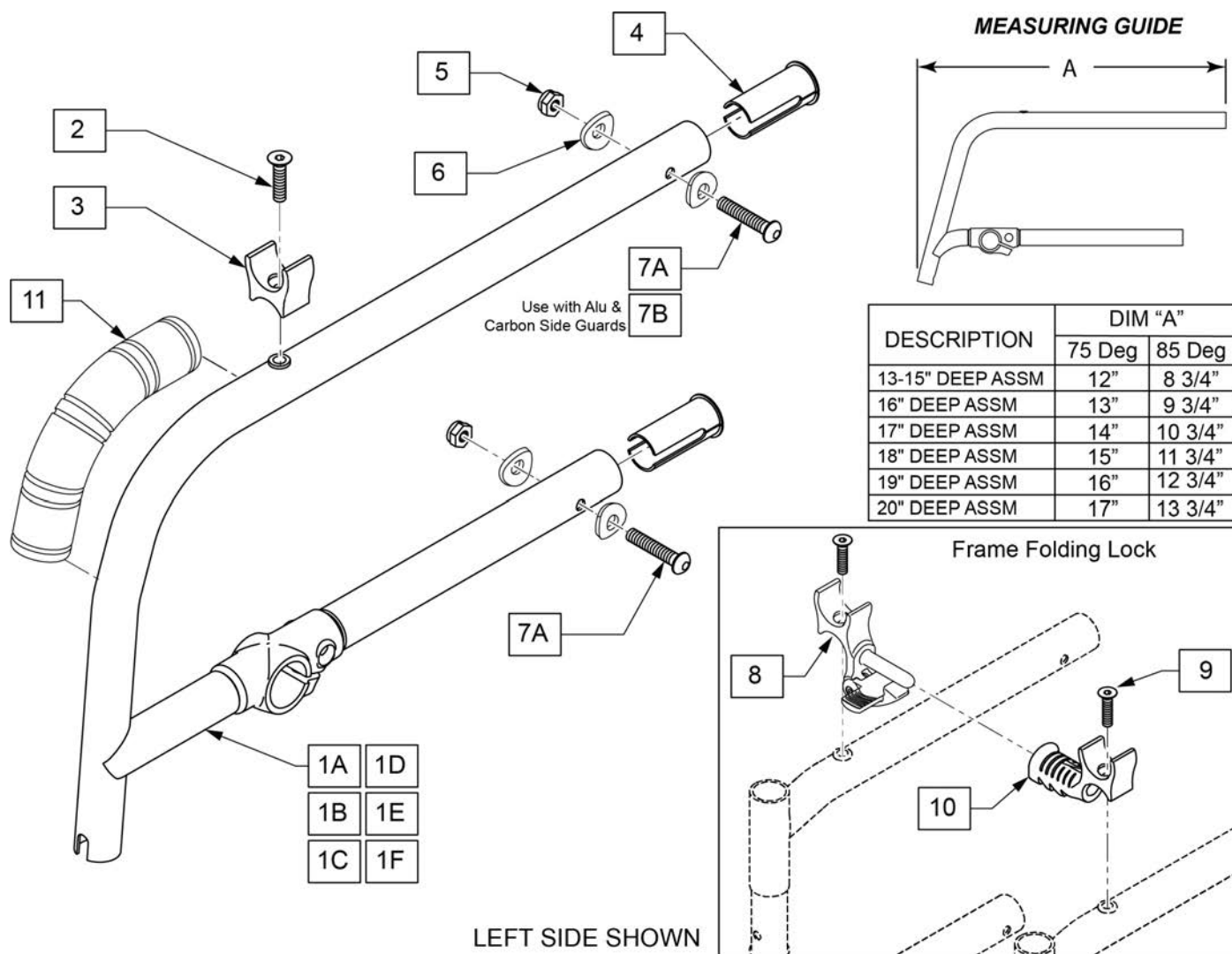


Pos.	Item Number	Description	Remarks
1A-7	169749S	FF FRM FX 75 ASSM13-15D QS5X	
1B-7	169750S	FF FRM FX 75 ASSM 16 D QS5X	
1C-7	169751S	FF FRM FX 75 ASSM 17 D QS5X	
1D-67	169752S	FF FRM FX 75 ASSM 18 D QS5X	
1E-7	169753S	FF FRM FX 75 ASSM 19 D QS5X	
1F-7	169754S	FF FRM FX 75 ASSM 20 D QS5X	
1G-7	169755S	FF FRM 85 FX ASSM13-15D QS5X	
1H-7	169756S	FF FRM 85 FX ASSM 16 D QS5X	
1I-7	169757S	FF FRM 85 FX ASSM 17 D QS5X	
1J-7	169758S	FF FRM 85 FX ASSM 18 D QS5X	
1K-7	169759S	FF FRM 85 FX ASSM 19 D QS5X	
1L-7	169760S	FF FRM FX 75 ASSM 20 D QS5X	
1M-7	169761S	FF FRM FX 13-15IN 85DEG 1I L	Left 1" INSET
1M-7	169762S	FF FRM FX 13-15IN 85DEG 1I R	Right 1" INSET
1N-7	169763S	FF FRM FX 16IN 85DEG 1I L	Left 1" INSET
1N-7	169764S	FF FRM FX 16IN 85DEG 1I R	Right 1" INSET
1O-7	169765S	FF FRM FX 17IN 85DEG 1I L	Left 1" INSET
1O-7	169766S	FF FRM FX 17IN 85DEG 1I R	Right 1" INSET
1P-7	169767S	FF FRM FX 18IN 85DEG 1I L	Left 1" INSET


Pos.	Item Number	Description	Remarks
1P-7	169768S	FF FRM FX 18IN 85DEG 1I R	Right 1" INSET
1Q-7	169769S	FF FRM FX 19IN 85DEG 1I L	Left 1" INSET
1Q-7	169770S	FF FRMFX 19IN 85DEG 1I R	Right 1" INSET
1R-7	169771S	FF FRM FX 20IN 85DEG 1I L	Left 1" INSET
1R-7	169772S	FF FRM FX 20IN 85DEG 1I R	Right 1" INSET
2,3	169747	SADDLE KIT FOLDING	
4	170134	FRAME BUSHING KIT	Pair
5-7A,7B	169748	FRONT FRAME FASTENER KIT QS5X	
7B	102262-006	BHCS,M6X1X40,FULL,NP	
8-10	170204	FRAME FOLD LOCK KIT STD	
11	71201061	Fixed Front touch point	

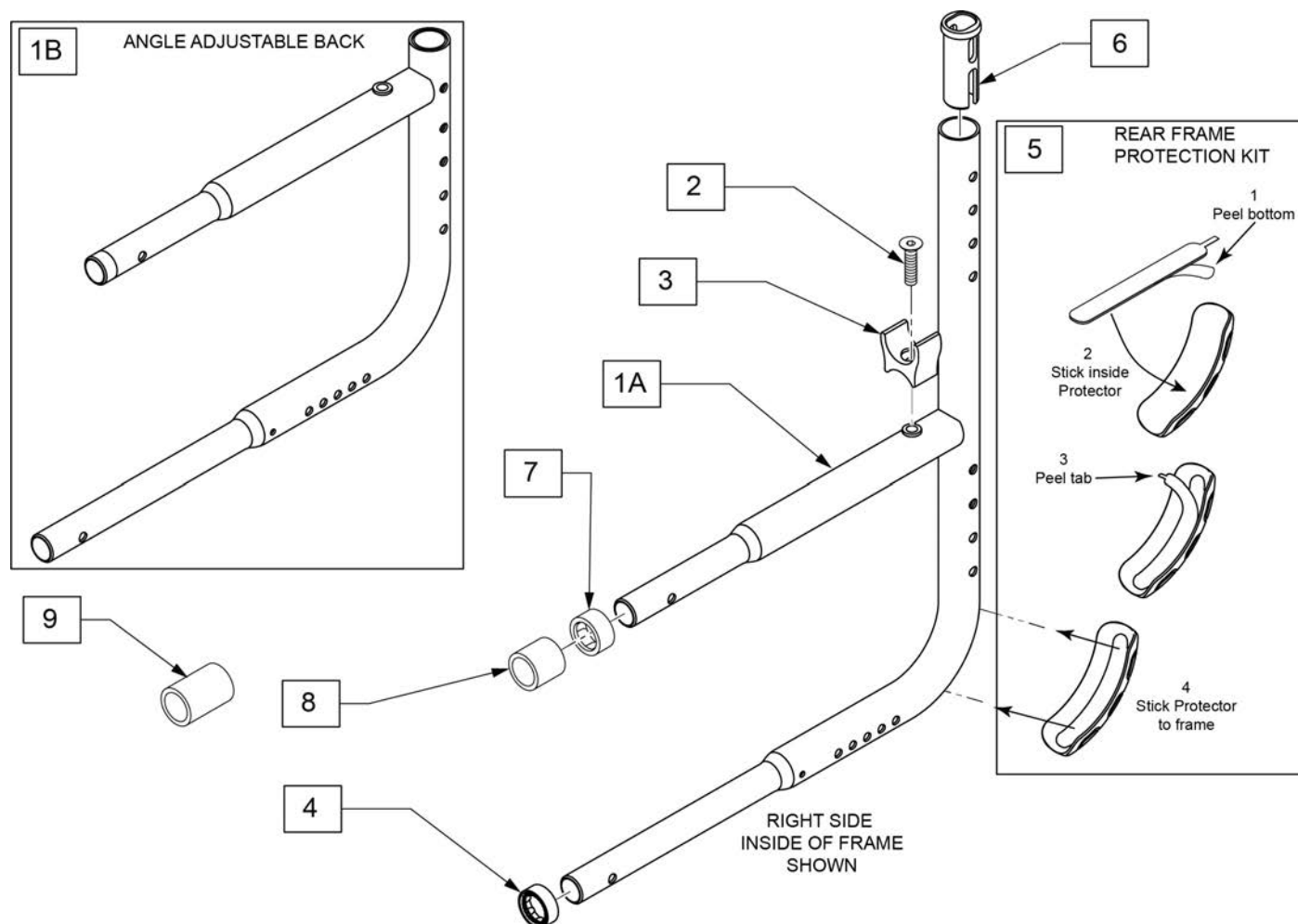

Pos.	Item Number	Description	Remarks
5	170135	REAR FRAME PROTECTION KIT	Includes left & right
1A-8	169773S	REAR SIDE FRAME ASSM STD LH	Left
1A-6,9	169774S	REAR SIDE FRAME ASSM STD RH	Right
1B-8	169775S	REAR SIDE FRAME ASSM HT ADJ LH	Left
1B-6,9	169776S	REAR SIDE FRAME ASSM HT ADJ RH	Right
2,3	169747	SADDLE KIT FOLDING	
4,7-9	170137	SPACER AND BUSHING KIT 5X	

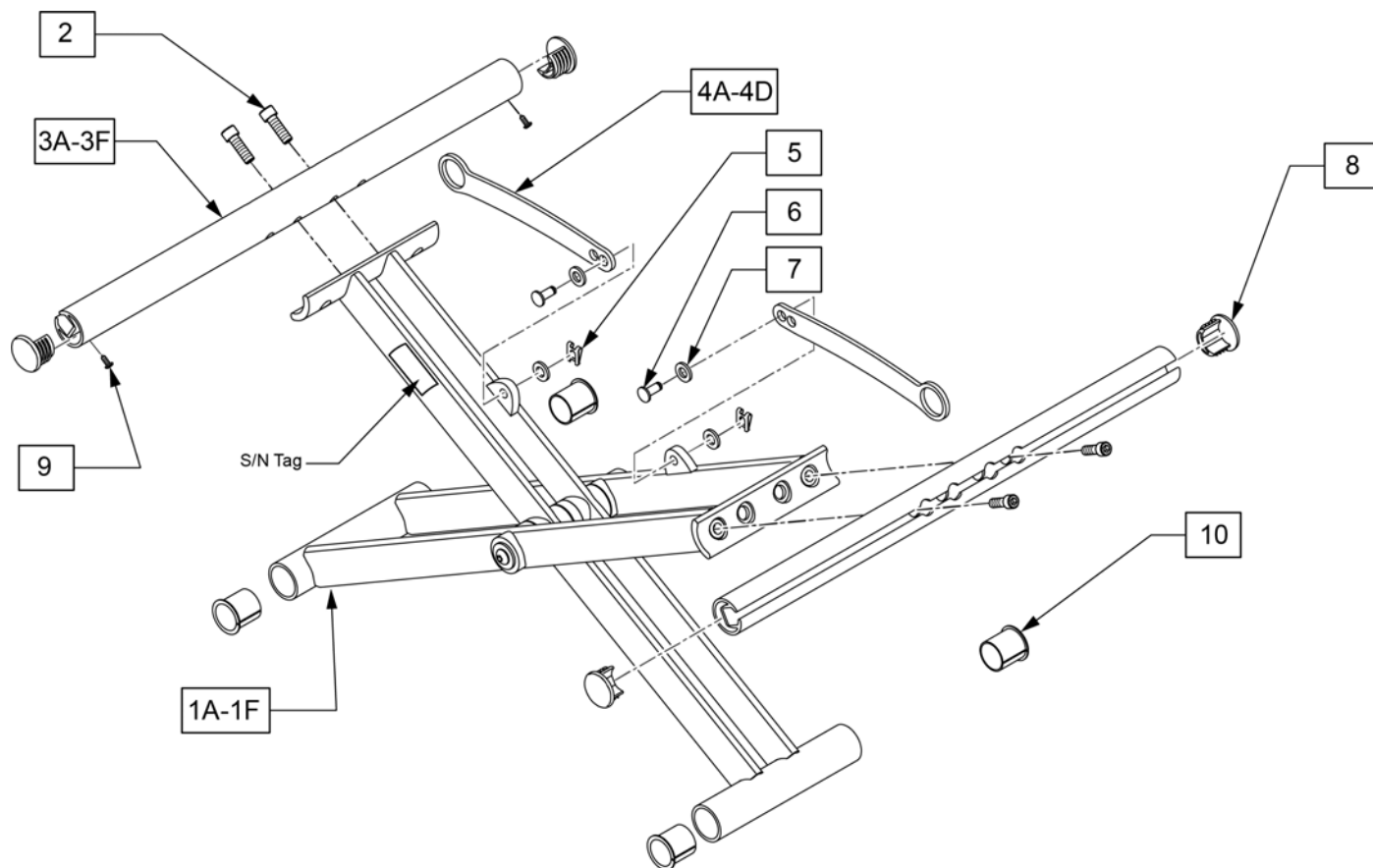

Pos.	Item Number	Description	Remarks
1A,4A,5-7	169783S	QS5X X BRACE ASSM 13W	
1B,4A,5-7	169784S	QS5X X BRACE ASSM 14W	
1C,4B,5-7	169785S	QS5X X BRACE ASSM 15W	
1D,4B,5-7	169786S	QS5X X BRACE ASSM 16W	
1E,4C,5-7	169787S	QS5X X BRACE ASSM 17W	
1F,4C,5-7	169788S	QS5X X BRACE ASSM 18W	
1G,4D,5-7	169789S	QS5X X BRACE ASSM 19W	
1H,4D,5-7	169790S	QS5X X BRACE ASSM 20W	
2,3A,8	169777	QS5X SEAT RAIL KIT 13-15D	Pair
2,3B,8	169778	QS5X SEAT RAIL KIT 16D	Pair
2,3C,8	169779	QS5X SEAT RAIL KIT 17D	Pair
2,3D,8	169780	QS5X SEAT RAIL KIT 18D	Pair
2,3E,8	169781	QS5X SEAT RAIL KIT 19D	Pair
2,3F,8	169782	QS5X SEAT RAIL KIT 20D	Pair
4A-6	169791	X-HINGE KIT 13-14W	
4B-6	169792	X-HINGE KIT 15-16W	
4C-6	169793	X-HINGE KIT 17-18W	
4D-6	169794	X-HINGE KIT 19-20W	
8,9	108117	SEAT RAIL END PLUG KIT (4)	Set of 4
10	560402S	3/4" NY-LINER	

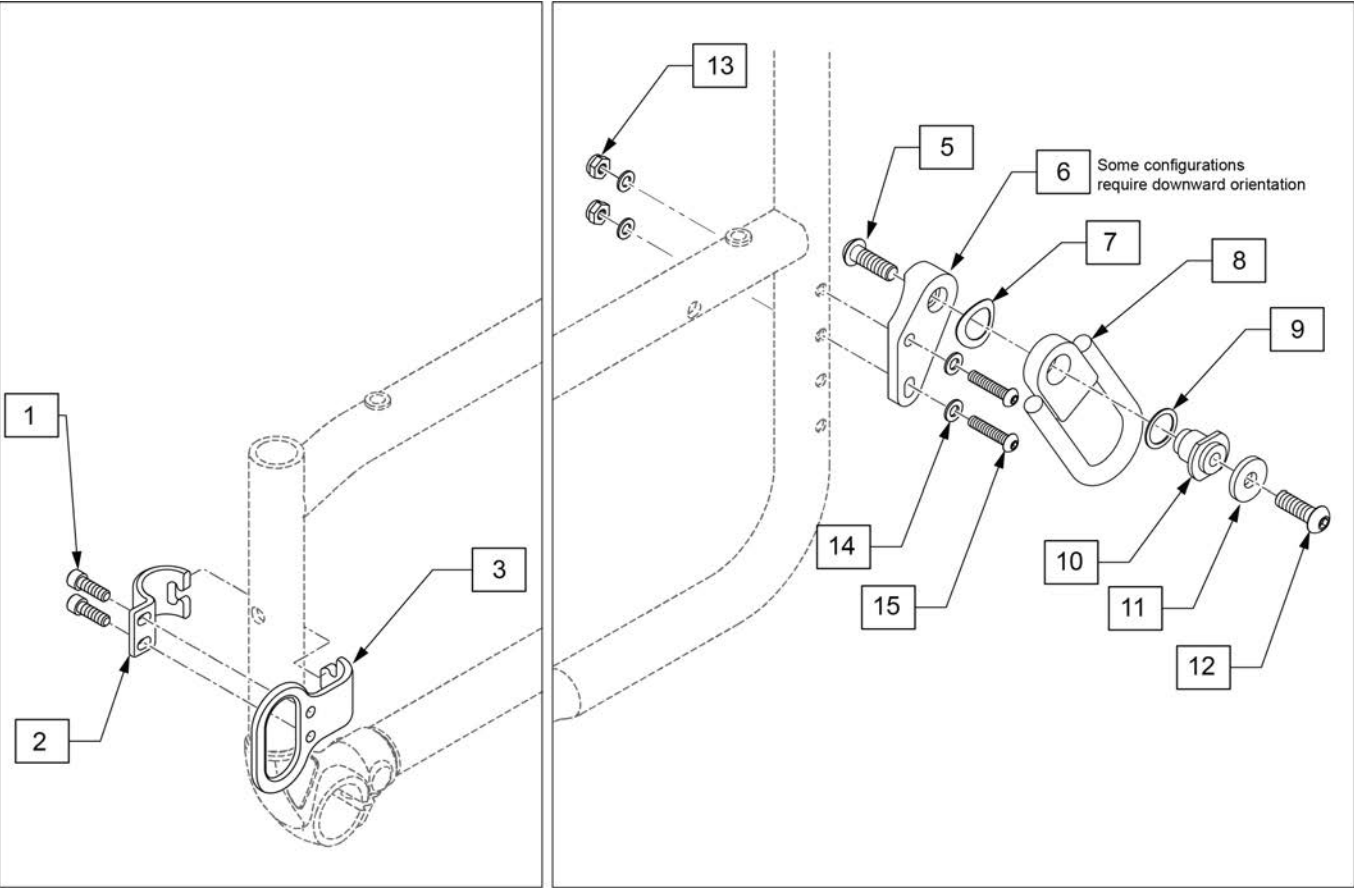


Left outside view



Pos.	Item Number	Description	Remarks
1-3	169801	FRONT TRANSIT KIT QS5X	
5-15	169802	REAR TRANSIT KIT QS5X	
13-15	169803	FASTENER KIT QS5X	for one side - 2 of each

Camber Guide


Camber	Upper Spacer	Lower Spacer	Lower SHCS
0 deg w/o ht adj armrests	none	none	10A
0 deg w/ ht adj armrests	1 item 12	1 item 12	10A
2 deg w/o ht adj armrests	none	1 item 12	10A
2 deg w/ ht adj armrests	1 item 12	1 item 14	10B
4 deg	1 item 12	1 item 12+1 item 14	10C

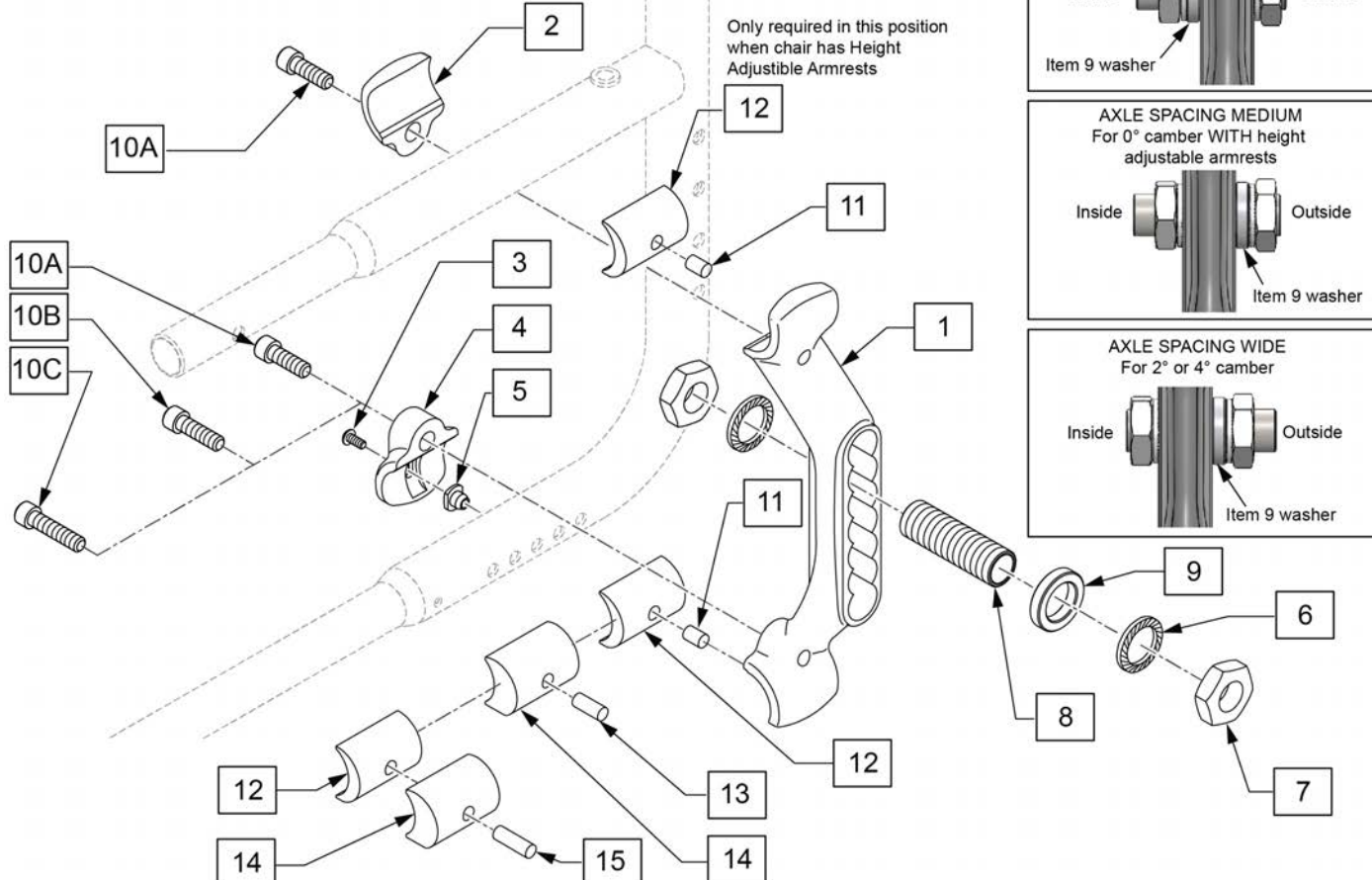

Pos.	Item Number	Description	Remarks
6-9	169800	AXLE SLEEVE KIT	ONE SIDE
1-10A	169797	AXLE PLATE KIT RH	right
1-10A	169798	AXLE PLATE KIT LH	left
10A,11,12	169799	CAMBER KIT 2 DEG	2 degree, std spacing, for 2 sides
10A,10C,11,12,14,15	170138	CAMBER KIT 4 DEG	4 degree, all spacing, for 2 sides
10A,10B,11-14		CAMBER KIT 2 DEG WIDE	4 degree, wide spacing, for 2 sides
10A,11,12		CAMBER KIT 0 DEG WIDE	0 degree, wide spacing, for 2 sides

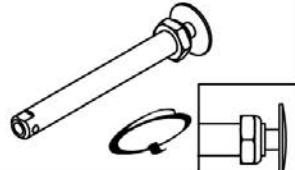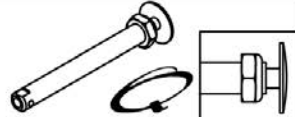
Wheel Type	Axle Type		Axle P/N	Spacer P/N
Proton Spinergy Wheels 5 Spoke Mag	Quick Release	1A 	109385 A=4 1/2"	N/A
5 Spoke Mag	Quad Quick Release	1B 	922170 A=4 1/2"	N/A
Lite Spoke	Quick Release	1C 	107268 A=4 1/4"	N/A

Quick Release Measuring Guide

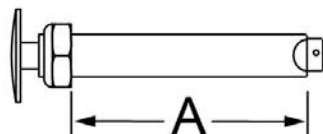


1/2" Axle Diameter

Pos.	Item Number	Description
1A	109385S	QR AXLE/NUT LONG BG BTTN
1B	922170	QR AXLE MAG W/QUAD NUT
1C	107268S	QR AXLE AND NUT SHORT BIG BTTN


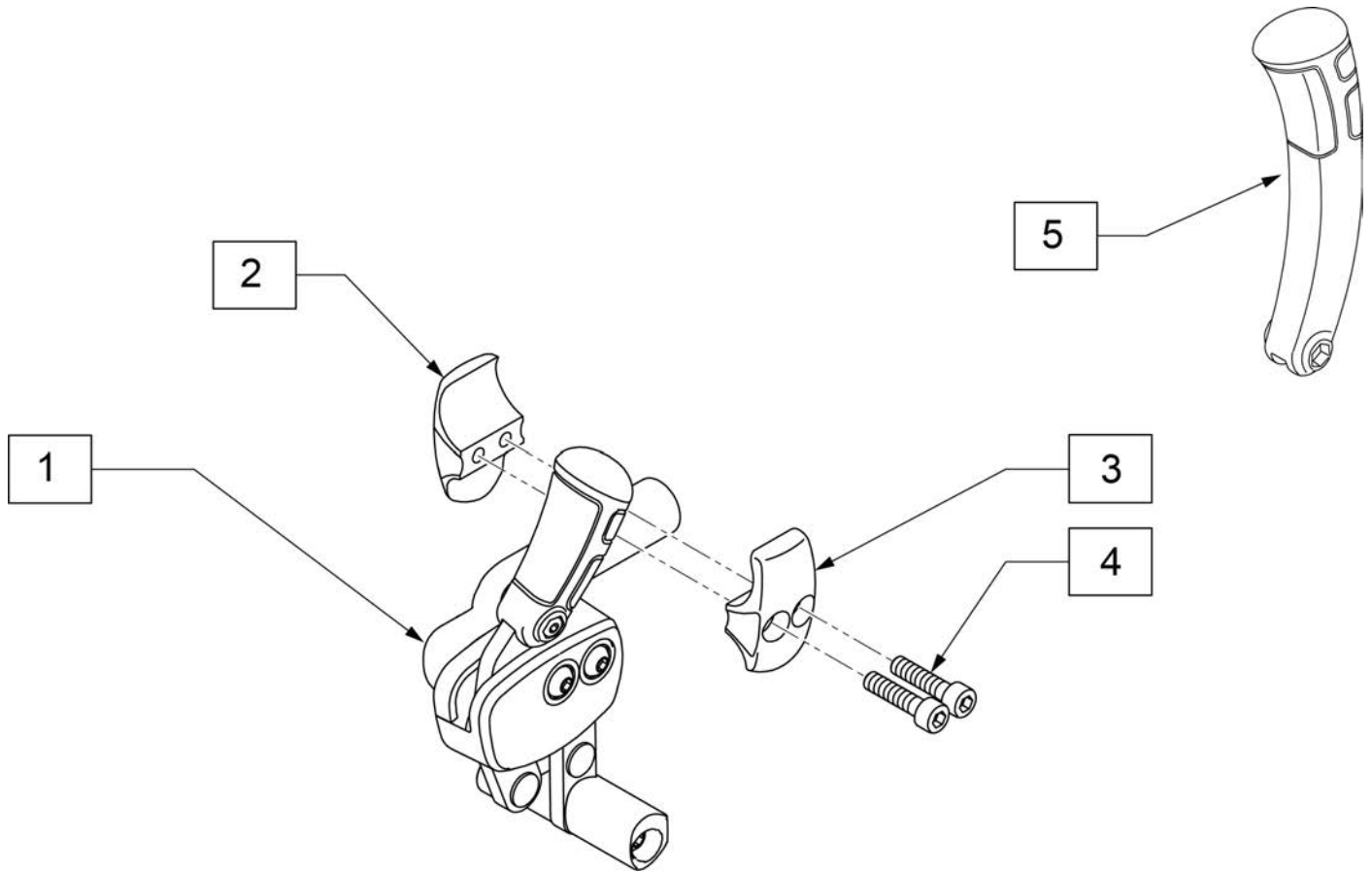


Pos.	Item Number	Description
1	71206000-002S	KNEE LEVER WHEEL LOCK RIGHT
1	71206000-001S	KNEE LEVER WHEEL LOCK LEFT
1-4	169804	FOLD DW PUSH WL RH
1-4	169805	FOLD DW PUSH WL LH
2-4	169806	WHEEL LOCK CLAMP ASSM
5	71206035	BRAKE LEVER EXTENSION

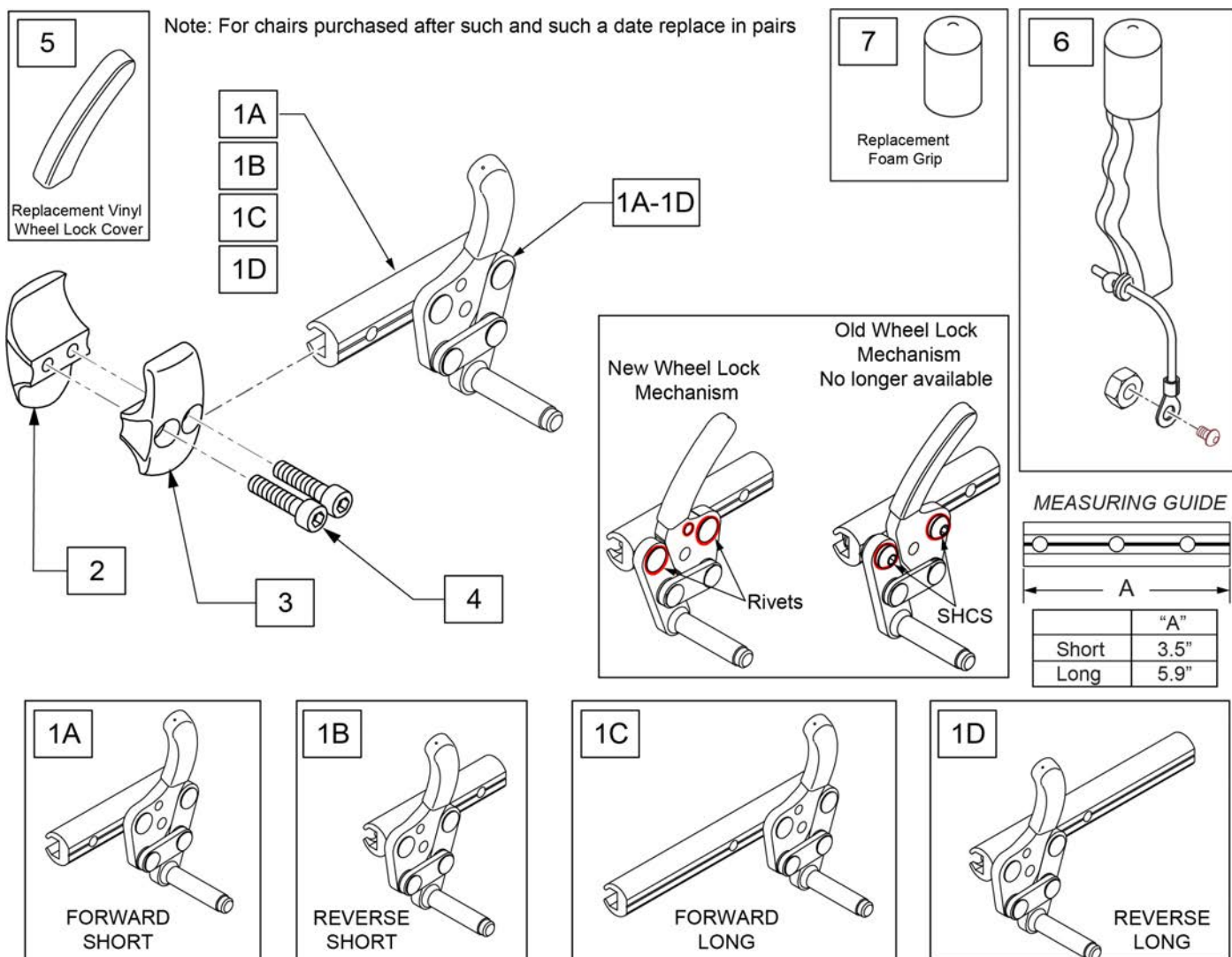


Pos.	Item Number	Description	Remarks
1A	259041S	WHEEL LOCK ASSEMBLY PUSH RIGHT	RIGHT
1A	259042S	WHEEL LOCK ASSEMBLY PUSH LEFT	LEFT
1B	259043S	WHEEL LOCK PUSH R REV SHT	RIGHT
1B	259044S	WHEEL LOCK PUSH L REV SHT	LEFT
1C	259045S	WHEEL LOCK PUSH R FWD LNG	RIGHT
1C	259046S	WHEEL LOCK PUSH L FWD LNG	LEFT
1D	259047S	WHEEL LOCK PUSH R REV LNG	RIGHT
1D	259048S	WHEEL LOCK PUSH L REV LNG	LEFT
1A,6	170376	WHL LOCK PUSH R FWD SHT W EXT	RIGHT
1A,6	170377	WHL LOCK PUSH L FWD SHT W EXT	LEFT
1B,6	170378	WHL LOCK PUSH R REV SHT W EXT	RIGHT
1B,6	170379	WHL LOCK PUSH L REV SHT W EXT	LEFT
1C,6	170380	WHL LOCK PUSH R FWD LNG W EXT	RIGHT
1C,6	170381	WHL LOCK PUSH L FWD LNG W EXT	LEFT
1D,6	170382	WHL LOCK PUSH R REV LNG W EXT	RIGHT
1D,6	170383	WHL LOCK PUSH L REV LNG W EXT	LEFT
2-4	169806	WHEEL LOCK CLAMP ASSM	
5	104141S	COVER CURVED WHEEL LOCK VINYL	

Pos.	Item Number	Description	Remarks
6	161930	WHEEL LOCK EXT ASSY (PAIR) KIT	PAIR
7	560600S	FOAM GRIP 1 1/2 X 0.705 X 0.1	
1A-4	169807	PUSH WHEEL LOCK ASSM RH	RIGHT
1A-4	169808	PUSH WHEEL LOCK ASSM LH	LEFT
1B-4	169809	PUSH WHEEL LOCK REV ASSM RH	RIGHT
1B-4	169810	PUSH WHEEL LOCK REV ASSM LH	LEFT
1C-4	169811	PUSH WHL LOCK LNG ASSM RH	RIGHT
1C-4	169812	PUSH WHL LOCK LNG ASSM LH	LEFT
1D-4	169813	PUSH WHL LOCK LNG REV ASSM RH	RIGHT
1D-4	169814	PUSH WHL LOCK LNG REV ASSM LH	LEFT

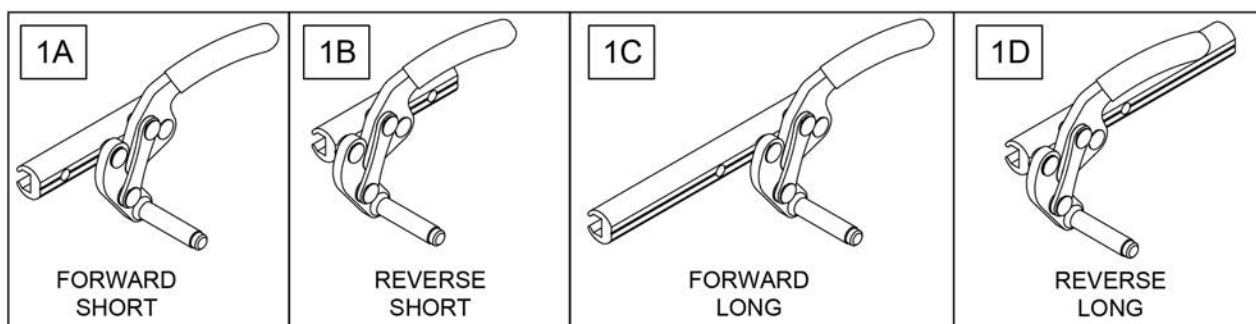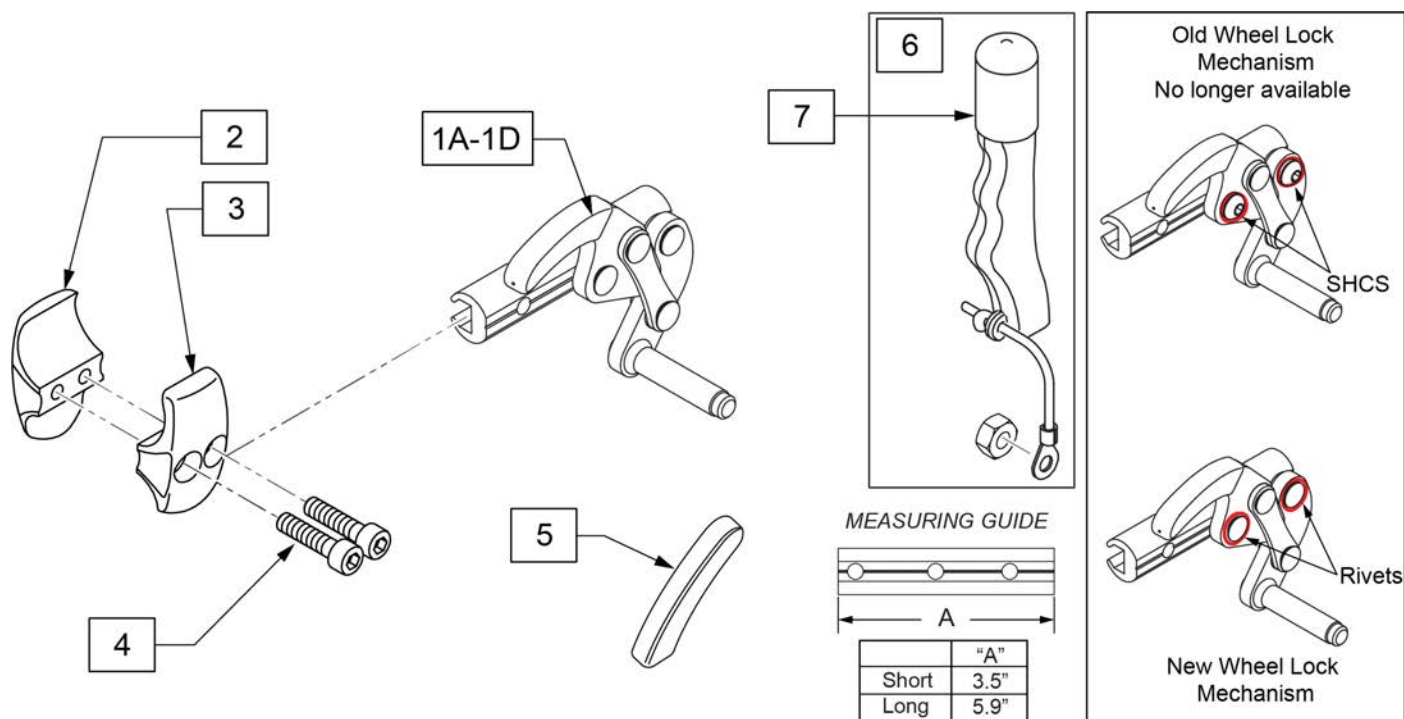


Pos.	Item Number	Description	Remarks
1A	259049S	WHEEL LOCK PULL R FWD SHT	Right-forward-short
1A	259051S	WHEEL LOCK PULL L FWD SHT	Left-forward-short
1B	259052S	WHEEL LOCK PULL R REV SHT	Right-reverse-short
1B	259053S	WHEEL LOCK PULL L REV SHT	Left-reverse-short
1C	259054S	WHEEL LOCK PULL R FWD LNG	Right-forward-long
1C	259055S	WHEEL LOCK PULL L FWD LNG	Left-forward-long
1D	259056S	WHEEL LOCK PULL R REV LNG	Right-reverse-long
1D	259057S	WHEEL LOCK PULL L REV LNG	Left-reverse-long
2-4	169806	WHEEL LOCK CLAMP ASSM	
5	104141S	COVER CURVED WHEEL LOCK VINYL	
6	161929	WHEEL LOCK EXTENSION ASSY KIT	EACH
7	560600S	FOAM GRIP 1 1/2 X 0.705 X 0.1	
1A-4	169815	PULL WHEEL LOCK ASSM RH	RIGHT
1A-4	169816	PULL WHEEL LOCK ASSM LH	LEFT
1B-4	169817	PULL WHEEL LOCK REV ASSM RH	RIGHT
1B-4	169818	PULL WHEEL LOCK REV ASSM LH	LEFT
1C-4	169819	PULL WHL LOCK LNG ASSM RH	RIGHT
1C-4	169820	PULL WHL LOCK LNG ASSM LH	LEFT
1D-4	169821	PULL WHL LOCK LNG REV ASSM RH	RIGHT


Pos.	Item Number	Description	Remarks
1A,6	170384	WHL LOCK PULL R FWD SHT W EXT	Right-forward-short
1A,6	170385	WHL LOCK PULL L FWD SHT W EXT	Left-forward-short
1B,6	170386	WHL LOCK PULL R REV SHT W EXT	Right-reverse-short
1B,6	170387	WHL LOCK PULL L REV SHT W EXT	Left-reverse-short
1C,6	170388	WHL LOCK PULL R FWD LNG W EXT	Right-forward-long
1C,6	170389	WHL LOCK PULL L FWD LNG W EXT	Left-forward-long
1D,6	170390	WHL LOCK PULL R REV LNG W EXT	Right-reverse-long
1D,6	170391	WHL LOCK PULL L REV LNG W EXT	Left-reverse-long

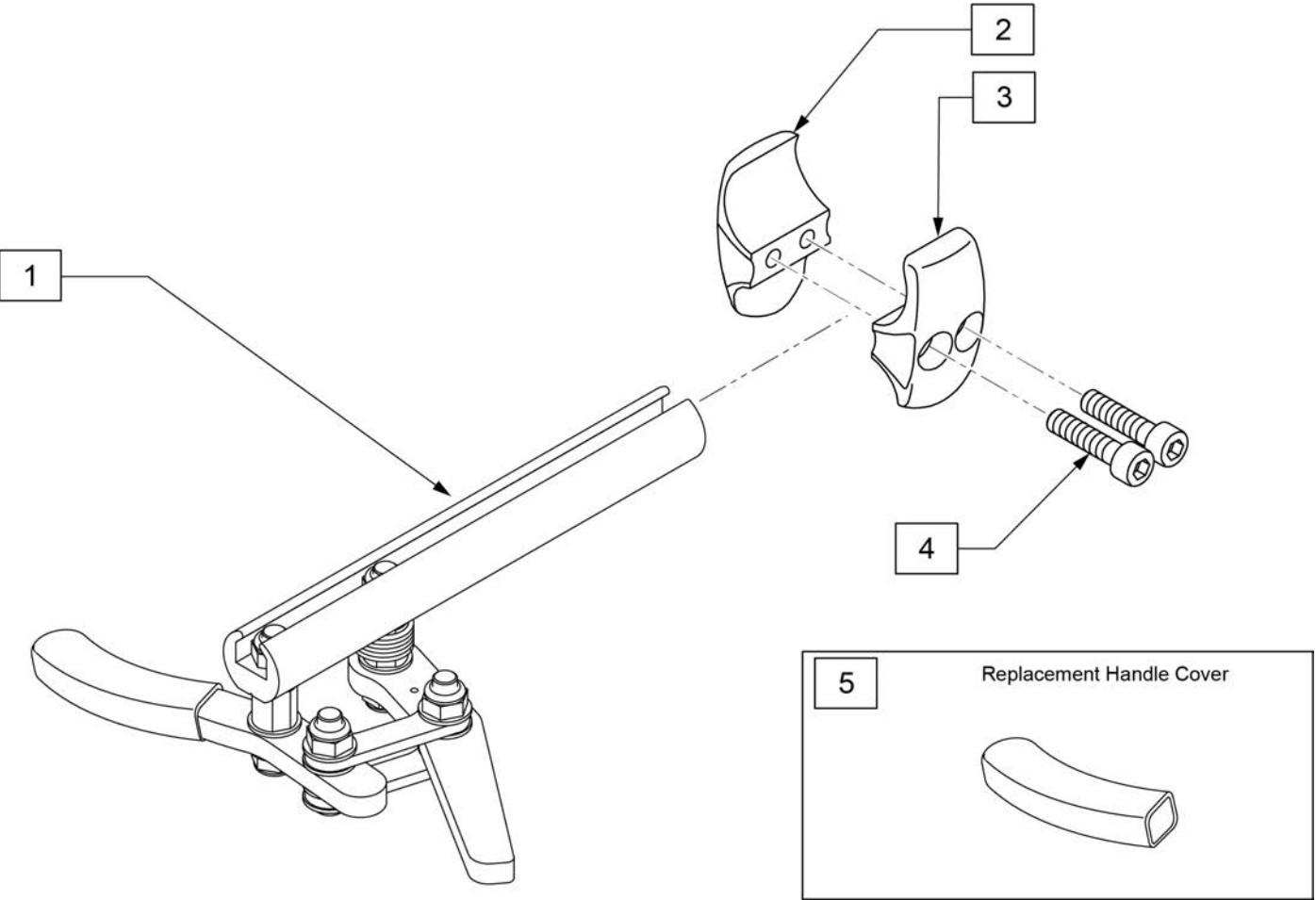


Pos.	Item Number	Description	Remarks
1	104726-001S	ASSY WHL LOCK SCISSOR ERGO	RIGHT
1	104726-002S	ASSY WHL LOCK SCISSOR ERGO	LEFT
2-4	169806	WHEEL LOCK CLAMP ASSM	
5	104141S	COVER CURVED WHEELLOCK VIN	
1-4	169823	ERGO SCISSOR WHL LOCK ASSM RH	
1-4	169824	ERGO SCISSOR WHL LOCK ASSM LH	

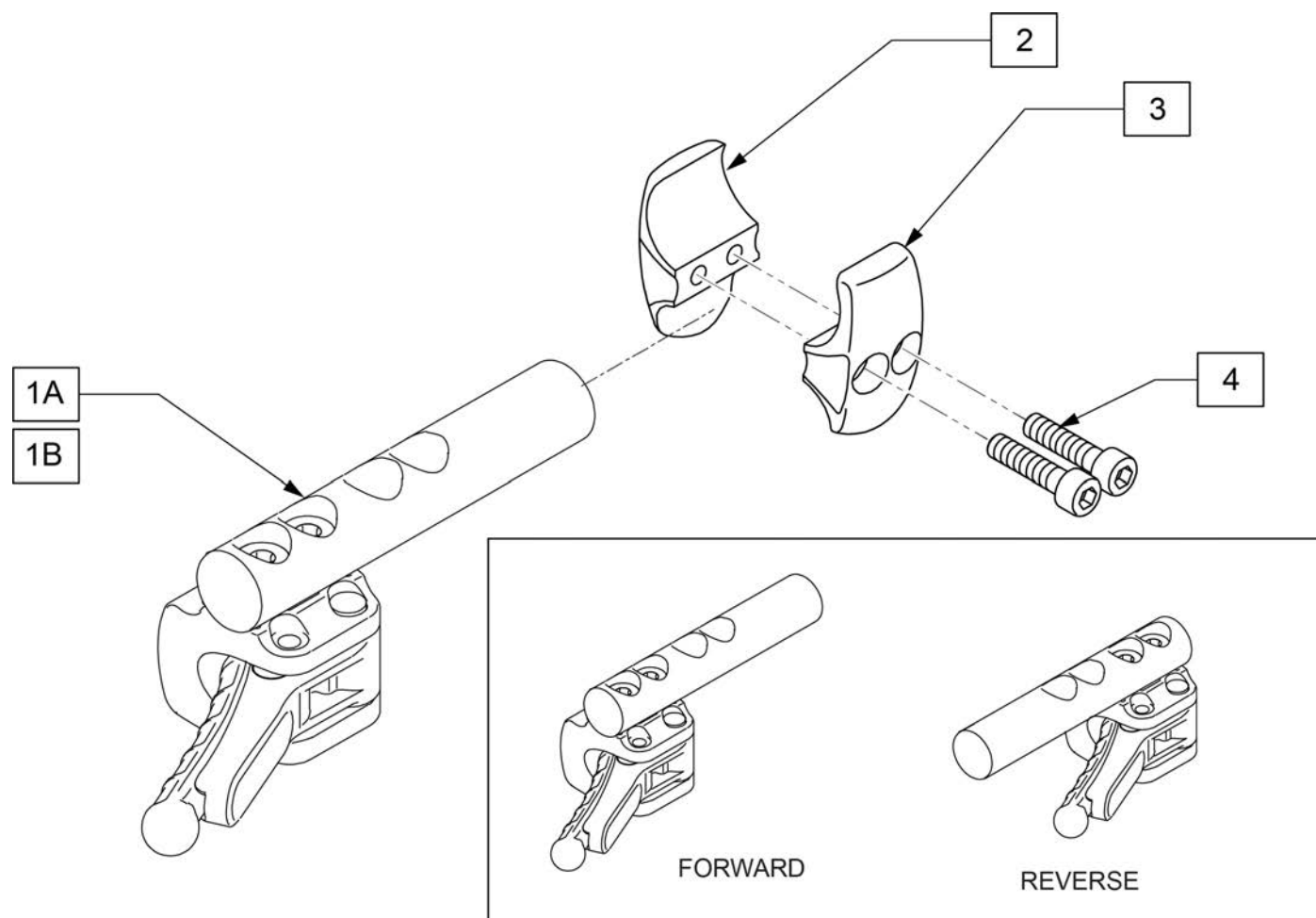

Pos.	Item Number	Description	HCPC
1A	66006100-302S	COMPACT BRAKE STD FRONT RIGHT	Forward Mounted
1A	66006100-301S	COMPACT BRAKE STD FRONT LEFT	Forward Mounted
1B	66006100-402S	COMPACT BRAKE STD BACK RIGHT	Reverse Mounted
1B	66006100-401S	COMPACT BRAKE STD BACK LEFT	Reverse Mounted
2-4	169806	WHEEL LOCK CLAMP ASSM	
1A-4	169825	COMPACT WHEEL LOCK ASSM FWD RH	Forward Mounted
1A-4	169826	COMPACT WHEEL LOCK ASSM FWD LH	Forward Mounted
1B-4	169827	COMPACT WHEEL LOCK ASSM RVS RH	Reverse Mounted
1B-4	169828	COMPACT WHEEL LOCK ASSM RVS LH	Reverse Mounted

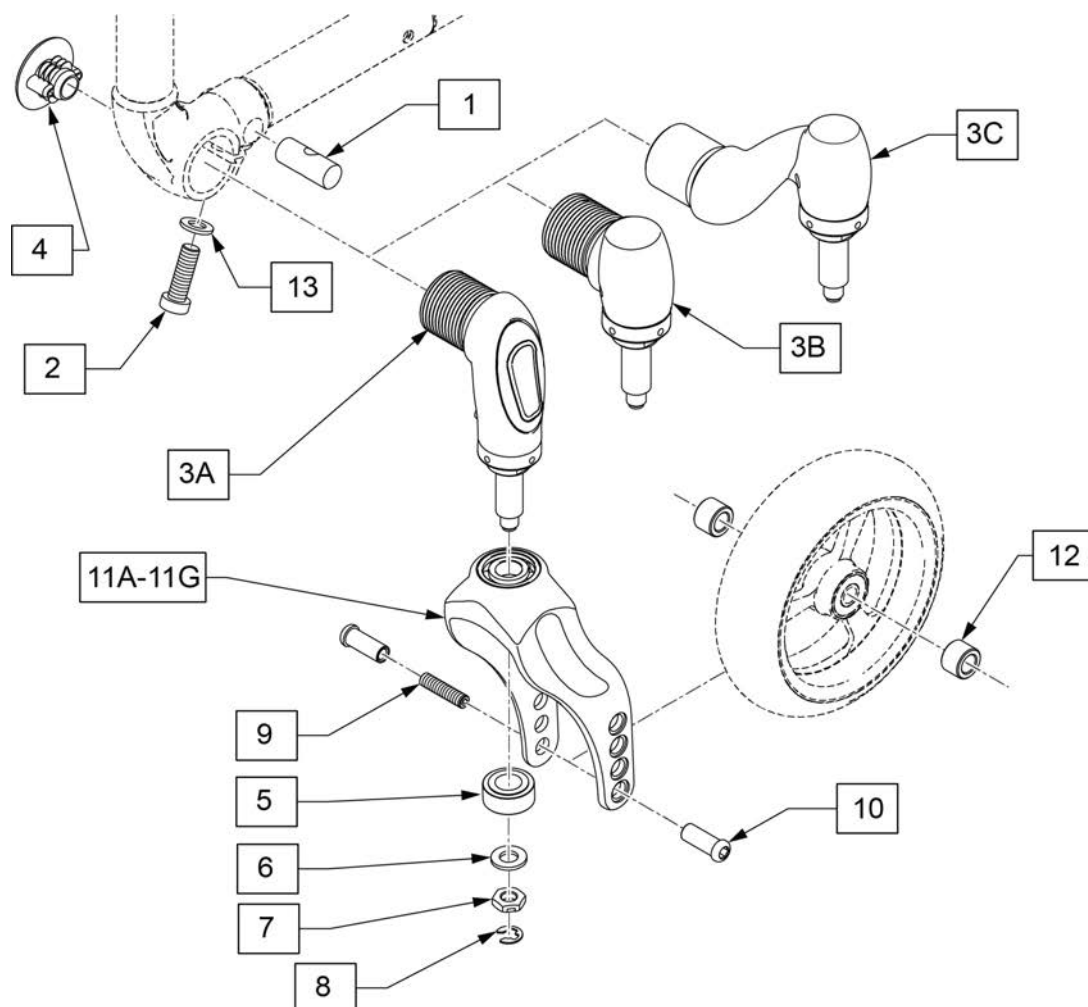


MEASURING GUIDE

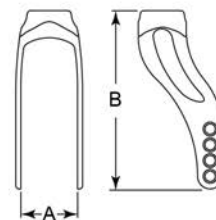


Decription	Dim "A"	Dim "B"
Fork 115x45mm	45mm	125mm
Fork 131x45mm	45mm	141mm
Fork 158x45mm	45mm	168mm
Fork 131x32mm	32mm	141mm
Fork 115x32mm	32mm	125mm
Fork 90x32mm	32mm	100mm
Fork 90x45mm	45mm	100mm

Alternate position for Spacer (item 5) is above fork to increase STF by 1/2".

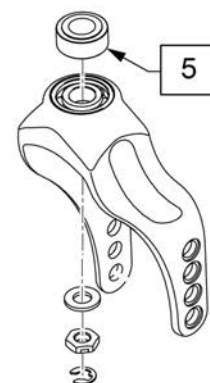


Pos.	Item Number	Description	Remarks
1,2,13	169830	CASTER LINK FASTENERS	For 1 side
4	170139	LINK CAP KIT	Pair
3A,5-8	169832	DESIGN CASTER LINK KIT LH	
3A,5-8	169833	DESIGN CASTER LINK KIT RH	
3B,5-8	169835	SHORT CASTER LINK KIT LH	
3B,5-8	169836	SHORT CASTER LINK KIT RH	
3C,5-8	169834	HEMI CASTER LINK KIT	
5-8	169831	FORK STEM FASTENERS	
9,10	169829	AXLE KIT WITH SPACERS	
9,10,12	170140	AXLE KIT WITH SPACERS 45MM	
9,10,11C	169839S	MP FORK 131,5 x 32 KIT	
9,10,11E	169841S	MP FORK 115x32 KIT	
9,10,11F	169842S	MP FORK 90x32 KIT	
9,10,11A,12	169837S	MP FORK 158 x 45 KIT	Used with 1.25 & 1.5 caster widths
9,10,11B,12	169838S	MP FORK 131,5 x 45 KIT	Used with 1.25 & 1.5 caster widths
9,10,11D,12	169840S	MP FORK 115x45 KIT	Used with 1.25 & 1.5 caster widths
9,10,11G,12	169843S	MP FORK 90x45 KIT	Used with 1.25 & 1.5 caster widths

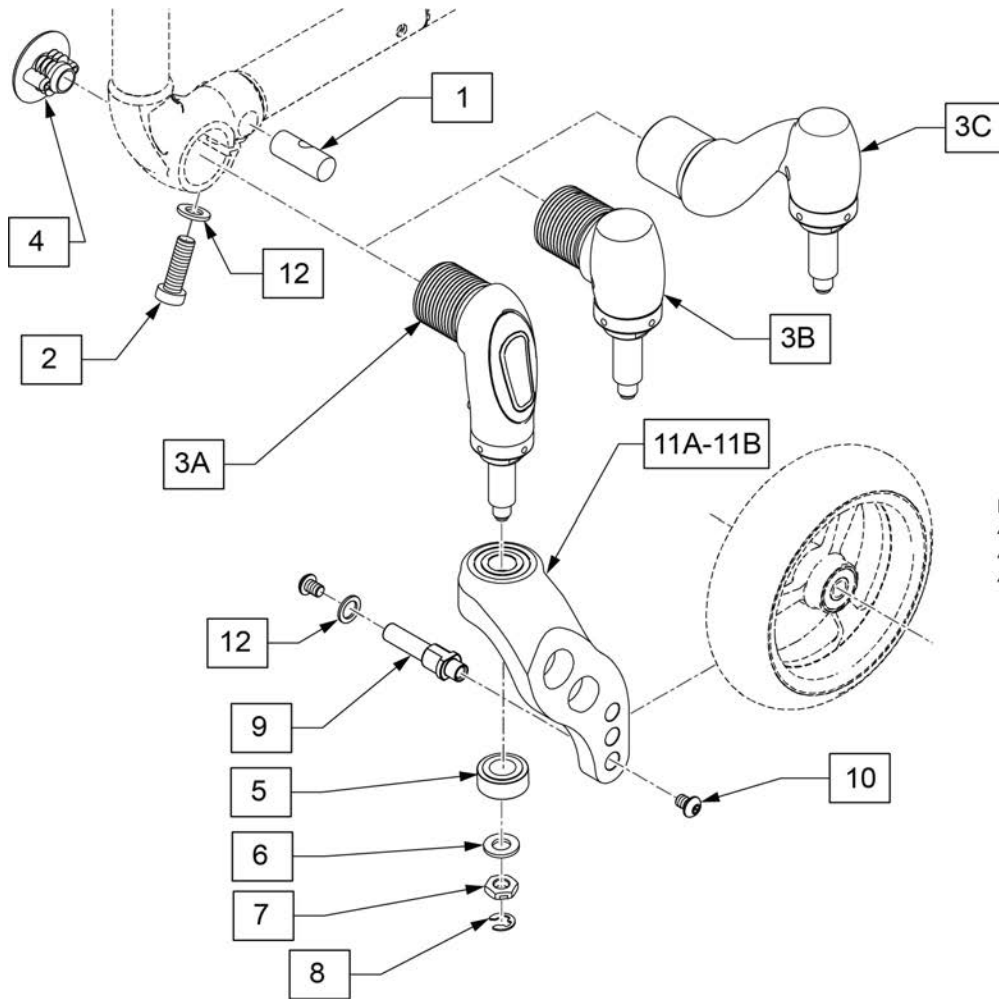

FORK MEASURING GUIDE

PART #	DISCRIPTION	"A"
112672	LOW, LH	4
112673	LOW, RH	4
112242	STD, LH	4 7/8
112243	STD, RH	4 7/8

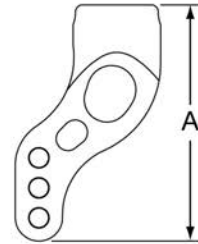


LOW Fits 3" Micro, 3" Micro Lighted, 4" Low Profile Poly, 4" Micro Lighted, 4x1 Alu Poly, 3x1.5 Alu Soft Roll, 4x1.25 Semi Pneu, 4x1.5 Alu Soft Roll

STD Fits 5" Low Profile Poly, 5" Micro Lighted, 5x1 Alu Poly, 5x1.5 Semi Pneu, 5x1.5 Alu Soft Roll, 6" Poly, 6" Pneu, 6x1 Alu Poly, 6x1.5 Semi Pneu, 6x1.5 Alu Soft Roll

Pos.	Item Number	Description	Remarks
1,2,12	169830	CASTER LINK FASTENERS	
3A,5-8	169832	DESIGN CASTER LINK KIT LH	Left
3A,5-8	169833	DESIGN CASTER LINK KIT RH	Right
3B,5-8	169834	HEMI CASTER LINK KIT	Right or Left
3C,5-8	169835	SHORT CASTER LINK KIT LH	Left
3C,5-8	169836	SHORT CASTER LINK KIT RH	Right
4	170139	LINK CAP	
5-8	169831	FORK STEM FASTENERS	
9,10,12	159437	AXLE KIT SINGLE SIDE FORK	
11A	112672	ASSM FORK LOW SNGL SD LH W/BRG	Black
11A	112673	ASSM FORK LOW SNGL SD RH W/BRG	Black
11B	112242	ASSM FORK STD SNGL SD LH W/BRG	Black
11B	112243	ASSM FORK STD SNGL SD RH W/BRG	Black
11A	117176S	FORK SNGL LOW CSTM ANO L	Choose custom anodized color
11A	117177S	FORK SNGL LOW CSTM ANO R	Choose custom anodized color
11B	117178S	FORK SNGL STD CSTM ANO L	Choose custom anodized color
11B	117179S	FORK SNGL STD CSTM ANO R	Choose custom anodized color


Note: Casters Available: 4" Low Profile Poly, 4x1.25 Semi Pneu, 4x1.5 Alu Soft Roll, 5" Low Profile Poly, 5x1.5 Semi Pneu, 5x1.5 Alu Soft Roll, 6" Poly, 6" Pneu, 6x1.5 Semi Pneu, 6x1.5 Alu Soft Roll.

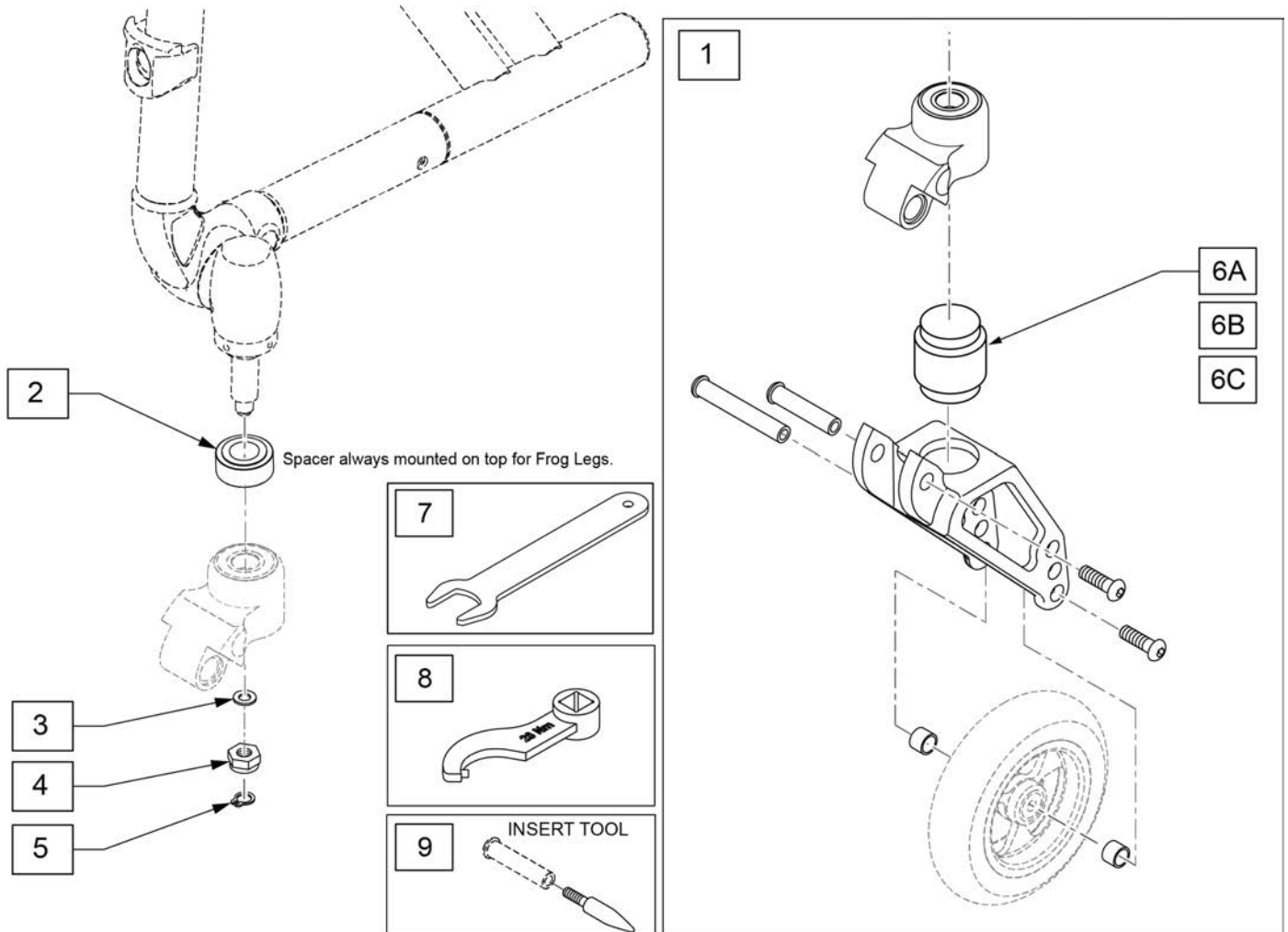

Pos.	Item Number	Description
1,6A	050225-001S	FORK FROGLEGS INT BRNG SOFT
1,6B	050225-002S	FORK FROGLEGS INT BRNG MED
1,6C	050225-003S	FORK FROGLEGS INT BRNG FIRM
2-5	169831	FORK STEM FASTENERS
7	66009205S	SPANNER WRENCH AF18
8	66009203S	TORQUE WRENCH WELDING
9	116351	FROG LEG FORK ASSEMBLY TOOL


Note: 7" and 8" Casters are not available with Integral Bearing Forks.

Note: When 6" casters or larger casters are used, Anti Flutter Kit must be installed. See Standard Multi Position Fork page.

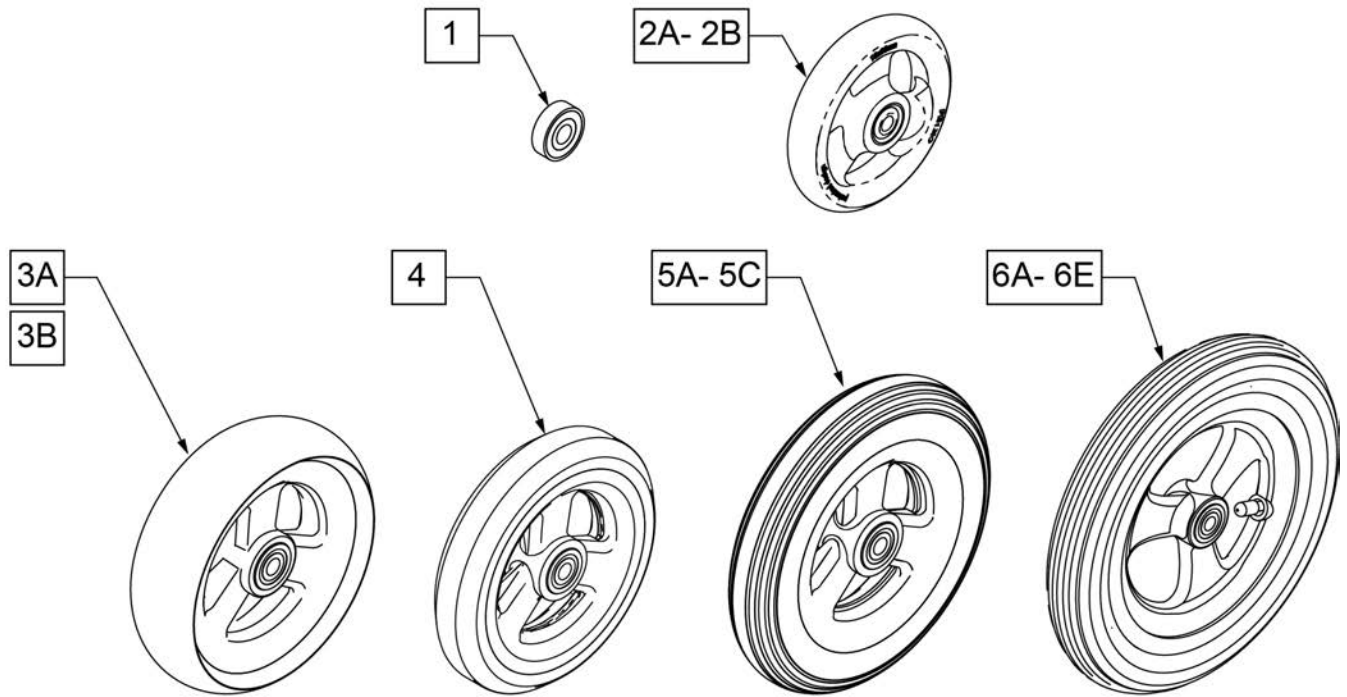

**CASTER
MEASURING GUIDE**


PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH	PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
101957	4.3	1.0	101256	5.75	1.0
101098	4.0	1.0	101257	5.9	1.0
101259	4.9	1.0	400212	8.0	1.0
101255	5.7	1.0	400213	7.8	1.0
101254	4.9	1.0	400214	6.9	1.0
			400215	7.25	1.0

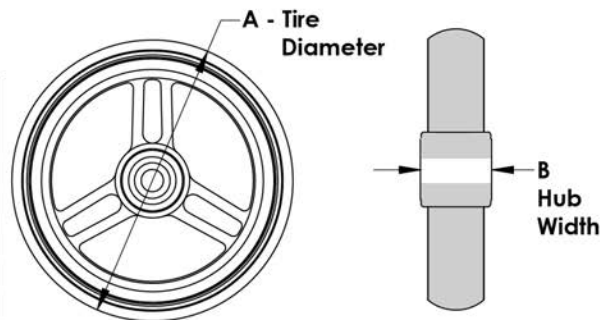


Pos.	Item Number	Description	Remarks
1	349502S	7/8"x 5/16" BEARING	
2A	101957S	4"X1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098S	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259S	5"X1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255S	6"X1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254S	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256S	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257S	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212S	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213S	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214S	CASTER 7" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255S	8" PNEUMATIC TIRE & TUBE	
6E	400215S	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY



SIDE VIEW

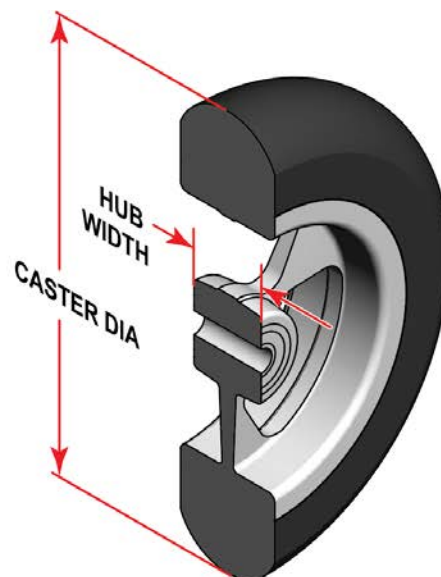

1A 1B 1C

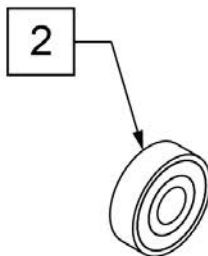


CASTER MEASURING GUIDE

CASTER DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

Pos.	Item Number	Description	HCPC	Remarks
1A	251732	CASTR ASSM ULTRLT SFTRL 4X1.5"	K0077	4" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1A	251733	CASTR ASSM ULTRLT SFTRL 5X1.5"	K0077	5" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1A	251734	CASTR ASSM ULTRLT SFTRL 6X1.5"	K0077	6" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1		ANODIZED		
1B	162609S	WHL CSTR 4" ULTRALT SFTRLL ANO	K0077	4" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
1B	162611S	WHL CSTR 5" ULTRALT SFTRLL ANO	K0077	5" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
1B	162613S	WHL CSTR 6" ULTRALT SFTRLL ANO	K0077	6" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
1		PAINTED		
1C	162603S	WHL CSTR 4" ULTRALT SFTRLL PNT	K0077	4" x 1.5" Aluminum Soft Roll - Black Tire - Painted Hub
1C	162605S	WHL CSTR 5" ULTRALT SFTRLL PNT	K0077	5" x 1.5" Aluminum Soft Roll - Black Tire - Painted Hub
1C	162607S	WHL CSTR 6" ULTRALT SFTRLL PNT	K0077	6" x 1.5" Aluminum Soft Roll - Black Tire - Painted Hub
2	349502	7/8" X 5/16" BEARING		

Standard Anti-Tip Selection Guide


RSH	Seat Squeeze		
	0"-0.5"	1"-1.5"	2"+
14.0	6B, 7A	6A, 7A	6A, 7A
14.5	6B, 7A	6B, 7A	6B, 7A
15.0	6B, 7B	6B, 7A	6B, 7A
15.5	6C, 7A	6B, 7B	6B, 7B
16.0	6C, 7A	6C, 7A	6C, 7A
16.5	6C, 7B	6C, 7A	6C, 7A
17.0	6D, 7B	6C, 7B	6C, 7B
17.5	6D, 7B	6D, 7B	6D, 7B
18.0	6C, 7C	6D, 7B	6D, 7B
18.5	6C, 7C	6C, 7C	6C, 7C
19.0	6C, 7C	6C, 7C	6C, 7C
19.5	6D, 7C	6C, 7C	6C, 7C
20.0	6D, 7C	6D, 7C	6D, 7C

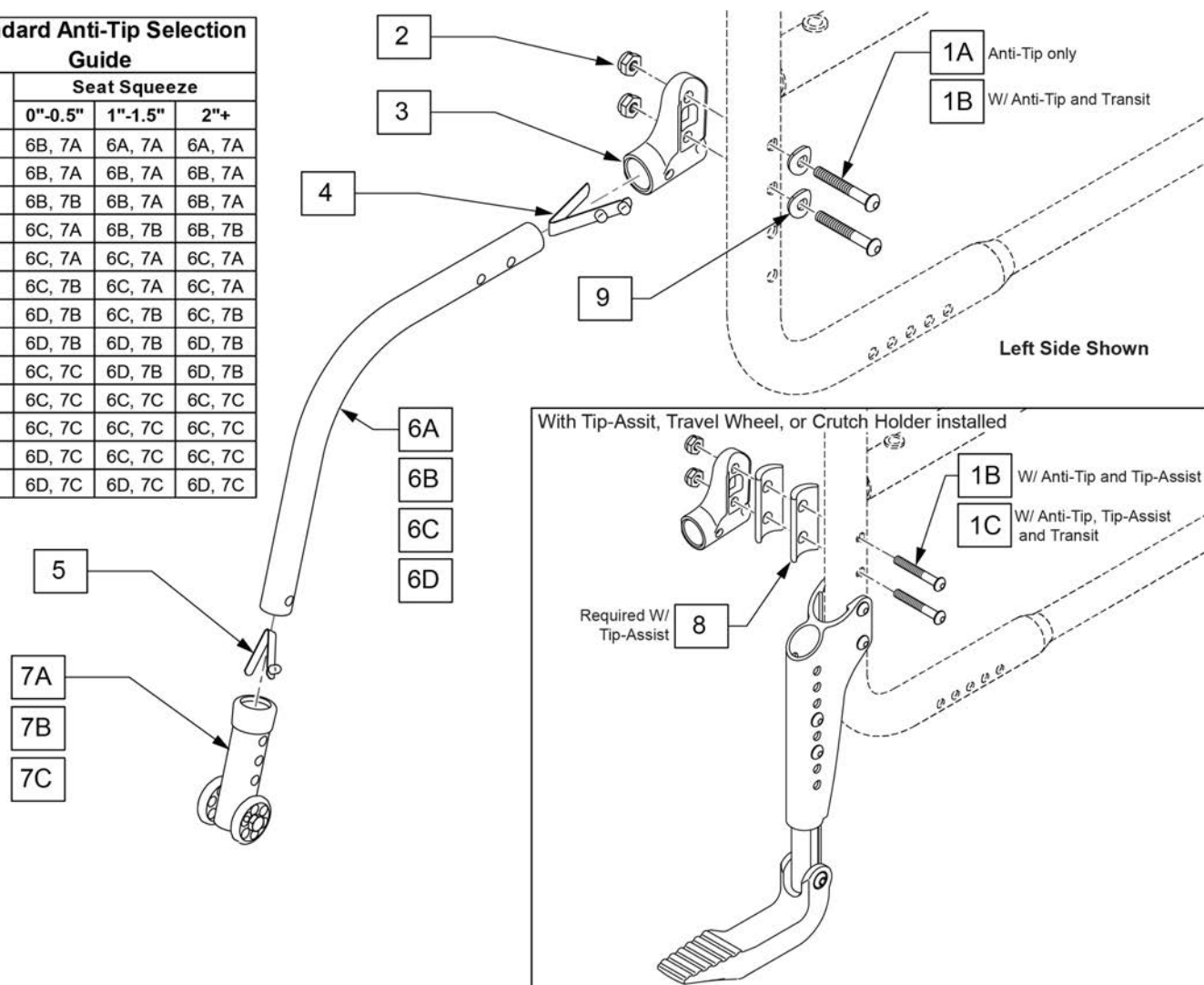

Pos.	Item Number	Description	Remarks
4,5,6A	882898S	ANTI-TIP TB KDZ R BL W/SNP BT	Right
4,5,6A	882899S	ANTI-TIP TB KDZ L BL W/SNP BT	Left
4,5,6B	104219-002S	ANTI TIP TUBE ASSY CRVD RH	Right
4,5,6B	104219-001S	ANTI TIP TUBE ASSY CRVD LH	Left
4,5,6C	882964S	ANTI-TIP TB SMALL R W/SNAP BTN	Right
4,5,6C	882965S	ANTI-TIP TB SMALL L W/SNAP BT	Left
4,5,6D	882896	ANTI-TIP TB REG R B W/SNAP BT	Right
4,5,6D	882897	ANTI-TIP TB REG L B W/SNAP BT	Left
7A	104157S	ANTI TIP ARM ADJUSTER	
7B	104639S	ANTI TIP ARM ADJ. MED	
7C	104640S	ANTI TIP ARM ADJ. LONG	
4,5,6A,7A	170206	ANTI TIP XS TUBE SHT SLV RH	Right
4,5,6A,7A	170207	ANTI TIP XS TUBE SHT SLV LH	Left
4,5,6B,7A	170208	ANTI TIP SHT TUBE SHT SLV RH	Right
4,5,6B,7A	170209	ANTI TIP SHT TUBE SHT SLV LH	Left
4,5,6C,7A	170210	ANTI TIP SHT TUBE, MED SLV RH	Right
4,5,6C,7A	170211	ANTI TIP SHT TUBE, MED SLV LH	Left
4,5,6B,7A	170212	ANTI TIP MED TUBE, SHT SLV RH	Right

Pos.	Item Number	Description	Remarks
4,5,6B,7A	170213	ANTI TIP MED TUBE, SHT SLV LH	Left
4,5,6B,7B	170214	ANTI TIP MED TUBE, MED SLV RH	Right
4,5,6B,7B	170215	ANTI TIP MED TUBE, MED SLEEVE LH	Left
4,5,6C,7B	170216	ANTI TIP LNG TUBE, MED SLV RH	Right
4,5,6C,7B	170217	ANTI TIP LNG TUBE, MED SLV LH	Left
4,5,6C,7C	170218	ANTI TIP MED TUBE, LNG SLV RH	Right
4,5,6C,7C	170219	ANTI TIP MED TUBE, LNG SLV LH	Left
4,5,6D,7C	170220	ANTI TIP LNG TUBE, LNG SLV RH	Right
4,5,6D,7C	170221	ANTI TIP LNG TUBE, LNG SLV LH	Left
1A,2,9	170222	ANTI TIP HARDWARE STD	1 side
1B,2	170223	ANTI TIP HARDWARE TRANSIT	1 side
1B,2,8,9	170224	ANTI TIP HARDWARE TIP ASST	1 side
1C,2,9	170225	ANTI TIP HARDWARE TRANST&TIP	1 side
1ABC,2,3,8,9	170226	ANTI TIP BRACKET KIT QS5X RH	Right, includes all fasteners
1ABC,2,3,8,9	170227	ANTI TIP BRACKET KIT QS5X LH	Left, includes all fasteners

Note: Not compatible with "Tip Assist" or "Crutch Holder".

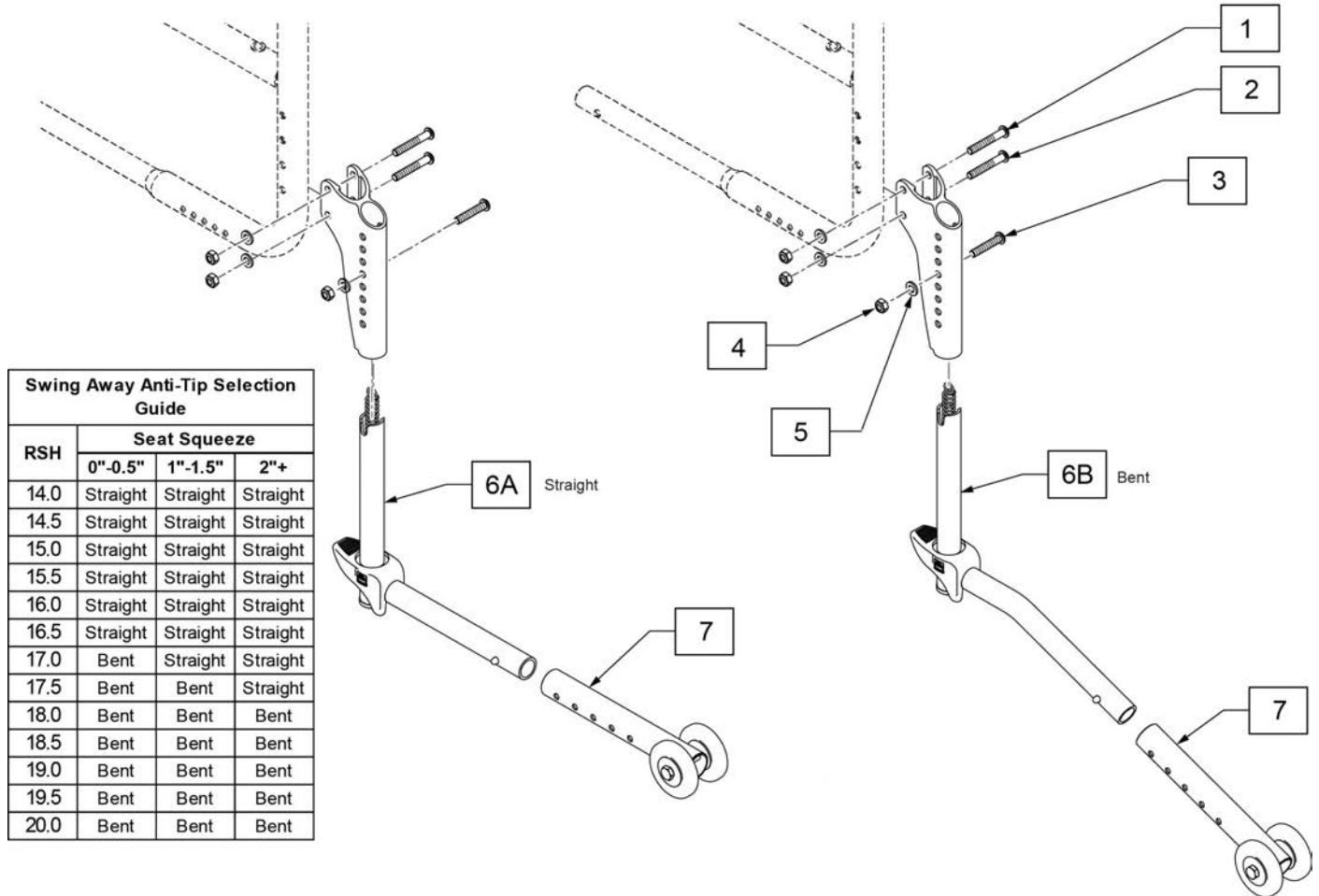

Pos.	Item Number	Description
1-5	170158	ACCESSORY BRKT KIT
2-5	170159	ACCESSORY BRKT FASTENERS
1-6A,7	170160	SA ANTI-TIP BENT RETRO KIT PR
1-6B,7	170161	SA ANTI-TIP STRT RETRO KIT PR
6A,7	170162	SA ANTI-TIP BENT REPLACENT KIT
6B,7	170163	SA ANTI-TIP STRT REPLACENT KIT

Note: 5 Spoke Mag is not compatible with One Arm Drive or Projection Handrims.

Note: 5 Spoke Mag utilizes exclusive screw mounted handrims.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1,2	128985S	20" 5 SPOKE MAG WHEEL	20" (451mm)
1,2	128986S	22" 5 SPOKE MAG WHEEL	22" (501mm)
1,2	126405S	24" MAG W/BRNG 5 SPOKE	24" (540mm)
2	349500S	1 1/8 X 1/2 BEARING	
3	129325S	20" MAG RIM STRIP	
3	129324S	22" MAG RIM STRIP	
3	127912S	24" MAG RIM STRIP	

Note: this Wheel utilizes Screw Mount Handrims; the Handrim has a threaded hole that mounts the Handrim to the Wheel through a spacer and fastener.

Note: the Lite Spoke Wheel has a total of 30 spokes.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: when ordering replacement bearings, if dust shield on bearing has the following markings: MBL-Z2 6001 2RS, then order P/N 114097.



Pos.	Item Number	Description	Remarks
1A	118755S	WHEEL ASSM GALLOP 20IN	20" (451)
1B	118756S	WHEEL ASSM GALLOP 22IN	22" (501)
1C	107131	WHEEL RIM/SPOKE 24" ASSM	24" (540)
1D	118764S	25 LT SPK W-HNDRM ANO ASSM GLP	25" (559)
1E	118760S	WHEEL ASSM GALLOP 26IN	26" (590)
2A	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide; used prior to 11/29/10
2B	349500S	1_125 X .5 BEARING	Beginning 11/29/10
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1	109242	SPINERGY STD WHL 22" BLK SPK	22"(489mm)
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500S	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1A	108858	SPINERGY LT EX WHL 22" BLK	22"(489mm); 12 Spoke
1A	109215	SPINERGY LT EX WHL 22" BLU	22"(489mm); 12 Spoke
1A	109216	SPINERGY LT EX WHL 22" RED	22"(489mm); 12 Spoke
1A	109217	SPINERGY LT EX WHL 22" WHT	22"(489mm); 12 Spoke
1A	109218	SPINERGY LT EX WHL 22" YLW	22"(489mm); 12 Spoke
1A	101989-001	SPINERGY LT EX WHEEL 540 BLK	24"(540mm); 12 Spoke
1A	101989-002	SPINERGY LT EX WHEEL 540 BLU	24"(540mm); 12 Spoke
1A	101989-003	SPINERGY LT EX WHEEL 540 RED	24"(540mm); 12 Spoke
1A	101989-005	SPINERGY LT EX WHEEL 540 WHT	24"(540mm); 12 Spoke
1A	101989-004	SPINERGY LT EX WHEEL 540 YLW	24"(540mm); 12 Spoke
1A	101991-001	SPINERGY LT EX WHEEL 559 BLK	25"(559mm); 12 Spoke
1A	101991-002	SPINERGY LT EX WHEEL 559 BLU	25"(559mm); 12 Spoke
1A	101991-003	SPINERGY LT EX WHEEL 559 RED	25"(559mm); 12 Spoke
1A	101991-004	SPINERGY LT EX WHEEL 559 YLW	25"(559mm); 12 Spoke
1A	101993-001	SPINERGY LT EX WHEEL 590 BLK	26"(590mm); 12 Spoke
1A	101993-004	SPINERGY LT EX WHEEL 590 YLW	26"(590mm); 12 Spoke
2	349500S	1 1/8 X 1/2 BEARING	
		SPINERGY LX WHEEL WITH FLEX HAND RIM	
1B	109232	SPINERGY FLEXRIM LT 12 SPOKE	24"(540mm) Black
1B	109237	RIM FLEX 25" SPINERGY BLK	25"(559mm) Black

Note: this Wheel utilizes Tab Mount Handrims (6 Tab).

Note: weight capacity is 220 lbs.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description
1A	066511-003S	PROTON WHEEL 24 X 1
1B	066511-004S	PROTON WHEEL 25 X 1




Pos.	Item Number	Description	Remarks
1	385237	16" X 1.75" PNEU TIRE	
1	385250S	20" PNEUMATIC TIRE	
1	385275S	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300S	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325S	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614S	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6



Pos.	Item Number	Description	Remarks
1	386232S	20" POLY TIRE STD	20"
1	386242S	22" POLY TIRE STD	22" (501mm)
1	386252S	24" POLY TIRE STD	24" (540mm)

Note: Shox Tire do not fit Topolino Wheels due to width.

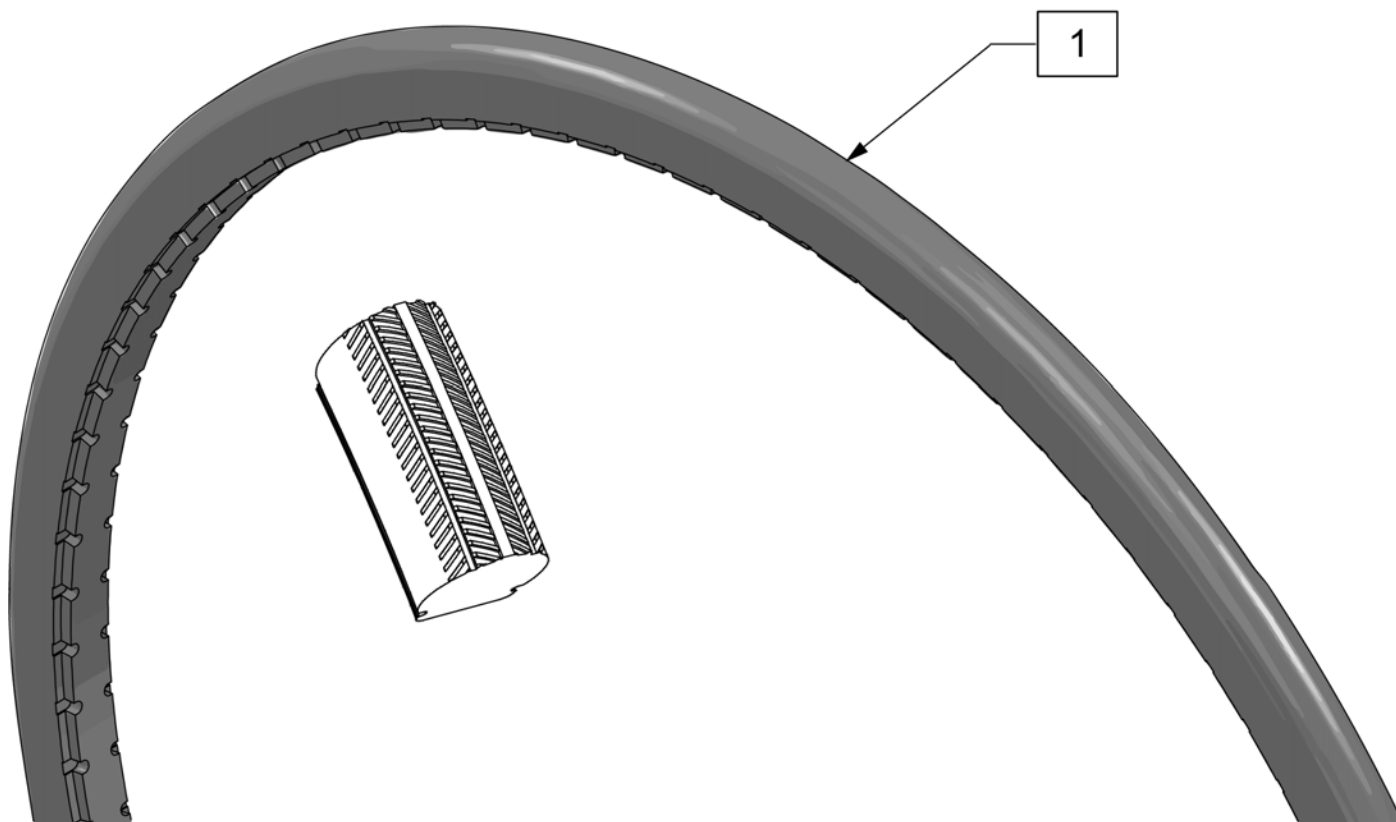

Pos.	Item Number	Description	Remarks
1	129924S	TIRE SHOX CEW20"	20"(457mm) Black
1	129925S	TIRE SHOX CEW22"	22"(501mm) Black
1	129926S	TIRE SHOX CEW24"	24"(540mm) Black
1	129927	TIRE SHOX CEW25"	25"(559mm) Black



Pos.	Item Number	Description	Remarks
1	254902S	TIRE, POLY SOLID, ORION, 24"	24"(540mm)





Pos.	Item Number	Description
1	129903S	22X1 HI PRESSURE CLINCHER BLK
1	129904S	24X1 HI PRESSURE CLINCHER BLK
1	129905S	26X1 HI PRESSURE CLINCHER BLK




Pos.	Item Number	Description	Remarks
1	102066-004S	TIRE SCHWALBE MP 489 BLK	22" (489mm)
1	102066-001S	TIRE SCHWBL MPE 540 BLK	24" (540mm)
1	102066-002S	TIRE SCHWBL MPE 559 BLK	25" (559mm)
1	102066-003S	TIRE SCHWBL MPE 590 BLK	26" (590mm)




Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>	LITE SPOKE	>>>>>
1,2A,3A	159508	20IN LT SPK AA HNDRLM W/HDWR	20" LITE SPOKE
1,2A,3A	159509	22IN LT SPK AA HNDRLM W/HDWR	22" (501) LITE SPOKE
1,2A,3A	159510S	24IN LT SPK AA HNDRLM W/HDWR	24" (540) LITE SPOKE
1,2A,3A	159511	25IN LT SPK AA HNDRLM W/HDWR	25" (559) LITE SPOKE
1,2A,3A	159512	26IN LT SPK AA HNDRLM W/HDWR	26" (590) LITE SPOKE
2A,3A	159513	LT SPK AL AN HNDRLM HDWR	LITE SPOKE, SET OF 6
>>>>>	>>>>>>>>	SPOKE	>>>>>
1,2B,3B	425931S	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941S	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000S	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B,3B	159514	SPK AL AN HNDRLM HDWR	SPOKE, SET OF 6
>>>>>	>>>>>>>>	MAG	>>>>>
1,3C	425934S	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951S	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009S	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175S	10-32 X 2 1/8 PAN PHIL SS	MAG
>>>>>	>>>>>>>>	M6 SPOKE	>>>>>
1,2B,3B	425941S	22" HR 6SMT W/HDWR	24" (540) SPOKE

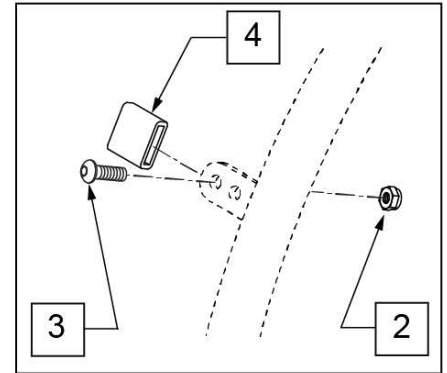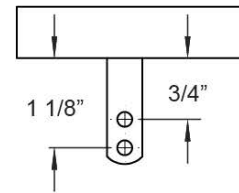**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1	529529S	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050S	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>	SPINERGY & PROTON	>>>>>
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020S	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030S	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2-4	159515	AA TAB MOUNT HANDRIM HARDWARE	SET OF 6

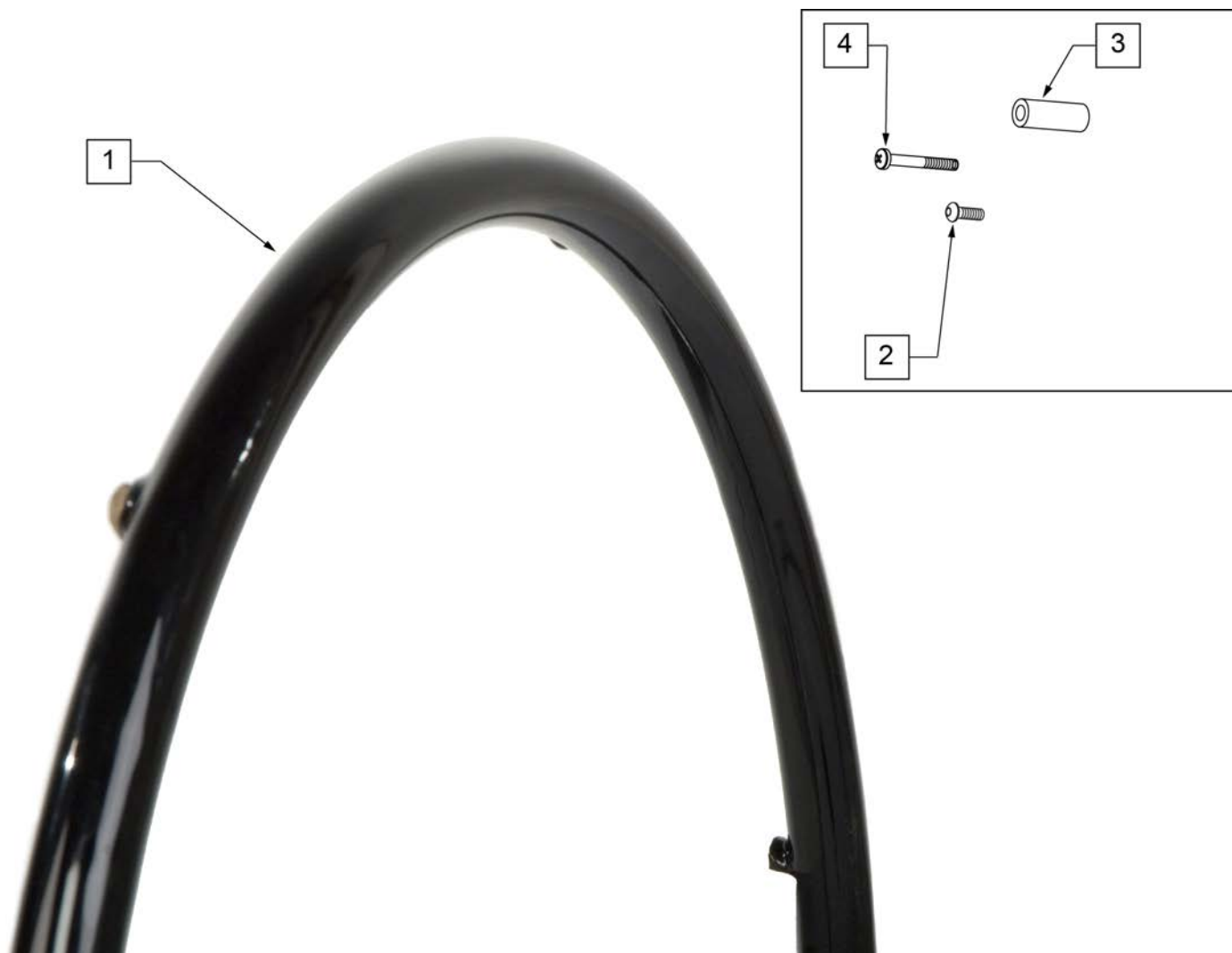

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	SPOKE & LITE SPOKE	>>>>>
1-3	425933	20" HR 6SMT PLST W/HDWR	20" SPOKE
1-3	425943	22" HR 6SMT PLST W/HDWR	22" (501) SPOKE
1-3	426004	24" HR 6SMT PLST W/HDWR	24" (540) SPOKE
>>>>>	>>>>>>>>>>	MAG	>>>>>
1,4	425935	20" HR 6SMT 6MAG PLST W/HDWR	20" MAG
1,4	425953	22" HR 6SMT 6MAG PLST W/HDWR	22" (501) MAG
1,4	426013	24" HR 6SMT 6MAG PLST W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>>	M6 SPOKE	>>>>>
1-3	425943	22" HR 6SMT 6 MAG PLST W/ HDWR	24" (540) M6 SPOKE
2,3	159534	PLST COATED HNDRM HARDWARE	SPOKE; SET OF 6
4	281175S	10 -32 x 2 1/8 PAN PHIL SS	6 SPOKE MAG

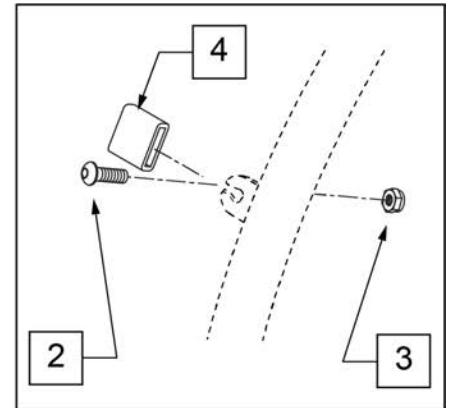


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	MOUNTAIN	>>>>>
1,2	425022	22" HR 4T PLST W/ HDWR	24" (540)
>>>>>	>>>>>>>>>>	SPINERGY & PROTON	>>>>>
1,2	425039	24"HR 6T BLK PLST W/HDWR	24" (540)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>>	QP (QUICKIE PERFORMANCE)	>>>>>
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
1,2	426060	QP 26" HR W/ HDWR (6T) PLASTIC	26" (700)
2	159535	PLST COATED TAB HNDRM HDWR	

NOTE: fits 24" 5 Spoke Wheel only.



Pos.	Item Number	Description	Remarks
1B,2A	170424	24 NF HR MAG W/ SUP THG ASM	24" (540)- Super Grip Thumb Guard
1A,2B	170422	24 NF HR MAG W/ STD THG ASM	24" (540)- Standard Grip Thumb Guard
2A	102168-004S	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2B	102168-003S	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Standard Grip Thumb Guard
3	101932S	SPCR .194ID X .375 OD X .750	
4	281175S	10-32 X 2 1/8 PAN PHIL SS	


Note: Screw Mount Handrims are utilized with Lite Spoke, Spoke and 6 Spoke Mag Wheels.

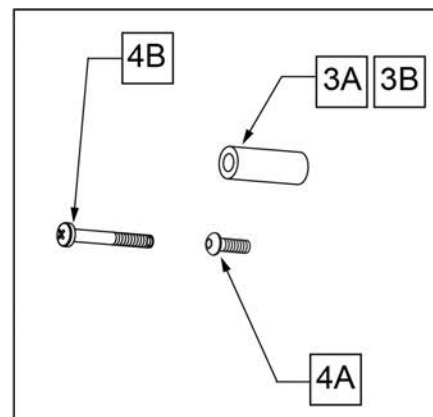

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD	>>>>>
2A	102168-001S	GUARD THUMB NAT FIT 22 PC BLK	22" (501)- Std Grip Thumb Guard
2A	102168-003S	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Std Grip Thumb Guard
2A	102168-005S	GUARD THUMB NAT FIT 25 PC BLK	25" (559)- Std Grip Thumb Guard
2A	102168-007S	GUARD THUMB NAT FIT 26 PC BLK	26" (590)- Std Grip Thumb Guard
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
2B	102168-002S	GUARD THUMB NAT FIT 22 SG BLK	22" (501)- Super Grip Thumb Guard
2B	102168-004S	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2B	102168-006S	GUARD THUMB NAT FIT 25 SG BLK	25" (559)- Super Grip Thumb Guard
2B	102168-008S	THUMB GUARD NAT FIT 26 SG BLK	26" (590)- Super Grip Thumb Guard
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD Wheel Assemblies	>>>>>
1,2A,3A,4A	170412	22 NF HR SCREW W/ STD THG ASM	22" (501)
1,2A,3A,4A	170413	24 NF HR SCREW W/ STD THG ASM	24" (540)
1,2A,3A,4A	170414	25 NF HR SCREW W/ STD THG ASM	25" (559)
1,2A,3A,4A	170415	26 NF HR SCREW W/ STD THG ASM	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD Wheel Assemblies	>>>>>
1,2B,3A,4A	170416	22 NF HR SCREW W/ SUP THG ASM	22" (501)
1,2B,3A,4A	170417	24 NF HR SCREW W/ SUP THG ASM	24" (540)
1,2B,3A,4A	170418	25 NF HR SCREW W/ SUP THG ASM	25" (559)

Pos.	Item Number	Description	Remarks
1,2B,3A,4A	170419	26 NF HR SCREW W/ SUP THG ASM	26" (590)
3A,4A	159524	NAT-FIT HNDRLM LT HDWR SET 6	USED W/ LITE SPOKE & SPOKE; SET OF 6
3B	101932S	SPCR .194ID X .375 OD X .750	USED W/ MAG
4B	281175S	10-32 X 2 1/8 PAN PHIL SS	USED W/ MAG


Note: Tab Mount Handrims are utilized with New Ultra Light, Spinnergy, Proton, and Mountain Wheels.

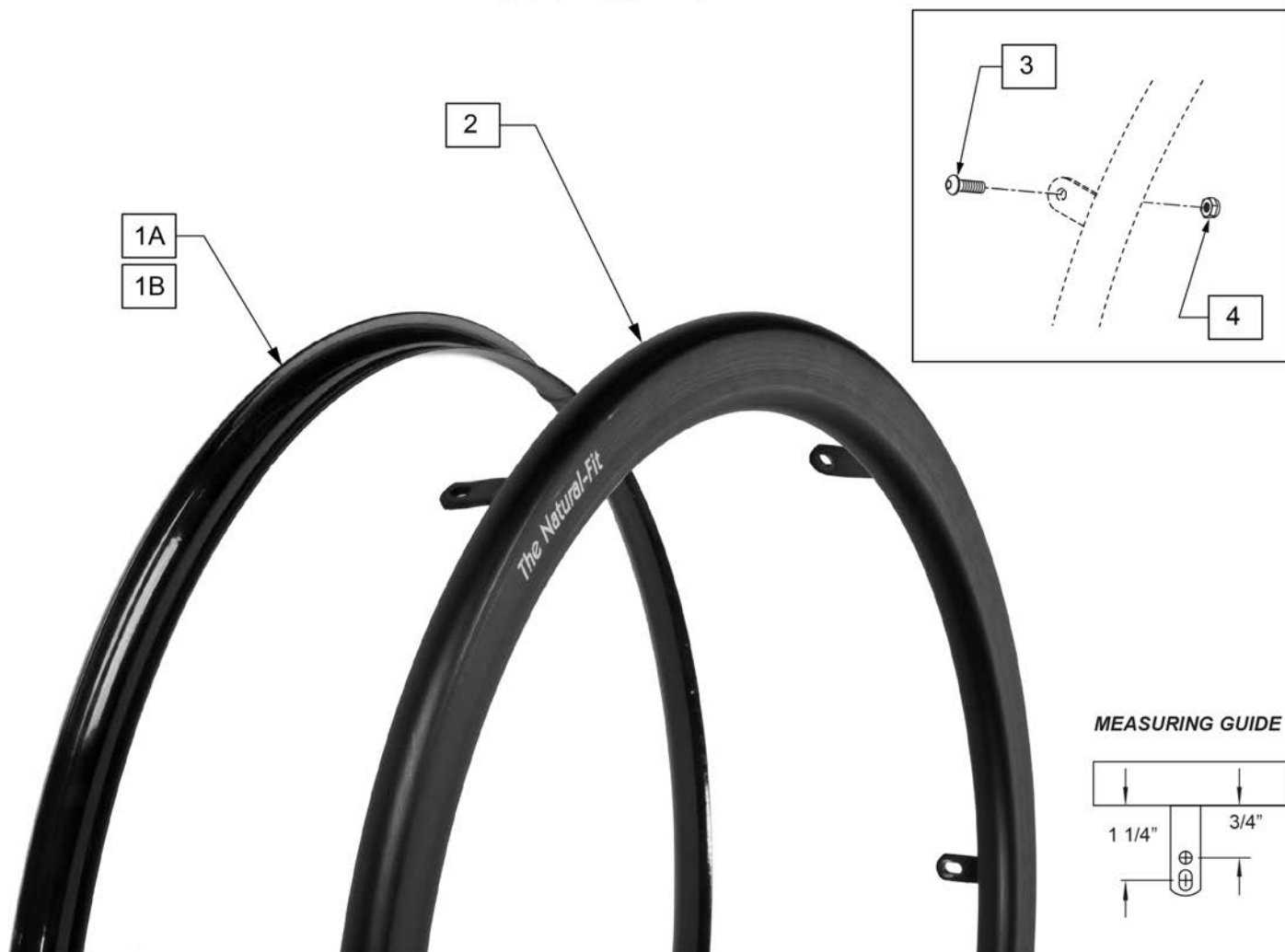

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GRIP	>>>>>
1A	102168-001S	GUARD THUMB NAT FIT 22 PC BLK	22" (501)
1A	102168-003S	GUARD THUMB NAT FIT 24 PC BLK	24" (540)
1A	102168-005S	GUARD THUMB NAT FIT 25 PC BLK	25" (559)
1A	102168-007S	GUARD THUMB NAT FIT 26 PC BLK	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
1B	102168-002S	GUARD THUMB NAT FIT 22 SG BLK	22" (501)
1B	102168-004S	GUARD THUMB NAT FIT 24 SG BLK	24" (540)
1B	102168-006S	GUARD THUMB NAT FIT 25 SG BLK	25" (559)
1B	102168-008S	THUMB GUARD NAT FIT 26 SG BLK	26" (590)
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD Wheel Assemblies	>>>>>
1A-4	170404	22 NF HR TAB W/ STD THG ASM	22" (501)
1A-4	170405	24 NF HR TAB W/ STD THG ASM	24" (540)
1A-4	170406	25 NF HR TAB W/ STD THG ASM	25" (559)
1A-4	170407	26 NF HR TAB W/ STD THG ASM	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD Wheel Assemblies	>>>>>
1B-4	170408	22 NF HR TAB W/ SUP THG ASM	22" (501)
1B-4	170409	24 NF HR TAB W/ SUP THG ASM	24" (540)
1B-4	170410	25 NF HR TAB W/ SUP THG ASM	25" (559)


Pos.	Item Number	Description	Remarks
1B-4	170411	26 NF HR TAB W/ SUP THG ASM	26" (590)
3,4	159533	NAT-FIT TAB MNT HARDWARE	SET OF 6

Note: Handrim is Latex free.
Note: used on Lite Spoke Wheels.


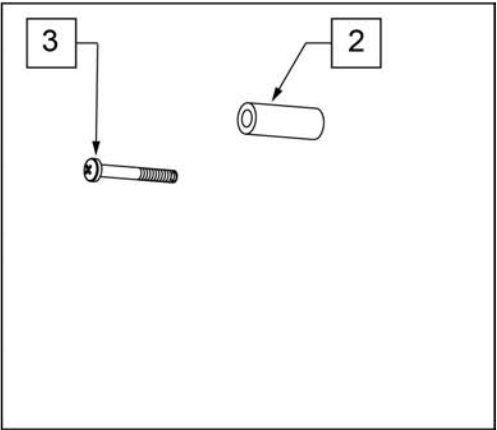


Pos.	Item Number	Description	Remarks
1-3	159504	24 HNDRIM RVNUT Q-GRIP W/HDWR	
1-3	159505	25 HNDRIM RVNUT Q-GRIP W/HDWR	
2,3	159506	Q-GRIP HARDWARE	set of 6

Note: Handrim is Latex free.
Note: used on Spinergy SPOX and LX wheels.



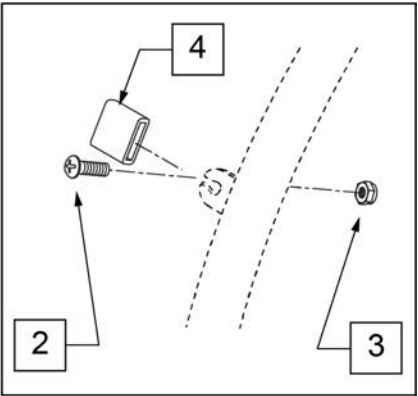


Pos.	Item Number	Description	Remarks
1-4	159501	24IN HANDRIM TAB Q-GRIP W/HDWR	
1-4	159502	25IN HANDRIM TAB Q-GRIP W/HDWR	
2-4	159503	HANDRIM TAB Q-GRIP HARDWARE	set of 6

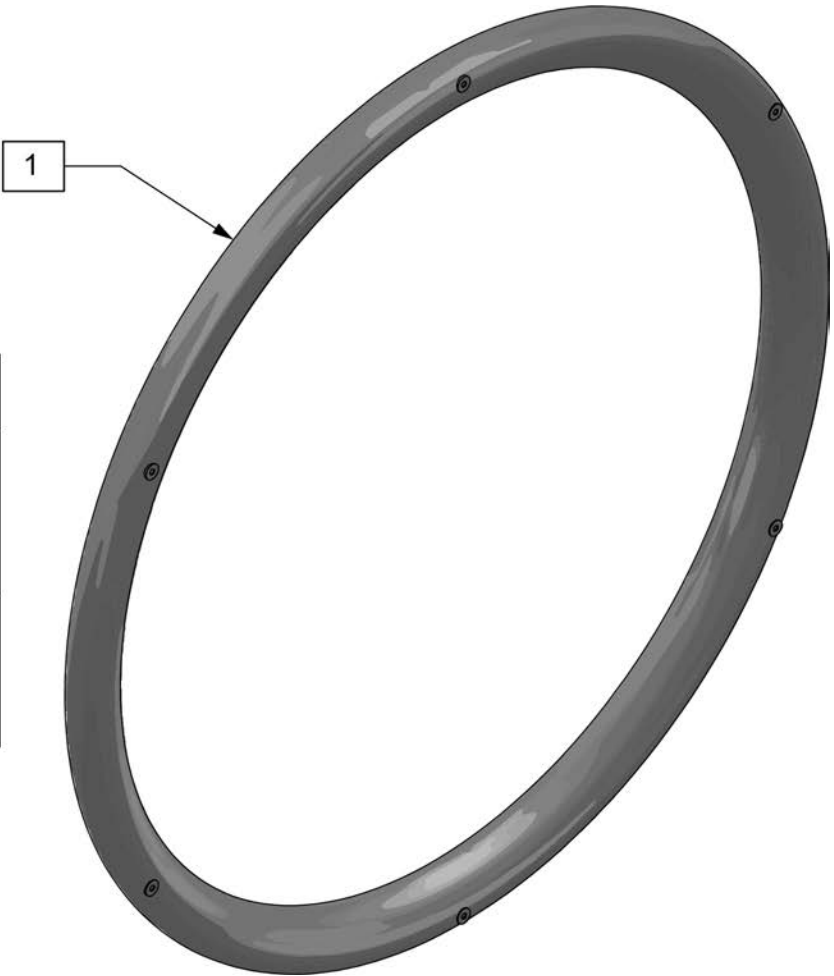
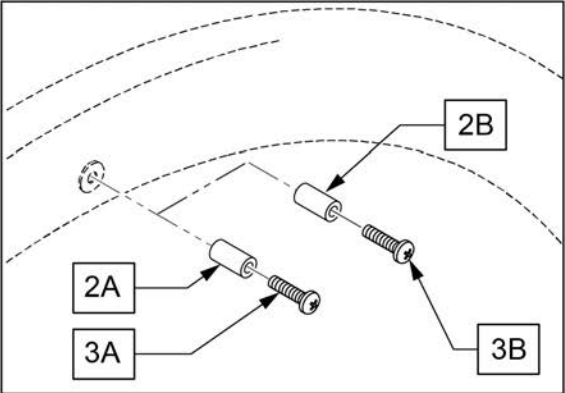


Pos.	Item Number	Description	Remarks
1	256748S	HR CURVE L TETRA SCREW 24" 515	
1	256749S	HR CURVE L TETRA SCREW 25" 539	
1, 2A, 3A	170118	HR CRV L TETRA SCREW 24 W/HDWR	
1, 2A, 3A	170120	HR CRV L TETRA SCREW 25 W/HDWR	
2A, 3A	159524	Screw Mount Hardware Only	Used /W Lite Spote & Sopke; SET OF 6

MEASURING GUIDE

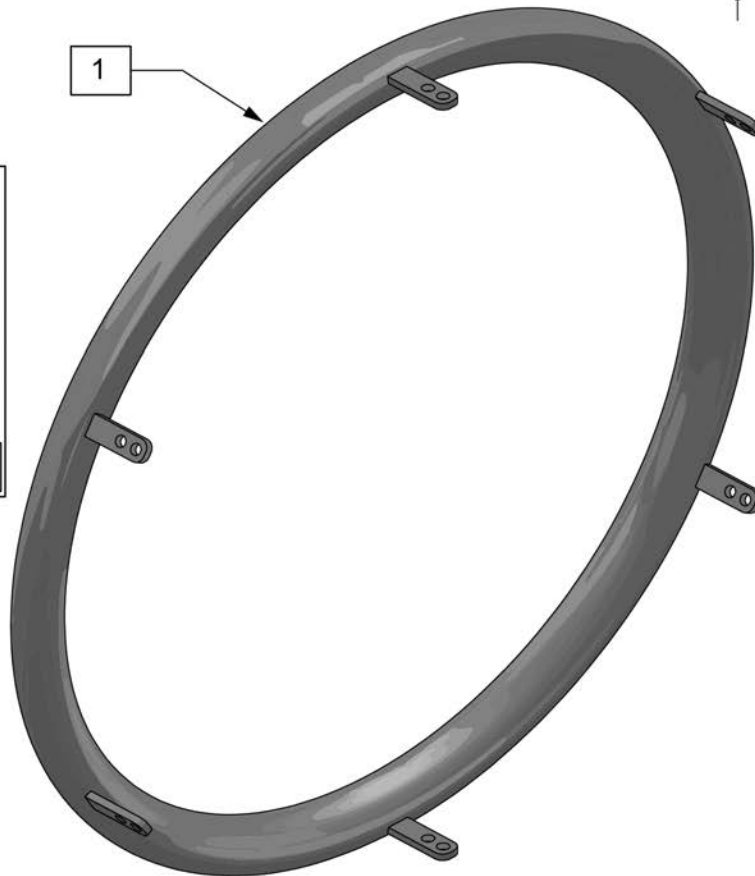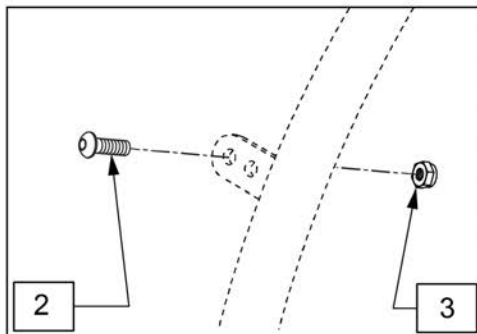
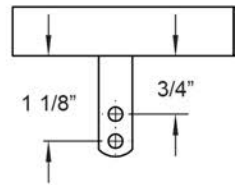


Pos.	Item Number	Description	Remarks
1	256743S	HR CURVE L TETRA TAB 24" 512	
1	256745S	HR CURVE L TETRA TAB 25" 535	
1-3	170116	HR CURVE L TETRA TAB 24 W/HDWR	
1-3	170117	HR CURVE L TETRA TAB 25 W/HDWR	
2,3	159533	TAB MOUNT HARDWARE	SET OF 6

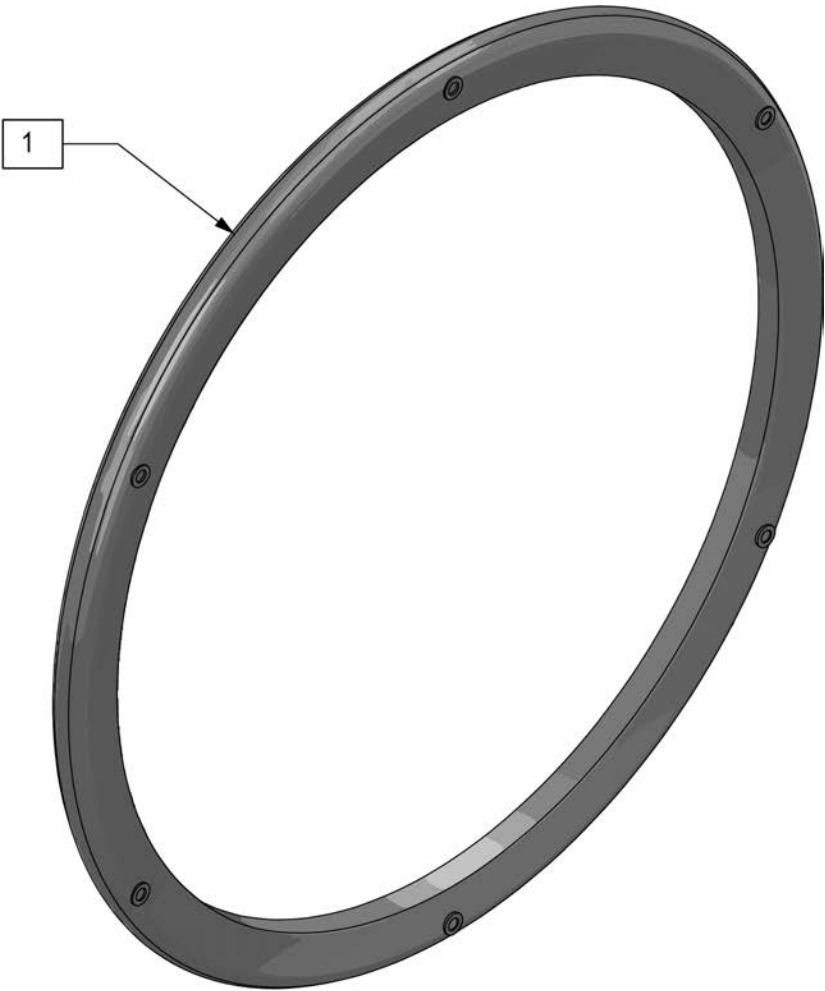
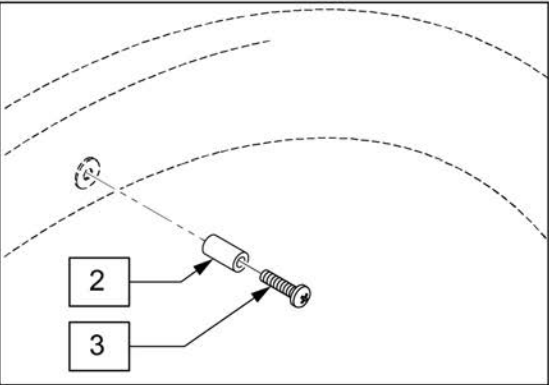


Pos.	Item Number	Description	Remarks
1	256584S	24" HANDRIM 6 RIVNUT ELIPSE	
1	256585S	25" HANDRIM 6 RIVNUT ELIPSE	
1-3	170124	HR ELLIPSE 3R SCREW 24 W/HDWR	
1-3	170125	HR ELLIPSE 3R SCREW 25 W/HDWR	
2,3	159524	Screw Mount Hardware Only	SET OF 6




MEASURING GUIDE

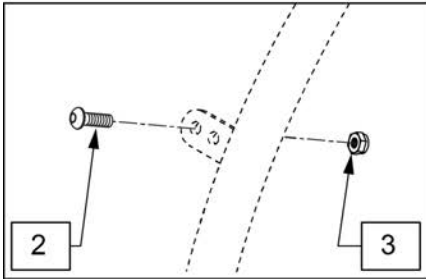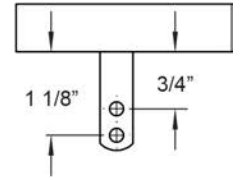



Pos.	Item Number	Description	Remarks
1	256580S	24" HANDRIM 6 TAB ELIPSE	
1	256581S	25" HANDRIM 6 TAB ELIPSE	
1-3	170122	HR ELIPSE 24 TAB W/HDWR	
1-3	170123	HR ELIPSE 25 TAB W/HDWR	
2,3	159533	TAB MOUNT HARDWARE	SET OF 6

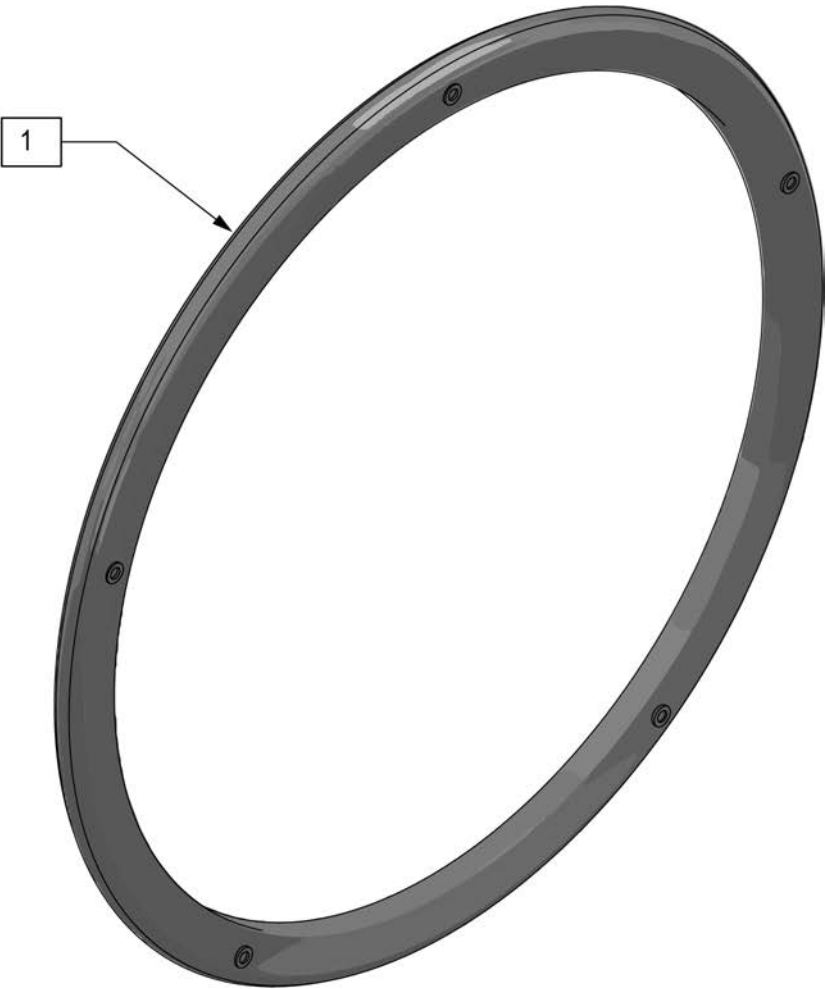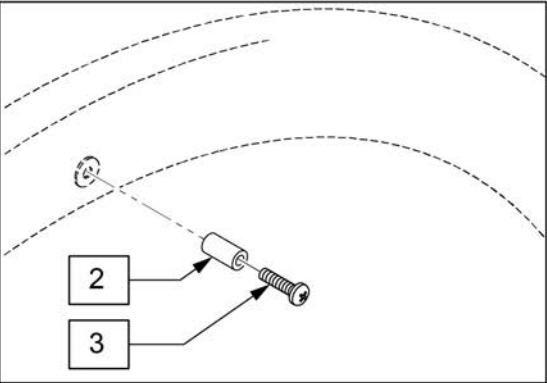




Pos.	Item Number	Description	Remarks
1	256980S	24" HANDRIM 5 RIVNUT ELLIPSE	
1-3	170126	ELLIPSE 3R SCREW 24 W/HDWR 5MG	
2,3	159507	5 SPK HNDRM HDWR	SET OF 5

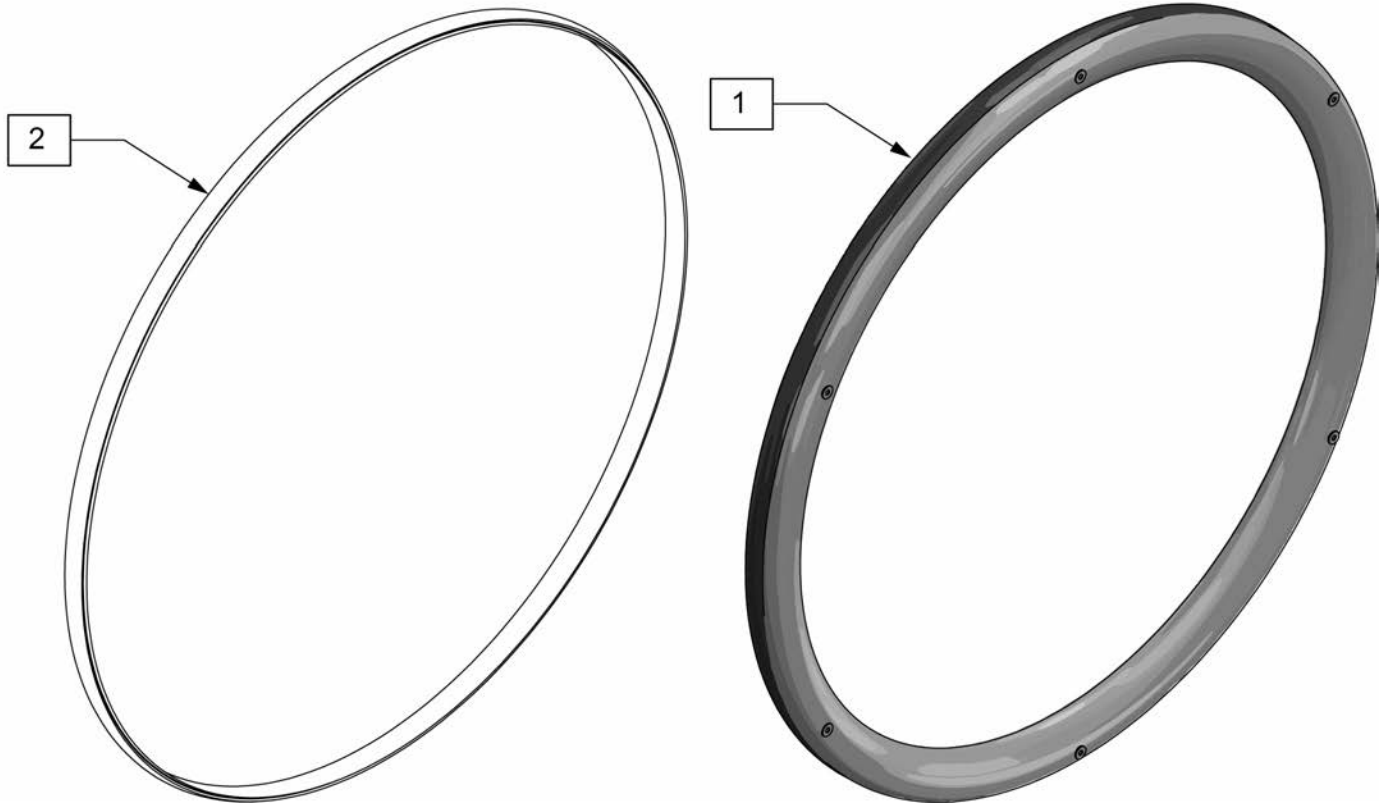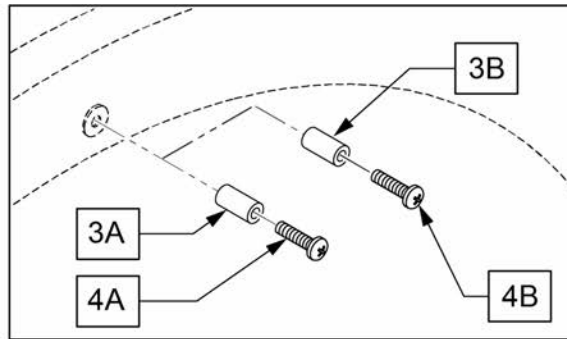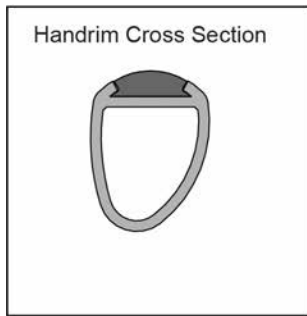

Pos.	Item Number	Description	Remarks
1	256731S	HR GEKKO SCREW 24" 515	Silicone Strap Included
1	256732S	HR GEKKO SCREW 25" 539	Silicone Strap Included
1, 3A, 4A	170111	HR GEKKO SCREW 24 W HDWR	Silicone Strap Included
1, 3A, 4A	170113	HR GEKKO SCREW 25 W HDWR	Silicone Strap Included
2	256726S	HR GEKKO STRIP 24"	
2	256727S	HR GEKKO STRIP 25"	
3A, 4A	159524	Screw Mount Hardware Only	Used /W Lite Spote & Sopke; SET OF 6

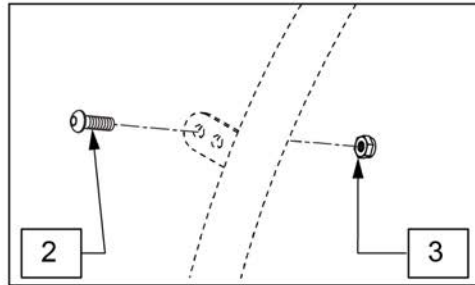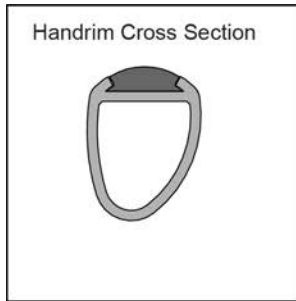


MEASURING GUIDE

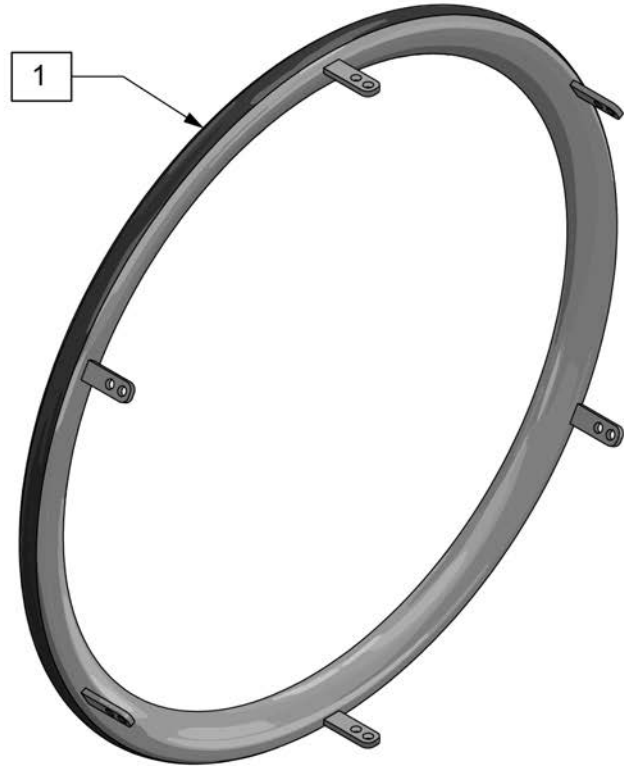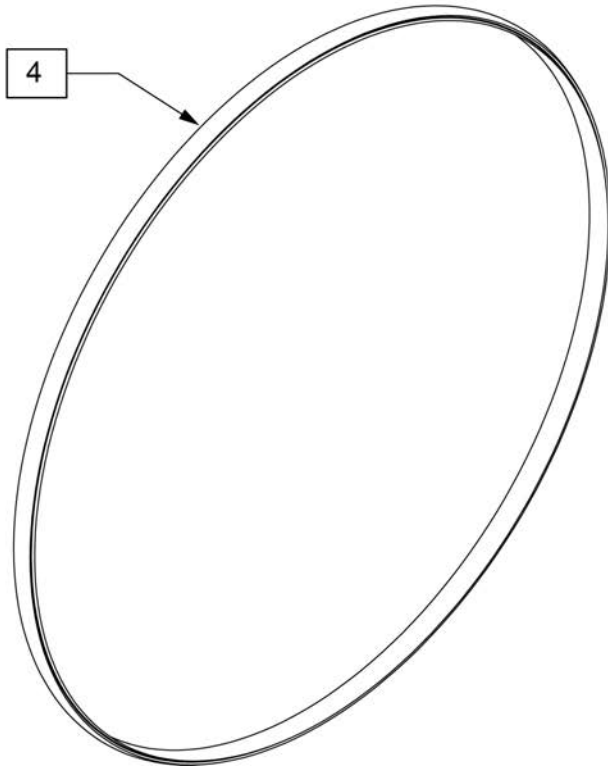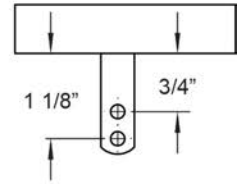

Pos.	Item Number	Description	Remarks
1	256721S	HR GEKKO TAB 24" 512	Silicone Strap Included
1	256723S	HR GEKKO TAB 25" 535	Silicone Strap Included
1-3	170109	HR GEKKO TAB 24" W/ HDWR	Silicone Strap Included
1-3	170110	HR GEKKO TAB 25" W/ HDWR	Silicone Strap Included
4	256726S	HR GEKKO STRIP 24"	
4	256727S	HR GEKKO STRIP 25"	
2,3	159533	TAB MOUNT HARDWARE	SET OF 6




Pos.	Item Number	Description	Remarks
1A	Discontinued	Presta Valves Discontinued	
1A	384736	INNER TUBE 12-1/2" X 2-1/4"	
1A	384738	16" X 1.75" INNER TUBE	
1A	382255S	TUBE 18" TIRE	18" X 1-3/4" TUBE
1A	382275S	20" TUBE	20" 1-3/8" TUBE
1A	382300S	22" TUBE	22" X 1-3/8" TUBE
1A	382336S	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1A	382325S	24 X 1.3/8IN TUBE REG	
1A	382334S	TUBE INLINE 24 X 1.75	
1A	384551S	26" X 1" TUBE SCHRADER	
1A	382350S	26" TUBE	26" X 1-3/8" TUBE
1A	382361S	TUBE 26 X 1.5/1.95/ S/V	
1A	382365	26 X 1.25/1.5 TUBE SHRADER	
1A	121122	TUBE 25" TIRE	25" X 1"
1A	382375S	24" x 2.125 TUBE	
1B	102972-001S	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1B	102973-001S	16IN AIRLESS INSERT	16" X 1-3/4"
1B	102973-002S	18IN AIRLESS INSERT	18" X 1-3/4"
1B	102973-003S	20IN AIRLESS INSERT	20" X 1-3/8"
1B	102972-002S	AIRLESS INSERT 20 X 2-1/8 P3	
1B	102973-004S	22IN AIRLESS INSERT	22" X 1-3/8"
1B	102973-005S	24IN AIRLESS INSERT	24" X 1-3/8"
1B	102972-003S	AIRLESS INSERT 24 X 2-1/8	
1B	108259S	26" AIRLESS INSERT	26" X 1-3/8"
1C	254910S	INSERT, ETPU AIRLESS, 24"	

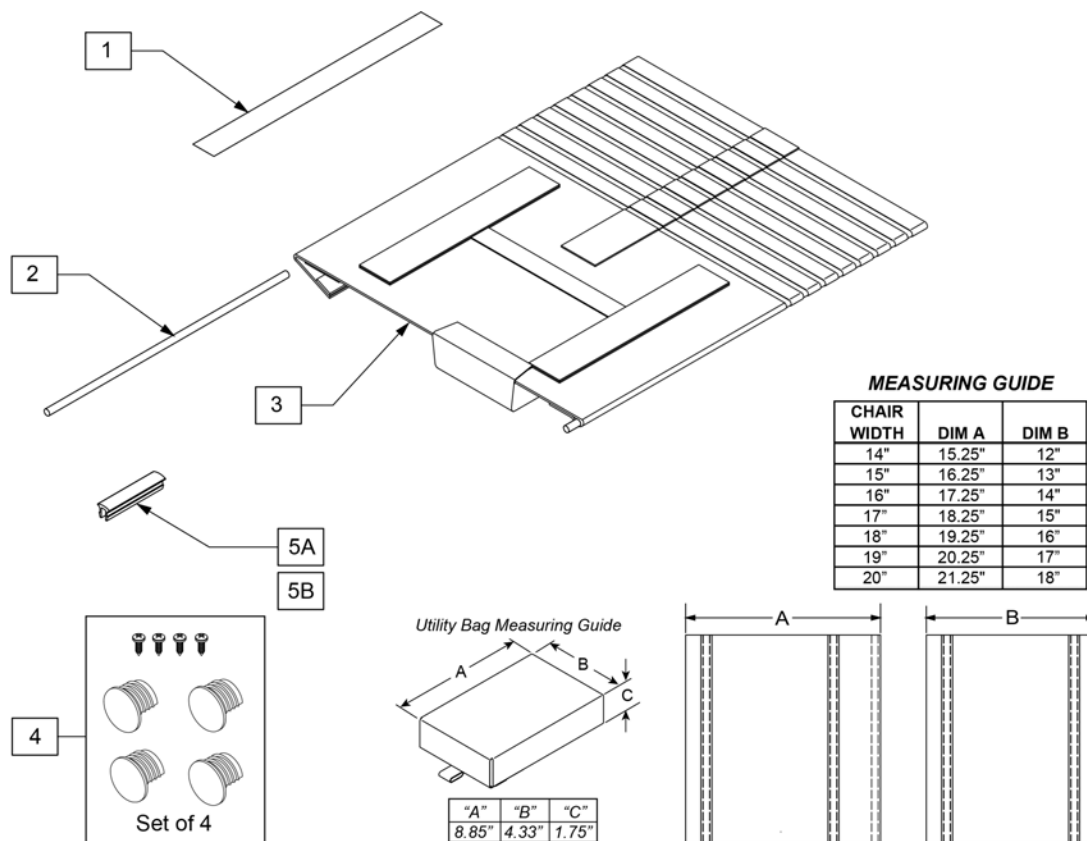


Pos.	Item Number	Description	Remarks
1	901169	SEAT SLING VELCRO STRIP ® 2"	
2,5A	170141	SEAT SLING ROD KIT 13	13"
2,5B	170142	SEAT SLING ROD KIT 14	14"
2	106587S	SEAT SLING ROD 15	15"
2	106588S	SEAT SLING ROD 16	16"
2	106589S	SEAT SLING ROD 17	17"
2	106590S	SEAT SLING ROD 18	18"
2	106591S	SEAT SLING ROD 19	19"
2	106592S	SEAT SLING ROD 20	20"
3	256178S	SEAT SLING 13W MULTI DEPTH	sling with utility bag
3	256179S	SEAT SLING 14W MULTI DEPTH	sling with utility bag
3	256180S	SEAT SLING 15W MULTI DEPTH	sling with utility bag
3	256181S	SEAT SLING 16W MULTI DEPTH	sling with utility bag
3	256182S	SEAT SLING 17W MULTI DEPTH	sling with utility bag
3	256183S	SEAT SLING 18W MULTI DEPTH	sling with utility bag
3	256186S	SEAT SLING 19W MULTI DEPTH	sling with utility bag
3	256187S	SEAT SLING 20W MULTI DEPTH	sling with utility bag
4	108117	SEAT RAIL END PLUG KIT	


Standard Push Handle

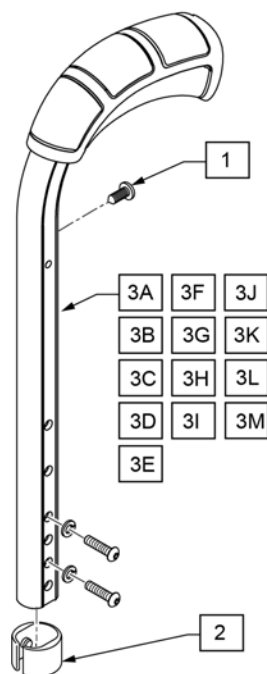


Folding Push Handle

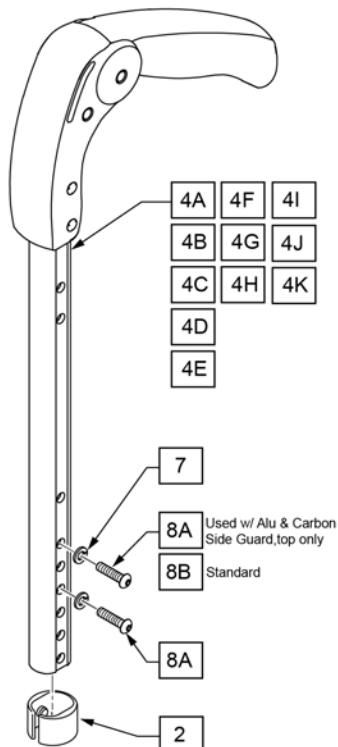


Omit Push Handle

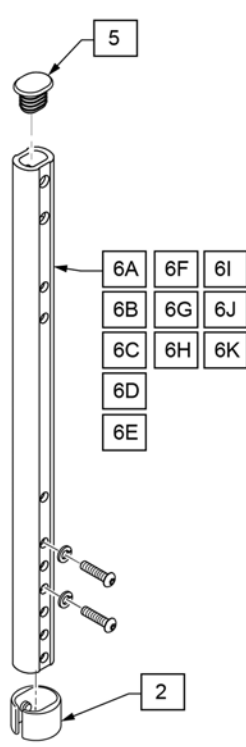


Measuring Guide

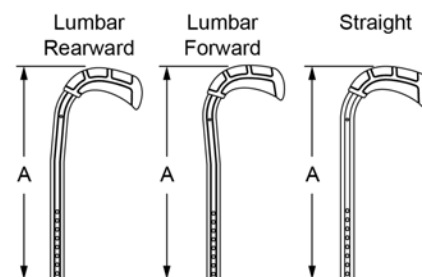

Back Height	"A"
10-12"	10"
12-14"	12"
14-16"	14"
16-18"	16"
18-20"	18"

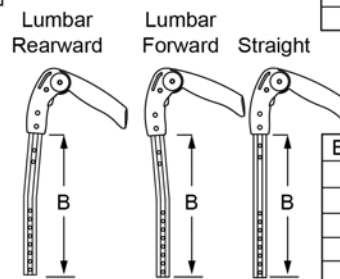


Back Height	"B"
10-12"	6 1/4"
12-14"	8 1/4"
14-16"	10 1/4"
16-18"	12 1/4"
18-20"	14 1/4"

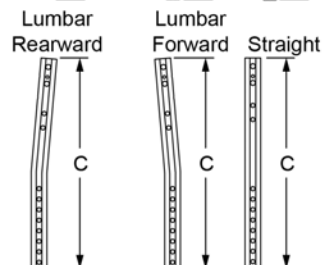


Back Height	"C"
10-12"	8"
12-14"	10"
14-16"	12"
16-18"	14"
18-20"	16"


Pos.	Item Number	Description	Pos.	Item Number	Description
1	107384S	PAN HD PHIL 10/32 x 5/8 ROHS	2,4G,7,8A	170237	LUMBAR FWD FOLD PH KIT 16-18
1,2,3A,7,8A	169854	QS5X STD PUSH HANDLE 10-12	2,4H,7,8A	170238	LUMBAR FWD FOLD PH KIT 18-20
1,2,3B,7,8A	169855	QS5X STD PUSH HANDLE 12-14	2,4I,7,8A	170239	LUMBAR RWRD FOLD PH KIT 14-16
1,2,3C,7,8A	169856	QS5X STD PUSH HANDLE 14-16	2,4J,7,8A	170240	LUMBAR RWRD FOLD PH KIT 16-18
1,2,3D,7,8A	169857	QS5X STD PUSH HANDLE 16-18	2,4K,7,8A	170241	LUMBAR RWRD FOLD PH KIT 18-20
1,2,3E,7,8A	169858	QS5X STD PUSH HANDLE 18-20	2,5-7,6A,7,8A	169864	QS5X NON-PUSH HNDL 10-12
1,2,3F,7,8A	170228	LUMBAR FWD STD PH KIT 12-14	2,5-7,6B,7,8A	169865	QS5X NON-PUSH HNDL 12-14
1,2,3G,7,8A	170229	LUMBAR FWD STD PH KIT 14-16	2,5-7,6C,7,8A	169866	QS5X NON-PUSH HNDL 14-16
1,2,3H,7,8A	170230	LUMBAR FWD STD PH KIT 16-18	2,5-7,6D,7,8A	169867	QS5X NON-PUSH HNDL 16-18
1,2,3I,7,8A	170231	LUMBAR FWD STD PH KIT 18-20	2,5-7,6E,7,8A	169868	QS5X NON-PUSH HNDL 18-20
1,2,3J,7,8A	170232	LUMBAR RWRD STD PH KIT 12-14	2,5-7,6F,7,8A	170242	LUMBAR FWD OMIT PH KIT 14-16
1,2,3K,7,8A	170233	LUMBAR RWRD STD PH KIT 14-16	2,5-7,6G,7,8A	170243	LUMBAR FWD OMIT PH KIT 16-18
1,2,3L,7,8A	170234	LUMBAR RWRD STD PH KIT 16-18	2,5-7,6H,7,8A	170244	LUMBAR FWD OMIT PH KIT 18-20
1,2,3M,7,8A	170235	LUMBAR RWRD STD PH KIT 18-20	2,5-7,6I,7,8A	170245	LUMBAR RWRD OMIT PH KIT 14-16
2,4B,9,1,7,8A	169860	QS5X FLD PUSH HNDL 12-14	2,5-7,6J,7,8A	170246	LUMBAR RWRD OMIT PH KIT 16-18
2,4C,9,7,8A	169861	QS5X FLD PUSH HNDL 14-16	2,5-7,6K,7,8A	170247	LUMBAR RWRD OMIT PH KIT 18-20
2,4D,7,8A	169862	QS5X FLD PUSH HNDL 16-18	7,8A	169853	PUSH HANDLE FASTENER KIT
2,4E,7,8A	169863	QS5X FLD PUSH HNDL 18-20	8B	103185-011	BHCS,M6X1X30,FULL,PL
2,4F,7,8A	170236	LUMBAR FWD FOLD PH KIT 14-16			

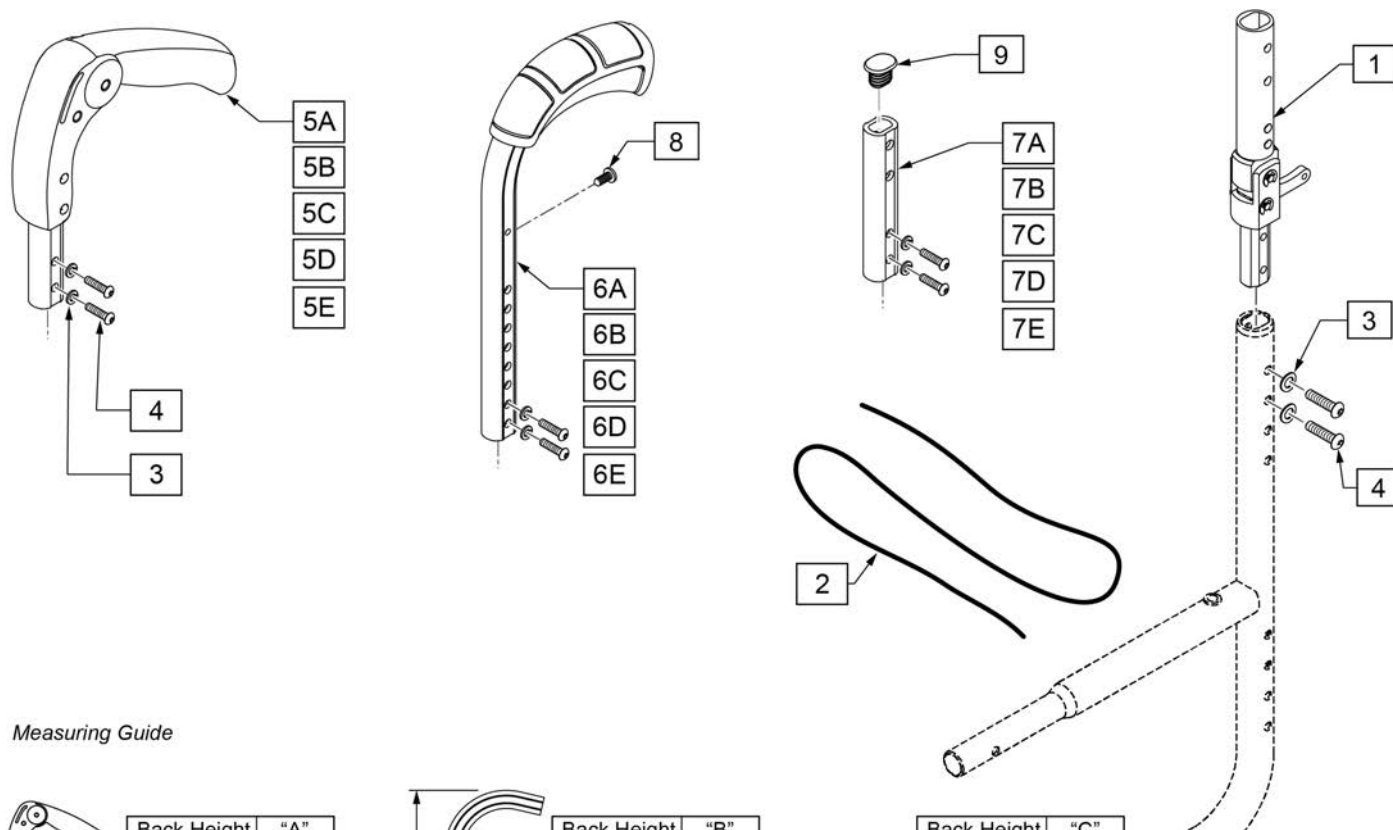

Measuring Guide

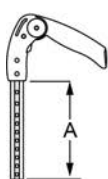

Back Height	"A"
14, 15 & 16"	2.25"
17"	3.25"
18"	4.25"
19"	5.25"
20"	6.5"

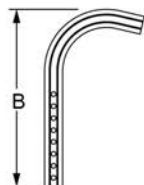


Back Height	"B"
14"	10"
15"	12"
16, 17 & 18"	14"
19"	16"
20"	18"




Back Height	"C"
14 - 16"	3.8"
17"	4.8"
18"	5.8"
19"	6.8"
20"	8"

Pos.	Item Number	Description	Remarks
1-4	76803016	BACK FOLDING MECHANISM	
3,4	169853	PUSH HANDLE FASTENER KIT	
3-5A	170143	FOLD DOWN PH FOR FLD BCK 14-16	for 14,15 and16 Ht
3-5B	170144	FOLD DOWN PH FOR FLD BCK 17	for 17
3-5C	170145	FOLD DOWN PH FOR FLD BCK 18	for 18
3-5D	170146	FOLD DOWN PH FOR FLD BCK 19	for 19
3-5E	170147	FOLD DOWN PH FOR FLD BCK 20	for 20
3,4,6A,8	170148	STD PH FOR FLD BCK 14	14 Ht
3,4,6B,8	170149	STD PH FOR FLD BCK 15	15
3,4,6C,8	170150	STD PH FOR FLD BCK 16-18	16 & 17 & 18
3,4,6D,8	170151	STD PH FOR FLD BCK 19	19
3,4,6E,8	170152	STD PH FOR FLD BCK 20	20
3,4,7A,9	170153	OMIT PH FOR FLD BCK 15, 16	15 & 16
3,4,7B,9	170154	OMIT PH FOR FLD BCK 17	17
3,4,7C,9	170155	OMIT PH FOR FLD BCK 18	18
3,4,7D,9	170156	OMIT PH FOR FLD BCK 19	19
3,4,7E,9	170157	OMIT PH FOR FLD BCK 20	20
8	107384S	PAN HD PHIL 10/32 x 5/8 ROHS	

NOTE: Retro fit requires Angle Adjustable Back version of the Rear Side Frame.

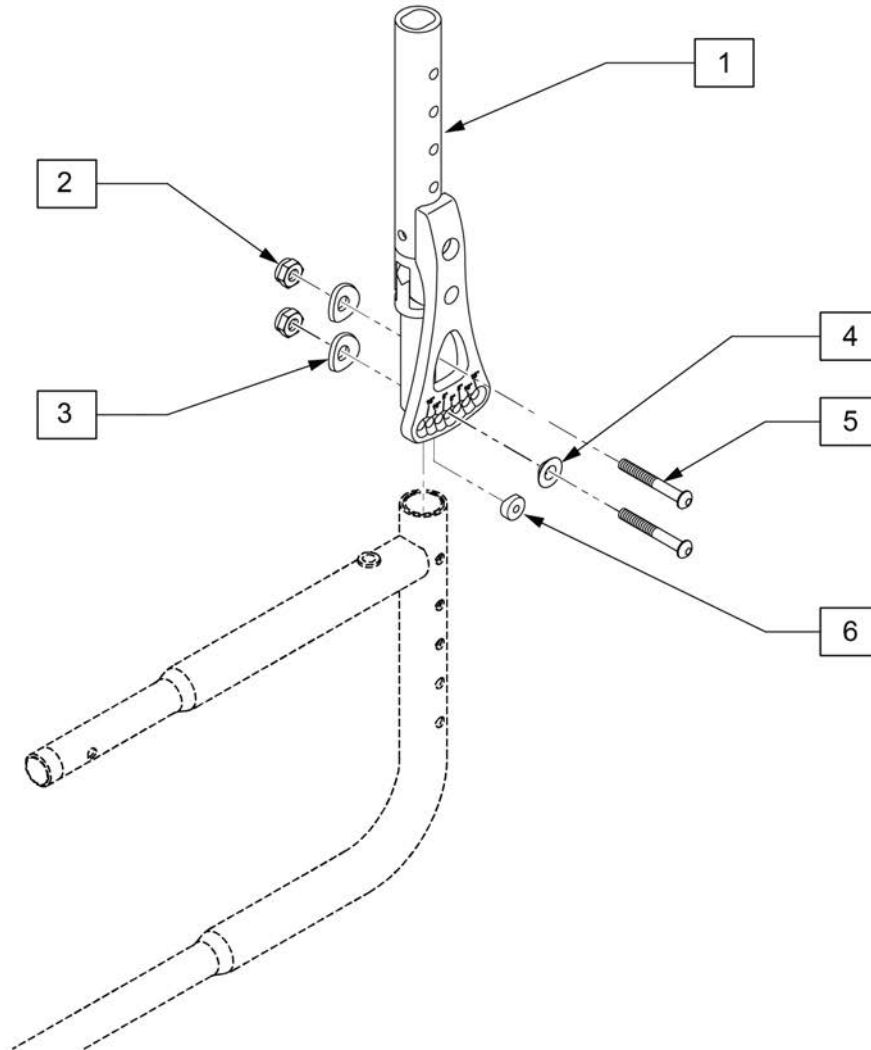

Pos.	Item Number	Description	Remarks
2-6	169875	QS5X ANGLE ADJ BACK HDWR	
1-6	169874	QS5X ANGLE ADJ BACK MECH	Fits Left or Right side

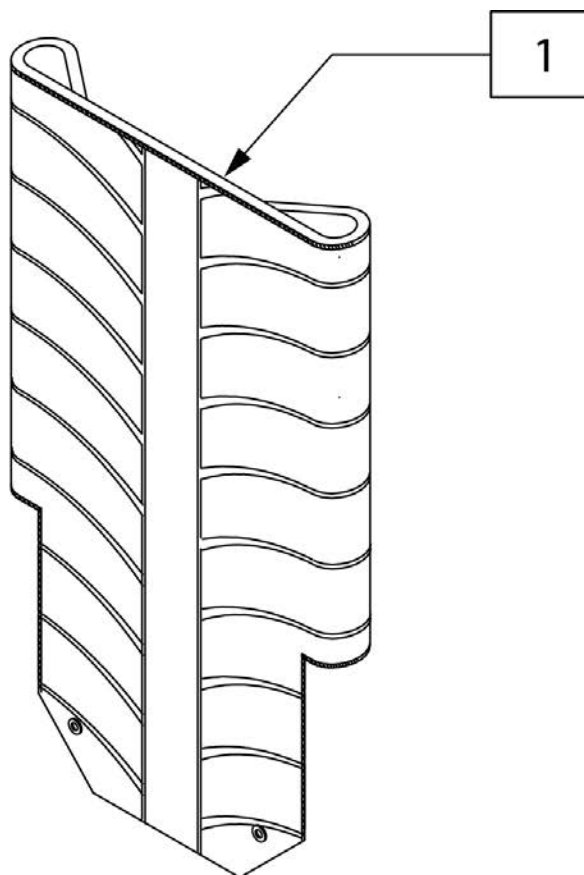


Pos.	Item Number	Description	Remarks
1	241275	BACK UPH EXO FLDG 14WD MD	Black
1	241276	BACK UPH EXO FLDG 14WD TL	Black
1	241277	BACK UPH EXO FLDG 15WD MD	Black
1	241278	BACK UPH EXO FLDG 15WD TL	Black
1	241279	BACK UPH EXO FLDG 16WD MD	Black
1	241280	BACK UPH EXO FLDG 16WD TL	Black
1	241281	BACK UPH EXO FLDG 17WD MD	Black
1	241282	BACK UPH EXO FLDG 17WD TL	Black
1	241283	BACK UPH EXO FLDG 18WD MD	Black
1	241284	BACK UPH EXO FLDG 18WD TL	Black
1	241285	BACK UPH EXO FLDG 19WD MD	Black
1	241286	BACK UPH EXO FLDG 19WD TL	Black
1	241287	BACK UPH EXO FLDG 20WD MD	Black
1	241288	BACK UPH EXO FLDG 20WD TL	Black
1	243650	BK UPH EXO FDG 11-12 LW BLK725	Black
1	243651	BK UPH EXO FDG 11-12 MD BLK725	Black
1	243652	BK UPH EXO FDG 11-12 TL BLK725	Black
1	243653	BK UPH EXO FDG 13W LW BLK725	Black
1	243654	BK UPH EXO FDG 13W MD BLK725	Black
1	243655	BK UPH EXO FDG 13W TL BLK725	Black
1	243656	BK UPH EXO FDG 14W LW BLK725	Black
1	243657	BK UPH EXO FDG 14W M-L BLK725	Black
1	243658	BK UPH EXO FDG 15W LW BLK725	Black
1	243659	BK UPH EXO FDG 16W LW BLK725	Black
1	243660	BK UPH EXO FDG 16W M-L BLK725	Black


Pos.	Item Number	Description	Remarks
1	243661	BK UPH EXO FDG 17W LW BLK725	Black
1	243662	BK UPH EXO FDG 18W LW BLK725	Black
1	243663	BK UPH EXO FDG 18W M-L BLK725	Black
1	243664	BK UPH EXO FDG 19W LW BLK725	Black
1	243665	BK UPH EXO FDG 20W LW BLK725	Black
1	243666	BK UPH EXO FDG 20W M-L BLK725	Black
1	242667	BACK UPH EXO FDG 14W MD ORANGE139	
1	242668	BACK UPH EXO FDG 14W TL ORANGE139	
1	242669	BACK UPH EXO FDG 15W MD ORANGE139	
1	242670	BACK UPH EXO FDG 15W TL ORANGE139	
1	242671	BACK UPH EXO FDG 16W MD ORANGE139	
1	242672	BACK UPH EXO FDG 16W TL ORANGE139	
1	242673	BACK UPH EXO FDG 17W MD ORANGE139	
1	242674	BACK UPH EXO FDG 17W TL ORANGE139	
1	242675	BACK UPH EXO FDG 18W MD ORANGE139	
1	242676	BACK UPH EXO FDG 18W TL ORANGE139	
1	242677	BACK UPH EXO FDG 19W MD ORANGE139	
1	242678	BACK UPH EXO FDG 19W TL ORANGE139	
1	242679	BACK UPH EXO FDG 20W MD ORANGE139	
1	242680	BACK UPH EXO FDG 20W TL ORANGE139	
1	243671	BK UPH EXO FDG 13W MD ORG139	
1	243672	BK UPH EXO FDG 13W TL ORG139	
1	243673	BK UPH EXO FDG 14W LW ORG139	
1	243674	BK UPH EXO FDG 14W M-L ORG139	
1	243675	BK UPH EXO FDG 15W LW ORG139	
1	243676	BK UPH EXO FDG 16W LW ORG139	
1	243677	BK UPH EXO FDG 16W M-L ORG139	
1	243678	BK UPH EXO FDG 17W LW ORG139	
1	243679	BK UPH EXO FDG 18W LW ORG139	
1	243680	BK UPH EXO FDG 18W M-L ORG139	
1	243681	BK UPH EXO FDG 19W LW ORG139	
1	243682	BK UPH EXO FDG 20W LW ORG139	
1	243683	BK UPH EXO FDG 20W M-L ORG139	
1	242737	BACK UPH EXO FDG 14W MD SILVER 702	
1	242738	BACK UPH EXO FDG 14W TL SILVER 702	
1	242739	BACK UPH EXO FDG 15W MD SILVER 702	
1	242740	BACK UPH EXO FDG 15W TL SILVER 702	
1	242741	BACK UPH EXO FDG 16W MD SILVER 702	
1	242742	BACK UPH EXO FDG 16W TL SILVER 702	
1	242743	BACK UPH EXO FDG 17W MD SILVER 702	
1	242744	BACK UPH EXO FDG 17W TL SILVER 702	
1	242745	BACK UPH EXO FDG 18W MD SILVER 702	
1	242746	BACK UPH EXO FDG 18W TL SILVER 702	
1	242747	BACK UPH EXO FDG 19W MD SILVER 702	
1	242748	BACK UPH EXO FDG 19W TL SILVER 702	
1	242749	BACK UPH EXO FDG 20W MD SILVER 702	
1	242750	BACK UPH EXO FDG 20W TL SILVER 702	
1	243684	BK UPH EXO FDG 11-12 LW SLV702	
1	243685	BK UPH EXO FDG 11-12 MD SLV702	
1	243686	BK UPH EXO FDG 11-12 TL SLV702	
1	243687	BK UPH EXO FDG 13W LW SLV702	
1	243688	BK UPH EXO FDG 13W MD SLV702	
1	243689	BK UPH EXO FDG 13W TL SLV702	
1	243690	BK UPH EXO FDG 14W LW SLV702	
1	243691	BK UPH EXO FDG 14W M-L SLV702	
1	243692	BK UPH EXO FDG 15W LW SLV702	
1	243693	BK UPH EXO FDG 16W LW SLV702	

Pos.	Item Number	Description	Remarks
1	243694	BK UPH EXO FDG 16W M-L SLV702	
1	243695	BK UPH EXO FDG 17W LW SLV702	
1	243696	BK UPH EXO FDG 18W LW SLV702	
1	243697	BK UPH EXO FDG 18W M-L SLV702	
1	243698	BK UPH EXO FDG 19W LW SLV702	
1	243699	BK UPH EXO FDG 20W LW SLV702	
1	243700	BK UPH EXO FDG 20W M-L SLV702	
1	242806	BACK UPH EXO FDG 14W MD BLUE 570	
1	242807	BACK UPH EXO FDG 14W TL BLUE 570	
1	242808	BACK UPH EXO FDG 15W MD BLUE 570	
1	242809	BACK UPH EXO FDG 15W TL BLUE 570	
1	242810	BACK UPH EXO FDG 16W MD BLUE 570	
1	242811	BACK UPH EXO FDG 16W TL BLUE 570	
1	242812	BACK UPH EXO FDG 17W MD BLUE 570	
1	242813	BACK UPH EXO FDG 17W TL BLUE 570	
1	242814	BACK UPH EXO FDG 18W MD BLUE 570	
1	242815	BACK UPH EXO FDG 18W TL BLUE 570	
1	242816	BACK UPH EXO FDG 19W MD BLUE 570	
1	242817	BACK UPH EXO FDG 19W TL BLUE 570	
1	242818	BACK UPH EXO FDG 20W MD BLUE 570	
1	242819	BACK UPH EXO FDG 20W TL BLUE 570	
1	243701	BK UPH EXO FDG 11-12 LW BLU570	
1	243702	BK UPH EXO FDG 11-12 MD BLU570	
1	243703	BK UPH EXO FDG 11-12 TL BLU570	
1	243704	BK UPH EXO FDG 13W LW BLU570	
1	243705	BK UPH EXO FDG 13W MD BLU570	
1	243706	BK UPH EXO FDG 13W TL BLU570	
1	243707	BK UPH EXO FDG 14W LW BLU570	
1	243708	BK UPH EXO FDG 14W M-L BLU570	
1	243709	BK UPH EXO FDG 15W LW BLU570	
1	243710	BK UPH EXO FDG 16W LW BLU570	
1	243711	BK UPH EXO FDG 16W M-L BLU570	
1	243712	BK UPH EXO FDG 17W LW BLU570	
1	243713	BK UPH EXO FDG 18W LW BLU570	
1	243714	BK UPH EXO FDG 18W M-L BLU570	
1	243715	BK UPH EXO FDG 19W LW BLU570	
1	243716	BK UPH EXO FDG 20W LW BLU570	
1	243717	BK UPH EXO FDG 20W M-L BLU570	
1	242929	BACK UPH EXO FDG 14W MD RED 320	
1	242930	BACK UPH EXO FDG 14W TL RED 320	
1	242931	BACK UPH EXO FDG 15W MD RED 320	
1	242932	BACK UPH EXO FDG 15W TL RED 320	
1	242933	BACK UPH EXO FDG 16W MD RED 320	
1	242934	BACK UPH EXO FDG 16W TL RED 320	
1	242935	BACK UPH EXO FDG 17W MD RED 320	
1	242936	BACK UPH EXO FDG 17W TL RED 320	
1	242937	BACK UPH EXO FDG 18W MD RED 320	
1	242938	BACK UPH EXO FDG 18W TL RED 320	
1	242939	BACK UPH EXO FDG 19W MD RED 320	
1	242940	BACK UPH EXO FDG 19W TL RED 320	
1	242941	BACK UPH EXO FDG 20W MD RED 320	
1	242942	BACK UPH EXO FDG 20W TL RED 320	
1	243718	BK UPH EXO FDG 11-12 LW RED320	
1	243719	BK UPH EXO FDG 11-12 MD RED320	
1	243720	BK UPH EXO FDG 11-12 TL RED320	
1	243721	BK UPH EXO FDG 13W LW RED320	
1	243722	BK UPH EXO FDG 13W MD RED320	

Pos.	Item Number	Description	Remarks
1	243723	BK UPH EXO FDG 13W TL RED320	
1	243724	BK UPH EXO FDG 14W LW RED320	
1	243725	BK UPH EXO FDG 14W M-L RED320	
1	243726	BK UPH EXO FDG 15W LW RED320	
1	243727	BK UPH EXO FDG 16W LW RED320	
1	243728	BK UPH EXO FDG 16W M-L RED320	
1	243729	BK UPH EXO FDG 17W LW RED320	
1	243730	BK UPH EXO FDG 18W LW RED320	
1	243731	BK UPH EXO FDG 18W M-L RED320	
1	243732	BK UPH EXO FDG 19W LW RED320	
1	243733	BK UPH EXO FDG 20W LW RED320	
1	243734	BK UPH EXO FDG 20W M-L RED320	

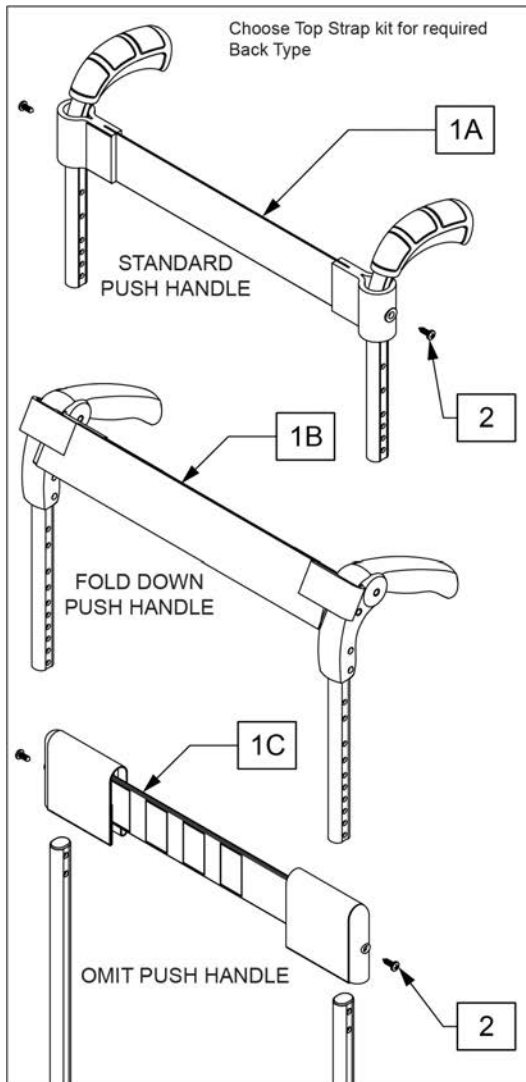


Choose number of Tension Straps for required Back Height

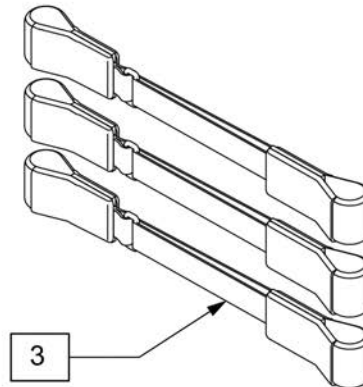

Back straps needed per height		
Back Height	STD/Fold-down /Omit	Height Adj.
10	3	N/A
11	3	N/A
12	4	N/A
13	4	3
14	5	3
15	5	4
16	5	4
17	6	5
18	6	5
19	6	5
20	7	6

Choose Upholstery for required Back Height, Back Width and Trim Color

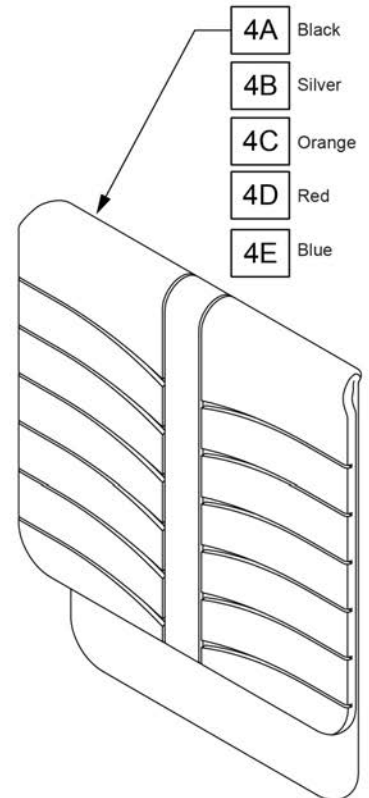


Back Height	Size
10-12"	XS
12-14"	SH
14-16"	MD
19-18"	TL
18-20"	XT


Pos.	Item Number	Description	Pos.	Item Number	Description
1A,2	170322	EX PRO TA TOP STRAP STD PH	4A	257731S	BACK UPH EXO TA BLK 14W XT
1B	06804026-000	UPPER BACK BELT	4A	257732S	BACK UPH EXO TA BLK 15W XS
1C,2	170323	EX PRO TA TOP STRAP OMIT PH	4A	257733S	BACK UPH EXO TA BLK 15W SH
3	066853	SINGLE BACK BELT	4A	257734S	BACK UPH EXO TA BLK 15W MD
4A	257717S	BACK UPH EXO TA BLK 12W XS	4A	257735S	BACK UPH EXO TA BLK 15W TL
4A	257718S	BACK UPH EXO TA BLK 12W SH	4A	257736S	BACK UPH EXO TA BLK 15W XT
4A	257719S	BACK UPH EXO TA BLK 12W MD	4A	257737S	BACK UPH EXO TA BLK 16W XS
4A	257720S	BACK UPH EXO TA BLK 12W TL	4A	257738S	BACK UPH EXO TA BLK 16W SH
4A	257721S	BACK UPH EXO TA BLK 12W XT	4A	257739S	BACK UPH EXO TA BLK 16W MD
4A	257722S	BACK UPH EXO TA BLK 13W XS	4A	257740S	BACK UPH EXO TA BLK 16W TL
4A	257723S	BACK UPH EXO TA BLK 13W SH	4A	257741S	BACK UPH EXO TA BLK 16W XT
4A	257724S	BACK UPH EXO TA BLK 13W MD	4A	257742S	BACK UPH EXO TA BLK 17W XS
4A	257725S	BACK UPH EXO TA BLK 13W TL	4A	257743S	BACK UPH EXO TA BLK 17W SH
4A	257726S	BACK UPH EXO TA BLK 13W XT	4A	257744S	BACK UPH EXO TA BLK 17W MD
4A	257727S	BACK UPH EXO TA BLK 14W XS	4A	257745S	BACK UPH EXO TA BLK 17W TL
4A	257728S	BACK UPH EXO TA BLK 14W SH	4A	257746S	BACK UPH EXO TA BLK 17W XT
4A	257729S	BACK UPH EXO TA BLK 14W MD	4A	257747S	BACK UPH EXO TA BLK 18W XS
4A	257730S	BACK UPH EXO TA BLK 14W TL	4A	257748S	BACK UPH EXO TA BLK 18W SH

Pos.	Item Number	Description	Pos.	Item Number	Description
4A	257749S	BACK UPH EXO TA BLK 18W MD	4B	258297S	BACK UPH EXO TA SLV 20W XT
4A	257750S	BACK UPH EXO TA BLK 18W TL	4C	258298S	BACK UPH EXO TA ORG 12W XS
4A	257751S	BACK UPH EXO TA BLK 18W XT	4C	258299S	BACK UPH EXO TA ORG 12W SH
4A	257752S	BACK UPH EXO TA BLK 19W XS	4C	258300S	BACK UPH EXO TA ORG 12W MD
4A	257753S	BACK UPH EXO TA BLK 19W SH	4C	258301S	BACK UPH EXO TA ORG 12W TL
4A	257754S	BACK UPH EXO TA BLK 19W MD	4C	258302S	BACK UPH EXO TA ORG 12W XT
4A	257755S	BACK UPH EXO TA BLK 19W TL	4C	258303S	BACK UPH EXO TA ORG 13W XS
4A	257756S	BACK UPH EXO TA BLK 19W XT	4C	258304S	BACK UPH EXO TA ORG 13W SH
4A	257757S	BACK UPH EXO TA BLK 20W XS	4C	258305S	BACK UPH EXO TA ORG 13W MD
4A	257758S	BACK UPH EXO TA BLK 20W SH	4C	258306S	BACK UPH EXO TA ORG 13W TL
4A	257759S	BACK UPH EXO TA BLK 20W MD	4C	258307S	BACK UPH EXO TA ORG 13W XT
4A	257760S	BACK UPH EXO TA BLK 20W TL	4C	258308S	BACK UPH EXO TA ORG 14W XS
4A	257761S	BACK UPH EXO TA BLK 20W XT	4C	258309S	BACK UPH EXO TA ORG 14W SH
4B	258253S	BACK UPH EXO TA SLV 12W XS	4C	258310S	BACK UPH EXO TA ORG 14W MD
4B	258254S	BACK UPH EXO TA SLV 12W SH	4C	258311S	BACK UPH EXO TA ORG 14W TL
4B	258255S	BACK UPH EXO TA SLV 12W MD	4C	258312S	BACK UPH EXO TA ORG 14W XT
4B	258256S	BACK UPH EXO TA SLV 12W TL	4C	258313S	BACK UPH EXO TA ORG 15W XS
4B	258257S	BACK UPH EXO TA SLV 12W XT	4C	258314S	BACK UPH EXO TA ORG 15W SH
4B	258258S	BACK UPH EXO TA SLV 13W XS	4C	258315S	BACK UPH EXO TA ORG 15W MD
4B	258259S	BACK UPH EXO TA SLV 13W SH	4C	258316S	BACK UPH EXO TA ORG 15W TL
4B	258260S	BACK UPH EXO TA SLV 13W MD	4C	258317S	BACK UPH EXO TA ORG 15W XT
4B	258261S	BACK UPH EXO TA SLV 13W TL	4C	258318S	BACK UPH EXO TA ORG 16W XS
4B	258262S	BACK UPH EXO TA SLV 13W XT	4C	258319S	BACK UPH EXO TA ORG 16W SH
4B	258263S	BACK UPH EXO TA SLV 14W XS	4C	258320S	BACK UPH EXO TA ORG 16W MD
4B	258264S	BACK UPH EXO TA SLV 14W SH	4C	258321S	BACK UPH EXO TA ORG 16W TL
4B	258265S	BACK UPH EXO TA SLV 14W MD	4C	258322S	BACK UPH EXO TA ORG 16W XT
4B	258266S	BACK UPH EXO TA SLV 14W TL	4C	258323S	BACK UPH EXO TA ORG 17W XS
4B	258267S	BACK UPH EXO TA SLV 14W XT	4C	258324S	BACK UPH EXO TA ORG 17W SH
4B	258268S	BACK UPH EXO TA SLV 15W XS	4C	258325S	BACK UPH EXO TA ORG 17W MD
4B	258269S	BACK UPH EXO TA SLV 15W SH	4C	258326S	BACK UPH EXO TA ORG 17W TL
4B	258270S	BACK UPH EXO TA SLV 15W MD	4C	258327S	BACK UPH EXO TA ORG 17W XT
4B	258271S	BACK UPH EXO TA SLV 15W TL	4C	258328S	BACK UPH EXO TA ORG 18W XS
4B	258272S	BACK UPH EXO TA SLV 15W XT	4C	258329S	BACK UPH EXO TA ORG 18W SH
4B	258273S	BACK UPH EXO TA SLV 16W XS	4C	258330S	BACK UPH EXO TA ORG 18W MD
4B	258274S	BACK UPH EXO TA SLV 16W SH	4C	258331S	BACK UPH EXO TA ORG 18W TL
4B	258275S	BACK UPH EXO TA SLV 16W MD	4C	258332S	BACK UPH EXO TA ORG 18W XT
4B	258276S	BACK UPH EXO TA SLV 16W TL	4C	258333S	BACK UPH EXO TA ORG 19W XS
4B	258277S	BACK UPH EXO TA SLV 16W XT	4C	258334S	BACK UPH EXO TA ORG 19W SH
4B	258278S	BACK UPH EXO TA SLV 17W XS	4C	258335S	BACK UPH EXO TA ORG 19W MD
4B	258279S	BACK UPH EXO TA SLV 17W SH	4C	258336S	BACK UPH EXO TA ORG 19W TL
4B	258280S	BACK UPH EXO TA SLV 17W MD	4C	258337S	BACK UPH EXO TA ORG 19W XT
4B	258281S	BACK UPH EXO TA SLV 17W TL	4C	258338S	BACK UPH EXO TA ORG 20W XS
4B	258282S	BACK UPH EXO TA SLV 17W XT	4C	258339S	BACK UPH EXO TA ORG 20W SH
4B	258283S	BACK UPH EXO TA SLV 18W XS	4C	258340S	BACK UPH EXO TA ORG 20W MD
4B	258284S	BACK UPH EXO TA SLV 18W SH	4C	258341S	BACK UPH EXO TA ORG 20W TL
4B	258285S	BACK UPH EXO TA SLV 18W MD	4C	258342S	BACK UPH EXO TA ORG 20W XT
4B	258286S	BACK UPH EXO TA SLV 18W TL	4D	258343S	BACK UPH EXO TA RED 12W XS
4B	258287S	BACK UPH EXO TA SLV 18W XT	4D	258344S	BACK UPH EXO TA RED 12W SH
4B	258288S	BACK UPH EXO TA SLV 19W XS	4D	258345S	BACK UPH EXO TA RED 12W MD
4B	258289S	BACK UPH EXO TA SLV 19W SH	4D	258346S	BACK UPH EXO TA RED 12W TL
4B	258290S	BACK UPH EXO TA SLV 19W MD	4D	258347S	BACK UPH EXO TA RED 12W XT
4B	258291S	BACK UPH EXO TA SLV 19W TL	4D	258348S	BACK UPH EXO TA RED 13W XS
4B	258292S	BACK UPH EXO TA SLV 19W XT	4D	258349S	BACK UPH EXO TA RED 13W SH
4B	258293S	BACK UPH EXO TA SLV 20W XS	4D	258350S	BACK UPH EXO TA RED 13W MD
4B	258294S	BACK UPH EXO TA SLV 20W SH	4D	258351S	BACK UPH EXO TA RED 13W TL
4B	258295S	BACK UPH EXO TA SLV 20W MD	4D	258352S	BACK UPH EXO TA RED 13W XT
4B	258296S	BACK UPH EXO TA SLV 20W TL	4D	258353S	BACK UPH EXO TA RED 14W XS

Pos.	Item Number	Description	Pos.	Item Number	Description
4D	258354S	BACK UPH EXO TA RED 14W SH	4E	258394S	BACK UPH EXO TA BLU 13W SH
4D	258355S	BACK UPH EXO TA RED 14W MD	4E	258395S	BACK UPH EXO TA BLU 13W MD
4D	258356S	BACK UPH EXO TA RED 14W TL	4E	258396S	BACK UPH EXO TA BLU 13W TL
4D	258357S	BACK UPH EXO TA RED 14W XT	4E	258397S	BACK UPH EXO TA BLU 13W XT
4D	258358S	BACK UPH EXO TA RED 15W XS	4E	258398S	BACK UPH EXO TA BLU 14W XS
4D	258359S	BACK UPH EXO TA RED 15W SH	4E	258399S	BACK UPH EXO TA BLU 14W SH
4D	258360S	BACK UPH EXO TA RED 15W MD	4E	258400S	BACK UPH EXO TA BLU 14W MD
4D	258361S	BACK UPH EXO TA RED 15W TL	4E	258401S	BACK UPH EXO TA BLU 14W TL
4D	258362S	BACK UPH EXO TA RED 15W XT	4E	258402S	BACK UPH EXO TA BLU 14W XT
4D	258363S	BACK UPH EXO TA RED 16W XS	4E	258403S	BACK UPH EXO TA BLU 15W XS
4D	258364S	BACK UPH EXO TA RED 16W SH	4E	258404S	BACK UPH EXO TA BLU 15W SH
4D	258365S	BACK UPH EXO TA RED 16W MD	4E	258405S	BACK UPH EXO TA BLU 15W MD
4D	258366S	BACK UPH EXO TA RED 16W TL	4E	258406S	BACK UPH EXO TA BLU 15W TL
4D	258367S	BACK UPH EXO TA RED 16W XT	4E	258407S	BACK UPH EXO TA BLU 15W XT
4D	258368S	BACK UPH EXO TA RED 17W XS	4E	258408S	BACK UPH EXO TA BLU 16W XS
4D	258369S	BACK UPH EXO TA RED 17W SH	4E	258409S	BACK UPH EXO TA BLU 16W SH
4D	258370S	BACK UPH EXO TA RED 17W MD	4E	258410S	BACK UPH EXO TA BLU 16W MD
4D	258371S	BACK UPH EXO TA RED 17W TL	4E	258411S	BACK UPH EXO TA BLU 16W TL
4D	258372S	BACK UPH EXO TA RED 17W XT	4E	258412S	BACK UPH EXO TA BLU 16W XT
4D	258373S	BACK UPH EXO TA RED 18W XS	4E	258413S	BACK UPH EXO TA BLU 17W XS
4D	258374S	BACK UPH EXO TA RED 18W SH	4E	258414S	BACK UPH EXO TA BLU 17W SH
4D	258375S	BACK UPH EXO TA RED 18W MD	4E	258415S	BACK UPH EXO TA BLU 17W MD
4D	258376S	BACK UPH EXO TA RED 18W TL	4E	258416S	BACK UPH EXO TA BLU 17W TL
4D	258377S	BACK UPH EXO TA RED 18W XT	4E	258417S	BACK UPH EXO TA BLU 17W XT
4D	258378S	BACK UPH EXO TA RED 19W XS	4E	258418S	BACK UPH EXO TA BLU 18W XS
4D	258379S	BACK UPH EXO TA RED 19W SH	4E	258419S	BACK UPH EXO TA BLU 18W SH
4D	258380S	BACK UPH EXO TA RED 19W MD	4E	258420S	BACK UPH EXO TA BLU 18W MD
4D	258381S	BACK UPH EXO TA RED 19W TL	4E	258421S	BACK UPH EXO TA BLU 18W TL
4D	258382S	BACK UPH EXO TA RED 19W XT	4E	258422S	BACK UPH EXO TA BLU 18W XT
4D	258383S	BACK UPH EXO TA RED 20W XS	4E	258423S	BACK UPH EXO TA BLU 19W XS
4D	258384S	BACK UPH EXO TA RED 20W SH	4E	258424S	BACK UPH EXO TA BLU 19W SH
4D	258385S	BACK UPH EXO TA RED 20W MD	4E	258425S	BACK UPH EXO TA BLU 19W MD
4D	258386S	BACK UPH EXO TA RED 20W TL	4E	258426S	BACK UPH EXO TA BLU 19W TL
4D	258387S	BACK UPH EXO TA RED 20W XT	4E	258427S	BACK UPH EXO TA BLU 19W XT
4E	258388S	BACK UPH EXO TA BLU 12W XS	4E	258428S	BACK UPH EXO TA BLU 20W XS
4E	258389S	BACK UPH EXO TA BLU 12W SH	4E	258429S	BACK UPH EXO TA BLU 20W SH
4E	258390S	BACK UPH EXO TA BLU 12W MD	4E	258430S	BACK UPH EXO TA BLU 20W MD
4E	258391S	BACK UPH EXO TA BLU 12W TL	4E	258431S	BACK UPH EXO TA BLU 20W TL
4E	258392S	BACK UPH EXO TA BLU 12W XT	4E	258432S	BACK UPH EXO TA BLU 20W XT
4E	258393S	BACK UPH EXO TA BLU 13W XS			

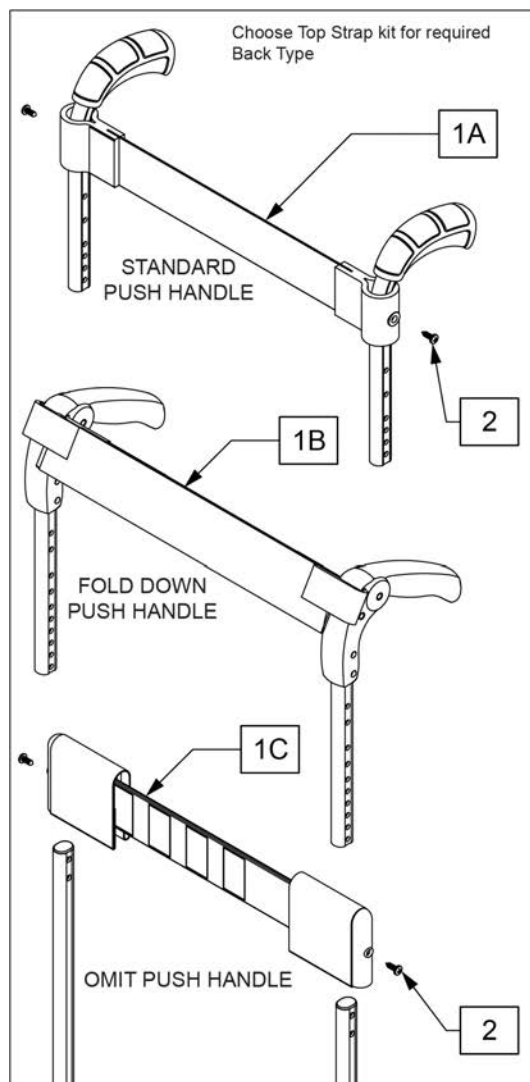


Choose number of Tension Straps for required Back Height

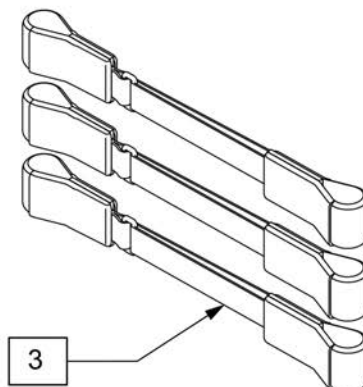

Back straps needed per height		
Back Height	STD/Fold-down /Omit	Height Adj.
10	3	N/A
11	3	N/A
12	4	N/A
13	4	3
14	5	3
15	5	4
16	5	4
17	6	5
18	6	5
19	6	5
20	7	6

Choose Upholstery for required Back Height, Back Width and Trim Color

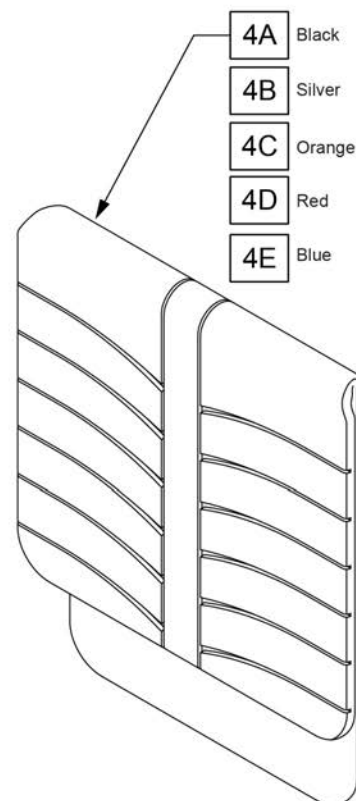


Back Height	Size
10-12"	XS
12-14"	SH
14-16"	MD
19-18"	TL
18-20"	XT

Pos.	Item Number	Description	Pos.	Item Number	Description
1A,2	170322	EX PRO TA TOP STRAP STD PH	4A	257650S	BACK UPH EXO PRO TA BLK 14W XT
1B	06804026-000	UPPER BACK BELT	4A	257651S	BACK UPH EXO PRO TA BLK 15W XS
1C,2	170323	EX PRO TA TOP STRAP OMIT PH	4A	257652S	BACK UPH EXO PRO TA BLK 15W SH
3	066853	SINGLE BACK BELT	4A	257653S	BACK UPH EXO PRO TA BLK 15W MD
4A	257636S	BACK UPH EXO PRO TA BLK 12W XS	4A	257654S	BACK UPH EXO PRO TA BLK 15W TL
4A	257637S	BACK UPH EXO PRO TA BLK 12W SH	4A	257655S	BACK UPH EXO PRO TA BLK 15W XT
4A	257638S	BACK UPH EXO PRO TA BLK 12W MD	4A	257656S	BACK UPH EXO PRO TA BLK 16W XS
4A	257639S	BACK UPH EXO PRO TA BLK 12W TL	4A	257657S	BACK UPH EXO PRO TA BLK 16W SH
4A	257640S	BACK UPH EXO PRO TA BLK 12W XT	4A	257658S	BACK UPH EXO PRO TA BLK 16W MD
4A	257641S	BACK UPH EXO PRO TA BLK 13W XS	4A	257659S	BACK UPH EXO PRO TA BLK 16W TL
4A	257642S	BACK UPH EXO PRO TA BLK 13W SH	4A	257660S	BACK UPH EXO PRO TA BLK 16W XT
4A	257643S	BACK UPH EXO PRO TA BLK 13W MD	4A	257661S	BACK UPH EXO PRO TA BLK 17W XS
4A	257644S	BACK UPH EXO PRO TA BLK 13W TL	4A	257662S	BACK UPH EXO PRO TA BLK 17W SH
4A	257645S	BACK UPH EXO PRO TA BLK 13W XT	4A	257663S	BACK UPH EXO PRO TA BLK 17W MD
4A	257646S	BACK UPH EXO PRO TA BLK 14W XS	4A	257664S	BACK UPH EXO PRO TA BLK 17W TL
4A	257647S	BACK UPH EXO PRO TA BLK 14W SH	4A	257665S	BACK UPH EXO PRO TA BLK 17W XT
4A	257648S	BACK UPH EXO PRO TA BLK 14W MD	4A	257666S	BACK UPH EXO PRO TA BLK 18W XS
4A	257649S	BACK UPH EXO PRO TA BLK 14W TL	4A	257667S	BACK UPH EXO PRO TA BLK 18W SH

Pos.	Item Number	Description	Pos.	Item Number	Description
4A	257668S	BACK UPH EXO PRO TA BLK 18W MD	4B	258117S	BACK UPH EXO PRO TA SLV 20W XT
4A	257669S	BACK UPH EXO PRO TA BLK 18W TL	4C	258118S	BACK UPH EXO PRO TA ORG 12W XS
4A	257670S	BACK UPH EXO PRO TA BLK 18W XT	4C	258119S	BACK UPH EXO PRO TA ORG 12W SH
4A	257671S	BACK UPH EXO PRO TA BLK 19W XS	4C	258120S	BACK UPH EXO PRO TA ORG 12W MD
4A	257672S	BACK UPH EXO PRO TA BLK 19W SH	4C	258121S	BACK UPH EXO PRO TA ORG 12W TL
4A	257673S	BACK UPH EXO PRO TA BLK 19W MD	4C	258122S	BACK UPH EXO PRO TA ORG 12W XT
4A	257674S	BACK UPH EXO PRO TA BLK 19W TL	4C	258123S	BACK UPH EXO PRO TA ORG 13W XS
4A	257675S	BACK UPH EXO PRO TA BLK 19W XT	4C	258124S	BACK UPH EXO PRO TA ORG 13W SH
4A	257676S	BACK UPH EXO PRO TA BLK 20W XS	4C	258125S	BACK UPH EXO PRO TA ORG 13W MD
4A	257677S	BACK UPH EXO PRO TA BLK 20W SH	4C	258126S	BACK UPH EXO PRO TA ORG 13W TL
4A	257678S	BACK UPH EXO PRO TA BLK 20W MD	4C	258127S	BACK UPH EXO PRO TA ORG 13W XT
4A	257679S	BACK UPH EXO PRO TA BLK 20W TL	4C	258128S	BACK UPH EXO PRO TA ORG 14W XS
4A	257680S	BACK UPH EXO PRO TA BLK 20W XT	4C	258129S	BACK UPH EXO PRO TA ORG 14W SH
4B	258073S	BACK UPH EXO PRO TA SLV 12W XS	4C	258130S	BACK UPH EXO PRO TA ORG 14W MD
4B	258074S	BACK UPH EXO PRO TA SLV 12W SH	4C	258131S	BACK UPH EXO PRO TA ORG 14W TL
4B	258075S	BACK UPH EXO PRO TA SLV 12W MD	4C	258132S	BACK UPH EXO PRO TA ORG 14W XT
4B	258076S	BACK UPH EXO PRO TA SLV 12W TL	4C	258133S	BACK UPH EXO PRO TA ORG 15W XS
4B	258077S	BACK UPH EXO PRO TA SLV 12W XT	4C	258134S	BACK UPH EXO PRO TA ORG 15W SH
4B	258078S	BACK UPH EXO PRO TA SLV 13W XS	4C	258135S	BACK UPH EXO PRO TA ORG 15W MD
4B	258079S	BACK UPH EXO PRO TA SLV 13W SH	4C	258136S	BACK UPH EXO PRO TA ORG 15W TL
4B	258080S	BACK UPH EXO PRO TA SLV 13W MD	4C	258137S	BACK UPH EXO PRO TA ORG 15W XT
4B	258081S	BACK UPH EXO PRO TA SLV 13W TL	4C	258138S	BACK UPH EXO PRO TA ORG 16W XS
4B	258082S	BACK UPH EXO PRO TA SLV 13W XT	4C	258139S	BACK UPH EXO PRO TA ORG 16W SH
4B	258083S	BACK UPH EXO PRO TA SLV 14W XS	4C	258140S	BACK UPH EXO PRO TA ORG 16W MD
4B	258084S	BACK UPH EXO PRO TA SLV 14W SH	4C	258141S	BACK UPH EXO PRO TA ORG 16W TL
4B	258085S	BACK UPH EXO PRO TA SLV 14W MD	4C	258142S	BACK UPH EXO PRO TA ORG 16W XT
4B	258086S	BACK UPH EXO PRO TA SLV 14W TL	4C	258143S	BACK UPH EXO PRO TA ORG 17W XS
4B	258087S	BACK UPH EXO PRO TA SLV 14W XT	4C	258144S	BACK UPH EXO PRO TA ORG 17W SH
4B	258088S	BACK UPH EXO PRO TA SLV 15W XS	4C	258145S	BACK UPH EXO PRO TA ORG 17W MD
4B	258089S	BACK UPH EXO PRO TA SLV 15W SH	4C	258146S	BACK UPH EXO PRO TA ORG 17W TL
4B	258090S	BACK UPH EXO PRO TA SLV 15W MD	4C	258147S	BACK UPH EXO PRO TA ORG 17W XT
4B	258091S	BACK UPH EXO PRO TA SLV 15W TL	4C	258148S	BACK UPH EXO PRO TA ORG 18W XS
4B	258092S	BACK UPH EXO PRO TA SLV 15W XT	4C	258149S	BACK UPH EXO PRO TA ORG 18W SH
4B	258093S	BACK UPH EXO PRO TA SLV 16W XS	4C	258150S	BACK UPH EXO PRO TA ORG 18W MD
4B	258094S	BACK UPH EXO PRO TA SLV 16W SH	4C	258151S	BACK UPH EXO PRO TA ORG 18W TL
4B	258095S	BACK UPH EXO PRO TA SLV 16W MD	4C	258152S	BACK UPH EXO PRO TA ORG 18W XT
4B	258096S	BACK UPH EXO PRO TA SLV 16W TL	4C	258153S	BACK UPH EXO PRO TA ORG 19W XS
4B	258097S	BACK UPH EXO PRO TA SLV 16W XT	4C	258154S	BACK UPH EXO PRO TA ORG 19W SH
4B	258098S	BACK UPH EXO PRO TA SLV 17W XS	4C	258155S	BACK UPH EXO PRO TA ORG 19W MD
4B	258099S	BACK UPH EXO PRO TA SLV 17W SH	4C	258156S	BACK UPH EXO PRO TA ORG 19W TL
4B	258100S	BACK UPH EXO PRO TA SLV 17W MD	4C	258157S	BACK UPH EXO PRO TA ORG 19W XT
4B	258101S	BACK UPH EXO PRO TA SLV 17W TL	4C	258158S	BACK UPH EXO PRO TA ORG 20W XS
4B	258102S	BACK UPH EXO PRO TA SLV 17W XT	4C	258159S	BACK UPH EXO PRO TA ORG 20W SH
4B	258103S	BACK UPH EXO PRO TA SLV 18W XS	4C	258160S	BACK UPH EXO PRO TA ORG 20W MD
4B	258104S	BACK UPH EXO PRO TA SLV 18W SH	4C	258161S	BACK UPH EXO PRO TA ORG 20W TL
4B	258105S	BACK UPH EXO PRO TA SLV 18W MD	4C	258162S	BACK UPH EXO PRO TA ORG 20W XT
4B	258106S	BACK UPH EXO PRO TA SLV 18W TL	4D	258163S	BACK UPH EXO PRO TA RED 12W XS
4B	258107S	BACK UPH EXO PRO TA SLV 18W XT	4D	258164S	BACK UPH EXO PRO TA RED 12W SH
4B	258108S	BACK UPH EXO PRO TA SLV 19W XS	4D	258165S	BACK UPH EXO PRO TA RED 12W MD
4B	258109S	BACK UPH EXO PRO TA SLV 19W SH	4D	258166S	BACK UPH EXO PRO TA RED 12W TL
4B	258110S	BACK UPH EXO PRO TA SLV 19W MD	4D	258167S	BACK UPH EXO PRO TA RED 12W XT
4B	258111S	BACK UPH EXO PRO TA SLV 19W TL	4D	258168S	BACK UPH EXO PRO TA RED 13W XS
4B	258112S	BACK UPH EXO PRO TA SLV 19W XT	4D	258169S	BACK UPH EXO PRO TA RED 13W SH
4B	258113S	BACK UPH EXO PRO TA SLV 20W XS	4D	258170S	BACK UPH EXO PRO TA RED 13W MD
4B	258114S	BACK UPH EXO PRO TA SLV 20W SH	4D	258171S	BACK UPH EXO PRO TA RED 13W TL
4B	258115S	BACK UPH EXO PRO TA SLV 20W MD	4D	258172S	BACK UPH EXO PRO TA RED 13W XT
4B	258116S	BACK UPH EXO PRO TA SLV 20W TL	4D	258173S	BACK UPH EXO PRO TA RED 14W XS


Pos.	Item Number	Description	Pos.	Item Number	Description
4D	258174S	BACK UPH EXO PRO TA RED 14W SH	4E	258214S	BACK UPH EXO PRO TA BLU 13W SH
4D	258175S	BACK UPH EXO PRO TA RED 14W MD	4E	258215S	BACK UPH EXO PRO TA BLU 13W MD
4D	258176S	BACK UPH EXO PRO TA RED 14W TL	4E	258216S	BACK UPH EXO PRO TA BLU 13W TL
4D	258177S	BACK UPH EXO PRO TA RED 14W XT	4E	258217S	BACK UPH EXO PRO TA BLU 13W XT
4D	258178S	BACK UPH EXO PRO TA RED 15W XS	4E	258218S	BACK UPH EXO PRO TA BLU 14W XS
4D	258179S	BACK UPH EXO PRO TA RED 15W SH	4E	258219S	BACK UPH EXO PRO TA BLU 14W SH
4D	258180S	BACK UPH EXO PRO TA RED 15W MD	4E	258220S	BACK UPH EXO PRO TA BLU 14W MD
4D	258181S	BACK UPH EXO PRO TA RED 15W TL	4E	258221S	BACK UPH EXO PRO TA BLU 14W TL
4D	258182S	BACK UPH EXO PRO TA RED 15W XT	4E	258222S	BACK UPH EXO PRO TA BLU 14W XT
4D	258183S	BACK UPH EXO PRO TA RED 16W XS	4E	258223S	BACK UPH EXO PRO TA BLU 15W XS
4D	258184S	BACK UPH EXO PRO TA RED 16W SH	4E	258224S	BACK UPH EXO PRO TA BLU 15W SH
4D	258185S	BACK UPH EXO PRO TA RED 16W MD	4E	258225S	BACK UPH EXO PRO TA BLU 15W MD
4D	258186S	BACK UPH EXO PRO TA RED 16W TL	4E	258226S	BACK UPH EXO PRO TA BLU 15W TL
4D	258187S	BACK UPH EXO PRO TA RED 16W XT	4E	258227S	BACK UPH EXO PRO TA BLU 15W XT
4D	258188S	BACK UPH EXO PRO TA RED 17W XS	4E	258228S	BACK UPH EXO PRO TA BLU 16W XS
4D	258189S	BACK UPH EXO PRO TA RED 17W SH	4E	258229S	BACK UPH EXO PRO TA BLU 16W SH
4D	258190S	BACK UPH EXO PRO TA RED 17W MD	4E	258230S	BACK UPH EXO PRO TA BLU 16W MD
4D	258191S	BACK UPH EXO PRO TA RED 17W TL	4E	258231S	BACK UPH EXO PRO TA BLU 16W TL
4D	258192S	BACK UPH EXO PRO TA RED 17W XT	4E	258232S	BACK UPH EXO PRO TA BLU 16W XT
4D	258193S	BACK UPH EXO PRO TA RED 18W XS	4E	258233S	BACK UPH EXO PRO TA BLU 17W XS
4D	258194S	BACK UPH EXO PRO TA RED 18W SH	4E	258234S	BACK UPH EXO PRO TA BLU 17W SH
4D	258195S	BACK UPH EXO PRO TA RED 18W MD	4E	258235S	BACK UPH EXO PRO TA BLU 17W MD
4D	258196S	BACK UPH EXO PRO TA RED 18W TL	4E	258236S	BACK UPH EXO PRO TA BLU 17W TL
4D	258197S	BACK UPH EXO PRO TA RED 18W XT	4E	258237S	BACK UPH EXO PRO TA BLU 17W XT
4D	258198S	BACK UPH EXO PRO TA RED 19W XS	4E	258238S	BACK UPH EXO PRO TA BLU 18W XS
4D	258199S	BACK UPH EXO PRO TA RED 19W SH	4E	258239S	BACK UPH EXO PRO TA BLU 18W SH
4D	258200S	BACK UPH EXO PRO TA RED 19W MD	4E	258240S	BACK UPH EXO PRO TA BLU 18W MD
4D	258201S	BACK UPH EXO PRO TA RED 19W TL	4E	258241S	BACK UPH EXO PRO TA BLU 18W TL
4D	258202S	BACK UPH EXO PRO TA RED 19W XT	4E	258242S	BACK UPH EXO PRO TA BLU 18W XT
4D	258203S	BACK UPH EXO PRO TA RED 20W XS	4E	258243S	BACK UPH EXO PRO TA BLU 19W XS
4D	258204S	BACK UPH EXO PRO TA RED 20W SH	4E	258244S	BACK UPH EXO PRO TA BLU 19W SH
4D	258205S	BACK UPH EXO PRO TA RED 20W MD	4E	258245S	BACK UPH EXO PRO TA BLU 19W MD
4D	258206S	BACK UPH EXO PRO TA RED 20W TL	4E	258246S	BACK UPH EXO PRO TA BLU 19W TL
4D	258207S	BACK UPH EXO PRO TA RED 20W XT	4E	258247S	BACK UPH EXO PRO TA BLU 19W XT
4E	258208S	BACK UPH EXO PRO TA BLU 12W XS	4E	258248S	BACK UPH EXO PRO TA BLU 20W XS
4E	258209S	BACK UPH EXO PRO TA BLU 12W SH	4E	258249S	BACK UPH EXO PRO TA BLU 20W SH
4E	258210S	BACK UPH EXO PRO TA BLU 12W MD	4E	258250S	BACK UPH EXO PRO TA BLU 20W MD
4E	258211S	BACK UPH EXO PRO TA BLU 12W TL	4E	258251S	BACK UPH EXO PRO TA BLU 20W TL
4E	258212S	BACK UPH EXO PRO TA BLU 12W XT	4E	258252S	BACK UPH EXO PRO TA BLU 20W XT
4E	258213S	BACK UPH EXO PRO TA BLU 13W XS			

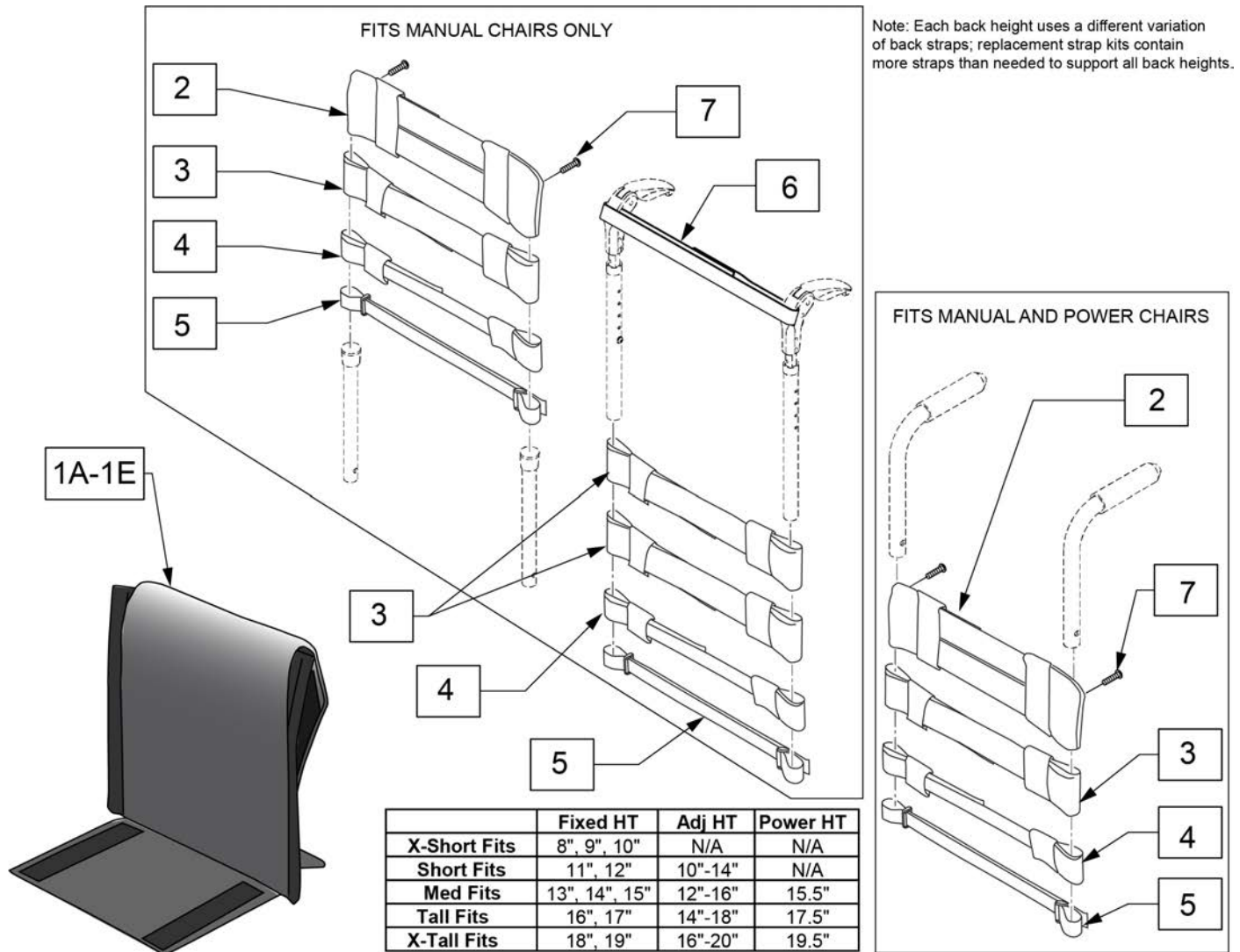


Pos.	Item Number	Description	Remarks
2	113712S	TOP CAP, STD, 11 -12"	Used for Omit Push Handle & Integral Push Handle only
2	113713S	TOP CAP, STD, 13 -14"	Used for Omit Push Handle & Integral Push Handle only
2	113714S	TOP CAP, STD, 15 -16"	Used for Omit Push Handle & Integral Push Handle only
2	113715S	TOP CAP, STD, 17 -18"	Used for Omit Push Handle & Integral Push Handle only
2	113716S	TOP CAP, STD, 19 -20"	Used for Omit Push Handle & Integral Push Handle only
2	119083S	TOP CAP, STD, 21-22	Used for Omit Push Handle & Integral Push Handle only
3	113727S	STRAP PADDED 2", 11 -12"	
3	113728S	STRAP PADDED 2", 13 -14"	
3	113729S	STRAP PADDED 2", 15 -16"	
3	113730S	STRAP PADDED 2", 17 -18"	
3	113731S	STRAP PADDED 2", 19 -20"	
3	119081S	STRAP PDDDED 2IN, 21-22	
4	113732S	STRAP PADDED 1", 11 -12"	
4	113733S	STRAP PADDED 1", 13 -14"	
4	113734S	STRAP PADDED 1", 15 -16"	
4	113735S	STRAP PADDED 1", 17 -18"	
4	113736S	STRAP PADDED 1", 19-20"	
4	119082S	STRAP PADDED 1IN, 21-22	

Pos.	Item Number	Description	Remarks
5	113717S	POSITION STRAP 1", 11 -12	
5	113718S	POSITION STRAP 1", 13 -14	
5	113719S	POSITION STRAP 1", 15 -16	
5	113720S	POSITION STRAP 1", 17 -18	
5	113721S	POSITION STRAP 1", 19 -20	
5	119080S	POSITION STRAP 1IN, 21-22	
6	052435-001S	UPH,VNTD ADJ.,FPH STRP,12-13"W	Fold Down Push Handles only
6	052435-002S	UPH,VNTD ADJ.,FPH STRP,14-15"W	Fold Down Push Handles only
6	052435-003S	UPH,VNTD ADJ.,FPH STRP,16-17"W	Fold Down Push Handles only
6	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold Down Push Handles only
6	052435-005S	UPH,VNTD ADJ.,FPH STRP,21-22"W	Fold Down Push Handles only
7	107384S	PAN HD PHIL 10/32 X 5/8 ROHS	
2-7	114879	ADJ UPH 11-12W STRAP KIT	
2-7	114880	ADJ UPH 13-14W STRAP KIT	
2-7	114881	ADJ UPH 15-16W STRAP KIT	
2-7	114882	ADJ UPH 17-18W STRAP KIT	
2-7	114883	ADJ UPH 19-20W STRAP KIT	
2-7	163951	ADJ UPH 21-22W STRAP KIT	
1A,2-7	113321-001	ADJ UP 12W X-SHT ASM	
1B,2-7	113321-002	ADJ UP 12W SHT ASM	
1C,2-7	113321-003	ADJ UP 12W MED ASM	
1D,2-7	113321-004	ADJ UP 12W TAL ASM	
1E,2-7	113321-005	ADJ UP 12W X-TAL ASM	
1A,2-7	113321-006	ADJ UP 13W X-SHT ASM	
1B,2-7	113321-007	ADJ UP 13W SHT ASM	
1C,2-7	113321-008	ADJ UP 13W MED ASM	
1D,2-7	113321-009	ADJ UP 13W TAL ASM	
1E,2-7	113321-010	ADJ UP 13W X-TAL ASM	
1A,2-7	113321-011	ADJ UP 14W X-SHT ASM	
1B,2-7	113321-012	ADJ UP 14W SHT ASM	
1C,2-7	113321-013	ADJ UP 14W MED ASM	
1D,2-7	113321-014	ADJ UP 14W TAL ASM	
1E,2-7	113321-015	ADJ UP 14W X-TAL ASM	
1A,2-7	113321-016	ADJ UP 15W X-SHT ASM	
1B,2-7	113321-017	ADJ UP 15W SHT ASM	
1C,2-7	113321-018	ADJ UP 15W MED ASM	
1D,2-7	113321-019	ADJ UP 15W TAL ASM	
1E,2-7	113321-020	ADJ UP 15W X-TAL ASM	
1A,2-7	113321-021	ADJ UP 16W X-SHT ASM	
1B,2-7	113321-022	ADJ UP 16W SHT ASM	
1C,2-7	113321-023	ADJ UP 16W MED ASM	
1D,2-7	113321-024	ADJ UP 16W TAL ASM	
1E,2-7	113321-025	ADJ UP 16W X-TAL ASM	
1A,2-7	113321-026	ADJ UP 17W X-SHT ASM	
1B,2-7	113321-027	ADJ UP 17W SHT ASM	
1C,2-7	113321-028	ADJ UP 17W MED ASM	
1D,2-7	113321-029	ADJ UP 17W TAL ASM	
1E,2-7	113321-030	ADJ UP 17W X-TAL ASM	
1A,2-7	113321-031	ADJ UP 18W X-SHT ASM	
1B,2-7	113321-032	ADJ UP 18W SHT ASM	
1C,2-7	113321-033	ADJ UP 18W MED ASM	
1D,2-7	113321-034	ADJ UP 18W TAL ASM	
1E,2-7	113321-035	ADJ UP 18W X-TAL ASM	
1A,2-7	113321-036	ADJ UP 19W X-SHT ASM	
1B,2-7	113321-037	ADJ UP 19W SHT ASM	
1C,2-7	113321-038	ADJ UP 19W MED ASM	
1D,2-7	113321-039	ADJ UP 19W TAL ASM	


Pos.	Item Number	Description	Remarks
1E,2-7	113321-040	ADJ UP 19W X-TAL ASM	
1A,2-7	113321-041	ADJ UP 20W X-SHT ASM	
1B,2-7	113321-042	ADJ UP 20W SHT ASM	
1C,2-7	113321-043	ADJ UP 20W MED ASM	
1D,2-7	113321-044	ADJ UP 20W TAL ASM	
1E,2-7	113321-045	ADJ UP 20W X-TAL ASM	
1A,2-7	163953	ADJ UPH 21 W X-SHT ASM	
1B,2-7	163955	ADJ UPH 21 W SHT ASM	
1C,2-7	163957	ADJ UPH 21 W MED ASM	
1D,2-7	163959	ADJ UPH 21 W TAL ASM	
1E,2-7	163961	ADJ UPH 21 W X-TAL ASM	
1A,2-7	163963	ADJ UPH 22 W X-SHT ASM	
1B,2-7	163965	ADJ UPH 22 W SHT ASM	
1C,2-7	163967	ADJ UPH 22 W MED ASM	
1D,2-7	163969	ADJ UPH 22 W TAL ASM	
1E,2-7	163971	ADJ UPH 22 W X-TAL ASM	

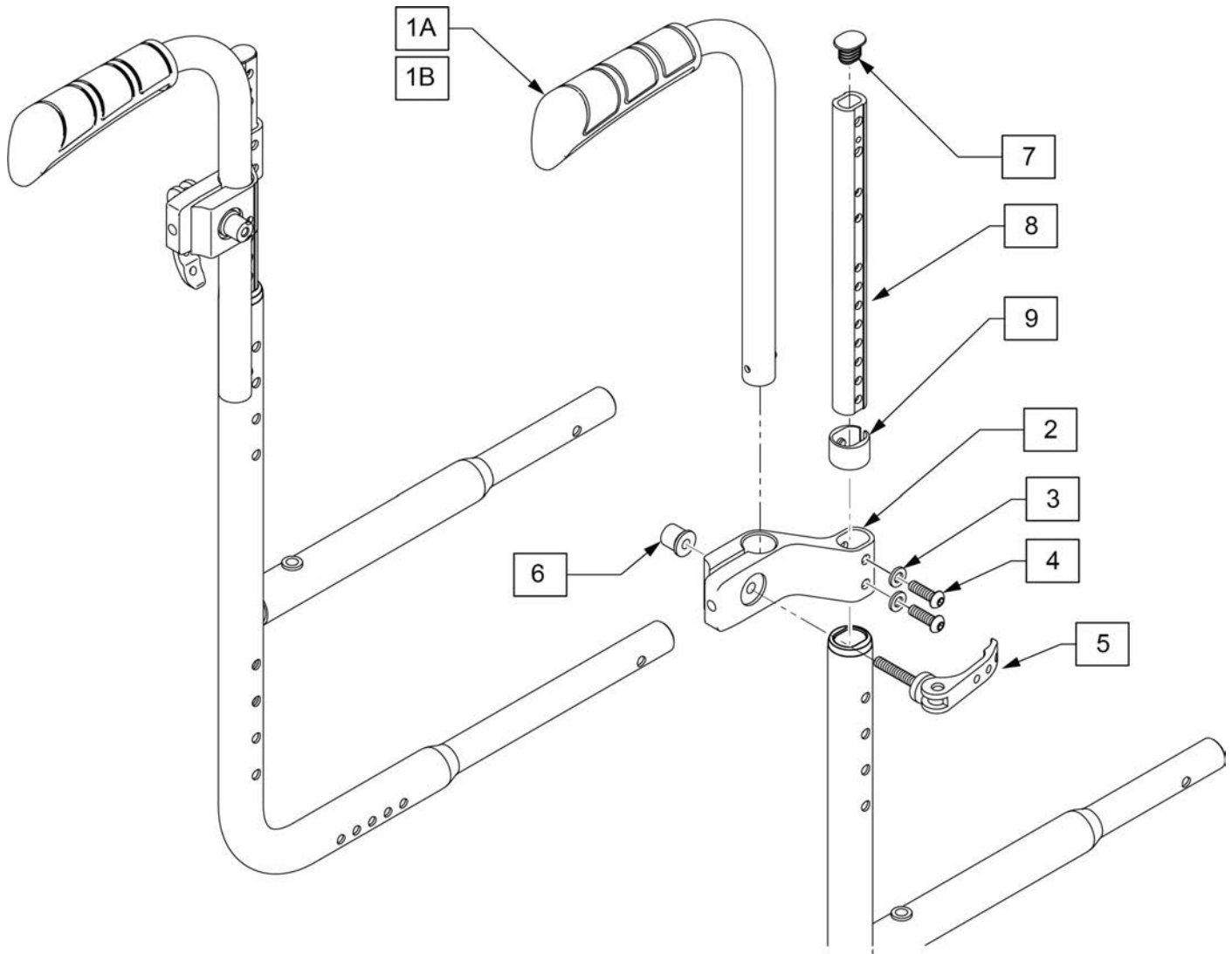


Pos.	Item Number	Description	Remarks
1A	257006	HT ADJ PH TUBE ASSY 10-14 IN	
1B	257007	HT ADJ PH TUBE ASSY 14-20 IN	
2-9	170320	HT ADJ PH CLAMP ASSM	1 side only, left or right compatible

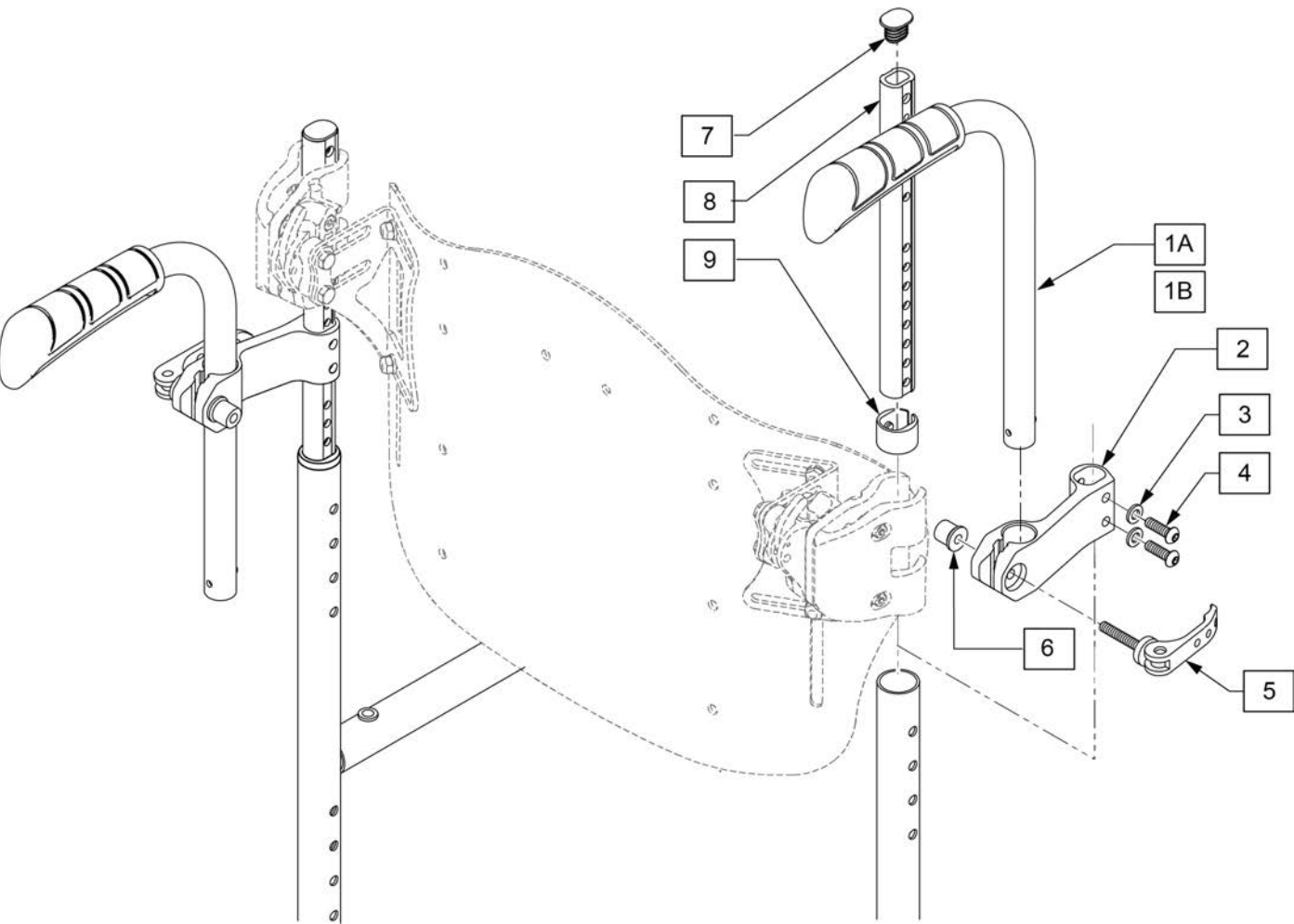


Pos.	Item Number	Description	Remarks
1A	257006	HT ADJ PH TUBE ASSY 10-14 IN	
1B	257007	HT ADJ PH TUBE ASSY 14-20 IN	
2-6	170321	HT ADJ PH CLAMP ASSM JAY	1 side only, left or right compatible

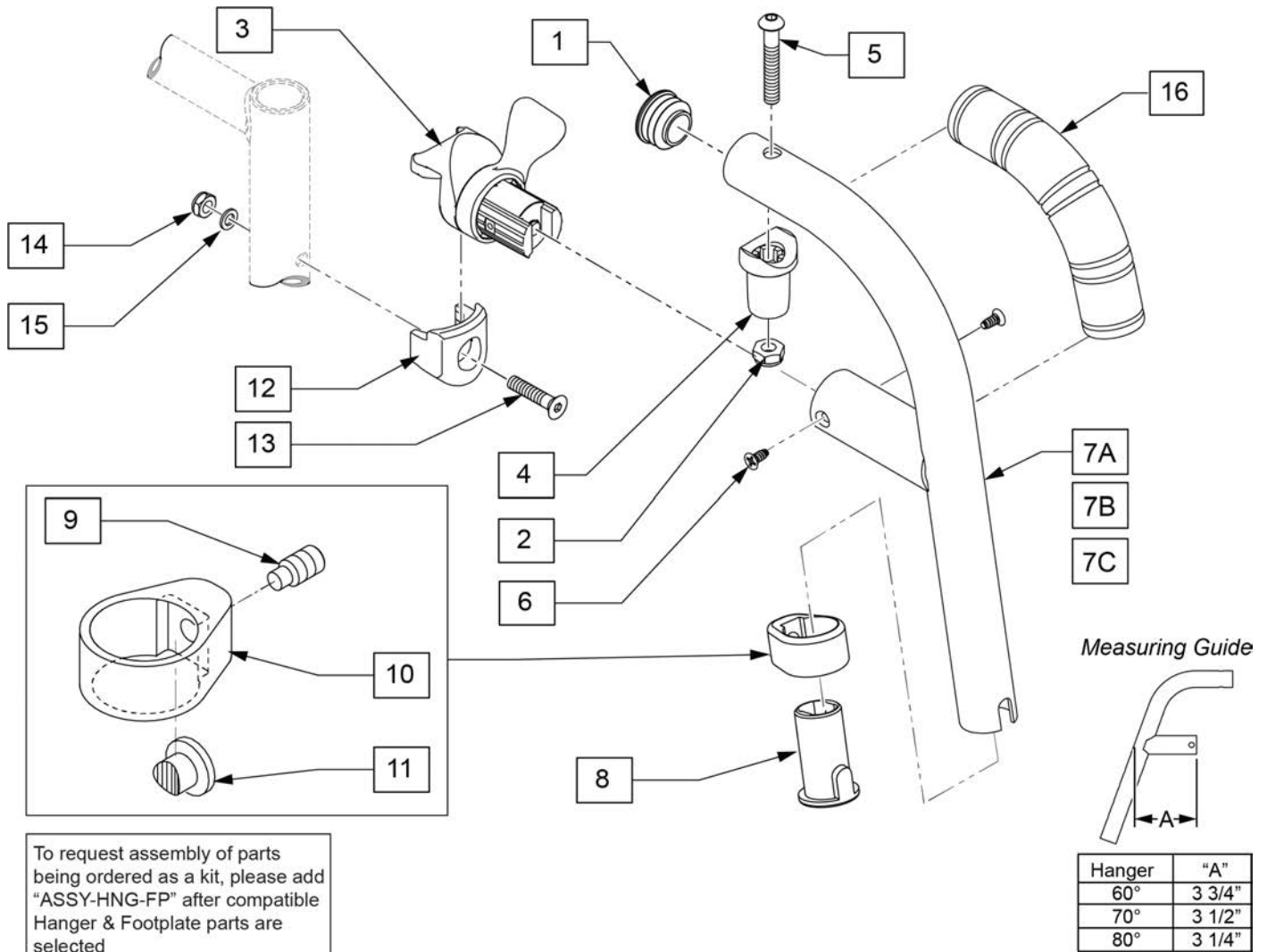

Pos.	Item Number	Description	Remarks
1	561062S	1" INSERT PLUG .065 RIBS	
8	104160S	BUSHING FOOTREST	
13	103500-010	FHCSM6X1X35FULLNP	
14	104381	NUT HEX NYLOCK M6X1.0	
15	104414S	WAVE WASHER 6 MM	
1,2,4,5	119518S	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-7A,8-11	169876S	60 DEG SWING I/O EXT HANGER RH	Right
1-7A,8-11	169877S	60 DEG SWING I/O EXT HANGER LH	Left
1-7B,8-11	169878S	70 DEG SWING I/O EXT HANGER RH	Right
1-7B,8-11	169879S	70 DEG SWING I/O EXT HANGER LH	Left
1-7C,8-11	169880S	80 DEG SWING I/O EXT HANGER RH	Right
1-7C,8-11	169881S	80 DEG SWING I/O EXT HANGER LH	Left
8-11	170101	FOOTREST CLAMP ASSM PAIR	
12-15	119514S	LATCH RECEIVER KIT	
16	71201061	Fixed Front touch point	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

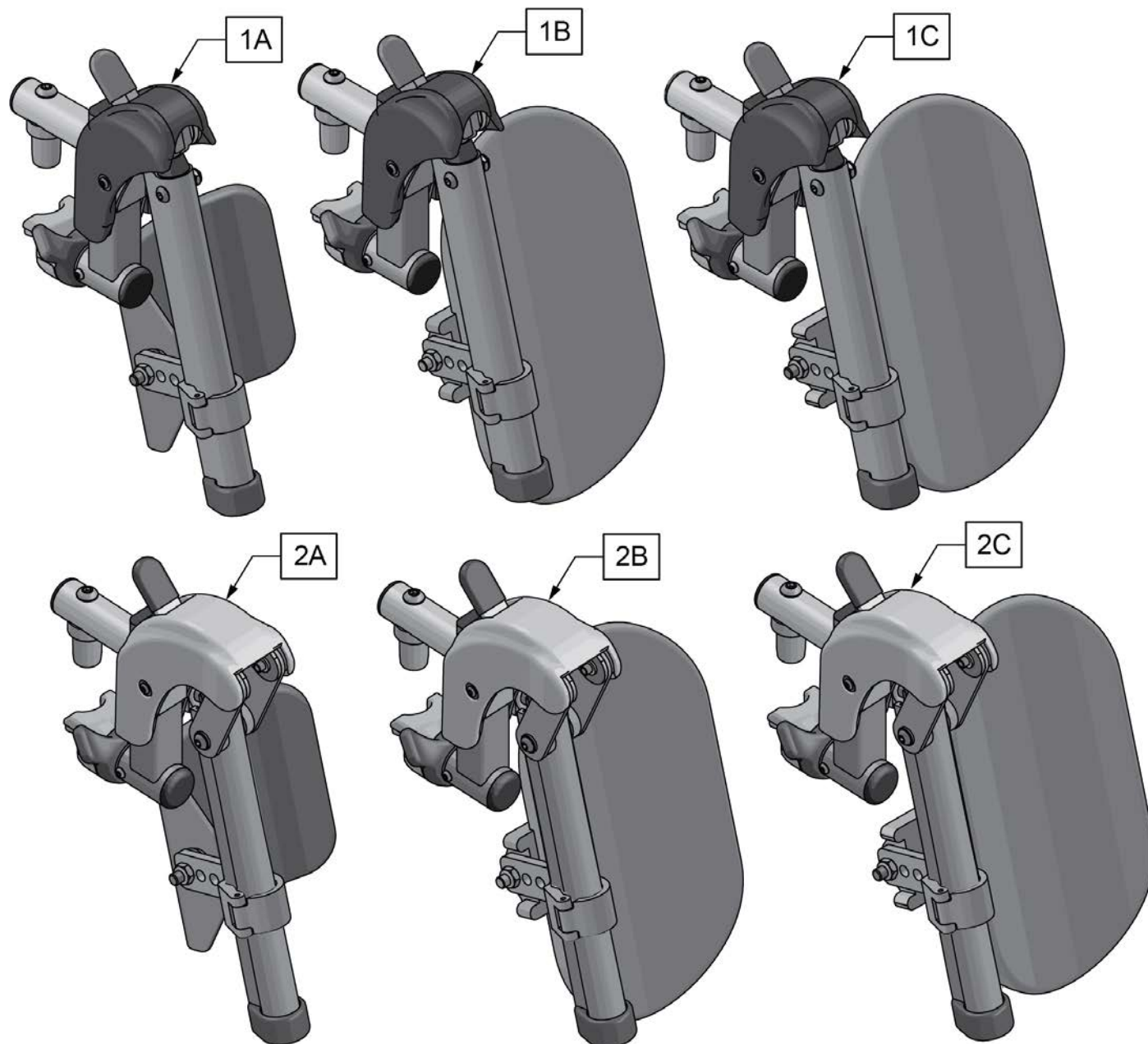


Pos.	Item Number	Description	Remarks
1A	161233S	SWNG I/O ELR ASM 14-15 W/PAD	Pair
1B	161234S	SWNG I/O ELR ASM 16-17 W/PAD	Pair
1C	161235S	SWNG I/O ELR ASM 18-20 W/PAD	Pair
2A	161236S	SWNG I/O ALR ASM 14-15 W/PAD	Pair
2B	161237S	SWNG I/O ALR ASM 16-17 W/PAD	Pair
2C	161238S	SWNG I/O ALR ASM 18-20 W/PAD	Pair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

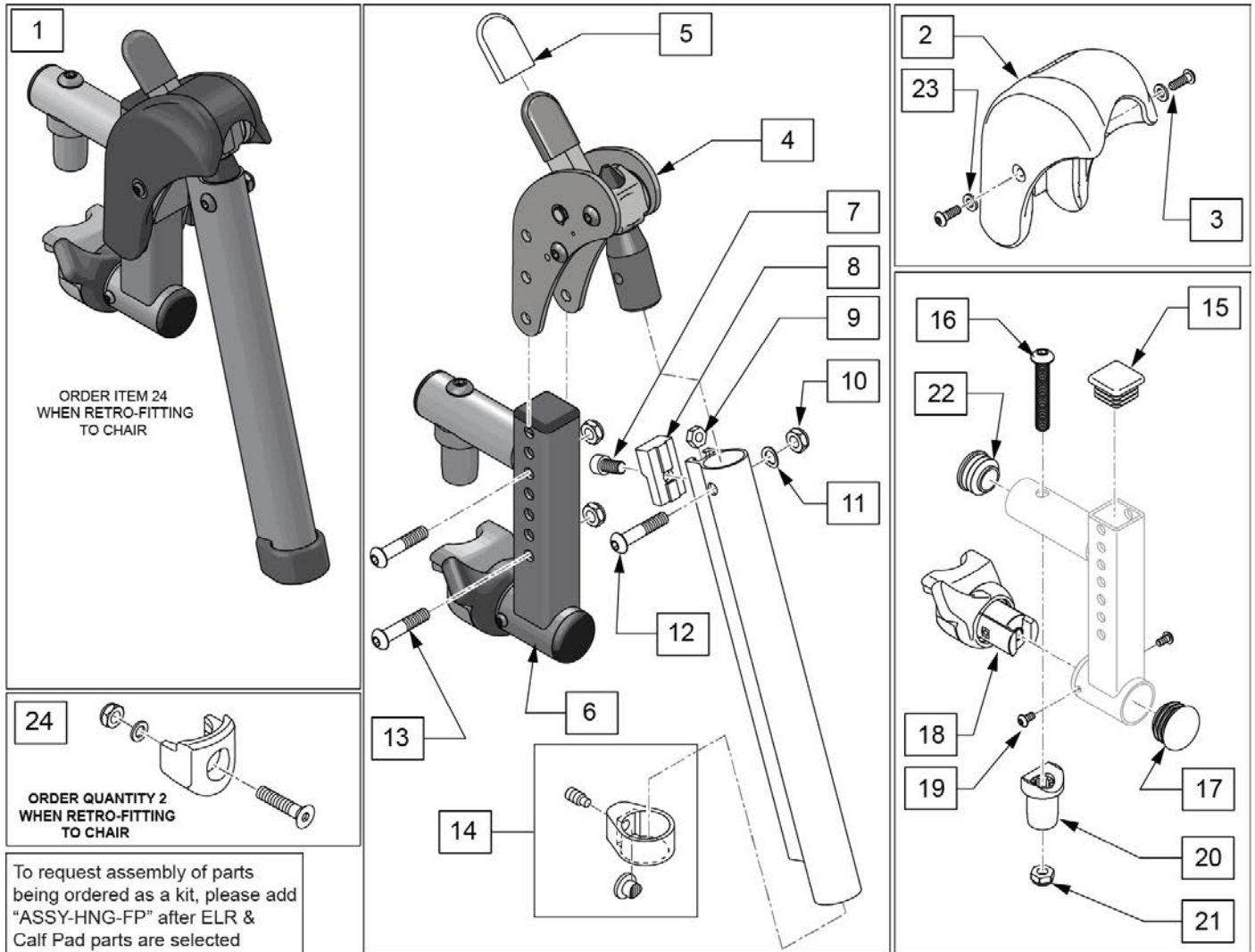


Pos.	Item Number	Description	Remarks
1	161239S	SWNG I/O ELR ASSM RH	Right
1	161240S	SWNG I/O ELR ASSM LH	Left
1	161241S	SWNG I/O ELR ASSM PR	Pair
2,3,23	129640	ELR SHROUD KIT	Kit w/ 2 screws
3	127135S	BHCS 8-32 X 1/2 BLK	
4	126992	HEMI ELR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188S	RECEIVER ASSY ELR ALR RH	
6	134186S	RECEIVER ASSY ELR ALR LH	
7	107350	SHCS 1/4-28 x 1/2	
8	127129	ANGLE STOP ELR	
9	107249	NUT HEX THIN 1/4-28	
10	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
11	104414S	WAVE WASHER 6 MM	
12	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
13	108623	BHCS 1/4-28 x 1-1/2	
14	119495	EXTENSION CLAMP KIT	
15	010615	SQUARE END CAP CNTR LEG	
16	226217	5/16-18 X 2" BH C/S BLK	

Pos.	Item Number	Description	Remarks
17	561062S	1" INSERT PLUG .065 RIBS	
18,19	161242S	ELR/ALR HANGER LATCH KIT RH	Right
18,19	161243S	ELR/ALR HANGER LATCH KIT LH	Left
20	561197	PLASTIC PIVOT SADDLE	
21	202102	5/16-18 NYLOCK NUT THIN	
22	560110S	ENDPLUG BACKREST TNT	
23	113282	WASHER FLAT AN #10 ROHS	
24	119514S	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

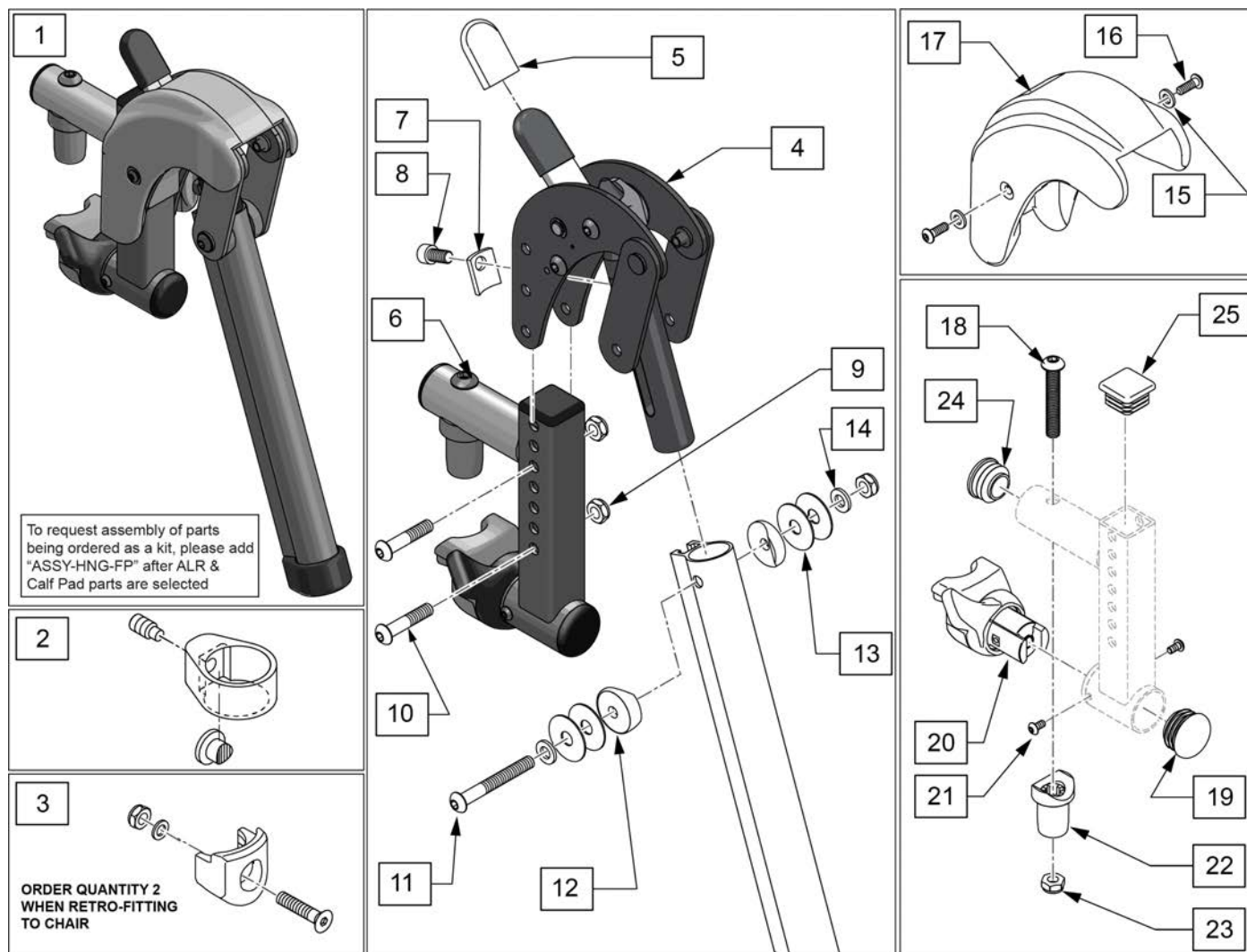

Pos.	Item Number	Description	Remarks
1	161244S	SWNG I/O ALR ASSM RH KIT	Right
1	161245S	SWNG I/O ALR ASSM LH KIT	Left
1	161246S	SWNG I/O ALR ASSM PR KIT	Pair
2	119495	EXTENSION CLAMP KIT	
3	119514S	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
4	127153	HEMI ALR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188S	RECEIVER ASSY ELR ALR RH	
6	134186S	RECEIVER ASSY ELR ALR LH	
7	127129	ANGLE STOP ELR	
8	107350	SHCS 1/4-28 x 1/2	
9	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
10	108623	BHCS 1/4-28 x 1-1/2	
11	106964	BHCS 1/4-28 X 2	
12	561086S	1" HALF SADDLE 1/4" HOLE	
13	113268S	WASHER NYLON 1IN OD X .312 ID	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	113282	WASHER FLAT AN #10 ROHS	
16	127135	BHCS 8-32 X 1/2 BLK	


Pos.	Item Number	Description	Remarks
15-17	129633	ALR SHROUD KIT	1 side
18	226217	5/16-18 X 2" BH C/S BLK	
19	560110S	ENDPLUG BACKREST TNT	
20,21	161242S	ELR/ALR HANGER LATCH KIT RH	Right
20,21	161243S	ELR/ALR HANGER LATCH KIT LH	Left
22	561197	PLASTIC PIVOT SADDLE	
23	202102	5/16-18 NYLOCK NUT THIN	
24	561062S	1" INSERT PLUG .065 RIBS	
25	010615	SQUARE END CAP CNTR LEG	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

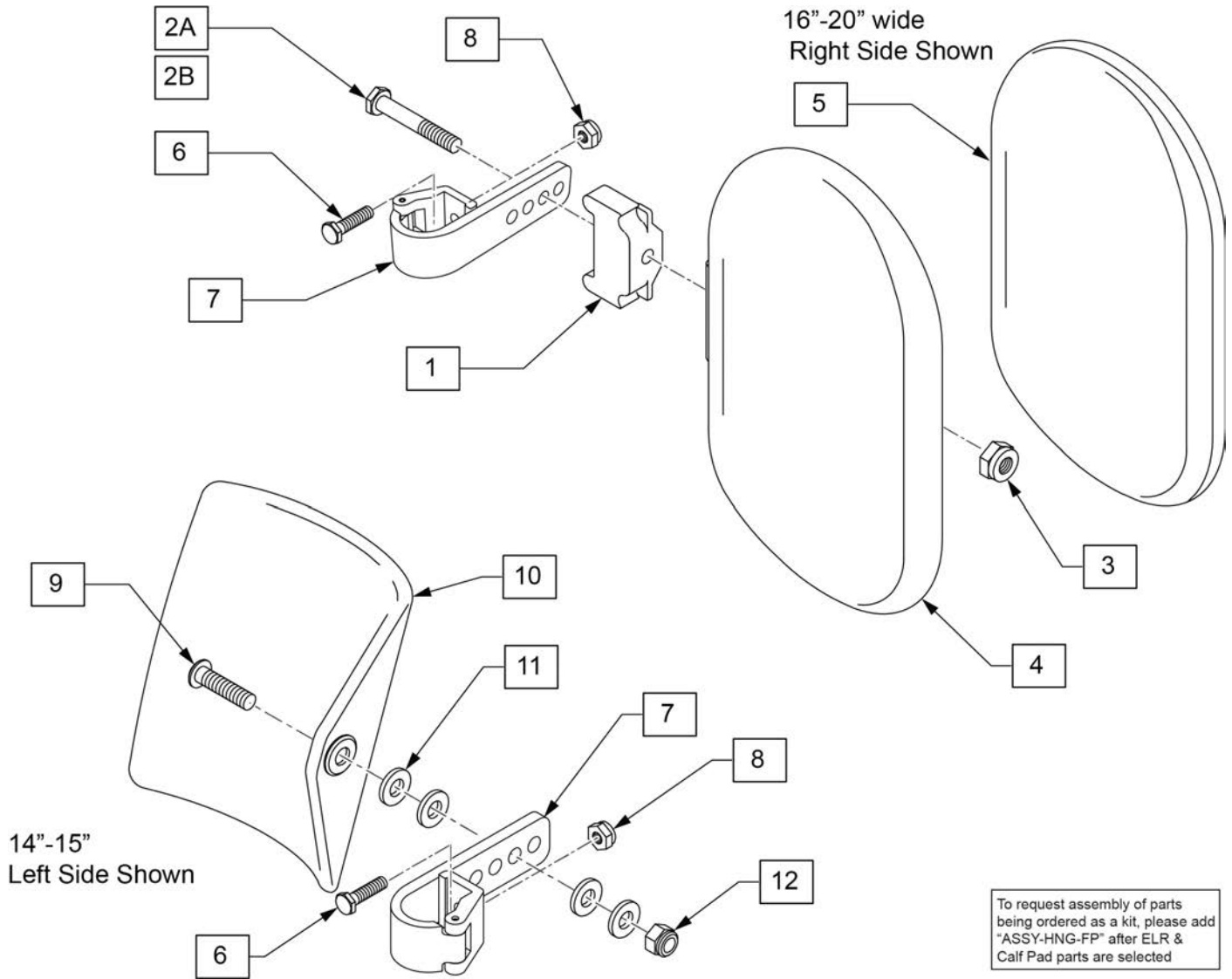


Pos.	Item Number	Description	Remarks
1	560199	ELR CALF HINGE SPACER	18"-20" wide only
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	106936S	NUT NYLOCK 5/16-18 ROHS	
4	560108	CALF PAD MOLDED 16"-20"	
5	751045	CALF PAD COVER	
6	107013	HHCS 1/4-28 X 5/8 ROHS	
7	881413	CALF PAD BRACKET ASSEMBLY	
8	106934	NUT NYLOCK 1/4-28 ROHS	
9	222108	5/16-18 X 1 1/4 BH C/S	
10	755014	CALF PAD TROUGH STYLE 14"/16"	
11	561016	1" WASHER 5/16" HOLE	
12	106937	NUT NYLOCK THIN 5/16-18	
2A,3-8	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-8	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	
6-12	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2" = Footrest Width
 "V" Hanger (Seat Width) - 4" = Footrest Width
 Offset Hanger Receiver (Seat Width) + 2" = Footrest Width

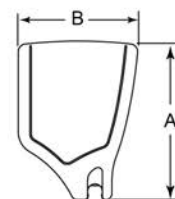FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION (FOOTREST WIDTH)	"A"	"B"
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"

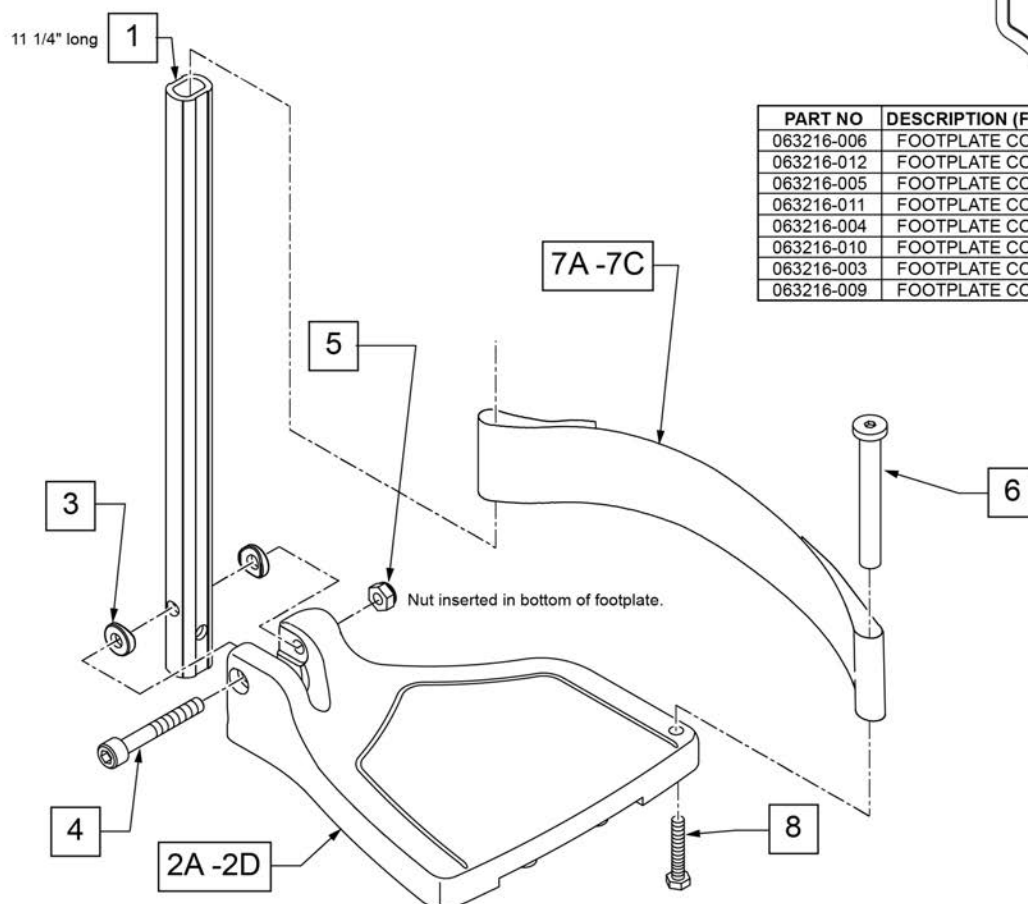

Pos.	Item Number	Description	Remarks
1,3-5	120662	LNG STRT EXT COMP HDWR	
2A	063216-006S	FOOTPLATE COMPOSITE 157 R	
2A	063216-012S	FOOTPLATE COMPOSITE 157 L	
2B	063216-005S	FOOTPLATE COMPOSITE 182 R	
2B	063216-011S	FOOTPLATE COMPOSITE 182 L	
2C	063216-004S	FOOTPLATE COMPOSITE 207 R	
2C	063216-010S	FOOTPLATE COMPOSITE 207 L	
2D	063216-003S	FOOTPLATE COMPOSITE 232 R	
2D	063216-009S	FOOTPLATE COMPOSITE 232 L	
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40, PART, NP	
5	132626	NUT AUTOMATION 1/4-28 ROHS	
6	104328	POST HEEL LOOP	
7A	104380-002S	ADJ FOOTREST STRAP 15-16W	
7B	104380-003	ADJ FOOTREST STRAP 17-18W	
7C	104380-004	ADJ FOOTREST STRAP 19-20W	
8	102268-001	HHCS M6 X 1 X 25 BLK	
1,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	
1,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	


Pos.	Item Number	Description	Remarks
1,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	
1,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	
1,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	
1,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	
1,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	
1,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.

Note: If using Short Ext Tube & Hemi Hanger with Composite Angle Adjustable Footplate use 1D (straight).

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

FOOTPLATE MEASURING GUIDE

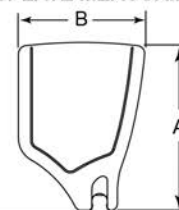


PART NO	DESCRIPTION	DIM - A-	DIM - B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

FOOTPLATE WIDTH GUIDE

	Footrest Width
2A	15"
2B	16"
2C	17"
2D	18"
2E	19"
2F	20"
2G	21" & 22"

FOOTPLATE STRAP GUIDE

	Footrest Width
10A	15"
10B	16"
10C	17"-18"
10D	19"-22"

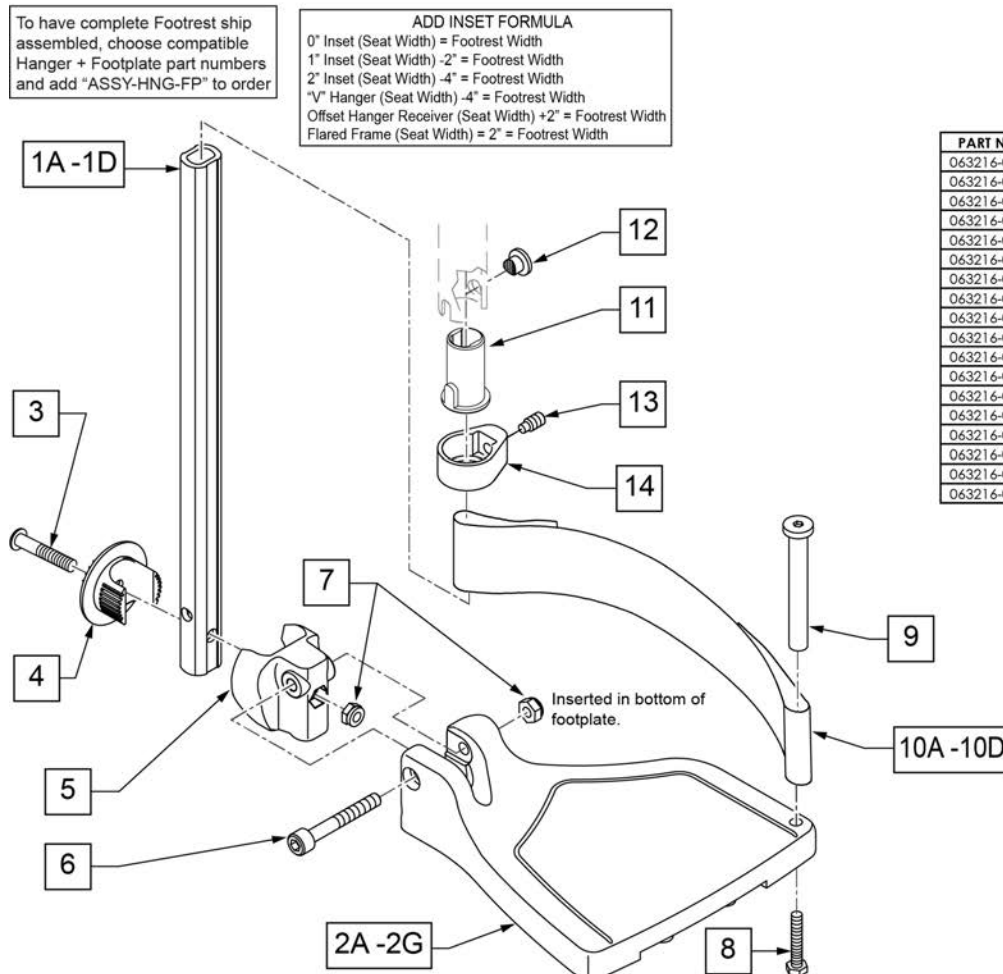


ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2" = Footrest Width
 2" Inset (Seat Width) - 4" = Footrest Width
 "V" Hanger (Seat Width) - 4" = Footrest Width
 Offset Hanger Receiver (Seat Width) + 2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width


Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	
1B,6,7	120661	MED STRT EXT COMP HDWR	
1C,6,7	120662	LNG STRT EXT COMP HDWR	
1D	128079S	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	063216-006S	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2A	063216-012S	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015S	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width
2B	063216-018S	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005S	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011S	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014S	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017S	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004S	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010S	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013S	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016S	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003S	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009S	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width

Pos.	Item Number	Description	Remarks
3	108233	BHCS M6 X 1 X 40	
4	054242S	ANGLE COVER	
5	054241S	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	132626	NUT AUTOMATION M6 X 1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001S	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002S	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160S	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 21&22 W- SHORT
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 21&22 W- SHORT
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 17 W- MED
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 21&22 W- MED
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 20 W- MED
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 21&22 W- MED
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 21&22 W- LONG
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 21&22 W- LONG
1D,2A,3-10A	129710	CAA FOOT KIT SHT HNGR 15W R	Right-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129712	CAA FOOT KIT SHT HNGR 16W R	Right-Hemi-16W; add Ext behind the Hemi


Pos.	Item Number	Description	Remarks
1D,2C,3-10B	129714	CAA FOOT KIT SHT HNGR 17W R	Right-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129716	CAA FOOT KIT SHT HNGR 18W R	Right-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129718	CAA FOOT KIT SHT HNGR 19W R	Right-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129720	CAA FOOT KIT SHT HNGR 20W R	Right-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129722	CAA FOOT KIT SHT HNGR 22W R	Right-Hemi-21&22W; add Ext behind the Hemi
1D,2A,3-10A	129711	CAA FOOT KIT SHT HNGR 15W L	Left-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129713	CAA FOOT KIT SHT HNGR 16W L	Left-Hemi-16W; add Ext behind the Hemi
1D,2C,3-10B	129715	CAA FOOT KIT SHT HNGR 17W L	Left-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129717	CAA FOOT KIT SHT HNGR 18W L	Left-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129719	CAA FOOT KIT SHT HNGR 19W L	Left-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129721	CAA FOOT KIT SHT HNGR 20W L	Left-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129723	CAA FOOT KIT SHT HNGR 22W L	Left-Hemi-21&22W; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

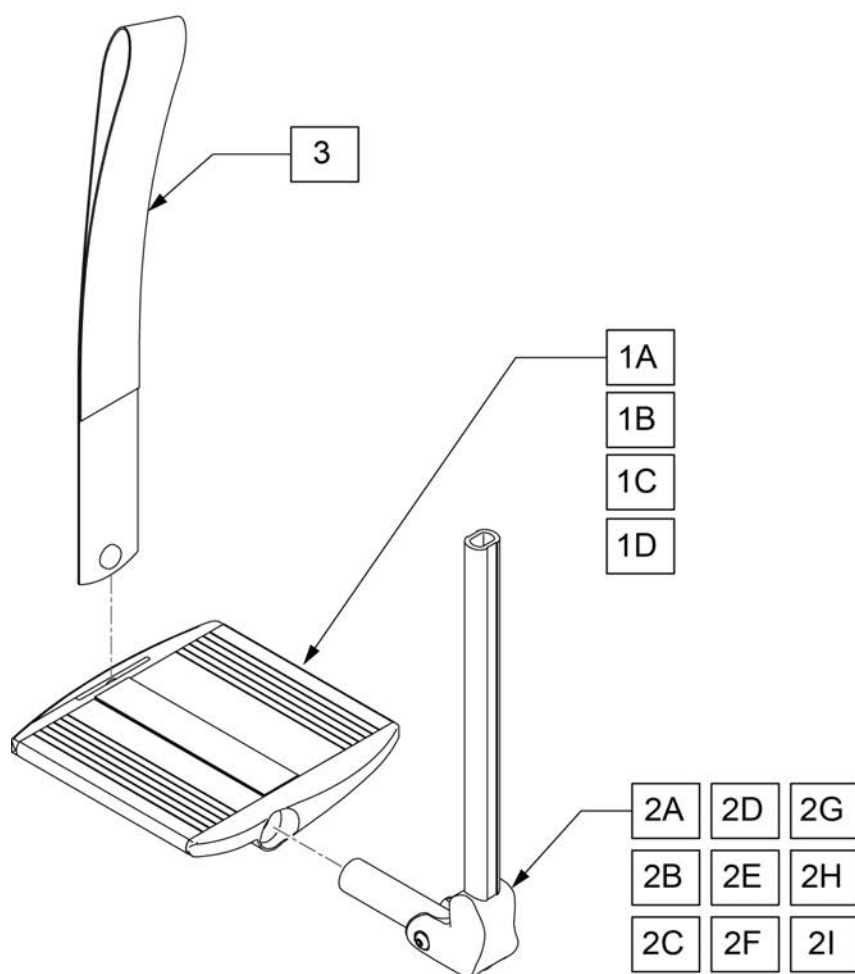

MEASURING GUIDE

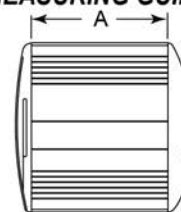

Footrest Width	Footplate Size	Dim A (inches)
13-14"	75mm	3
15-16"	100mm	4
17-18"	125mm	5
19-20"	150mm	6

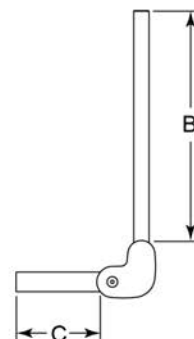

Ext. Tube	Dim B (inches)
Short	5.6
Medium	7.6
Long	8.7


Footrest Width	Footrest Pivot Size	Dim C (inches)
13-14"	125mm	3.1
15-18"	150mm	4.1
19-20"	200mm	6

Pos.	Item Number	Description	Remarks
1A	71205015-003S	DIVIDED FOOT PLATE LEFT 75MM	13-14
1A	71205015-004S	DIVIDED FOOT PLATE RIGHT 75MM	13-14
1B	71205015-005S	DIVIDED FOOT PLATE LEFT 100MM	15-16
1B	71205015-006S	DIVIDED FOOT PLATE RIGHT 100MM	15-16
1C	71205015-007S	DIVIDED FOOT PLATE LEFT 125MM	17-18
1C	71205015-008S	DIVIDED FOOT PLATE RIGHT 125MM	17-18
1D	71205015-009	DIVIDED FOOT PLATE LEFT 150MM	19-20
1D	71205015-010	DIVIDED FOOT PLATE RIGHT 150MM	19-20
2A		Part	13-14,Short
2B		Part	15-18,Short
2C		Part	19-20,Short
2D		Part	13-14,Med
2E		Part	15-18,Med
2F		Part	19-20,Med
2G	71205014-003	Footrest Pivot Assy 125mm	13-14,Long
2H	71205014-002	Footrest Pivot Assy 150mm	15-18,Long
2I	71205014-004	FOOTREST PIVOT ASSY 200MM	19-20,Long
1A,2A		PAINTED FTPLT 13-14IN LH SHT	Short


Pos.	Item Number	Description	Remarks
1A,2A		PAINTED FTPLT 13-14IN RH SHT	Short
1B,2B		PAINTED FTPLT 15-16IN LH SHT	Short
1B,2B		PAINTED FTPLT 15-16IN RH SHT	Short
1C,2B		PAINTED FTPLT 17-18IN LH SHT	Short
1C,2B		PAINTED FTPLT 17-18IN RH SHT	Short
1D,2C		PAINTED FTPLT 19-20IN LH SHT	Short
1D,2C		PAINTED FTPLT 19-20IN RH SHT	Short
1A,2D		PAINTED FTPLT 13-14IN LH MED	Medium
1A,2D		PAINTED FTPLT 13-14IN RH MED	Medium
1B,2E		PAINTED FTPLT 15-16IN LH MED	Medium
1B,2E		PAINTED FTPLT 15-16IN RH MED	Medium
1C,2E		PAINTED FTPLT 17-18IN LH MED	Medium
1C,2E		PAINTED FTPLT 17-18IN RH MED	Medium
1D,2F		PAINTED FTPLT 19-20IN LH MED	Medium
1D,2F		PAINTED FTPLT 19-20IN RH MED	Medium
1A,2G	169882S	PAINTED FOOTPLATE 13-14IN LH	Long
1A,2G	169883S	PAINTED FOOTPLATE 13-14IN RH	Long
1B,2H	169884S	PAINTED FOOTPLATE 15-16IN LH	Long
1B,2H	169885S	PAINTED FOOTPLATE 15-16IN RH	Long
1C,2H	169886S	PAINTED FOOTPLATE 17-18IN LH	Long
1C,2H	169887S	PAINTED FOOTPLATE 17-18IN RH	Long
1D,2I	169888S	PAINTED FOOTPLATE 19-20IN LH	Long
1D,2I	169889S	PAINTED FOOTPLATE 19-20IN RH	Long
3	760824031	CALF STRAP DIAGONAL	

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.
 Note: If using Short Ext Tube & Hemi Hanger with Adult Angle Adjustable Footplate use 1D (straight) .

21 EXT MNT HARDWARE 1 SIDE			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT AUTOMATION M6 X 1.0 ROHS	1
4		WASHER FLAT THIN AN 1/4	2
5		HANGER EXTENSION SADDLE	1
6		3/4" SADDLE 1" OD	1
8		WASHER FLAT SAE 1/4 ROHS	1
9		SHCS M6X1X45 12MM PART THD	1

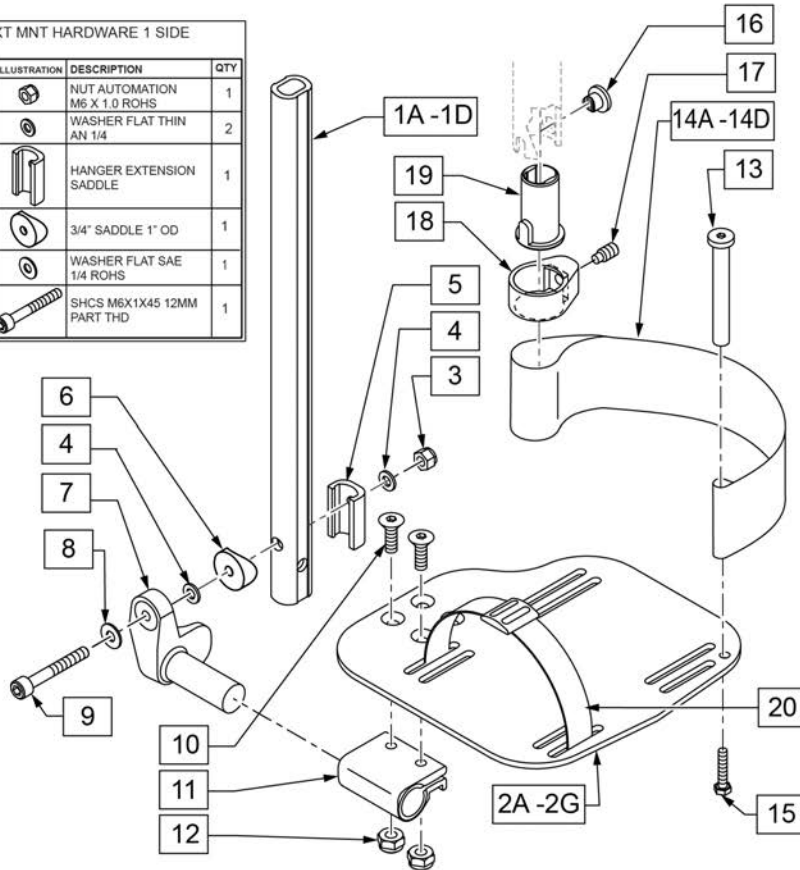

MEASURING GUIDE

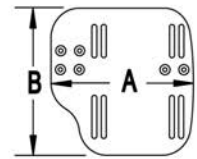

CHAIR WIDTH	"A"	"B"
10"-11"	3 1/2"	7 1/4"
12"-13"	4 1/2"	7 3/4"
14"-15"	5 1/2"	7 3/4"
16"-17"	6 1/2"	7 3/4"
18"-20"	7 1/2"	7 3/4"

To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

ADD INSET FORMULA


0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2" = Footrest Width
 2" Inset (Seat Width) - 4" = Footrest Width
 "V" Hanger (Seat Width) - 4" = Footrest Width
 Offset Hanger Receiver (Seat Width) + 2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

Pos.	Item Number	Description	Remarks
1A,3-6,8,9	120670	SHT STRT EXT AAA HDWR	
1B,3-6,8,9	120671	MED STRT EXT AAA HDWR	
1C,3-6,8,9	120672	LNG STRT EXT AAA HDWR	
1D	128079S	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	132626	NUT AUTOMATION M6 X 1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	561055	3/4" SADDLE 1" OD	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	

Pos.	Item Number	Description	Remarks
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001S	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002S	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
16-19		FOOTREST CLAMP ASSM	new metric
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
21	120659	EXT MNT HARDWARE 1 SIDE	
>>>>		60,70,80 STD HANGERS - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 18-22" W- SHORT
>>>>		60,70,80 STD HANGERS - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 18-22" W- SHORT
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 18-22" W- LONG
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 16-17" W- LONG
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 18-22" W- LONG
>>>>		60&70 HEMI HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	129756	AAA FOOT KIT SHT HNGR 12W R	Right- 12-13" W- SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129758	AAA FOOT KIT SHT HNGR 14W R	Right- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129760	AAA FOOT KIT SHT HNGR 16W R	Right- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129762	AAA FOOT KIT SHT HNGR 18W R	Right- 18-22" W- SHORT; add Ext behind the Hemi
>>>>		60&70 HEMI HANGERS - SHORT EXT -LEFT	
1D,2B,3-14A,15	129757	AAA FOOT KIT SHT HNGR 12W L	Left- 12-13" W-SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129759	AAA FOOT KIT SHT HNGR 14W L	Left- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129761	AAA FOOT KIT SHT HNGR 16W L	Left- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129763	AAA FOOT KIT SHT HNGR 18W L	Left- 18-22" W- SHORT; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit


To request assembly of parts being ordered as a kit, please add "ASSY-HNG-FP" after ELR or ALR & Calf Pad parts are selected

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

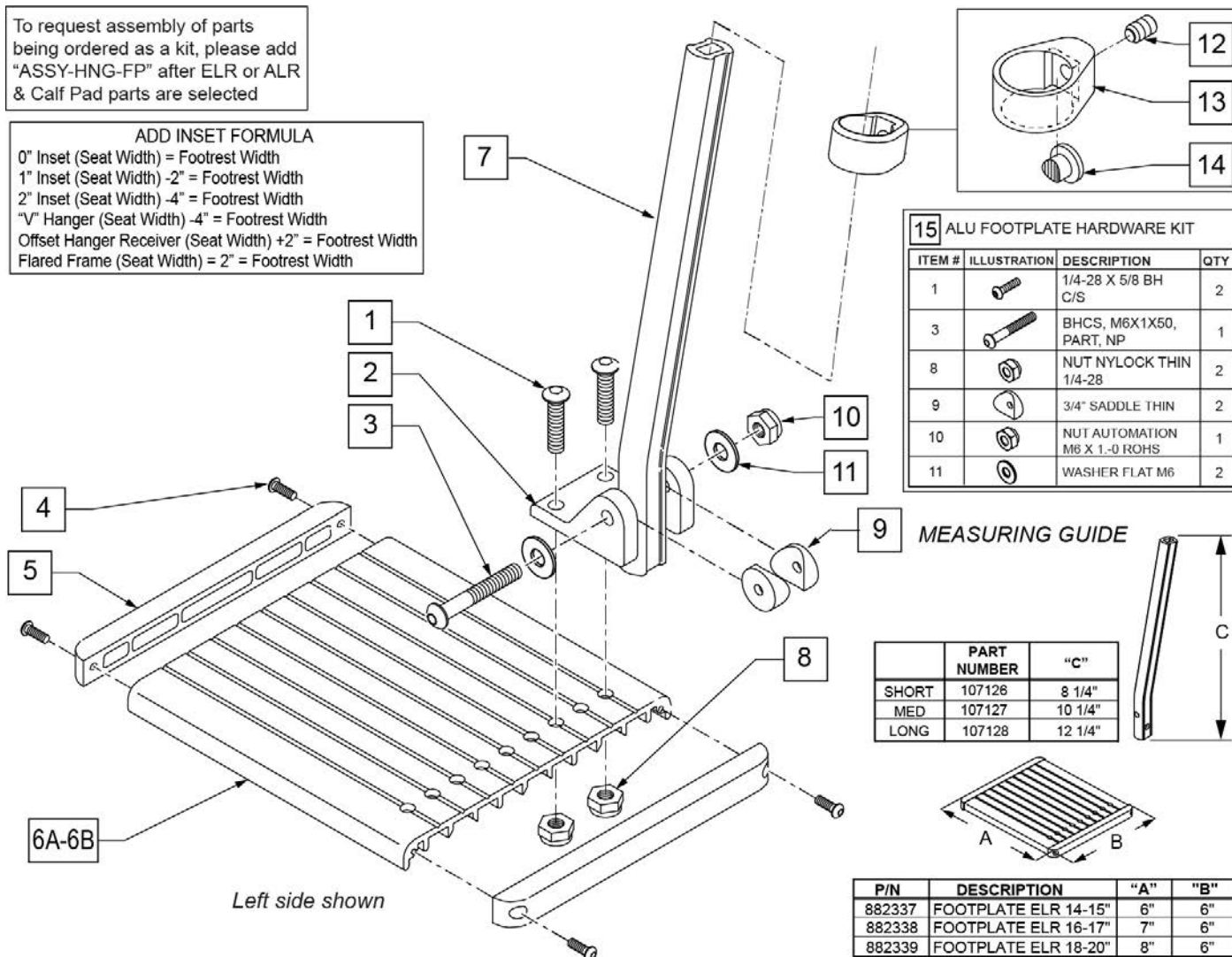


Pos.	Item Number	Description	Remarks
1	226090	1/4-28 x 5/8 BH C/S	
2	118093	BRACKET FOOTPLATE ALUM	
3	102262-008S	BHCS M6 X 1 X 50 INORGANIC	
4	281043	#4 x 5/8 PAN PHIL S/TAP	
5	560101S	END CAPS FOOTREST	
7	107126	SHORT BENT	
7	107127	MED BENT	
7	107128	LONG BENT	
7	128078	TUBE FOOTREST EXT SHT HEMI	Use for Hemi Extension option
8	106935	1/4-28 NYLOCK NUT THIN	
9	561003	3/4" SADDLE THIN	
10	132626	NUT AUTOMATION M6 X 1.0	
11	107383	WASHER FLAT M6 ROHS	
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	
12-14	119495	EXTENSION CLAMP KIT	
15	129775	ALU FOOTPLATE HDWARE KIT	Ext tube not included; bracket (118093) not included

Pos.	Item Number	Description	Remarks
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

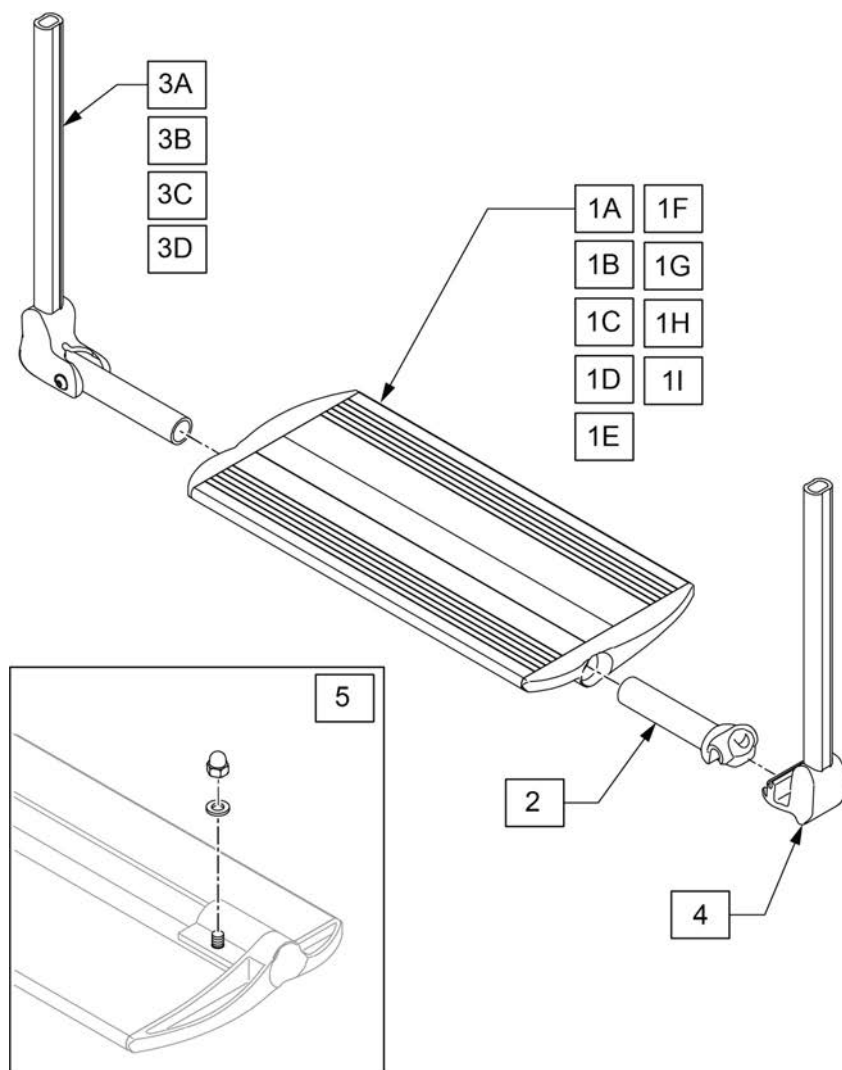

MEASURING GUIDE

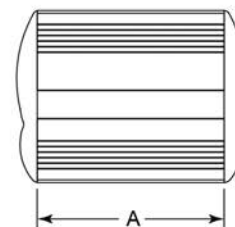


DESCRIPTION	DIM "A"
SINGLE FOOT PLATE ASSY 125MM	5"
SINGLE FOOT PLATE ASSY 150MM	6"
SINGLE FOOT PLATE 175MM	7"
SINGLE FOOT PLATE 200MM	8"
SINGLE FOOT PLATE 225MM	9"
SINGLE FOOT PLATE 250MM	10"
SINGLE FOOT PLATE 275MM	11"
SINGLE FOOT PLATE 300MM	12"
SINGLE FOOT PLATE 350MM	14"

INSET FORMULA

0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2" = Footrest Width


Pos.	Item Number	Description	Remarks
1A,5	71205029-001S	SINGLE FOOT PLATE ASSY 125MM	
1B,5	71205029-002S	SINGLE FOOT PLATE ASSY 150MM	
1C,5	71205029-003S	SINGLE FOOT PLATE 175MM	
1D,5	71205029-004S	SINGLE FOOT PLATE 200MM	
1E,5	71205029-005S	SINGLE FOOT PLATE 225MM	
1F,5	71205029-006S	SINGLE FOOT PLATE 250MM	
1G,5	71205029-007S	SINGLE FOOT PLATE 275MM	
1H,5	71205029-008S	SINGLE FOOT PLATE 300MM	
1I,5	71205029-009S	SINGLE FOOT PLATE 350MM	
3A	71205014-001	FOOTREST PIVOT ASSY 100MM	
3B	71205014-002	FOOTREST PIVOT ASSY 125MM	
3C	71205014-003	FOOTREST PIVOT ASSY 150MM	
3D	71205014-004	FOOTREST PIVOT ASSY 200MM	
2	71205033	FOOT REST FORK	
4	71205025	RECEIVER WITH FOOT REST TUBE	
5	169898	FOLDING PLTFRM FP FASTENERS	2 OF EACH
1A,2,3A-5	169899S	FOLDING PLTFRM FP NO-LCK 12W	
1B,2,3A-5	169900S	FOLDING PLTFRM FP NO-LCK 13W	
1C,2,3B-5	169901S	FOLDING PLTFRM FP NO-LCK 14W	

Pos.	Item Number	Description	Remarks
1D,2,3B-5	169902S	FOLDING PLTFRM FP NO-LCK 15W	
1E,2,3B-5	169903S	FOLDING PLTFRM FP NO-LCK 16W	
1F,2,3C-5	169904S	FOLDING PLTFRM FP NO-LCK 17W	
1G,2,3C-5	169905S	FOLDING PLTFRM FP NO-LCK 18W	
1H,2,3D-5	169906S	FOLDING PLTFRM FP NO-FLD 19W	
1I,2,3D-5	169907S	FOLDING PLTFRM FP NO-LCK 20W	

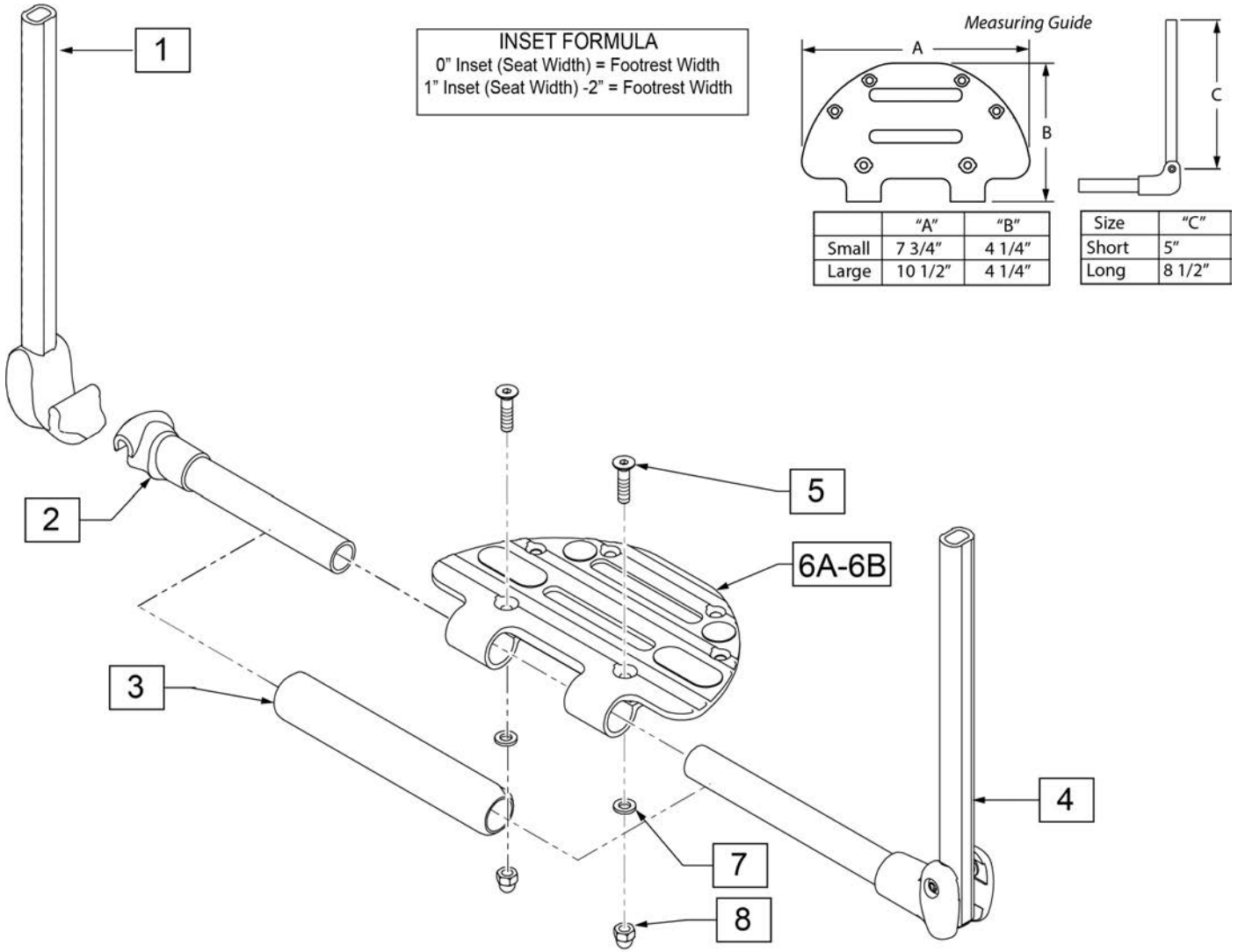


Pos.	Item Number	Description	Remarks
3,5,6A,7,8	162903	ULTRA LITE FOOT PLT SML	Small 32-38
3,5,6B,7,8	162904	ULTRA LITE FOOT PLT LRG	Large 40-50
3	066203-000	TUBE FRONT COVER	
5,7,8	169927	PLTFRM ULTRA LT FP HDWR	
1-5,6A,7,8	169928	UL LT PLATFORM FP 11	
1-5,6A,7,8	169929	UL LT PLATFORM FP 12	
1-5,6A,7,8	169930	UL LT PLATFORM FP 13	
1-5,6A,7,8	169931	UL LT PLATFORM FP 14	
1-5,6A,7,8	169932	UL LT PLATFORM FP 15	
1-5,6A,7,8	169933	UL LT PLATFORM FP 16	
1-5,6A,7,8	169934	UL LT PLATFORM FP 17	
1-5,6A,7,8	169935	UL LT PLATFORM FP 18	
1-5,6A,7,8	169936	UL LT PLATFORM FP 19-20	

Note: for Fixed Front Frame style only verify if inset is selected on the chair; Std/Long extension length is not available on Fixed Front w/ a 1cm inset due to interference with the bend in the frame to achieve that inset.

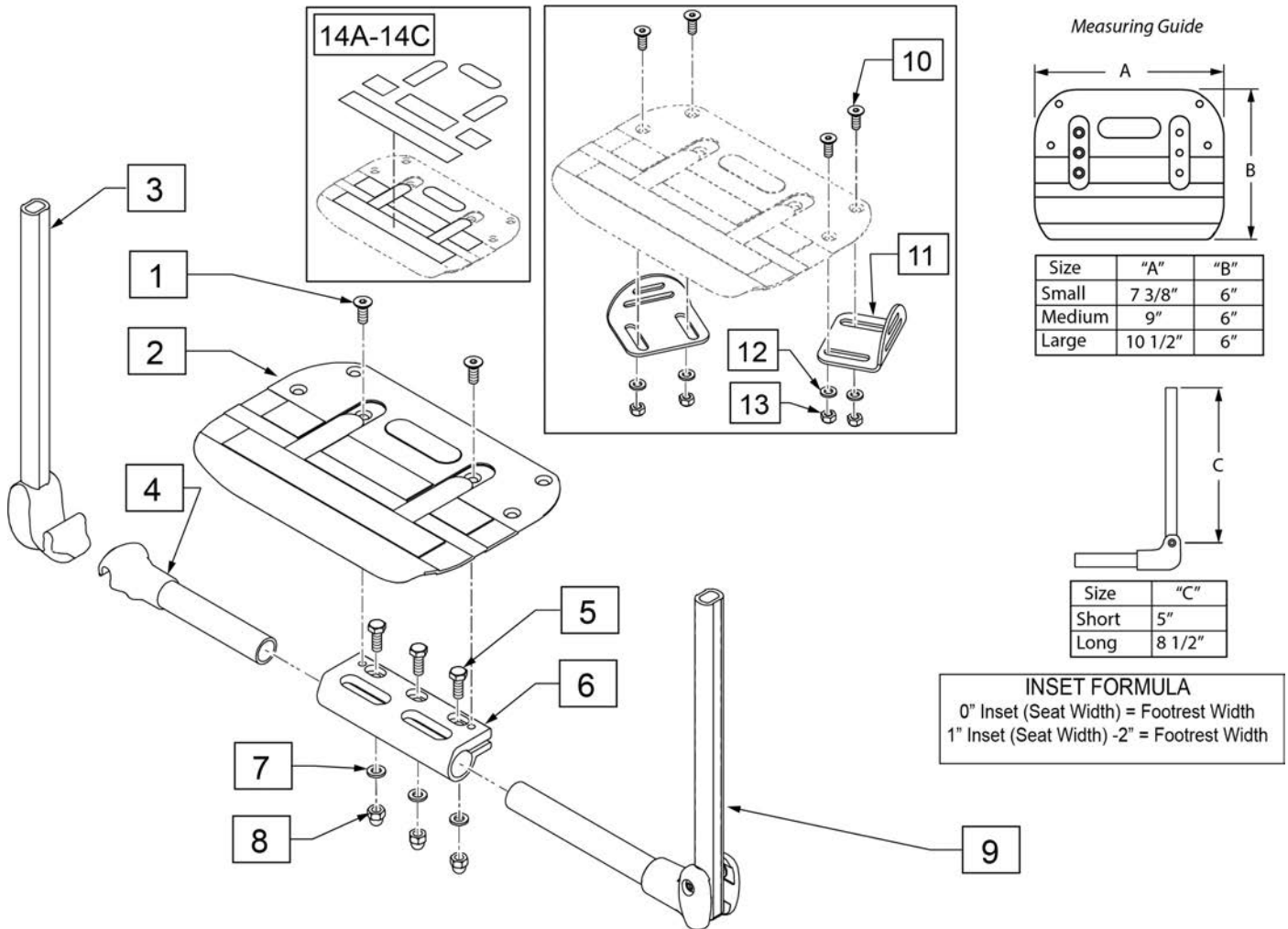


Pos.	Item Number	Description	Remarks
5-8	06802227	FOOTREST CLAMP	
10-13	068890	STOP BRACKET FOR FOOTBOARD	
14A	132004	ANTISLIP STCKR CRBN FTPT S KIT	11-12
14B	132005	ANTISLIP STCKR CRBN FTPT M KIT	13-14
14C	132006	ANTISLIP STCKR CRBN FTPT L KIT	15-20
1,2,14A	162925	PLTFRM CARB PLATE 32/34 CM	11-12
1,2,14B	162926	PLTFRM CARB PLATE 36/38 CM	13-14
1,2,14C	162927	PLTFRM CARB PLATE 40-50 CM	15-20
1-9	169937	COMP PLATFORM FP 11W	
1-9	169938	COMP PLATFORM FP 12W	
1-9	169939	COMP PLATFORM FP 13W	
1-9	169940	COMP PLATFORM FP 14W	
1-9	169941	COMP PLATFORM FP 15W	
1-9	169942	COMP PLATFORM FP 16W	
1-9	169943	COMP PLATFORM FP 17W	
1-9	169944	COMP PLATFORM FP 18W	
1-9	169945	COMP PLATFORM FP 19-20W	

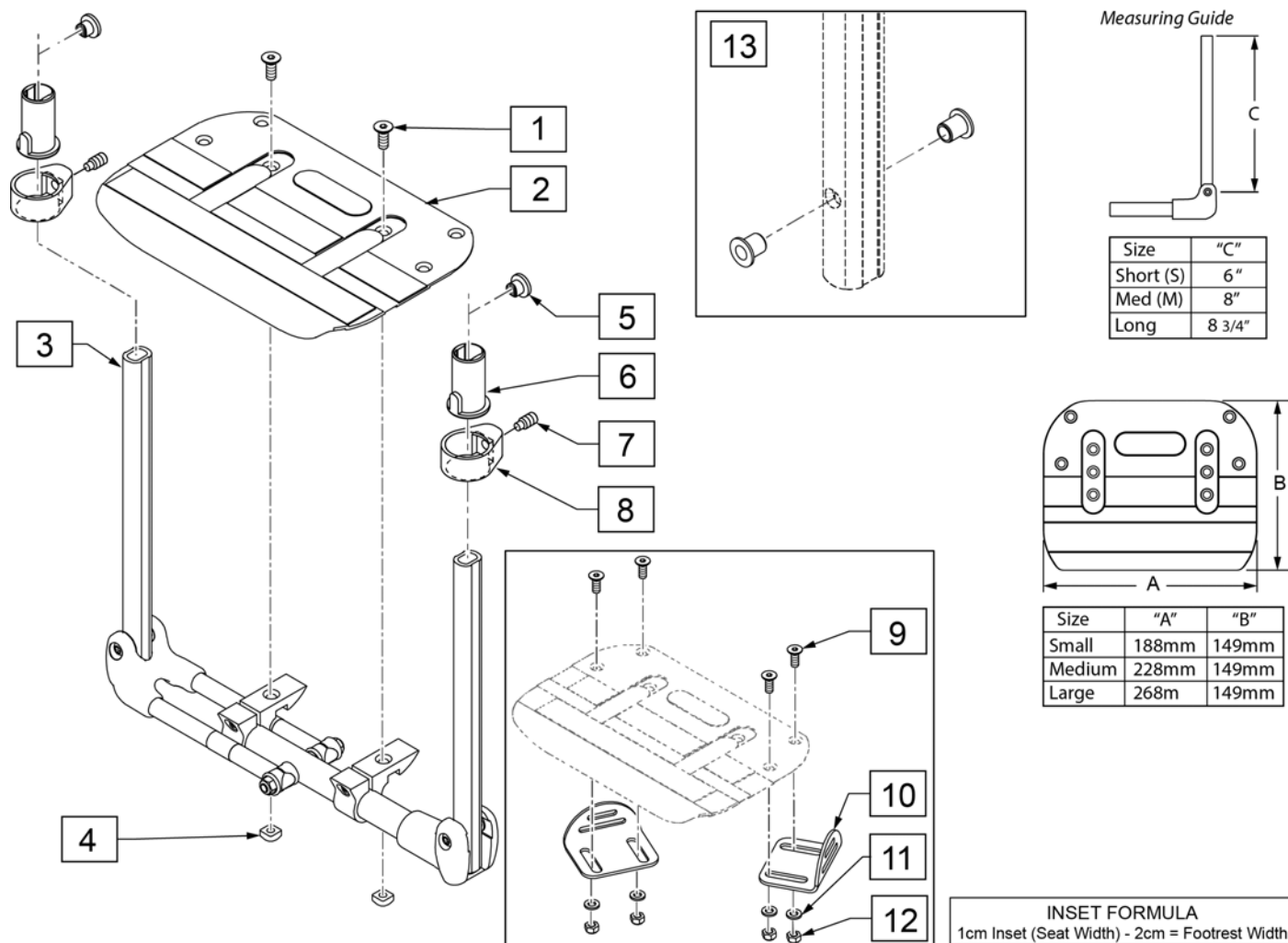


Pos.	Item Number	Description	Remarks
1,2,4	162926	PLTFRM CARB PLATE 36/38 CM	13 & 14 inch widths
1,2,4	162927	PLTFRM CARB PLATE 40-50 CM	15 inch wide and up
9-12	068890	STOP BRACKET FOR FOOTBOARD	2 pcs/pack
5-8	170101	FOOTREST CLAMP ASSM PAIR	
1-4	260332S	AUTOFOLD FOOTBOARD MECH 32W S	short extension tube
1-4	260333S	AUTOFOLD FOOTBOARD MECH 34W S	short extension tube
1-4	260334S	AUTOFOLD FOOTBOARD MECH 36W S	short extension tube
1-4	260335S	AUTOFOLD FOOTBOARD MECH 38W S	short extension tube
1-4	260336S	AUTOFOLD FOOTBOARD MECH 40W S	short extension tube
1-4	260337S	AUTOFOLD FOOTBOARD MECH 42W S	short extension tube
1-4	260338S	AUTOFOLD FOOTBOARD MECH 44W S	short extension tube
1-4	260339S	AUTOFOLD FOOTBOARD MECH 46W S	short extension tube
1-4	260340S	AUTOFOLD FOOTBOARD MECH 48W S	short extension tube
1-4	260341S	AUTOFOLD FOOTBOARD MECH 50W S	short extension tube
1-4	260342S	AUTOFOLD FOOTBOARD MECH 32W M	med extension tube
1-4	260343S	AUTOFOLD FOOTBOARD MECH 34W M	med extension tube
1-4	260344S	AUTOFOLD FOOTBOARD MECH 36W M	med extension tube
1-4	260345S	AUTOFOLD FOOTBOARD MECH 38W M	med extension tube
1-4	260346S	AUTOFOLD FOOTBOARD MECH 40W M	med extension tube
1-4	260347S	AUTOFOLD FOOTBOARD MECH 42W M	med extension tube

Pos.	Item Number	Description	Remarks
1-4	260348S	AUTOFOLD FOOTBOARD MECH 44W M	med extension tube
1-4	260349S	AUTOFOLD FOOTBOARD MECH 46W M	med extension tube
1-4	260350S	AUTOFOLD FOOTBOARD MECH 48W M	med extension tube
1-4	260351S	AUTOFOLD FOOTBOARD MECH 50W M	med extension tube
1-4	71102003-001S	AUTOFOLD FOOTBOARD MECH 32W L	large extension tube
1-4	71102003-002S	AUTOFOLD FOOTBOARD MECH 34W L	large extension tube
1-4	71102003-003S	AUTOFOLD FOOTBOARD MECH 36W L	large extension tube
1-4	71102003-004S	AUTOFOLD FOOTBOARD MECH 38W L	large extension tube
1-4	71102003-005S	AUTOFOLD FOOTBOARD MECH 40W L	large extension tube
1-4	71102003-006S	AUTOFOLD FOOTBOARD MECH 42W L	large extension tube
1-4	71102003-007S	AUTOFOLD FOOTBOARD MECH 44W L	large extension tube
1-4	71102003-008S	AUTOFOLD FOOTBOARD MECH 46W L	large extension tube
1-4	71102003-009S	AUTOFOLD FOOTBOARD MECH 48W L	large extension tube
1-4	71102003-010S	AUTOFOLD FOOTBOARD MECH 50W L	large extension tube

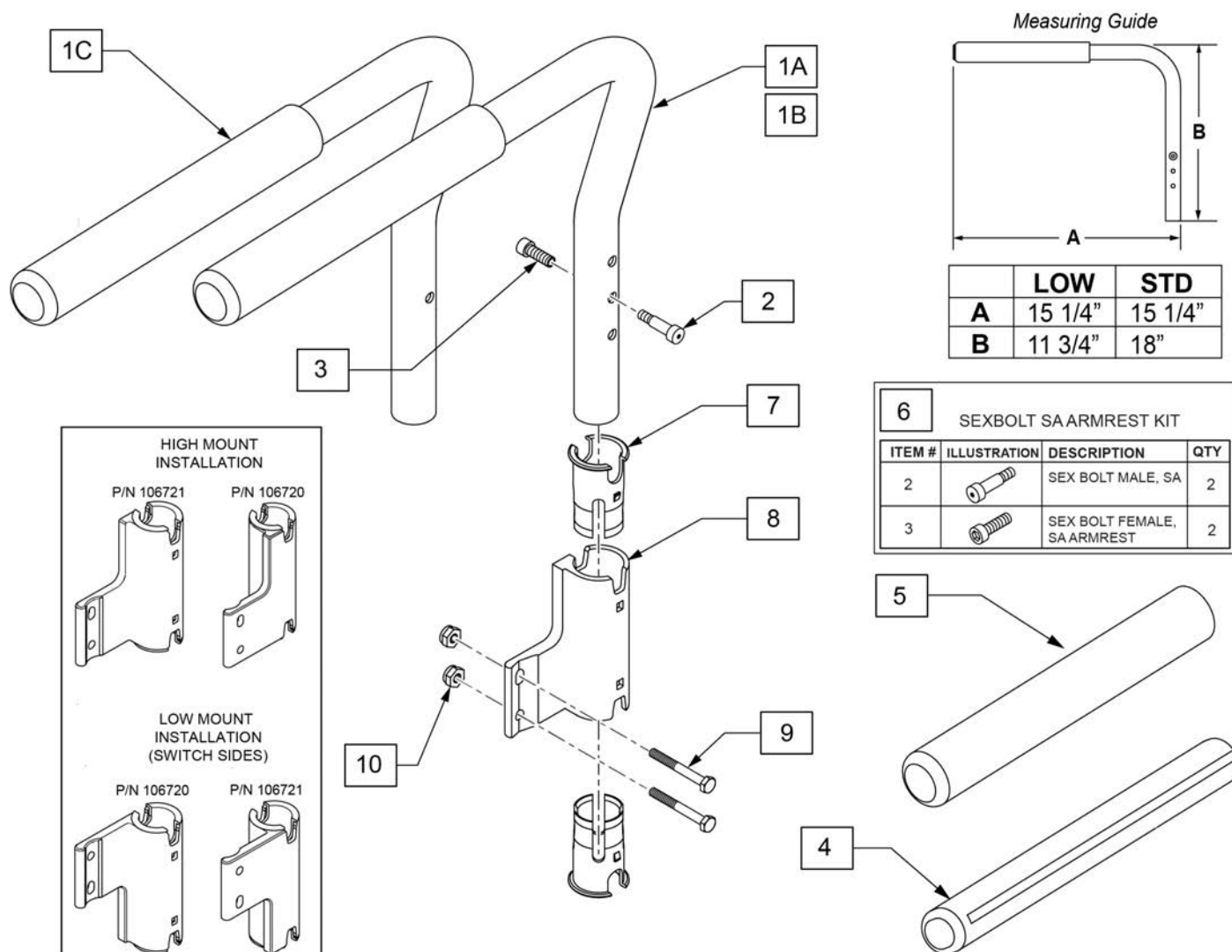


Pos.	Item Number	Description	Remarks
1A,2,3	111153	SWING-AWAY ARMREST LOW W/ BOLT R	RIGHT-LOW
1A,2,3	111154	SWING-AWAY ARMREST LOW W/ BOLT L	LEFT -LOW
1B,2,3	111155	SWING-AWAY ARMREST STD W/ BOLT R	RIGHT-STD
1B,2,3	111156	SWING-AWAY ARMREST STD W/ BOLT L	LEFT -STD
1C,2,3	162992	PADDED SA TUBE FXD W/BOLT RH	RIGHT - Fixed HT
1C,2,3	162993	PADDED SA TUBE FXD W/BOLT LH	LEFT - Fixed HT
4	108315S	GEL PAD S/ ARM	PAIR
5	560640S	9X.93X.155 FOAM GRIP	Single replacement grip
6	108118	SEXBOLTS ARMREST KIT	
7	106505S	SLEEVE SWING AWAY ARMREST	
7,8	106720S	RECEIVER, SA ASSY, RH	RIGHT
7,8	106721S	RECEIVER, SA ASSY, LH	LEFT- SHOWN
7-10	169946	PADDED SA RECEIVER KIT RH	
7-10	169947	PADDED SA RECEIVER KIT LH	
9,10	169948	PADDED SA RECEIVER HDWR	2 EA

Note: if switching from Desk to Full Length Armrest Pad OR switching from Full Length to Desk Armrest Pad, the following items are required:
1B/1C, 2, 3B/3C, 4 and 5 (item 5 used on Full Length only).

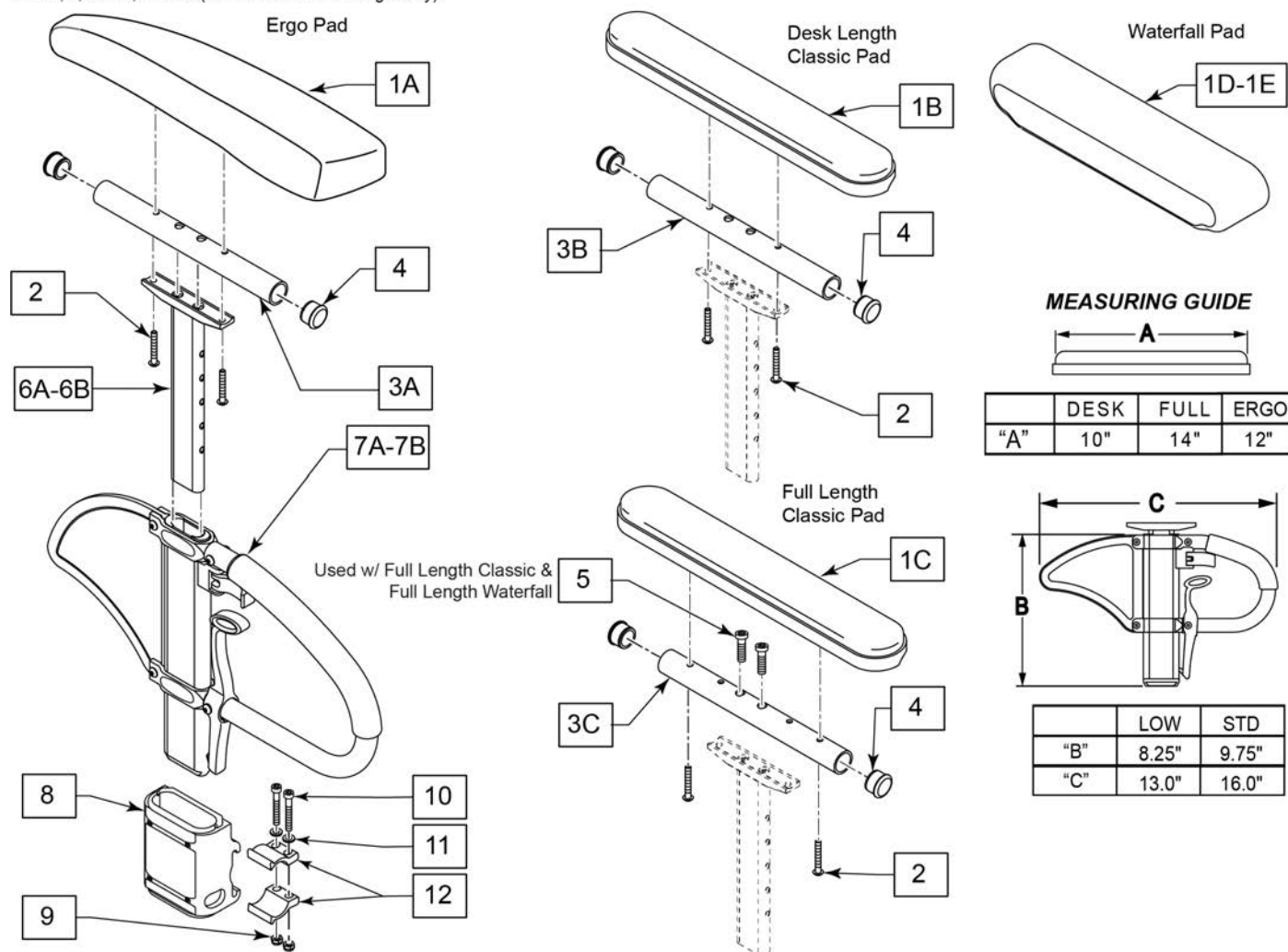


Pos.	Item Number	Description	Remarks
1A,2	161631	ERGO PAD RIGHT	with Fasteners
1A,2	161632	ERGO PAD LEFT	with Fasteners
1B	161633	CLASSIC PAD DESK	with Fasteners
1C	161634	CLASSIC PAD FULL	with Fasteners
1D	561280	ARMREST PAD 10" LEFT W/F	
1D	561281	ARMREST PAD 10" RIGHT W/F	
1E	561284	ARMREST PAD 14" LEFT W/F	
1E	561285	ARMREST PAD 14" RIGHT W/F	
2	107024	BHCS 10-32 X 1 1/2 ROHS	
3B	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
3C	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
4	560735S	7/8" ROUND TUBE PLUG	
5	108534	SHCS 10-32 X 1-1/4 PL	Used w/ Full Length Classic & Full Length Waterfall
6A	116313	SNGL POST UPPR ASSM STD	Standard
6B	116312	SNGL POST UPPR ASSM LOW	Low/Small
6A,7A	241619S	Q2 SPHA ARMREST ASSY	Standard
6B,7B	241664S	Q2 SPHA LOW ARMREST ASSY	Low/Small
1A,2,3A,4,6A	161637	ERGO ARM PAD W/POST RIGHT	Right
1A,2,3A,4,6A	161638	ERGO ARM PAD W/POST LEFT	Left

Pos.	Item Number	Description	Remarks
1A,2,3A,4,6B	161657	ERGO ARM PAD W/POST LOW RH	Right
1A,2,3A,4,6B	161658	ERGO ARM PAD W/POST LOW LH	Left
		<<COMPLETE ARMREST W/O RECEIVERS>>	
1A,2,3A,4,6B,7B	161561	SPHA LOW AERO PAD FULL RH	Right
1A,2,3A,4,6B,7B	161562	SPHA LOW AERO PAD FULL LH	Left
1B,2,3B,4,6B,7B	161557	SPHA LOW CLASSIC PAD DESK RH	Right
1B,2,3B,4,6B,7B	161558	SPHA LOW CLASSIC PAD DESK LH	Left
1B,2,3B,4,6A,7A	161559	SPHA STD CLASSIC PAD DESK RH	Right
1B,2,3B,4,6A,7A	161560	SPHA STD CLASSIC PAD DESK LH	Left
1C,2,3C,4,5,6B,7B	161553	SPHA LOW CLASSIC PAD FULL RH	Right
1C,2,3C,4,5,6B,7B	161554	SPHA LOW CLASSIC PAD FULL LH	Left
1C,2,3C,4,5,6A,7A	161555	SPHA STD CLASSIC PAD FULL RH	Right
1C,2,3C,4,5,6A,7A	161556	SPHA STD CLASSIC PAD FULL LH	Left
1D,2,3B,4,6B,7B	164089	SPHA LOW W/F DESK PAD R	Right
1D,2,3B,4,6B,7B	164090	SPHA LOW W/F DESK PAD L	Left
1E,2,3C,4,6B,7B	164091	SPHA LOW W/F FULL PAD R	Right
1E,2,3C,4,6B,7B	164092	SPHA LOW W/F FULL PAD L	Left
1D,2,3B,4,6A,7A	164093	SPHA STD W/F DESK PAD R	Right
1D,2,3B,4,6A,7A	164094	SPHA STD W/F DESK PAD L	Left
1E,2,3C,4,6A,7A	164095	SPHA STD W/F FULL PAD R	Right
1E,2,3C,4,6A,7A	164096	SPHA STD W/F FULL PAD L	Left
		<<RECEIVERS>>	
8-12	161651	SPHA FB LT RECEIVER FRNT	

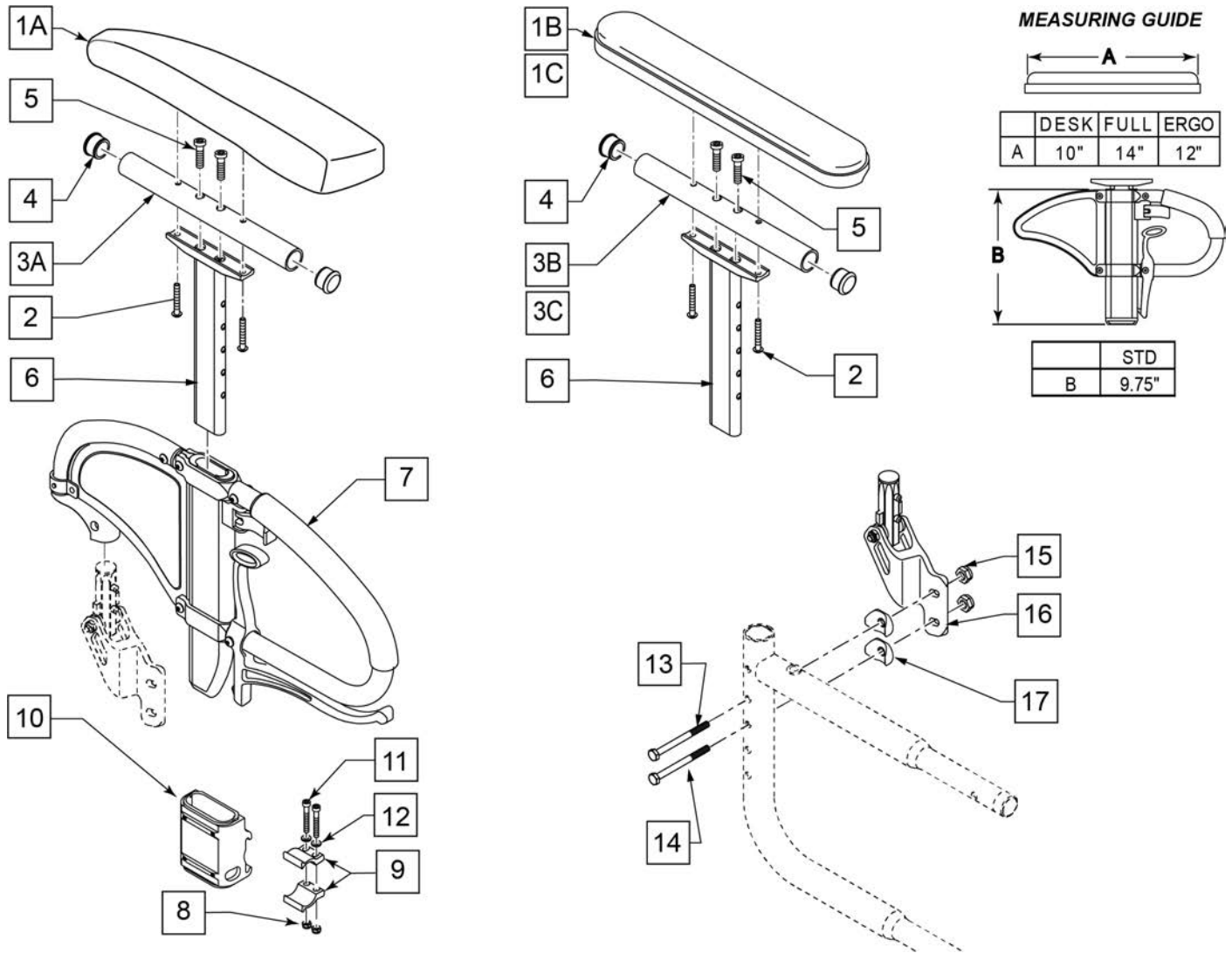


Pos.	Item Number	Description	Remarks
1A,2	161631S	ERGO PAD RIGHT	With Fasteners
1A,2	161632S	ERGO PAD LEFT	With Fasteners
1B,2	161633S	CLASSIC PAD DESK	With Fasteners
1C,2	161634S	CLASSIC PAD FULL	With Fasteners
2	107024S	10-32 X 1 1/2 BH C/S BLK	
3A	243044S	HEIGHT ADJ UPPER ARM TUBE MED	
3B	521812S	HT ADJ PD TUBE 10IN DESK POWE	
3C	521813S	HT ADJ PD TUBE 14IN FULL POWE	
4	560735S	7/8" ROUND TUBE PLUG	
5	226031	10-32 x 1 1/4 SOC C/S BLK	Full
6	116313S	SPHA ARM PAD MOUNT	
1A,2,3A,4-6	161637S	ERGO ARM PAD W/POST RIGHT	Right
1A,2,3A,4-6	161638S	ERGO ARM PAD W/POST LEFT	Left
6,7	241726S	Q2 VIP 2 FLIP UP RH	Right
6,7	241592S	Q2 VIP 2 FLIP UP LH	Left
8-12	161651S	SPHA FB LT RECEIVER FRNT	
1B,2,3B,4-7	161643S	SPHA FB LT CLASSIC DESK R	Right
1B,2,3B,4-7	161644S	SPHA FB LT CLASSIC DESK L	Left

Pos.	Item Number	Description	Remarks
1B,2,3C,4-7	161639S	SPHA FB LT CLASSIC FULL R	Right
1B,2,3C,4-7	161642S	SPHA FB LT CLASSIC FULL L	Left
1A,2,3A,4-7	161649S	SPHA FB LT ERGO R	Right
1A,2,3A,4-7	161650S	SPHA FB LT ERGO L	Left
13-17	169949	REAR FB ARM REC RH	Right
13-17	169950	REAR FB ARM REC LH	Left
8,11-15,17	169951	FB RECEIVER HARDWARE	One Side

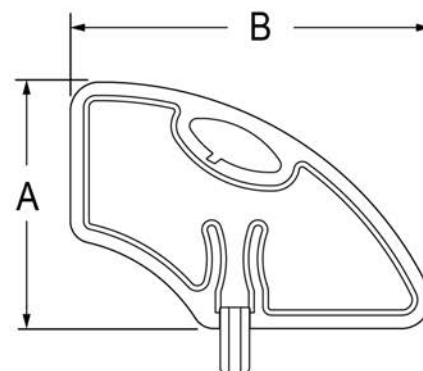
MEASURING GUIDE

	SMALL	STD
A	6.4"	8.4"
B	11"	12.3"

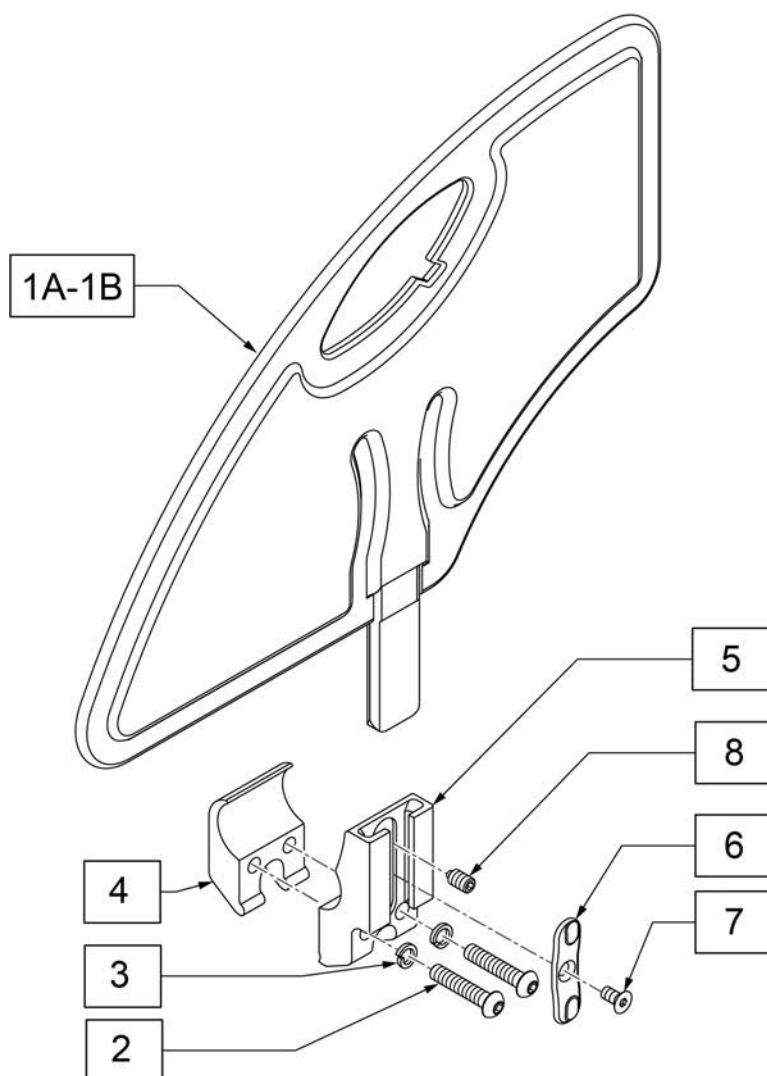


9 FASTENERS SIDE GUARD RECVR			
Pos.		Description	Qty
2		BHCS,M6X1X30,FULL,NP	2
3		WASHER LOCK HI COLLAR 1/4	2
7		FHCS #6-32 X 1/4 BLK	1
8		SETSCREW 1/4-28 X 5/16 OVAL PT	1

Pos.	Item Number	Description	Remarks
1A	104164-001S	SIDE GUARD ASSY STD	
1B	104164-002S	SIDE GUARD ASSY SMALL	
2	102262-005	BHCS,M6X1X30,FULL,NP	
5-8	108781S	RECEIVER SIDE GUARD ASSY	
2-8	169952	SIDE GUARD RECIEVER	one side, left or right
1A,2-8	169954	SIDE GUARD W/RECIEVER STD	one side, left or right
1B,2-8	169955	SIDE GUARD W/RECIEVER SMALL	one side, left or right
9	169953	FASTENERS SIDE GUARD RECVR	one side, left or right

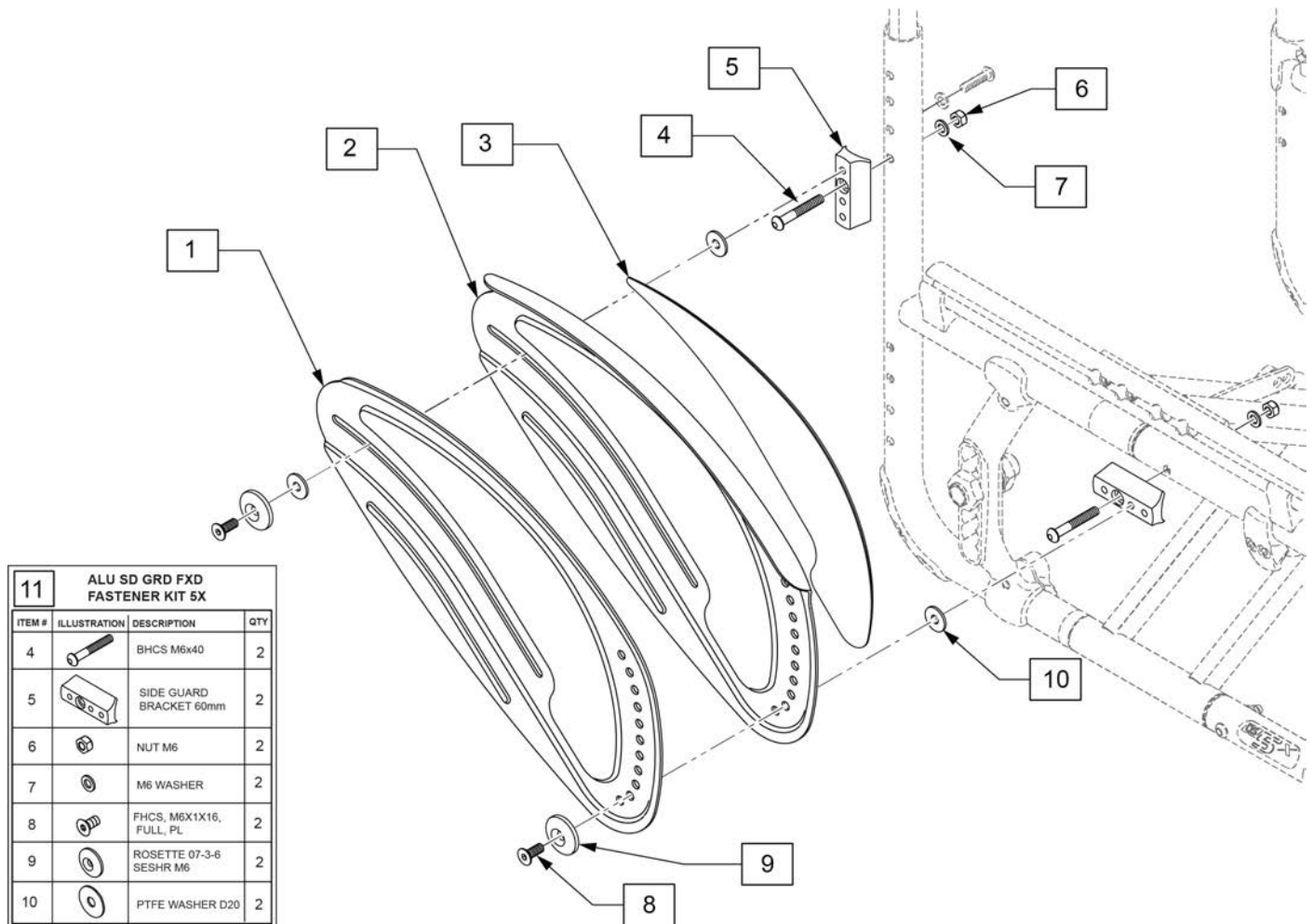


Pos.	Item Number	Description	Remarks
1,3	169967	ALU SIDE GRD PANEL PAINT RH	paint
1,3	169968	ALU SIDE GRD PANEL PAINT LH	
2,3	169969	ALU SIDE GRD FNDR PANEL PAINT RH	
2,3	169970	ALU SIDE GRD FNDR PANEL PAINT LH	
1,3-10	169971	ALU SIDE GRD W/O FENDER 5X RH	
1,3-10	169972	ALU SIDE GRD W/O FENDER 5X LH	
2-10	169973	ALU SIDE GRD W/FENDER 5X RH	
2-10	169974	ALU SIDE GRD W/FENDER 5X LH	
11	169975	ALU SD GRD FXD FASTENER KIT 5X	

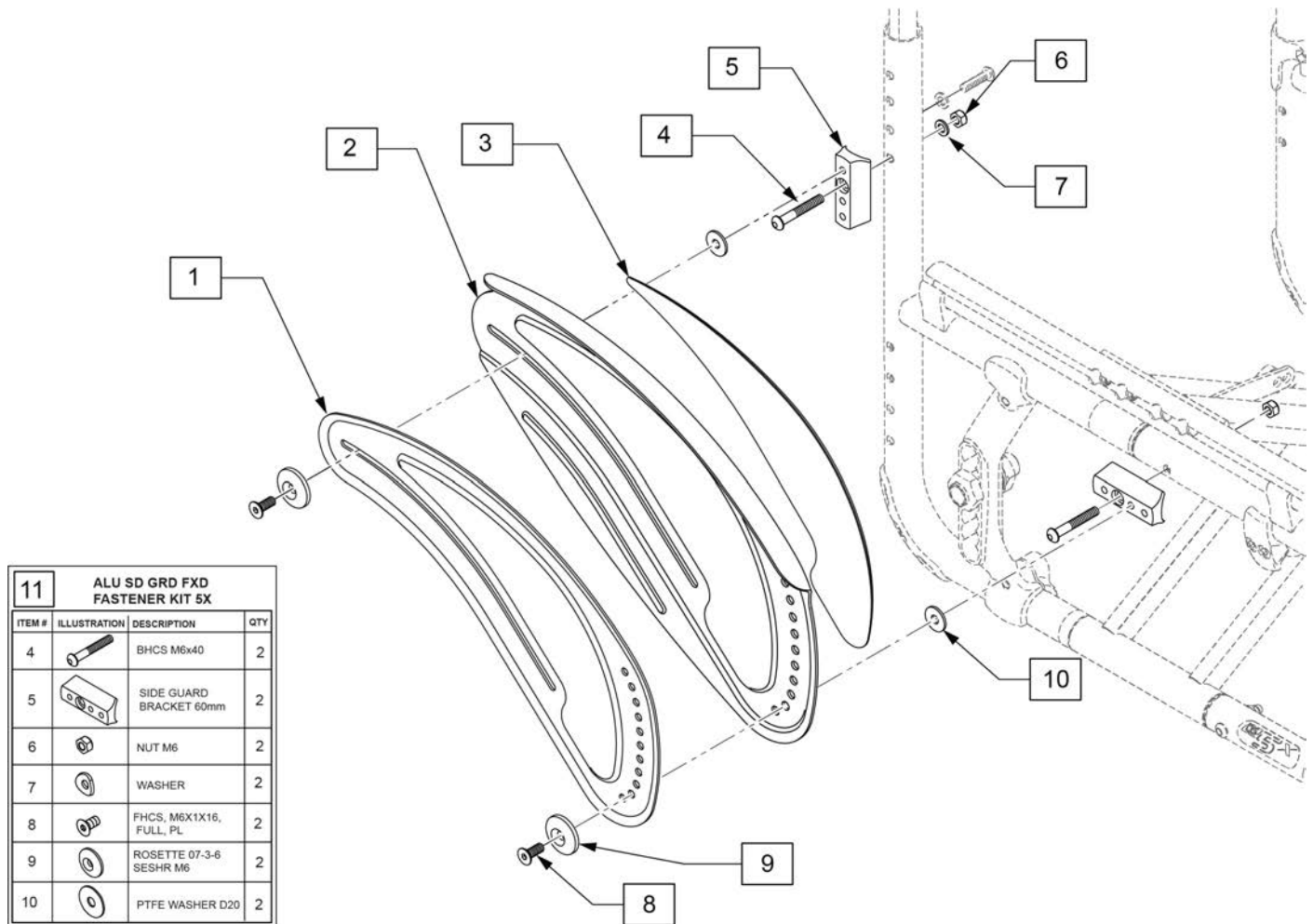


Pos.	Item Number	Description
1	058455-001S	SLIMSTYLE SIDEGUARD LEFT
1	058455-002S	SLIMSTYLE SIDEGUARD RIGHT
2,3	169981	CARBON FIBER SG LRG FNDR RH
2,3	169982	CARBON FIBER SG LRG FNDR LH
1,4-10	169983	CARBON SIDE GRD SLIM RH
1,4-10	169984	CARBON SIDE GRD SLIM LH
2-10	169985	CARBON SIDE GRD LRG RH
2-10	169986	CARBON SIDE GRD LRG LH
11	169975	ALU SD GRD FXD FASTENER KIT 5X


NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

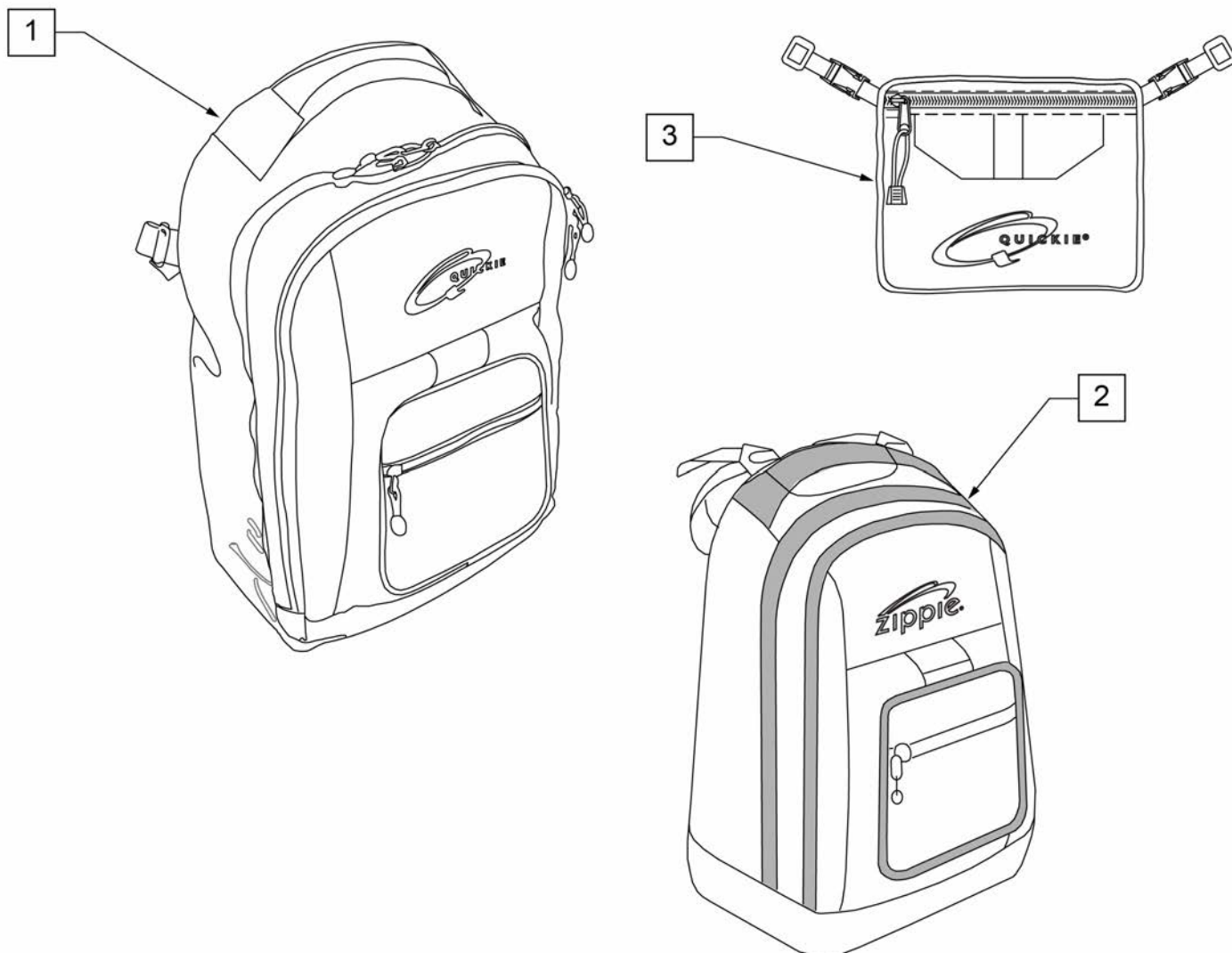

Pos.	Item Number	Description	Remarks
1	750115	BACK PACK QUICKIE	Overall dimensions: width 12.5", height 18.5", depth 6.5"
2	104572S	BACKPACK ZIPPIE	Overall dimensions: width 12", height 16.5", depth 4.5"
3	750307S	SEAT POUCH QUICKIE	Dimensions: width 8.5", height 6", depth top 2.5", depth bottom 3.5"

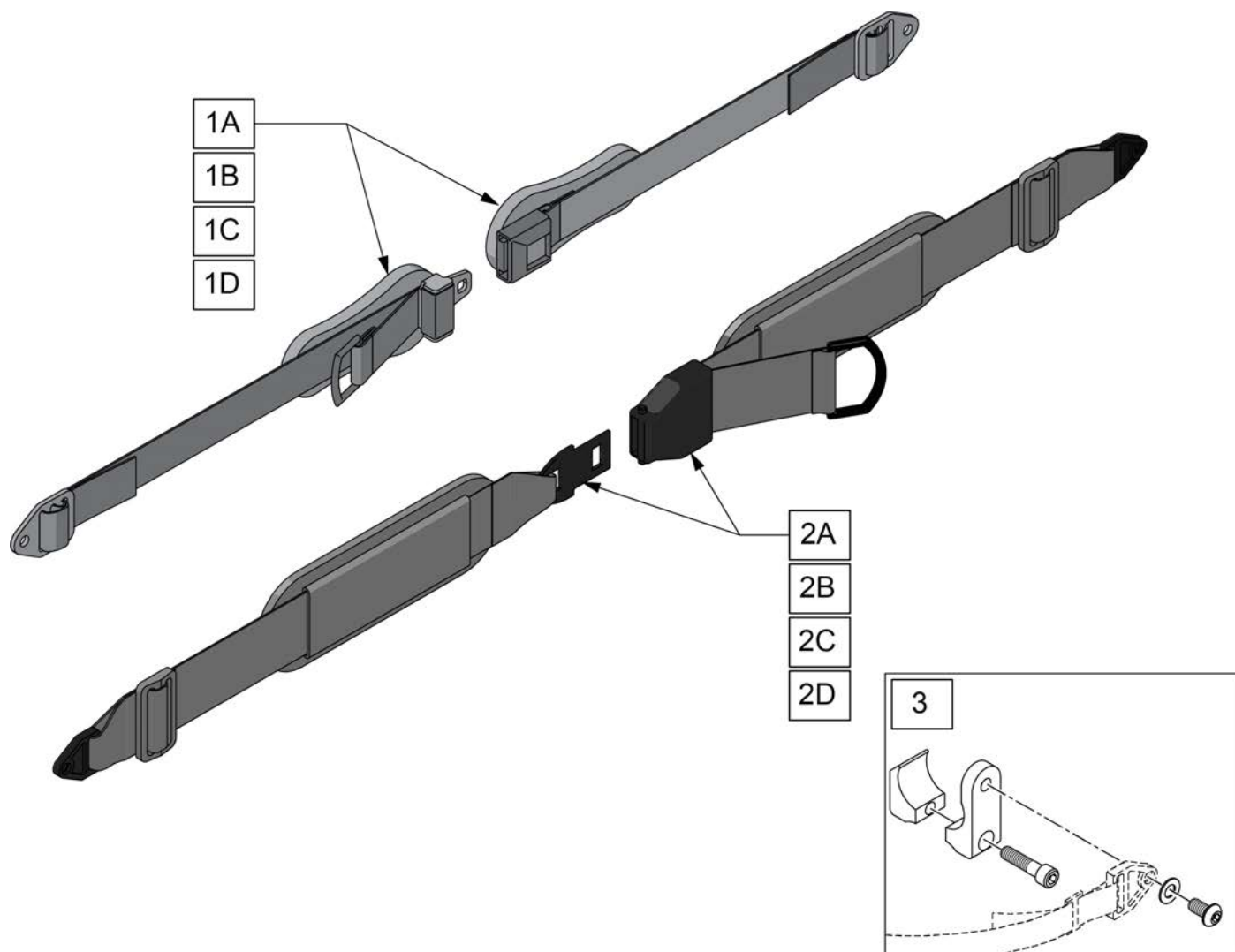


Pos.	Item Number	Description	Remarks
1A	APB7212	APB, 1.5" PB, SINGLE, SM PADS	(7.5"Pad), Push Button
1B	APB7213	APB, 1.5" PB, SINGLE, MED PADS	(9.5"Pad), Push Button
1C	APB7313	APB, 2" PB, SINGLE, MED PADS	(7.5"Pad), Push Button
1D	APB7314	APB, 2" PB, SINGLE, LG PADS	(9.5"Pad), Push Button
2A	450418-QU	BELT 2" AIRCRAFT BUCKLE	Max length of belt 64"
2B	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD	Max length of belt 64"
2C	450424	BELT 2" AIRCRAFT BKL LONG W/PAD	Max length of belt 72"
2D	450425	BELT 2" AIRCRAFT BKL LONG	Max length of belt 72"
3	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair

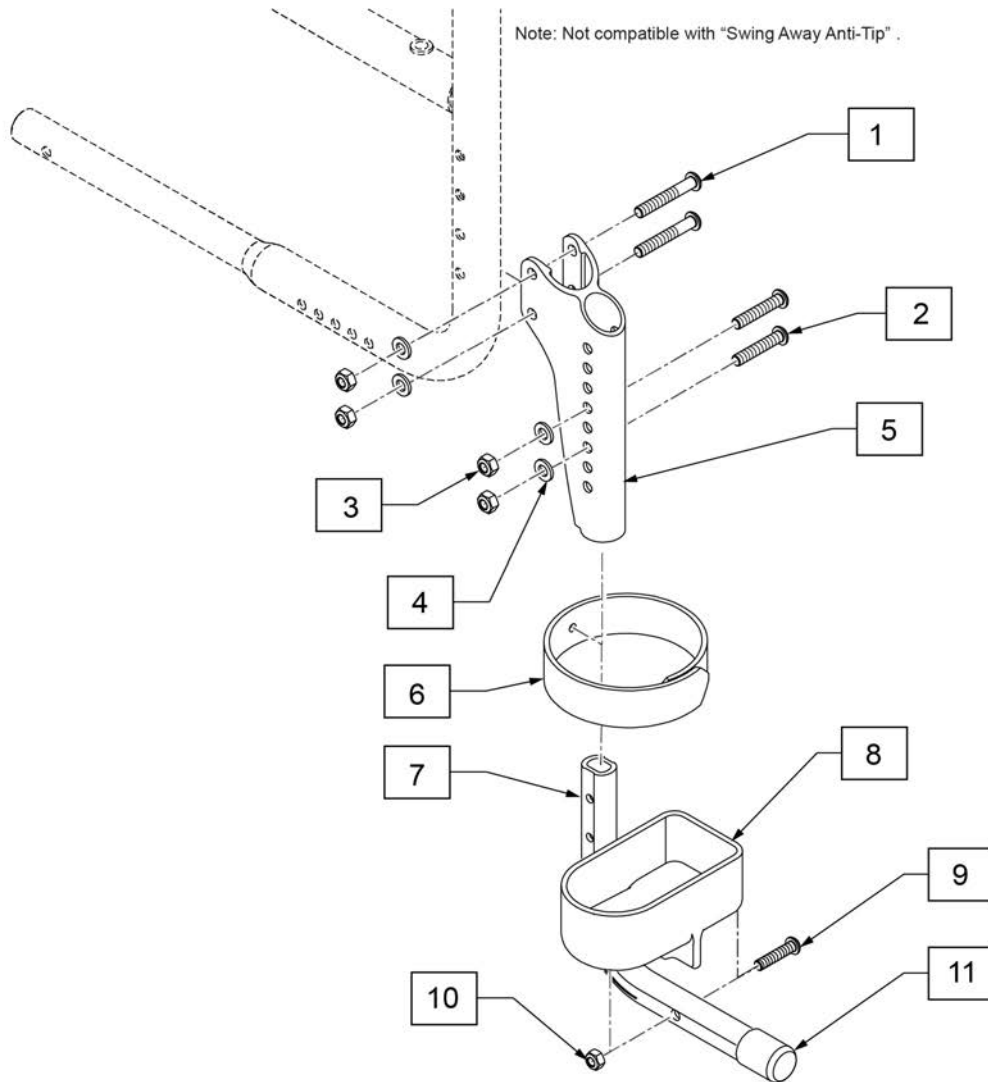


Pos.	Item Number	Description	Remarks
1-5	170158	ACCESSORY BRKT KIT	
1-4	170159	ACCESSORY BRKT FASTENERS	
6-11	740815120	CRUTCH HOLDER WITH VELCRO	
1-11	TBD	CRUTCH HOLDER RETRO KIT	Single
11	740815123	TIP ASSIST THREAD CAP	

Note: Not compatible with "Swing Away Anti-Tip" or "Crutch Holder".

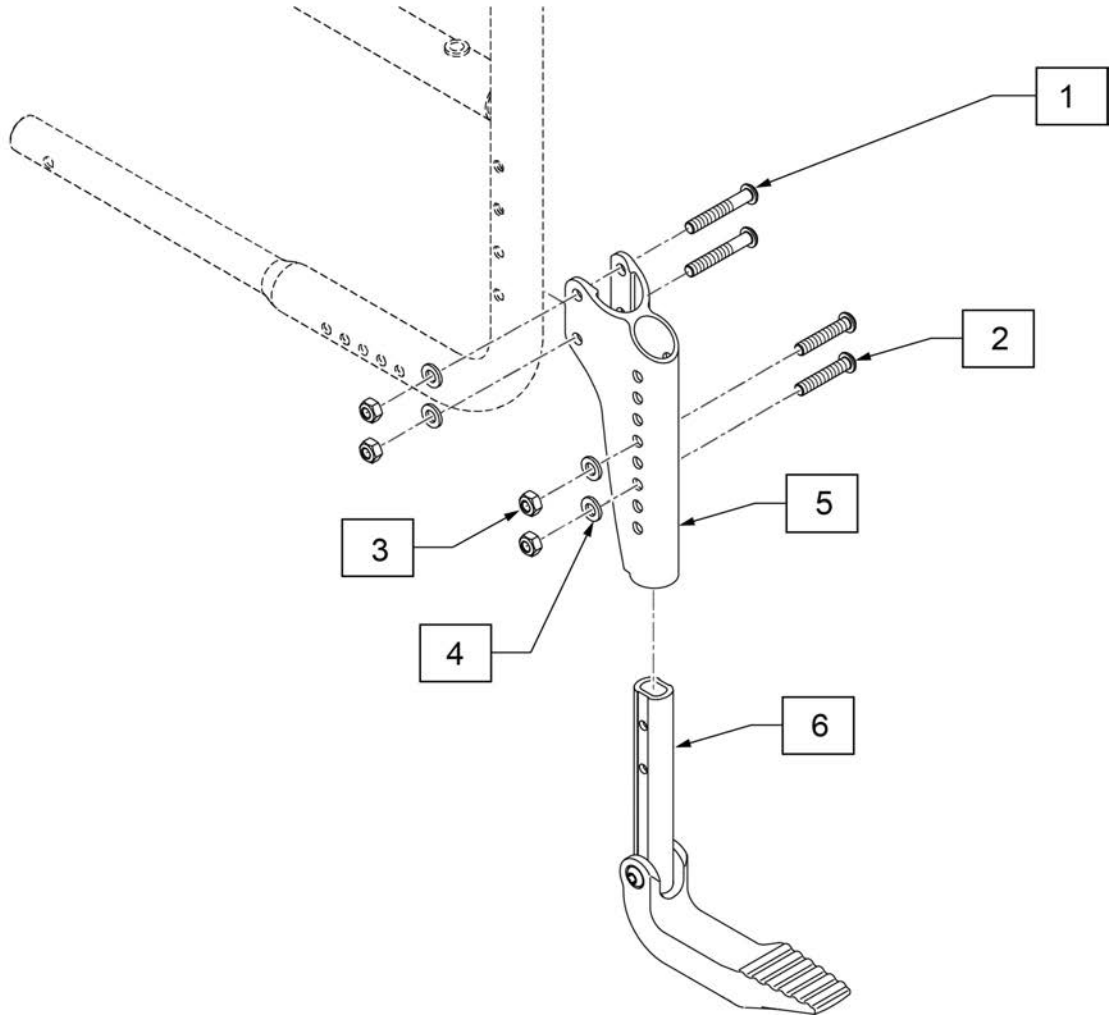


Pos.	Item Number	Description
1-5	170158	ACCESSORY BRKT KIT
1-4	170159	ACCESSORY BRKT FASTENERS
1-6	170164	TIP ASSIST KIT


Note: Not compatible with "Swing Away Anti-Tip" or "Crutch Holder".

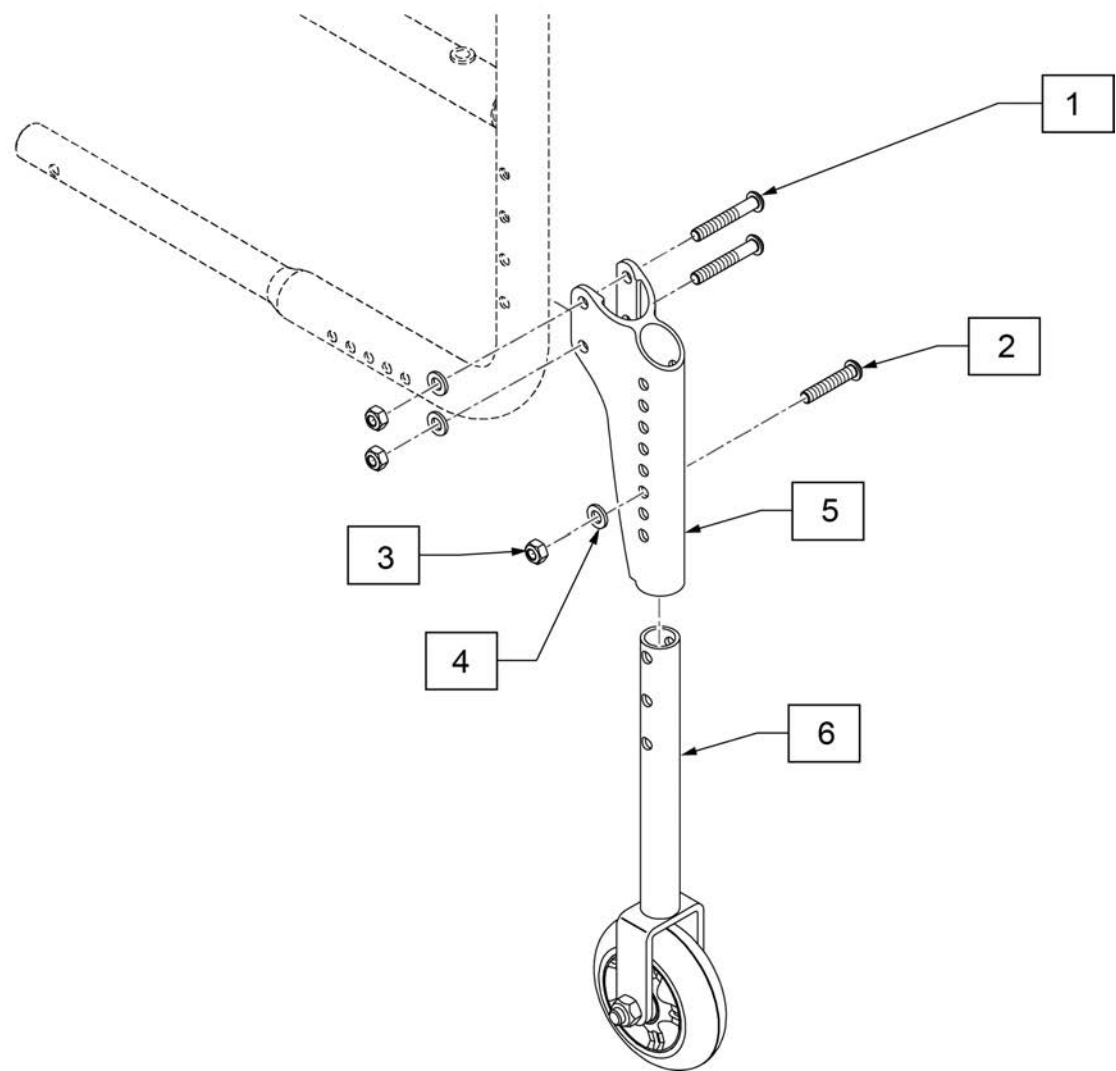


Pos.	Item Number	Description	Remarks
1-5	170158	ACCESSORY BRKT KIT	
1-4	170159	ACCESSORY BRKT FASTENERS	
1-6	170165	TRANSIT WHEEL KIT PR	Pair

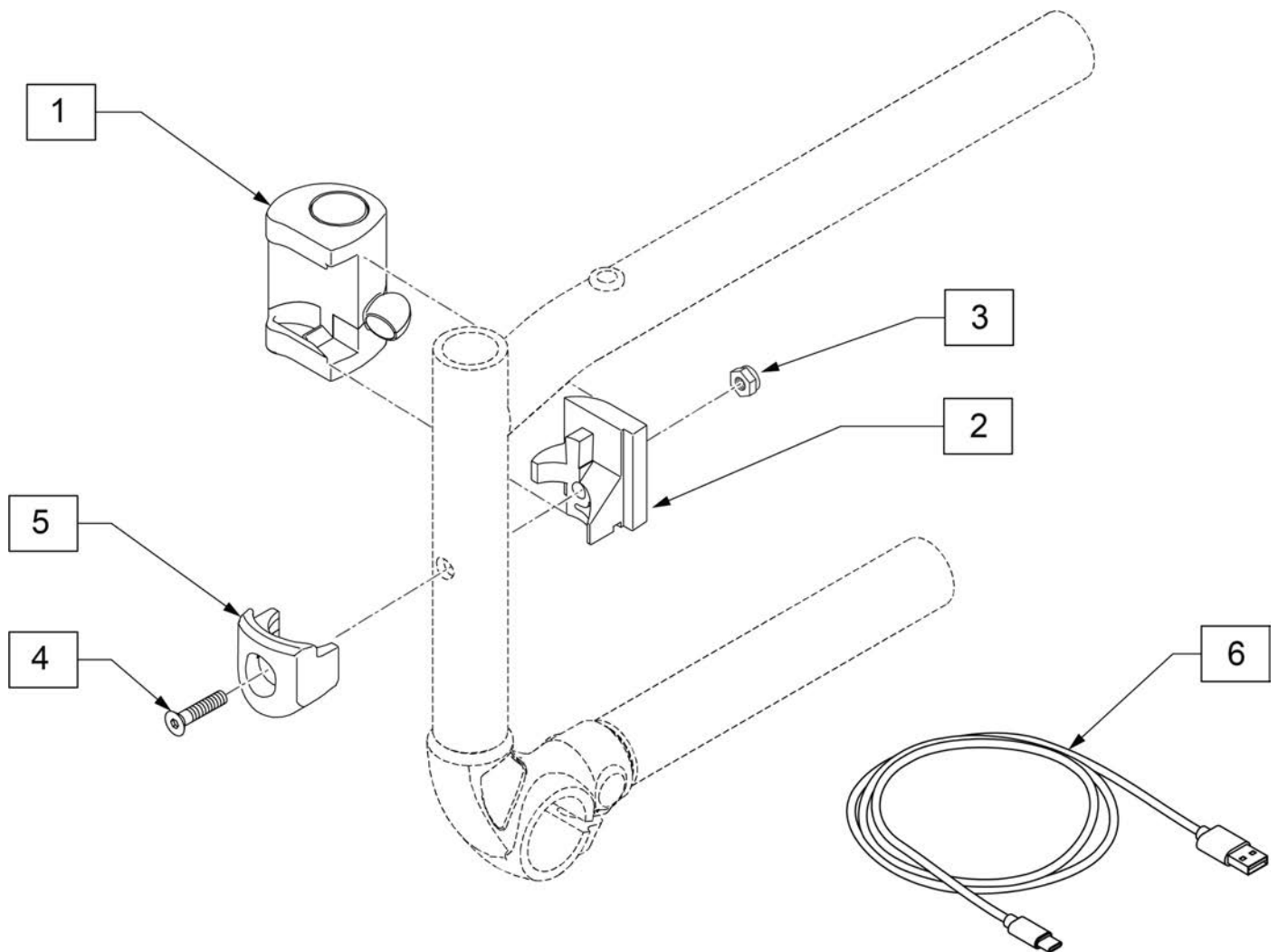


Pos.	Item Number	Description	Remarks
1-4	169956	LED LIGHT SET	PAIR
5	118696S	Hanger Latch Receiver	
6	71209037	Charging cable	

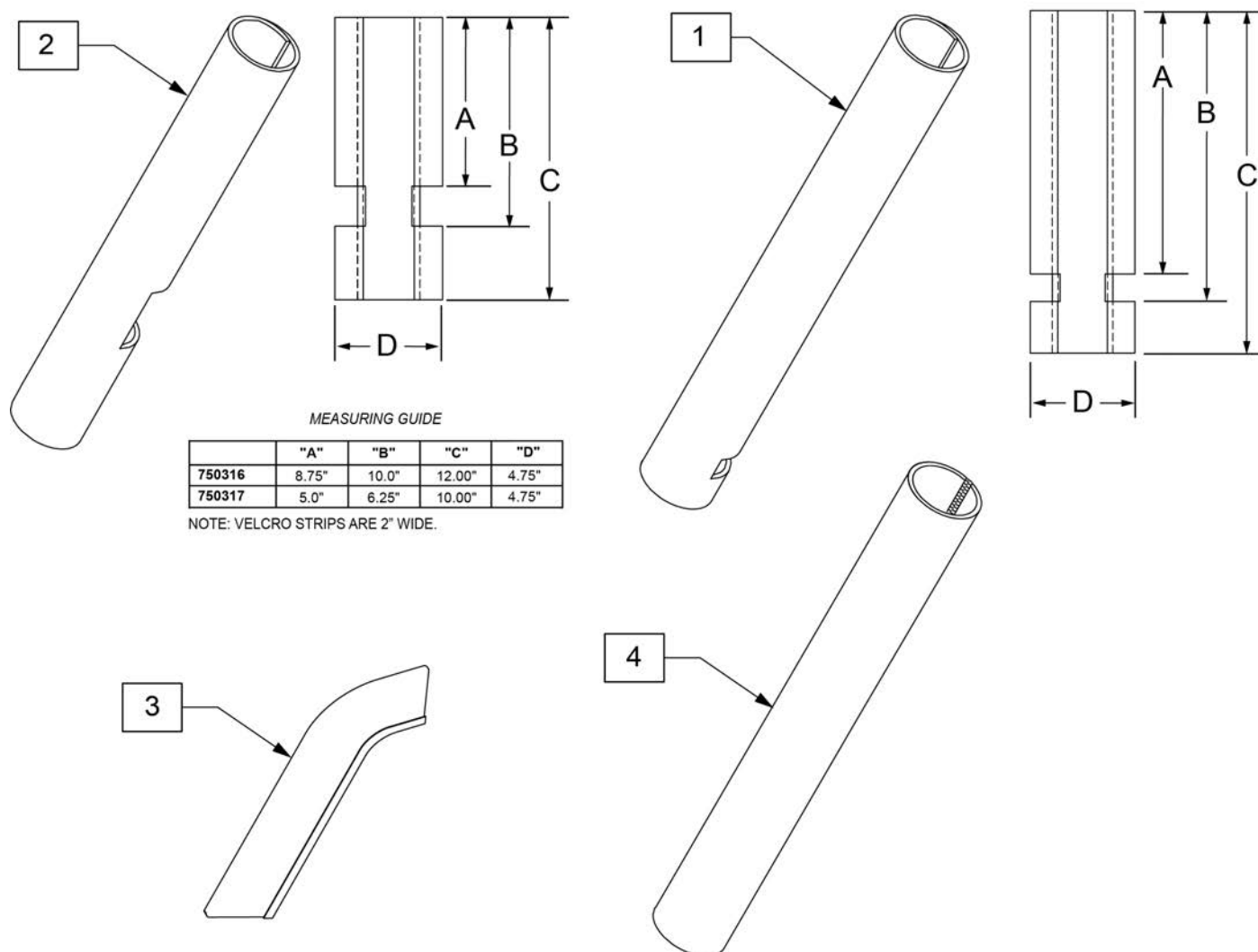

Pos.	Item Number	Description	Remarks
1	750316S	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317S	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231S	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame
4	108315S	GEL PAD SA ARM	Single, fits Padded SA Armrest

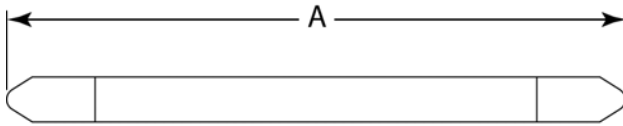MEASURING GUIDE

DESCRIPTION	DIM "A"
LEG STRAP 11"-12" BLK	27 1/2
LEG STRAP 13"-15" BLK	32 1/2
LEG STRAP 16"-18" BLK	39 1/2
LEG STRAP 19"-20" BLK	45 1/2

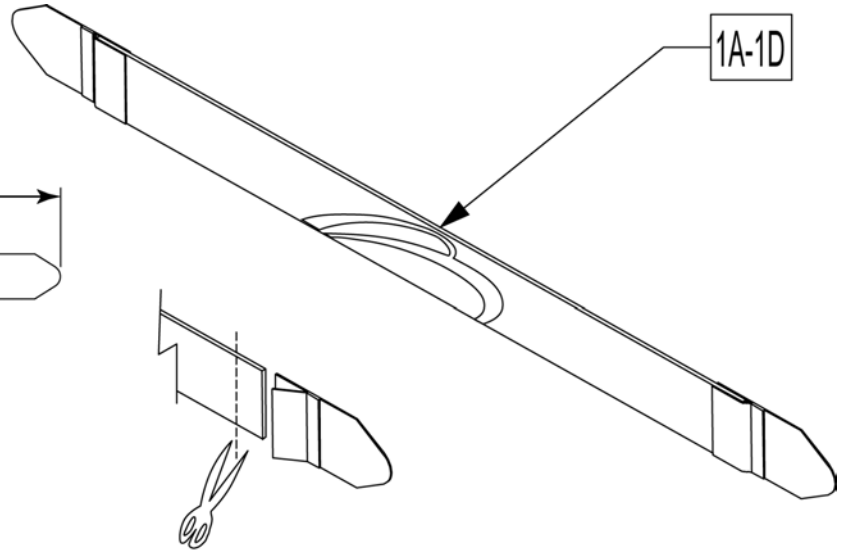


Leg Straps may be custom cut to a smaller size



Pos.	Item Number	Description
1A	113255S	LEG STRAP 11"-12" BLK
1B	113256S	LEG STRAP 13"-15" BLK
1C	113257S	LEG STRAP 16"-18" BLK
1D	113258S	LEG STRAP 19"-20" BLK

Note: Not available with 5 Spoke Mag or Xtender wheels. Not available with Projection style handrims.

1A

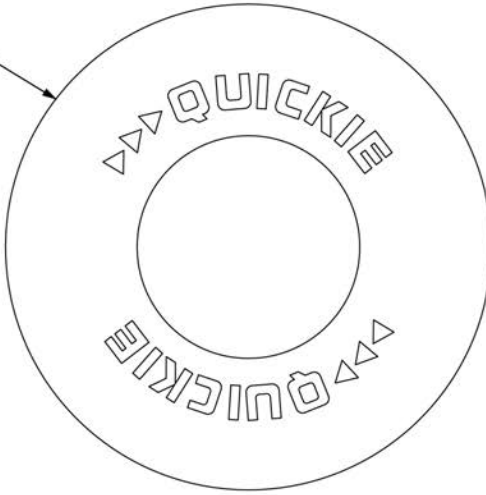

1B

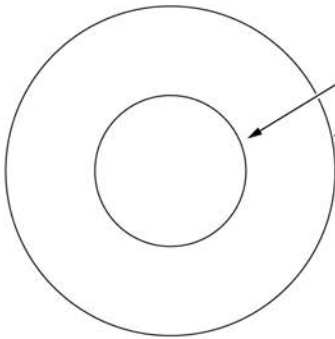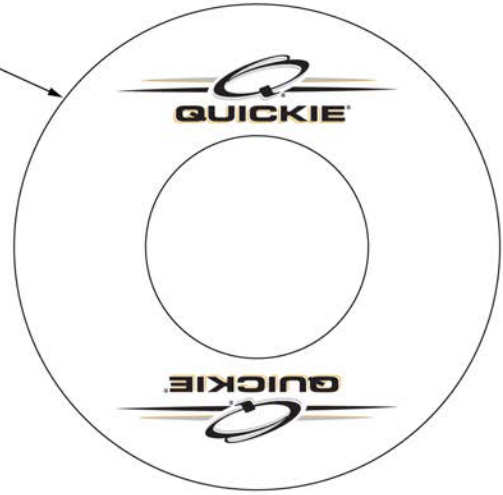

INSIDE DIAMETER


OUTSIDE DIAMETER

SPOKE GUARD MEASURING GUIDE


PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
902037	10 1/4	21 7/8
902038	10 1/4	23 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2
103734-001	10 1/4	19 1/4
103734-002	10 1/4	22
119336	10 1/4	19 1/4

Pos.	Item Number	Description	Remarks
>>>>		BLACK SPOKE GUARDS	
1A	902001	SPK GRDS 20" QUICKIE BLK	PR-Black-fits 20" Whls
1A	902011	SPK GRDS 22" QUICKIE BLK	PR-Black-fits 22" Whls, 24" (540) Whls w/ Proj HR, 24" Mountain Spoke Whls, M6 24" (540) Spoke Whls
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	PR-Black-fits 24" (540) Mag & Spoke Whls
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	PR-Black-fits 25" (559) Spk Whls & 26" Mountain Spoke Whls
1A	902037	SPK GRD 26" QUICKIE TAB MT BLK	PR-Black-fits 26" (590) Whls
1A	902038	SPK GRD 26" QUICKIE HT BLK	PR-Black-fits 27" (700) Whls
>>>>		CLEAR SPOKE GUARDS	
1B	103660-001S	CLEAR SPOKE GUARD 20"	PR-Clear-fits 20" Whls
1B	103660-002S	CLEAR SPOKE GUARD 22"	PR-Clear-fits 22" Whls, 24" (540) Whls w/ Proj HR, 24" Mountain Spoke Whls, M6 24" (540) Spoke Whls


Note: Water Bottle Cage clamp fits 3/4" to 1 1/2" round tubing, 7R/Q7 and Xenon oval tubing as well as Double "D" tubing; Max bottle diameter is 3 3/4".
 Note: Use rubbing alcohol to clean area of frame where adhesive rubber strip (item 3) is to be located prior to instalation to insure best adhesion.

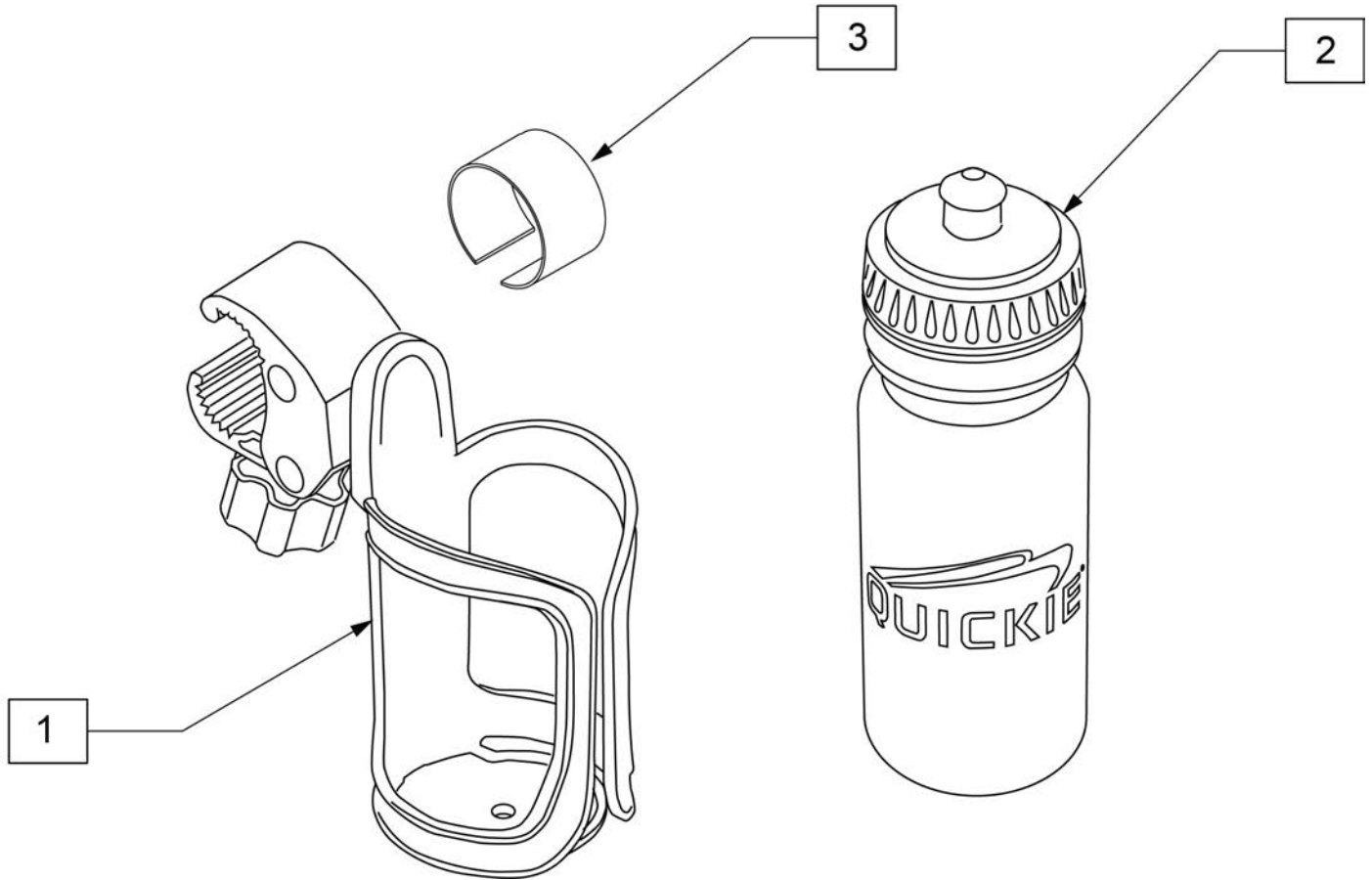


Pos.	Item Number	Description	Remarks
1	252135	WATER BOTTLE CAGE W/ CLAMP	Black
2	252136	QUICKIE WATER BOTTLE	Black
3	123526	ADHESIVE RUBBER PAD	
1-3	163680	WATER BOTTLE W/ CAGE AND CLAMP	

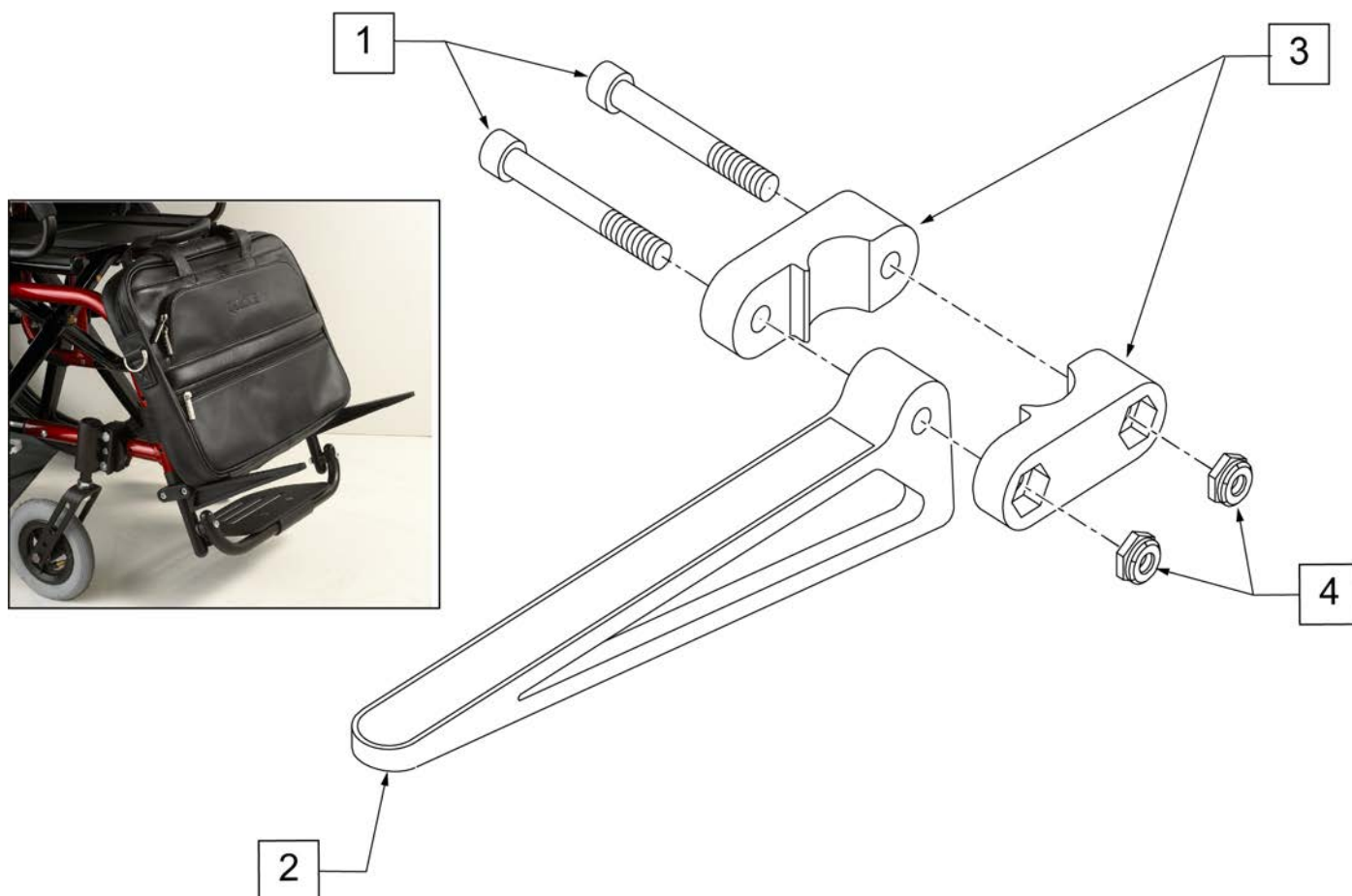

Pos.	Item Number	Description	Remarks
1	106975S	1/4-28 x 1 1/4 SOC C/S	
2	561105S	CADDY ARM	Clamps on to Extension Tube (3/4" round and D2); Caddy Arm length is 7 1/2"
3	561104S	CADDY CLAMP	Clamps on to Extension Tube (3/4" round and D2)
4	106935	1/4-28 NYLOCK NUT THIN	
1-4	902350S	QUICKIE CADDY (PR)	

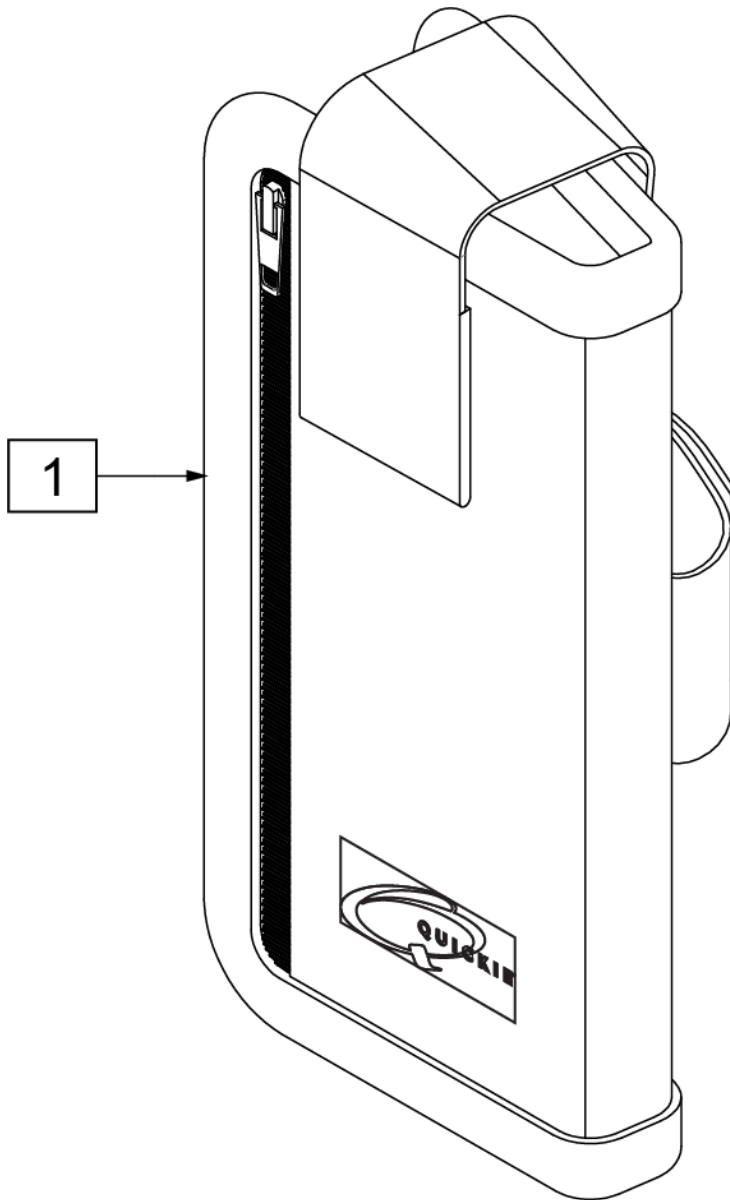


Measuring Guide

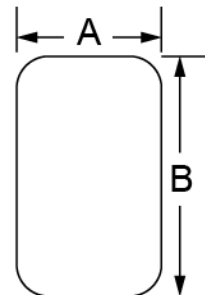

A	B
2 3/4"	6"

Pos.	Item Number	Description
1	000670004	CELLPHONE CASE W/LOGO

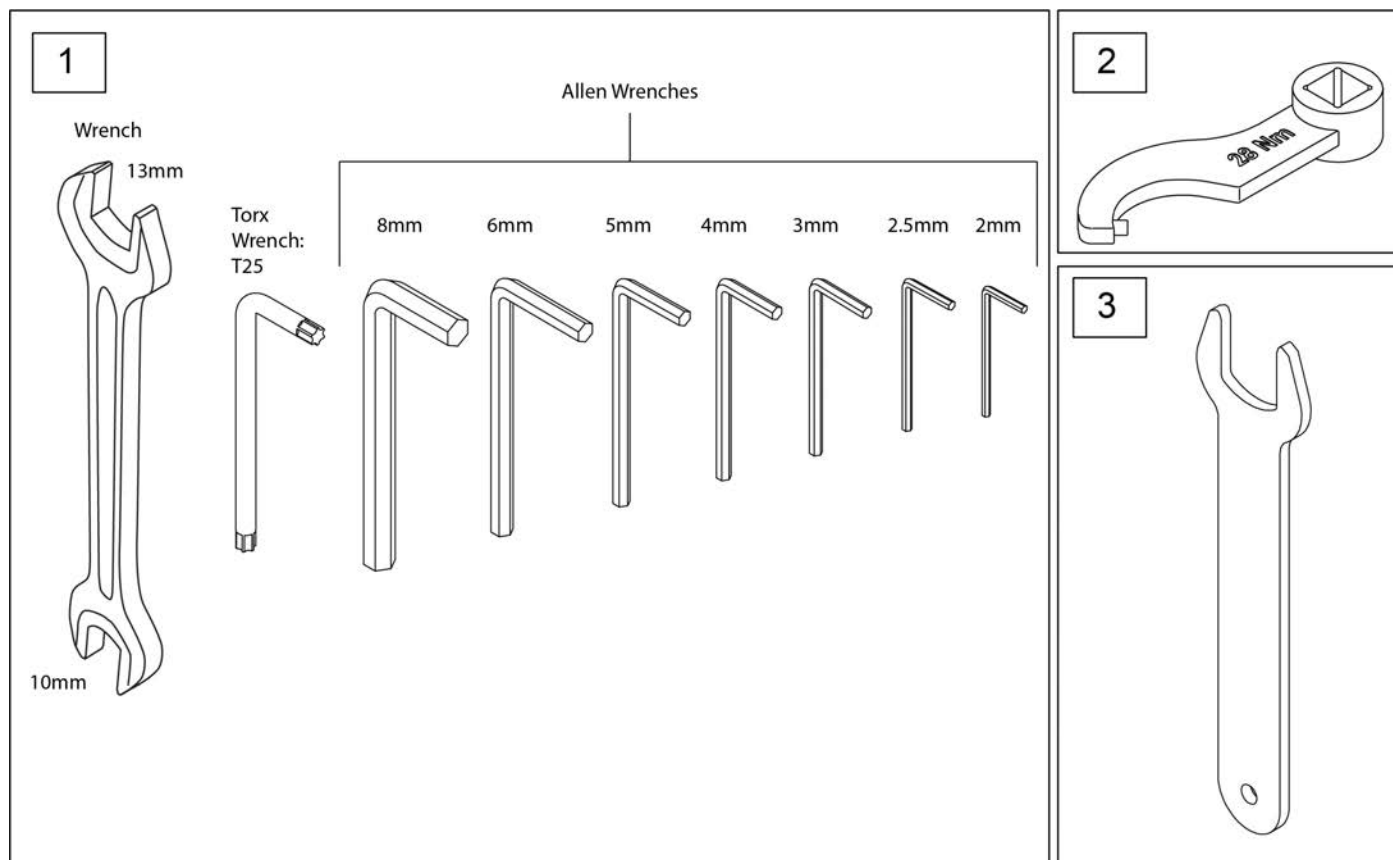

Pos.	Item Number	Description
1	000604270	TOOL SET
2	66009203S	torque wrench welding
3	66009205S	spanner wrench AF18