JAY®

J3 CUSHION

ROHO® DRY FLOATATION® INSERT

IMPORTANT CONSUMER INFORMATION

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NOTICE: This owner's manual is a supplement to the J3 Cushion Owner's Manual. The J3 Cushion owner's manual should be referenced in conjunction with this owner's manual prior to the installation, set-up or use of this product.

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end-user.

SUPPLIER: This manual must be given to the user of this product

USER: Before using this product, read this entire manual, and the J3 Cushion manual, and save the instructions for future reference.

GENERAL WARNINGS

WARNING

CHECK INFLATION AT LEAST ONCE A DAY!

WARNING

UNDER-INFLATION: DO NOT use an under-inflated cushion. Using a cushion that is under-inflated reduces or eliminates the cushion's benefits, increasing the risk to the skin and other soft tissue. If your cushion appears under-inflated or does not appear to be holding air, check to make sure that all valve(s) are closed by turning the inflation valve(s) clockwise. If your cushion is still not holding air, contact your clinical caregiver, supplier immediately.

WARNING

OVER-INFLATION: DO NOT use an over-inflated cushion. Using an over-inflated cushion will not allow you to sink into the cushion and will reduce or eliminate the cushion's benefits increasing risk to skin and other soft tissue.

WARNING

CUSHION AND COVER ORIENTATION: Product must be used with the air cells facing up.

WARNING

PRESSURE: Changes in altitude may require adjustment to your cushion. Check your cushion when changing altitude of 1,000 feet (300 meters) or greater.

WARNING

PUNCTURE: Keep your cushion away from sharp objects which may puncture the air cells and cause it to deflate.

WARNING

HEAT/FLAME: DO NOT expose your cushion to high heat, open flames or hot ashes.

WARNING

CLIMATE: If a cushion has been in temperatures less than 32°F / 0°C and experiences unusual stiffness, allow the cushion to warm to room temperature, then open air valve. Roll up the pad and then unroll it until the neoprene is soft and pliable again. Then repeat proper adjustment instructions before using.

WARNING

DO NOT use on top or in conjunction with another cushioning product other than the J3 Cushion with which it was supplied.

WARNING

OBSTRUCTIONS: DO NOT place any obstructions between the user and the cushion, as it will reduce product effectiveness.

WARNING

HANDLING: DO NOT use any valve as a handle for carrying or pulling the cushion.

WARNING

PRODUCT USE: DO NOT use your cushion as a water floatation device (e.g., a Life Preserver).

CAUTION– SOLUTIONS: DO NOT ALLOW OIL-BASED LOTIONS OR LANOLIN TO COME IN CONTACT WITH YOUR CUSHION, AS THEY MAY DEGRADE THE MATERIAL.

CAUTION– OZONE GENERATORS: THE US ENVIRONMENTAL PROTECTION AGENCY, HEALTH CANADA AND OTHER GOVERNMENT AGENCIES HAVE ISSUED WARNINGS REGARDING THE USE OF OZONE GENERATORS. PROLONGED EXPOSURE TO OZONE WILL DEGRADE RUBBER, FABRICS AND OTHER MATERIALS USED IN THE MANUFACTURE OF THE ROHO PRODUCT AND MAY AFFECT THE PERFORMANCE OF THE PRODUCT AND VOID THE PRODUCT WARRANTY.


CAUTION– DO NOT USE A PUMP OR REPAIR KIT OTHER THAN THOSE PROVIDED BY EITHER ROHO, INC. OR SUNRISE MEDICAL. DOING SO MAY VOID YOUR PRODUCT'S WARRANTY.

NOTICE: Please reference the J3 Cushion Owner's Manual for set-up and adjustment of the cushion. The following instructions pertain to the installation and adjustment of the DRY FLOATATION® Insert only.

INSTALLATION OF THE INFLATION TUBE(S)

Some configurations of the J3 Cushion with the DRY FLOATATION® insert come with the inflation tube(s) un-installed. If this is the case, you will have received the following parts:


- DRY FLOATATION® insert with a tube coupler at the end of the inserts truncated inflation tube
- A length of tube with one end terminated by an inflation valve, the other end open. A second tube will be included if the DRY FLOATATION® insert was a dual chamber insert. You can identify if the insert is single or dual chamber by the truncated inflation tubes at the rear of the insert. A coupling connector has already been installed in the truncated inflation tubes.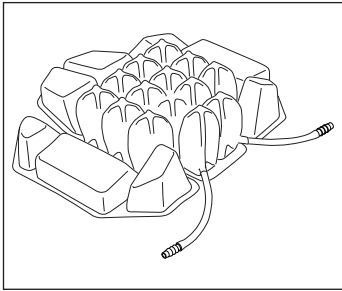


To install the inflation tube(s)

1. Install the DRY FLOATATION® insert into the J3 foam base as outlined in the instructions below.

2. Measure the length of inflation tube required for your specific cushion size. Do this by measuring the distance (l) from the front edge of the truncated inflation tube on the insert, to 1/2" (13mm) from the front edge of the cushion. You can use a flexible tape measure or string to take this measurement. (Fig. 1)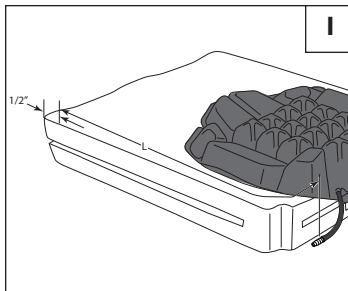

3. On the length of inflation tube, measure the distance (l) from the front of the valve to a location on the inflation tube that matches the measurement taken in step 2. (Fig. 2)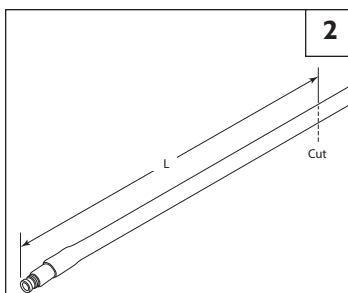


4. Cut the length of inflation tube as measured using scissors.

5. Attach the cut length of inflation tube to the coupling connector. Then follow the installation instructions Steps 2-6 following.


INSTALLATION OF DRY FLOATATION® INSERT INTO FOAM CUSHION

1. With the cover removed from the J3 foam base, install the DRY FLOATATION® insert into the Opti-Well. Ensure that the DRY FLOATATION® insert is aligned with the well and that the inflation tubes are to the rear of the cushion.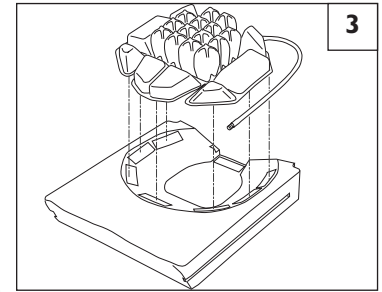


Ensure that the Velcro attached to the DRY FLOATATION® insert is aligned with the matching Velcro adhered to the foam base. If a PLA insert is being used, see the J3 Owner's Manual for instructions on its installation. (Fig. 3)

2. With the DRY FLOATATION® insert installed, the inflation tube(s) need to be secured to the foam base. Your DRY FLOATATION® insert may have one or two inflation tubes depending if it is a single chamber insert or a dual chamber insert. Two channels on the foam base are used to locate the inflation tube(s). (Fig. 4)

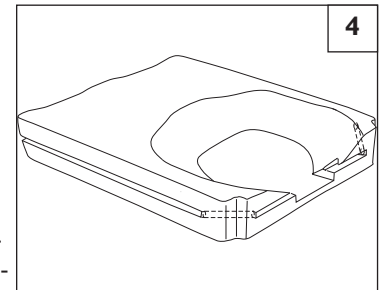
3. Begin by securing the inflation tube(s) in the channel in the rear of the foam cushion, then threading it through the hole which joins the back channel to the side channel. Once through, secure the remainder of the inflation tube to the side channel. (Fig. 5)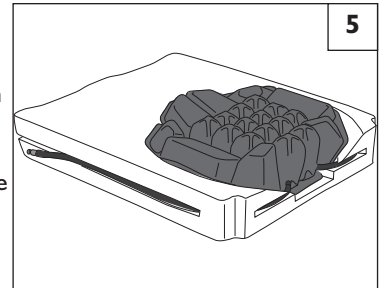

4. If the DRY FLOATATION® insert is dual chambered, do the same for the second inflation tube.
5. Re-install the cover by sliding the cover over the foam assembly. It is best to secure the inflation valve in place with your hand as you slide the cover over the assembly with your other hand.

6. Insert the valve through the side slot located in the cover behind the Jay logo. The valve is intended to protrude through the slot and be secured by the side band on the J3 Cushion cover. (Fig. 6)




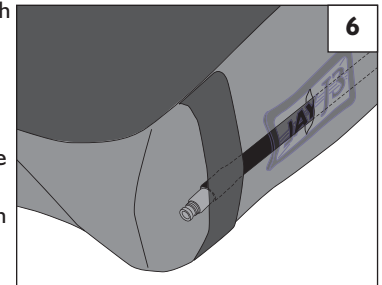


ADJUSTMENT

1. Open the air valve by twisting it counterclockwise. If your cushion comes with two valves, open both valves using the same technique. (Fig. 7)
2. Attach the rubber hose on the ROHO Pump to the end of the valve. Inflate the cushion insert with the pump until just before the cushion begins to arch. (Fig. 8)
3. Remove the pump hose from the valve and close it by twisting it clockwise. (Fig. 9)
4. Sit on the J3 Cushion, with the DRY FLOATATION® insert installed, in your normal sitting position with armrests and footrests properly positioned. (Fig. 10)
5. Slide your hand beneath your buttocks to locate your lowest bony prominence, usually the ischial tuberosities. If necessary, your clinician or caregiver should assist you. (Fig. 11)
6. Leaving one hand in place beneath the lowest bony prominence, use the other hand to open the valve. As air is released from the cushion insert, you will begin to sink into it. When your fingertips touch its base, close the valve. Approximately 1/2" (13 mm) of air should be between your lowest bony prominence and the base of the cushion. (Fig. 12)

IMPORTANT ADJUSTMENT NOTES

- Your J3 Cushion with DRY FLOATATION® insert may have been ordered with two valves, one on each side of the cushion. If you have two valves, steps 1-6 above should be repeated for each valve. Two valves indicate that your cushion insert is divided in the middle. You, your clinician or caregiver should locate the lowest bony prominence on the same side as the air valve used to adjust the cushion.
- You, your clinicians or caregivers should check the cushion insert daily to ensure that you have not bottomed out (ie. lost air and are now sitting on the base of the cushion). If you have bottomed out, the protective value of the cushion insert is lost. You should also check to be sure the cushion insert is not over-filled with air. The cushion insert will work best with approximately 1/2" (13mm) of air between you and its base.
- The DRY FLOATATION® insert may be affected by changes in altitude. You may have to repeat steps 1-6 above as you change altitudes.

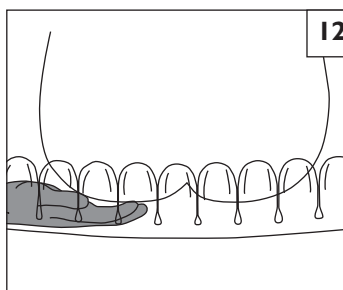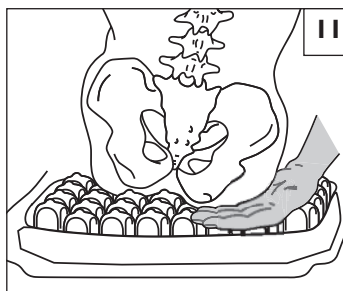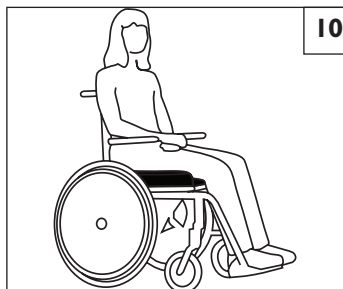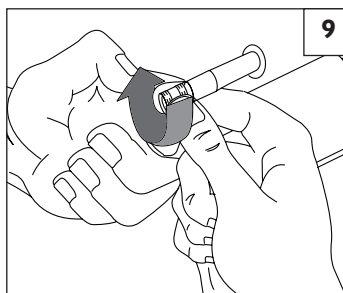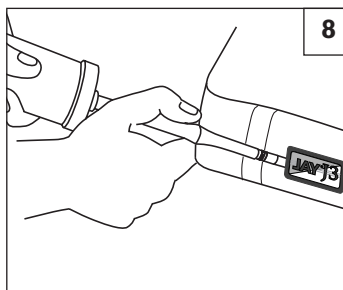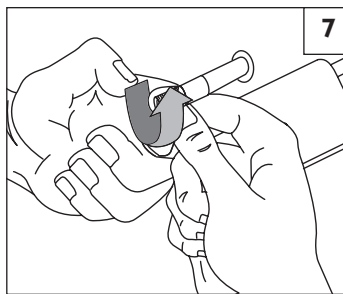CLEANING AND DISINFECTING YOUR J3 DRY FLOATATION® INSERT

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

CLEANING THE DRY FLOATATION® INSERT

To clean the DRY FLOATATION® insert, gently pull it out of the recess in the foam base, taking care to gently pull the valve through the foam. Deflate the cushion, close the valve(s) and place in a large sink. Gently scrub the air cells and the spaces between them with a soft plastic bristle brush, sponge or washcloth. Liquid dish-washing or laundry detergent or other general-purpose cleaning product is suggested. For institutional cleaning, a general house-keeping disinfectant cleaner may be used.

DISINFECTING THE DRY FLOATATION® INSERT

To disinfect the DRY FLOATATION® Insert, use a germicidal product that meets infection control needs. A disinfectant with tuberculocidal properties is recommended. Follow the manufacturer's dilution directions. For home disinfection, use solutions of approximately 1/2 cup liquid household bleach per quart of water (125 ml bleach to 1 liter of water). Keep the clean cushion wet with bleach solution for 10 minutes. Rinse thoroughly and dry.

WARNING

Disinfectants are not effective on porous surfaces, such as foam, if the cushion becomes soiled or contaminated. DO NOT use with multiple users.

CAUTION— DO NOT USE OIL-BASED LOTIONS OR LANOLIN ON YOUR CUSHION AS THEY MAY COMPROMISE THE INTEGRITY OF THE NEOPRENE RUBBER.

REPAIRS

Each J3 Cushion with a Dry Floation® insert is shipped complete with a repair kit and instructions for minor repairs. The following information can be used if repair instructions have been lost or misplaced.

If A Leak Occurs

1. Place your cushion cells down and over-inflate until all four corners touch.
2. Immerse the over-inflated cushion in water to locate the leak.
3. Once the leak has been located, mark the leak by inserting a toothpick into the hole
4. Allow the cushion to dry thoroughly.
5. Remove the toothpick, and clean the area around the hole with the alcohol wipe provided. Let the surface dry.
6. Peel the backing from the patch provided and, centering the patch, place it over the hole.
7. Press the patch down firmly until a good seal is achieved.
8. The cushion is now ready for use. Adjust the cushion for proper inflation following the instructions in this manual.

Although patches do not change the performance of the cushion, they should be monitored periodically to ensure effectiveness in addressing the leak. If the patch is not successful in addressing the problem, or if more extensive damage is present, please contact your supplier immediately.

Replacing Valves

1. Firmly grip the rubber valve stem and remove the old inflation valve by using pliers.

CAUTION— DO NOT REMOVE AN INFLATION VALVE BY HOLDING THE CUSHION AND PULLING THE VALVE, DOING SO MAY TEAR THE CUSHION.

2. Moisten the ribbed end of the replacement valve and insert it into the valve stem until the rubber is snug to the body of the valve.

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Sunrise Medical AB
Sunrise Medical B.V.