Jay[®] Focus Point Back

IMPORTANT CONSUMER INFORMATION

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

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Manuel d'instructions

SUNRISE MEDICAL LISTENS

Thank you for choosing a JAY Focus Back Support System. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Authorized Sunrise dealer. Please feel free to write or call us at the address and telephone number below:

Sunrise Medical (US) LLC
Customer Service Department
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA
(800) 333-4000

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this wheelchair. If you lose your warranty card, call or write and we will gladly send you a new one.

FOR ANSWERS TO YOUR QUESTIONS

Your Authorized Dealer knows your product best, and can answer most of your questions about safety, use and maintenance. For future reference, fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this product was made with Natural Rubber Latex.

**DISPOSAL AND RECYCLING INFORMATION**

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This product is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your wheelchair properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations

ENGLISH

Introduction.....	3
General Warnings.....	3
Use and Maintenance	4
Installation & Adjustment.....	5
Warranty.....	7

ESPAÑOL**FRANÇAIS****NOTICE– READ BEFORE USE****A. CHOOSE THE RIGHT SUPPORT & SAFETY OPTIONS**

This product is intended for single person use only. Selection of the type of support systems, options, and final adjustment rests solely with the user and their health care professional. Choosing the best product and set-up for safety and health depends on such things as:

1. Type of disability, strength, balance and coordination.
2. The types of hazards that must be overcome in daily use.
3. The need for options that will improve your positioning, safety and comfort (such as positioning belts, or special seating systems).

B. REVIEW THIS MANUAL OFTEN

Before using this system, you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

C. WARNINGS

The word “**WARNING**” refers to a hazard or unsafe practice that may cause **severe injury or death** to you or to other persons. The “Warnings” are in Three main sections, as follows:

1. GENERAL WARNINGS

A safety checklist and a summary of risks you need to be aware of before you use this equipment.

2. SAFETY WARNINGS: FALLS & TIP-OVERS

Learn about the safe use of your equipment, and how to avoid a fall or tip-over while performing daily activities.

3. WARNINGS– COMPONENTS & OPTIONS

Learn about the components and options that can be selected for safety and Positioning needs. Consult your Authorized Supplier and your Health care advisor to help you choose the best set-up and options for safe use.

NOTE - Where they apply, you will also find “Warnings” in other sections of this manual. Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

THE JAY FOCUS POINT BACK

The JAY Focus Point is designed to provide the important benefits of a proper back support for a wide variety of users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is lightweight, durable and easy to use.

Maximum user weight: 250 lbs. (113 kg).

III. GENERAL WARNINGS

A. CENTER OF GRAVITY

WARNING

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair, and may cause the wheelchair to tip backwards potentially resulting in injury to the rider.

B. ANTI-TIPS

WARNING

Anti-tip tubes can help keep your chair from tipping over backward in normal conditions.

1. Sunrise recommends the use of anti-tip tubes
2. ALWAYS assess the need for anti-tips or an amputee axle adapter brackets that could be added to the wheelchair to help increase the stability of the wheelchair and seating system.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

C. PUSHING OR LIFTING

WARNING

Do not use the Back to move or push the wheelchair.

1. Do not use any part of the back to lift the wheelchair. It is intended only for Back removal.
2. Improper use of the back by pushing or pulling may cause the back to unexpectedly detach from the wheelchair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

D. COMPATIBILITY

WARNING

The JAY Focus Point Back is designed to be compatible with most wheelchairs with the following exception:

1. Wheelchairs that are angle adjustable with recline or tilt, and result in a back angle greater than 60° from the vertical.

Basic back use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair. This could result in a fall or potential injury of the rider and/or caregiver.

⚠ WARNING

The owner of this equipment is responsible for making sure that it has been set up and adjusted by a Authorized Dealer under the advice of a healthcare professional. It may require periodic safety and function checks or certain in-use adjustments that can be performed by the owner, caregiver or Authorized Dealer if desired.

A. MAINTENANCE AND CLEANING

1. Proper maintenance and cleaning will improve performance and extend the useful life of your equipment.
2. Clean your equipment regularly. This will help you find loose or worn parts and make your equipment easier to use. You will need a mild detergent solution and cleaning rags.
3. If discovered, have loose, worn, bent or damaged parts replaced **before using the back support**.

B. CRITICAL SAFETY CHECKS

1. Fasteners:

Sunrise Medical recommends that all fasteners be checked for wear, such as loose bolts or broken components **Every 6 months**. Loose fasteners should be retightened according to the installation instructions.

2. Replacing components:

Contact your Authorized Dealer immediately to replace any components. Do not continue to use the system after identifying loose or broken components.

C. EASY CLEANING 1

1. Back Cover

- a. Remove the 3-part cover from the Focus Point Back.
- b. Remove the foam and turn the cover inside out.
- c. Zip the cover closed to avoid snagging.
- d. Machine wash in cold water.
- e. Air dry.

NOTE – Do not dry clean covers or use industrial washers or dryers to clean covers. Machine drying is not recommended. Do not steam autoclave. Do not bleach. Laundry care symbol definitions are shown (Fig. 1).

2. Foam Inserts

- a. Wipe with a damp cloth
- b. Allow to dry before reinstalling the cover.

3. Back Frame and Laterals

- a. Wipe clean with soap and water.

NOTE - Be sure not to use harsh abrasives or cleaners on any part of the Focus Point Back frame.

NOTE - The ABS plastic support shell is designed to be placed inside the cover to provide additional support to the back.

Laundry Care Symbols		1
	Machine wash in cold water (30° C)	
	Only non-chlorine bleach, when needed	
	Drip dry	
	Do not iron	
	Do not dry clean	

⚠ WARNING



Improper installation of the Focus Point Back and/or any of its hardware components, may result in injury. Be sure to follow all instructions for re-installation should the back be removed from the wheelchair.

A. VERIFYING BACK SIZE

Measure your wheelchair post width from the outside of one post to the outside of the other post. Verify that you have the proper width Jay Focus Point Back for your chair. Begin by removing armrests and any upholstery from your wheelchair's upright posts. Place whatever cushion you plan to use into your wheel-chair (preferably a Jay® J2 Recline or Jay® J2 Deep Contour Cushion).

B. INSTALLING THE BOTTOM POST CLAMP 2

1. Remove the screw (A) and nut (B) securing the clamp.
2. Two spacers (C) are provided for installing the clamp. If required, install the spacer onto the backrest tube.
 - If installing the backrest on 1.125" (2.9 cm) diameter backrest tube, no spacer is required.
 - If installing the backrest on 1" (2.5 cm) diameter backrest tubes, the 1" spacers are required.
 - If installing backrest on 7/8" (2.2cm) diameter backrest tubes, the 7/8" spacers are required.
3. Install the clamp onto the backrest tube with the hook cradle (D) facing upward or pin (E), as shown in figures 2 & 3. Install the clamp (around the spacer if required), so that the holes on the spacer are aligned with the holes on the clamp.
4. Install the screw through the hole on the clamp. Hold the nut in place while tightening the screw.
5. Orient the clamp, so that the hook cradle faces the front of the wheelchair.
6. Tighten the screw using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment).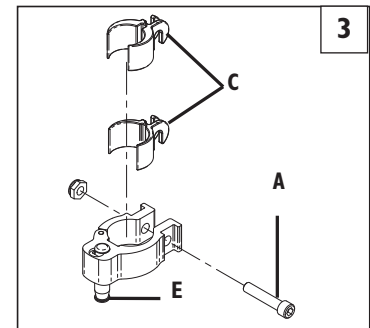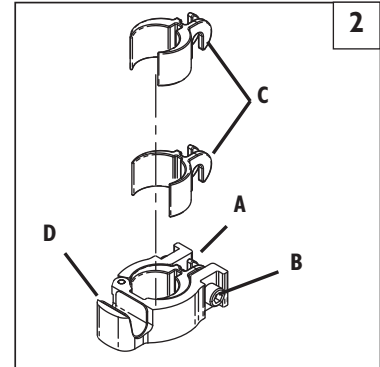


C. INSTALLING THE TOP DROP PIN CLAMP 3

1. Follow steps one through four above.
2. Orient the clamp as shown in figure 3, so that the drop pin (E) faces the front of the wheelchair.
3. Tighten the screw (A) using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment).

D. ADJUSTING FOR WHEELCHAIR WIDTH 4

The width of the hook brackets are adjustable to accommodate various widths of backrest tubes over a certain range.


1. Loosen the nuts (A) securing the left and right hinge plates. The nuts should be loose enough to allow the hinge plates to slide along the back pan.
2. Place the backrest against the wheelchair backrest tubes. Slide the hinge plates inward or outward so that the hooks will easily drop into the cradle of the bottom post clamps. Make certain that the position of the left and right hooks with respect to the backrest pan are the same (symmetrical), so that the pan is centered between the backrest tubes.
3. Tighten the nuts firmly.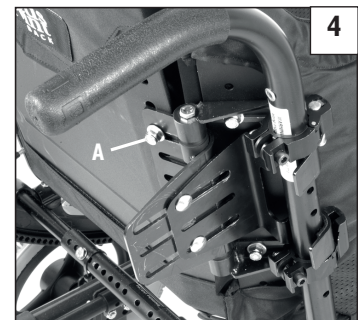

E. SETTING THE HEIGHT OF THE BACKREST 5


The Jay Focus Point Back must be mounted at the correct height on the backrest tubes. Sunrise Medical recommends that the initial fitting and positioning of the backrest take place under the supervision of a clinician, or a rehab technology supplier. The height of the backrest is adjusted by loosening the screws that secure the drop pin clamp and the bottom hook clamp.

1. Establish the height of the backrest by loosening the screws (E) that secure the bottom hook clamps. Raise or lower the clamps and re-tighten, checking that the backrest pan is at the proper height when the hooks are installed into the clamp hook cradle. Make certain that the left and right hardware is installed at the same height by measuring the height of each clamp.
2. Tighten the screws securing the bottom clamps firmly.
3. Determine the correct height of the drop pin clamps. Slide the pin in the receiver to the unlocked (raised) position. Slide the upper clamp hardware on the backrest tubes so that they reside just above the top hooks when the hooks cradle the backrest tubes. Allow for a small 1/16" (2mm) clearance between the top hooks and the drop pin clamps/quick release bracket hardware.
4. Tighten the screws firmly.

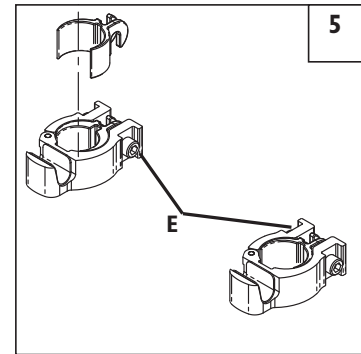**F. CHANGING THE HEIGHT POSITION OF THE FOCUS POINT BACK** 6

It is possible to change the height position of the back on the wheelchair without changing the position of the clamps. Such adjustment is needed when either of these parts is located in a position that interferes with other components mounted to the backrest tube.


1. Remove the back foam from the backrest pan.
2. If the width of the hinge plate has already been established (see instructions for width adjustment), mark their location using a pencil. This will assist in maintaining the correct width position when the brackets are moved to a different height.
3. Loosen and remove the nuts and washers (Fig. 4-A) securing the hinge plate and remove the backrest pan from the bracket.
4. Remove the nut plate (Fig. 6-B) from the holes of the backrest pan, and reposition with the hinge plate in the desired set of holes.
5. Install and secure the nuts and washers to the hinge plate and nut plate.
6. If necessary, follow the instructions for "Adjusting the Width of the Backrest".
7. Tighten the nuts firmly.

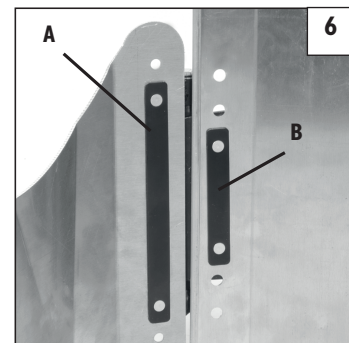**G. ADJUSTING THE DEPTH AND ANGLE** 7

The depth and angle of the Focus Point Back is easily adjusted by loosening (do not remove) the thin "jam" nuts attached to bolts A (Fig. 7).

1. Loosen both jam nuts on the left/right hinge plates.
2. Adjust Focus Point Backrest into desired position.
3. Retighten the jam nuts.

NOTE- The thin "jam" nuts used for this adjustment are unique to this application, and can not be used in other places on this product.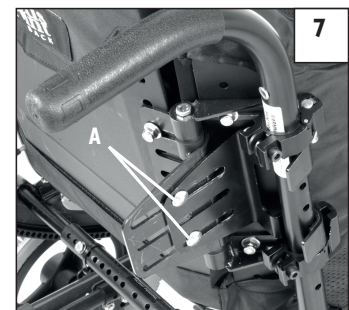


H. ADJUSTING THE LATERAL HEIGHT AND ANGLE 8

To adjust the angle of the laterals:

1. Loosen the two bolts (Fig. 8-A) just enough for the laterals to move.
2. Adjust the laterals into desired position.
3. Re-tighten bolts (A).

To adjust the height of the laterals:

1. Remove the nuts & washers (Fig. 8-B).
2. Remove lateral from backshell.
3. Remove the nut plate (Fig. 6-A) from the holes of the lateral and reposition.
4. Re-install and secure the nuts and washers (Fig. 8-B)

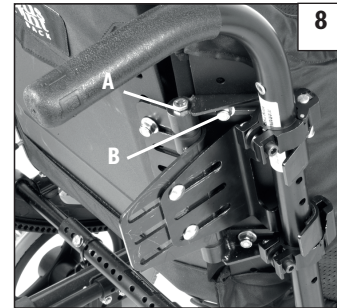I. RELEASING THE BACKREST FROM THE WHEELCHAIR

The Focus Point Back is designed so that the backrest can easily be removed from the backrest tubes. The drop pins must always be in their locked (lowered) position when the chair is in use. Improper use may cause the backrest to unexpectedly detach from the wheelchair. Never attempt to lift the wheelchair by lifting on the backrest pan or supporting hardware.

To remove the back:

1. Slide the drop pins to the unlocked (raised) position.
2. Remove the back by pulling forward and upward on the backrest pan.

To install the back:

1. Insert the bottom hooks into the hook cradles. Rotate the backrest so that the top hooks cradle the backrest tube.
2. Slide the drop pins to the locked (lowered) position so that the hooks are captured behind the pins.

VI. WARRANTY

24 MONTH LIMITED WARRANTY

1. Each Jay Focus Point Back is carefully inspected and tested to provide peak performance. Every Jay Focus Point Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the Back's removable cover.

2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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Sunrise Medical Canada Inc.