# Jay® Focus Point Back

Owner's Manual

# El Respaldo Focus Point Jay®

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# Le Dossier Jay® Focus Point

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**JAY®**

## HOW TO FIT THE JAY FOCUS POINT BACK

The Jay Focus Point Back is a versatile back designed to replace the sling upholstery. When properly utilized the back should:

- Offer you thoracics support and comfort
- Promote trunk alignment and lateral stability
- When used as a replacement for your existing sling upholstery, offer moderate shear reduction.
- Maximum user weight: 250 lbs. (113 kg.)

**CAUTION**— Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

### Verifying Back Size

Measure your wheelchair post width from the outside of one post to the outside of the other post. Verify that you have the proper width Jay Focus Point Back for your chair. Begin by removing armrests and any upholstery from your wheelchair's upright posts. Place whatever cushion you plan to use into your wheelchair (preferably a Jay® J2 Recline or Jay® J2 Deep Contour Cushion).

## INSTRUCTIONS FOR YOUR FOCUS POINT BACK

Thank you for selecting the Focus Point Back for your seating needs. By following the simple guidelines listed below, the Focus Point Back should provide you with years of maintenance-free comfort. Be sure to talk with your rehabilitation provider should you have any questions or concerns about the Focus Point Back.

**CAUTION**- Improper installation of the Focus Point Back and/or any of its hardware components, may result in injury. Be sure to follow all instructions for reinstallation should the back be removed from the wheelchair.

## CLEANING AND MAINTENANCE INSTRUCTIONS

### To Clean the Back Cover

Remove the 3-part cover from the Focus Point Back and remove foam. Machine wash cold and allow cover to fully air dry.

**NOTE**– Machine drying is not recommended. Do not bleach.

**NOTE**– It is advisable to purchase a backup cover for use while your primary cover is being washed.

### To Clean the Foam Inserts

Wipe with a damp cloth. Allow to dry before reinstalling the cover.

### To Clean the Back Frame and Laterals

The frame of the Focus Point Back may be wiped clean with soap and water.


**NOTE**– Be sure not to use harsh abrasives or cleaners on any part of the Focus Point Back frame.

## INSTALLING THE FOCUS POINT BACK

### INSTALLING THE BOTTOM POST CLAMP (Figure 1)

Required tools: 3/16" Allen wrench

1. Remove the screw (A) and nut (B) securing the clamp.
2. Two spacers (C) are provided for installing the clamp. If required, install the spacer onto the backrest tube.
  - If installing the backrest on 1.125" (2.9 cm) diameter backrest tube, no spacer is required.
  - If installing the backrest on 1" (2.5 cm) diameter backrest tubes, the 1" spacers are required.
  - If installing backrest on 7/8" (2.2cm) diameter backrest tubes, the 7/8" spacers are required.
3. Install the clamp onto the backrest tube with the hook cradle (D) facing upward or pin (E), as shown in figures 1 & 2. Install the clamp (around the spacer if required), so that the holes on the spacer are aligned with the holes on the clamp.
4. Install the screw through the hole on the clamp. Hold the nut in place while tightening the screw.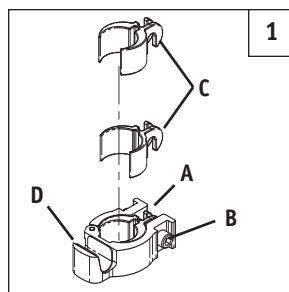


5. Orient the clamp, so that the hook cradle faces the front of the wheelchair.
6. Tighten the screw using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment on page 5).

### INSTALLING THE TOP DROP PIN CLAMP (Figure 2)

Required tools: 3/16" Allen wrench

1. Follow steps one through four above.
2. Orient the clamp as shown in figure 2, so that the drop pin (E) faces the front of the wheelchair.
3. Tighten the screw (A) using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment on page 5).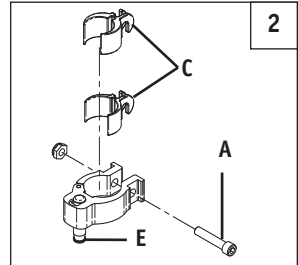

## ADJUSTING THE WIDTH OF THE BACKREST


### ADJUSTING THE WIDTH OF THE HINGE PLATE

Required tools: 7/16" wrench.

The width of the hook brackets are adjustable to accommodate various widths of backrest tubes over a certain range.

#### Adjusting for Wheelchair Width (Fig. 3)

1. Loosen the nuts (A) securing the left and right hinge plates. The nuts should be loose enough to allow the hinge plates to slide along the back pan.
2. Place the backrest against the wheelchair backrest tubes. Slide the hinge plates inward or outward so that the hooks will easily drop into the cradle of the bottom post clamps. Make certain that the position of the left and right hooks with respect to the backrest pan are the same (symmetrical), so that the pan is centered between the backrest tubes.
3. Tighten the nuts firmly.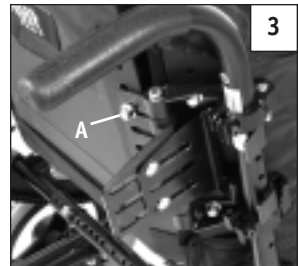


## SETTING THE HEIGHT OF THE BACKREST

### Required tools: 7/16" wrench, tape measure

The Jay Focus Point Back must be mounted at the correct height on the backrest tubes. Sunrise Medical recommends that the initial fitting and positioning of the backrest take place under the supervision of a clinician, or a rehab technology supplier. The height of the backrest is adjusted by loosening the screws that secure the drop pin clamp and the bottom hook clamp.

1. Establish the height of the backrest by loosening the screws (Fig. 4-E) that secure the bottom hook clamps. Raise or lower the clamps and retighten, checking that the backrest pan is at the proper height when the hooks are installed into the clamp hook cradle. Make certain that the left and right hardware is installed at the same height by measuring the height of each clamp.
2. Tighten the screws securing the bottom clamps firmly.
3. Determine the correct height of the drop pin clamps. Slide the pin in the receiver to the unlocked (raised) position. Slide the upper clamp hardware on the backrest tubes so that they reside just above the top hooks when the hooks cradle the backrest tubes. Allow for a small 1/16" (2mm) clearance between the top hooks and the drop pin clamps/quick release bracket hardware.
4. Tighten the screws firmly.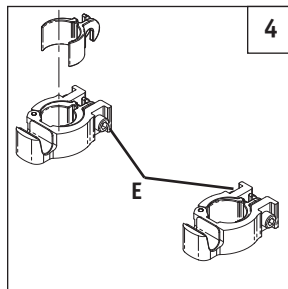


## CHANGING THE HEIGHT POSITION OF THE FOCUS POINT BACK

### Required tools: 7/16" wrench

It is possible to change the height position of the back on the wheelchair without changing the position of the clamps. Such adjustment is needed when either of these parts is located in a position that interferes with other components mounted to the backrest tube.

1. Remove the back foam from the backrest pan.
2. If the width of the hinge plate has already been established (see instructions for width adjustment), mark their location using a pencil. This will assist in maintaining the correct width position when the brackets are moved to a different height.
3. Loosen and remove the nuts and washers (Fig. 3-A) securing the hingeplate and remove the backrest pan from the bracket.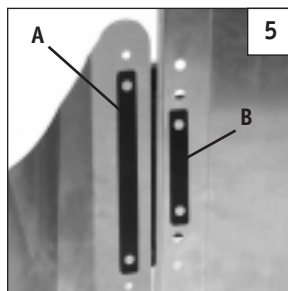


4. Remove the nut plate (Fig. 5-B) from the holes of the backrest pan, and reposition with the hinge plate in the desired set of holes.
5. Install and secure the nuts and washers to the hinge plate and nut plate.
6. If necessary, follow the instructions for "Adjusting the Width of the Backrest".
7. Tighten the nuts firmly.

## ADJUSTING THE DEPTH AND ANGLE OF THE FOCUS POINT BACK

### Required tools: 7/16" wrench

The depth and angle of the Focus Point Back is easily adjusted by loosening (do not remove) the thin "jam" nuts attached to bolts A (Fig. 6).

1. Loosen both jam nuts on the left/right hinge plates.
2. Adjust Focus Point Backrest into desired position.
3. Retighten the jam nuts.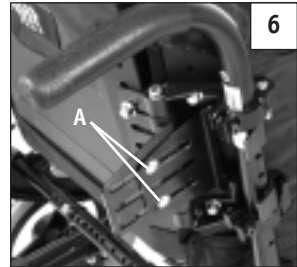

**NOTE**– The thin "jam" nuts used for this adjustment are unique to this application, and can not be used in other places on this product.

## ADJUSTING THE LATERAL HEIGHT AND ANGLE


### Required tools: 7/16" wrench

#### To adjust angle of laterals:

1. Loosen the two bolts (Fig. 7-A) just enough for the laterals to move.
2. Adjust the laterals into desired position.
3. Retighten bolts (A).

#### To adjust height of laterals:

1. Remove the nuts & washers (Fig. 7-B).
2. Remove lateral from backshell.
3. Remove the nut plate (Fig. 5-A) from the holes of the lateral and reposition.
4. Reinstall and secure the nuts and washers (Fig. 7-B)

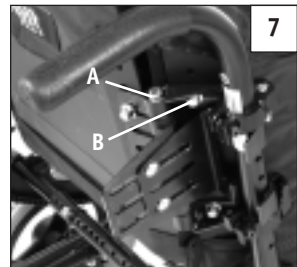## **RELEASING THE BACKREST FROM THE WHEELCHAIR**

The Focus Point Back is designed so that the backrest can easily be removed from the backrest tubes. The drop pins must always be in their locked (lowered) position when the chair is in use. Improper use may cause the backrest to unexpectedly detach from the wheelchair. Never attempt to lift the wheelchair by lifting on the backrest pan or supporting hardware.

### **To remove the back**

1. Slide the drop pins to the unlocked (raised) position.
2. Remove the back by pulling forward and upward on the backrest pan.

### **To install the back**

1. Insert the bottom hooks into the hook cradles. Rotate the backrest so that the top hooks cradle the backrest tube.
2. Slide the drop pins to the locked (lowered) position so that the hooks are captured behind the pins.

## **24 MONTH LIMITED WARRANTY**

Each Jay Focus Point Back is carefully inspected and tested to provide peak performance. Every Jay Focus Point Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the Back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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© 2006, Sunrise Medical 3.06
**Sunrise Medical, Inc.**