JAY®

LITE

JAY® Lite™ Cushion

IMPORTANT CONSUMER INFORMATION

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

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Manuel d'instructions

SUNRISE MEDICAL LISTENS

Thank you for choosing a JAY Cushion. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Authorized Sunrise dealer. Please feel free to write or call us at the address and telephone number below:

**Sunrise Medical (US) LLC
Customer Service Department
12002 Volunteer Blvd.
Mount Juliet, TN 37122
(800) 333-4000**

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this wheelchair. If you lose your warranty card, call or write and we will gladly send you a new one.

NOTE - Check all parts for shipping damage. In case of damage

DO NOT use. Contact Carrier/Sunrise for further instructions.

FOR ANSWERS TO YOUR QUESTIONS

Your Authorized Dealer knows your product best, and can answer most of your questions about safety, use and maintenance. For future reference, fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this product was made with Natural Rubber Latex.

DISPOSAL AND RECYCLING INFORMATION

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This product is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your wheelchair properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations

ENGLISH

Introduction.....2
General Warnings.....2
Use and Maintenance3
Warranty.....4

NOTICE– READ BEFORE USE

A. CHOOSE THE RIGHT CUSHION

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Cushion is appropriate for your specific needs. Cushions should only be installed by an authorized Sunrise Medical supplier.

B. CHECK SITTING PRESSURE OFTEN

Your JAY cushion was designed for sitting comfort and postural support. Users should inspect their skin daily for redness. The clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

C. REVIEW THIS MANUAL OFTEN

Before using this cushion, you and each person who may assist you, should read this entire Manual and make sure to follow all instructions. Review the Warnings often, until they are second nature to you.

D. WARNINGS

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons.

GENERAL WARNINGS

A. GENERAL WARNINGS

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Lite cushion is appropriate. Cushions should only be installed by an authorized Sunrise Medical supplier.

Caution: Persons at high risk of skin breakdown should be thoroughly evaluated by a therapist prior to purchasing the JAY Lite cushion. The JAY Lite cushion is no substitute for good skin care, including proper diet and cleanliness. The cushion should be tried for a few hours while a therapist or doctor inspects the skin to ensure that red pressure spots are not developing.

If skin develops redness, discontinue the use of the cushion and consult with a doctor or therapist. In addition, avoid sharp objects or exposure to excessive heat or open flame. If the cushion is left outside overnight at temperatures below 40° F (5° C), allow the cushion to warm to room temperature before using. Sitting on excessively hot or cold surfaces may cause skin damage.

Do not install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions, and/or Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.

⚠ WARNING

Hook and loop Velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro® to hold the cushion in place.

⚠ WARNING

Installing a cushion on a wheelchair may effect the center of gravity of the wheelchair. Installed incorrectly, it may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess if the addition of anti-tips, and/or an amputee axle adapter brackets may be needed to help increase stability after adding a cushion.

⚠ WARNING

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

⚠ WARNING

Your JAY cushion is designed to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The JAY cushion is not a substitute for good skin care including, proper diet, cleanliness, and regular pressure relief. Be sure that the Velcro is engaged and able to hold the cushion in place.

⚠ WARNING

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and

improper positioning may result. Replace stretched upholstery or purchase the JAY Adjustable Solid Seat for more support.

⚠ WARNING

Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise Medical that is appropriate for you. Contact an authorized supplier for a free demonstration of these back support products.

B. PROTECTING YOUR CUSHION

⚠ WARNING

Always avoid exposing your cushion to sharp objects, excessive heat or open flame, and prolonged exposure to environmental conditions like freezing temperatures and/or direct sunlight.

C. OBSTRUCTIONS

⚠ WARNING

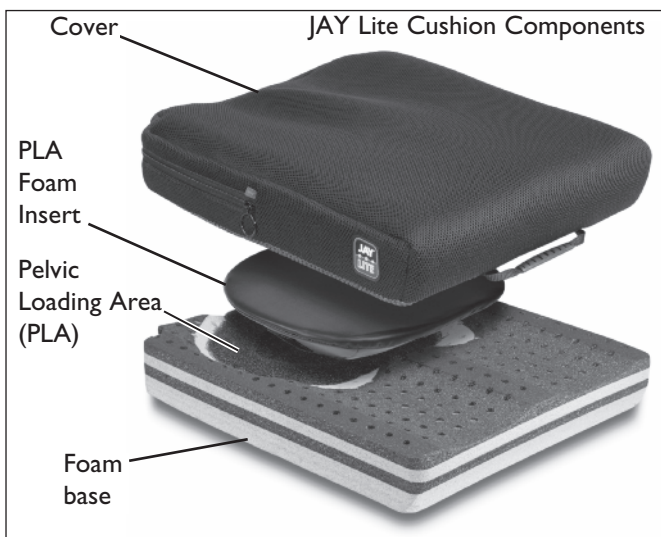
Make sure that there are never any obstructions between you and your cushion. Any obstruction will reduce product effectiveness and/or produce unnecessary pressure points for the user.

D. COVER ORIENTATION

⚠ WARNING

Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase the risk of skin and soft tissue injuries.

USE AND MAINTENANCE

THE JAY®LITE CUSHION

The JAY Lite Cushion is designed to have all the benefits of skin protection in a lightweight, comfortable and low-maintenance package. The JAY Lite incorporates a PLA foam insert, and oillex foam base, and has a microclimatic outer cover.

The JAY Lite cushion helps to provide pressure and shear reduction which may help reduce the incidence of decubitus ulcers. It is comfortable and doesn't restrict movement, allowing the user to change positions on the cushion.

Maximum user weight:
250 lbs (113kg).

A. FITTING THE JAY LITE CUSHION

Select a cushion to match your hip width when in the seated position. Place the cushion on the seat of the wheelchair. The cushion length should extend to within "1-2" (2.5cm-5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and pelvic loading area toward the rear. When fit correctly, the user's hips should be to the back of the chair. The Ischials(seat bones) should be centered in the pelvic loading area of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. The footrests should be adjusted low enough to allow the legs to rest firmly on the front of the Cushion.

NOTE - Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.

Assess the client. Are the hips level? Are the legs in good position? Is the client comfortable? If the answer to any of these questions is "no," you may want to explore other accessories to optimize the user's seated support, positioning, and comfort.

B. EASY MAINTENANCE AND CLEANING

Monthly cleaning and regular maintenance may help extend the life of your cushion. Before, during, and after cleaning, component inspection is recommended.

Check the cover for tears and excessive wear and or any other abnormalities.

Check the foam base to ensure foam consistency. Check the pelvic loading area to ensure ritculated foam consistency.

Cleaning the cover:

1. Remove the cover from the foam base and turn it inside out.
2. Machine wash in warm water and drip or tumble dry on low heat.

Do Not Iron the cover, excessive heat could damage the cover.

Do not dry clean covers or use industrial washers and dryers to clean the cover. Do not steam autoclave. Do not bleach.

Cleaning the foam base:

1. Remove the cover from the foam base.
2. Wipe lightly with a damp cloth. Do not use soap. Do not submerge in water.
3. Wipe off with a clean cloth. Let it completely air-dry before reassembly.

NOTE - Avoid submersion of the foam base in water.

To refit the cover after washing, Make sure the zipper side of the cover matches up with the rear of the cushion and stretch fit the cover back over the foam and PLA foam insert. Insert the foam base into the outer cover. If this cushion is utilized in an institutional setting, write the resident's name with permanent marker on the content label of the covers. This will help to ensure the return of the right cushion and cover to it's original fitted user.

Laundry Care Symbols	
	Machine wash in warm water 60° C
	Only non-chlorine bleach, when needed
	Drip Dry
	Do not Iron
	Do not dry clean

SUNRISE LIMITED WARRANTY

JAY LITE CUSHION WARRANTY

Every JAY Lite cushion base is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable outer cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Contact Sunrise Medical Customer Service at 1-800-333-4000

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to replacement or refund for a major failure & for compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality & the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights & remedies under a law in relation to the goods to which the warranty relates.

Record your serial number here for future reference:

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Sunrise Medical Canada Inc.