

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
						Jan-15
5" / 5X1.5" POLYURETHENE			Axle Position 1=Top			
STD HEIGHT FRAME			Axle Position 2=Bottom			
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
5" POLY	4.75"	20" LOW	15"	1	1	MED
5" POLY	4.75"	20" LOW	16"	2	2	MED
5" POLY	4.75"	20" FULL/PNEU	16"	2	1	MED
5"POLY	4.75"+1.5"EXT	20"LOW	17"			
5"POLY	4.75"+1.5"EXT	20" FULL/PNEU	17"			
5" POLY	4.75"+1.5"EXT	20" LOW	17.5"			
5" POLY	6"	20" LOW	16.5"	1	2	MED
5" POLY	6"	20" FULL/PNEU	16.5"	1	2	MED
5" POLY	4.75"+1.5" EXT	20" LOW	17"	2	2	MED
5" POLY	4.75"+1.5" EXT	20" FULL/PNEU	17.5"	2	2	MED
5" POLY	6" + 1.5" EXT	20" FULL/PNEU	18"			
5" POLY	4.75"	22" LOW	16"	2	1	MED
5" POLY	6"	22" LOW	16.5"	1	1	MED
5" POLY	4.75" +1.5 EXT	22" LOW	17"	2	2	MED
5" POLY	4.75" +1.5 EXT	22" FULL/PNEU	17"	1	1	MED
5" POLY	4.75" +1.5 EXT	22" LOW	17.5"			
5" POLY	4.75" +1.5 EXT	22" FULL/PNEU	17.5"			
5" POLY	6"+1.5" EXT	22" LOW	18"	1	2	MED
5" POLY	6"+1.5" EXT	22" FULL/PNEU	18"	1	2	MED
5" POLY	4.75" +1.5" EXT	24" LOW	17"	2	1	LARGE
5" POLY	4.75" +1.5" EXT	24" LOW	17.5"			
5" POLY	4.75" +1.5" EXT	24" FULL/PNEU	17.5"	2	1	LARGE
5" POLY	6"+1.5" EXT	24" LOW	18"	1	2	LARGE
5" POLY	6"+1.5" EXT	24" FULL/PNEU	18"	1	1	LARGE

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
						Jan-15
6"/6X1.5" POLYURETHENE			Axle Position 1=Top		Caster Position 1=Top	
STD HEIGHT FRAME			Axle Position 2=Bottom		Caster Position 2=Bottom	
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
6" POLY	6"	20" LOW	17"	1	2	MED
6" POLY	6"	20" FULL/PNEU	17"	1	2	MED
6" POLY	6"	22" LOW	17"	1	1	MED
6" POLY	6"	22" FULL/PNEU	17"	1	1	MED
6" POLY	6"	22" LOW	18"	2	2	LARGE
6" POLY	6"	22" FULL/PNEU	18"	2	2	LARGE
6" POLY	6"+1.5" EXT	22" FULL/PNEU	18.5"	1	2	LARGE
6" POLY	6"	24" LOW	17"	1	1	MED
6" POLY	6"	24" LOW	18"	2	2	LARGE
6" POLY	6"	24" FULL/PNEU	18"	2	1	LARGE
6" POLY	6"+1.5" EXT	24" LOW	18.5"	1	2	LARGE
6" POLY	6"+1.5" EXT	24" FULL/PNEU	18.5"	1	2	LARGE
6" POLY	6"+1.5" EXT	24" FULL/PNEU	19.5"	2	2	LARGE

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
						Jan-15
8"/8"X1.5" POLYURETHENE			Axle Position 1=Top		Caster Position 1=Top	
STD HEIGHT FRAME			Axle Position 2=Bottom		Caster Position 2=Bottom	
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
8" POLY	6"	20" LOW	17.5"	1	2	MED
8" POLY	6"	20" FULL/PNEU	17.5"	1	2	MED
8" POLY	6"	20" PNEU	18.5"	(FULL NOT RECOMMENDED)		
8" POLY	6"	22" LOW	17.5"	1	2	LARGE
8" POLY	6"	22" FULL/PNEU	17.5"	1	2	LARGE
8" POLY	6"	22" LOW	18.5"	2	2	LARGE
8" POLY	6"	22" FULL/PNEU	18.5"	2	2	LARGE
8" POLY	6"+1.5" EXT	22" FULL/PNEU	19"	1	2	LARGE
8" POLY	6"	24" LOW	17.5"	1	1	LARGE
8" POLY	6"	24" FULL/PNEU	17.5"	1	1	LARGE
8" POLY	6"	24" LOW	18.5"	2	2	LARGE
8" POLY	6"	24" FULL/PNEU	18.5"	2	2	LARGE
8" POLY	6"+1.5" EXT	24" LOW	19"	1	2	LARGE
8" POLY	6"+1.5" EXT	24" FULL/PNEU	19"	1	2	LARGE
8" POLY	6"+1.5" EXT	24" FULL/PNEU	20"	2	2	LARGE

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
						Jan-15
8" PNEUMATIC			Axle Position 1=Top		Caster Position 1=Top	
STD HEIGHT FRAME			Axle Position 2=Bottom		Caster Position 2=Bottom	
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
8" PNEU	6"	22" FULL/PNEU	19"	2	2	LARGE
8" PNEU	6"	24" LOW	19"*****	2	2	LARGE
8" PNEU	6"	24" FULL/PNEU	19"*****	2	2	LARGE
8" PNEU	6"+1.5" EXT	24" FULL/PNEU	19.5"	1	2	LARGE
8" PNEU	6"+1.5" EXT	24" FULL/PNEU	20.5"	2	2	LARGE
***** ONLY AVAILABLE WITH 60 DEGREE HANGERS						

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
			Axle Position 1=Top			Jan-15
5"/5X1.5" POLYURETHENE			Axle Position 2=Middle/Top		Caster Position 1=TOP	
HEMI HEIGHT FRAME			Axle Position 3=Middle/Bottom		Caster Position 2=Bottom	
			Axle Position 4=Bottom			
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
5" POLY	4.75"	20" LOW	13"	1	1	SMALL
5" POLY	4.75"	20" LOW	14"	2	2	SMALL
5" POLY	4.75"	20" FULL/PNEU	14"	2	1	SMALL
5" POLY	4.75"+1.5" EXT	20" LOW	14.5"	1	2	SMALL
5" POLY	4.75"+1.5" EXT	20" FULL/PNEU	14.5"	1	2	SMALL
5" POLY	4.75"+1.5" EXT	20" LOW	15.5"	2	3	MED
5" POLY	4.75"+1.5" EXT	20" FULL/PNEU	15.5"	2	3	MED
5" POLY	4.75"	22" LOW	14"	2	1	SMALL
5" POLY	4.75"+1.5" EXT	22" LOW	14.5"	1	2	MED
5" POLY	4.75"+1.5" EXT	22" FULL/PNEU	14.5"	1	1	MED
5" POLY	4.75"+1.5" EXT	22" LOW	15.5"	2	2	MED
5" POLY	4.75"+1.5" EXT	22" FULL/PNEU	15.5"	2	2	MED
5" POLY	4.75"+1.5" EXT	24" LOW	14.5"	1	1	MED
5" POLY	4.75"+1.5" EXT	24" LOW	15.5"	2	2	MED
5" POLY	4.75"+1.5" EXT	24" FULL/PNEU	15.5"	2	1	MED

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
			Axle Position 1=Top			Jan-15
6"/6X1.5" POLYURETHENE			Axle Position 2=Middle/Top		Caster Position 1=Top	
HEMI HEIGHT FRAME			Axle Position 3=Middle/Bottom		Caster Position 2=Bottom	
			Axle Position 4=Bottom			
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
6" POLY	4.75"	20" LOW	14.5"	2	2	SMALL
6" POLY	4.75"	20" FULL/PNEU	14.5"	2	2	SMALL
6" POLY	4.75"+1.5" EXT	20" LOW	16"	2	3	SMALL
6" POLY	4.75"+1.5" EXT	20" FULL/PNEU	16"	2	3	SMALL
6" POLY	4.75"	22" LOW	14.5"	2	2	SMALL
6" POLY	4.75"	22" FULL/PNEU	14.5"	2	1	SMALL
6" POLY	4.75"+1.5" EXT	22" LOW	16"	2	2	MED
6" POLY	4.75"+1.5" EXT	22" FULL/PNEU	16"	2	2	MED
6" POLY	4.75"+1.5" EXT	24" LOW	16"	2	2	MED
6" POLY	4.75"+1.5" EXT	24" FULL/PNEU	16"	2	1	MED

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
			Axle Position 1=Top			Jan-15
8"/8"X1.5" POLYURETHENE			Axle Position 2=Middle/Top		Caster Position 1=Top	
HEMI HEIGHT FRAME			Axle Position 3=Middle/Bottom		Caster Position 2=Bottom	
			Axle Position 4=Bottom			
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
8" POLY	4.75"	20" LOW	15"	2	3	MED
8" POLY	4.75"	20" FULL/PNEU	15"	2	2	MED
8" POLY	4.75"+1.5" EXT	20" LOW	16.5"	2	4	MED
8" POLY	4.75"+1.5" EXT	20" FULL/PNEU	16.5"	2	3	MED
8" POLY	6"	20" LOW	15.5"			
8" POLY	6"	20" FULL/PNEU	15.5"			
8" POLY	6"	20" LOW	16"			
8" POLY	6"	20" FULL/PNEU	16"			
8" POLY	6"+1.5" EXT	20"LOW	17"	1	4	
8" POLY	6"+1.5" EXT	20" FULL/PNEU	17"			
8" POLY	6"+1.5" EXT	20" FULL/PNEU	17.5"			
8" POLY	4.75"	22" LOW	15"	2	2	MED
8" POLY	4.75"	22" FULL/PNEU	15"	2	1	MED
8" POLY	6"	22" LOW	15.5"	1	2	MED
8" POLY	6"	22" FULL/PNEU	15.5"	1	2	MED
8" POLY	6"	22" LOW	16"	2	3	MED
8" POLY	6"	22" FULL/PNEU	16"	2	3	MED
8" POLY	4.75"+1.5" EXT	22" LOW	16.5"	2	3	MED
8" POLY	4.75"+1.5" EXT	22" FULL/PNEU	16.5"	2	3	MED
8" POLY	6"+1.5" EXT	22" LOW	17"	1	4	LARGE
8" POLY	6"+1.5" EXT	22" FULL/PNEU	17"	1	3	LARGE
8" POLY	6"+1.5" EXT	22" LOW	17.5"	2	4	LARGE
8" POLY	6"+1.5" EXT	22" FULL/PNEU	17.5"	2	4	LARGE
8" POLY	4.75"	24" FULL/PNEU	15"	2	1	MED
8" POLY	6"	24" LOW	15.5"	1	1	MED
8" POLY	6"	24" FULL/PNEU	15.5"	1	1	MED
8" POLY	6"	24" LOW	16"	2	2	MED

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
			Axle Position 1=Top			Jan-15
8"/8"X1.5" POLYURETHENE			Axle Position 2=Middle/Top		Caster Position 1=Top	
HEMI HEIGHT FRAME			Axle Position 3=Middle/Bottom		Caster Position 2=Bottom	
			Axle Position 4=Bottom			
CASTER SIZE	FORK/STEM COMBO	WHEEL SIZE	STD SEAT HEIGHT	CASTER POSITION	REAR AXLE POSITION	ANTI-TIP TUBES
8" POLY	6"	24" FULL/PNEU	16"	2	2	MED
8" POLY	4.75"+1.5" EXT	24" LOW	16.5"	2	3	MED
8" POLY	4.75"+1.5" EXT	24" FULL/PNEU	16.5"	2	2	MED
8" POLY	6"+1.5" EXT	24" LOW	17"	1	3	LARGE
8" POLY	6"+1.5" EXT	24" FULL/PNEU	17"	1	2	LARGE
8" POLY	6"+1.5" EXT	24" LOW	17.5"	2	3	LARGE
8" POLY	6"+1.5" EXT	24" FULL/PNEU	17.5"	2	3	LARGE

BREEZY 600 SEAT HEIGHT CHARTS

BREEZY 600 SEAT HEIGHT/AXLE/CASTER/ANTI-TIP MATRIX						
			Axle Position 1=Top			Jan-15
8" PNEUMATIC			Axle Position 2=Middle/Top		Caster Position 1=Top	
HEMI HEIGHT FRAME			Axle Position 3=Middle/Bottom		Caster Position 2=Bottom	
			Axle Position 4=Bottom			
CASTER	FORK/STEM	WHEEL	STD SEAT	CASTER	REAR AXLE	ANTI-TIP
SIZE	COMBO	SIZE	HEIGHT	POSITION	POSITION	TUBES
8" PNEU	6"	20" LOW	16"	1	4	MED
8" PNEU	6"	20" FULL/PNEU	16"	1	3	MED
8" PNEU	6"	20" LOW	17"	2	4	MED
8" PNEU	6"	20" FULL/PNEU	17"	2	3	MED
8" PNEU	6"+1.5" EXT	20" FULL/PNEU	17.5"			
8" PNEU	6"	22" LOW	16"	1	3	MED
8" PNEU	6"	22" FULL/PNEU	16"	1	2	MED
8" PNEU	6"	22" LOW	17"	2	4	LARGE
8" PNEU	6"	22" FULL/PNEU	17"	2	3	LARGE
8" PNEU	6"+1.5" EXT	22" LOW	17.5"	1	4	LARGE
8" PNEU	6"+1.5" EXT	22" FULL/PNEU	17.5"	1	3	LARGE
8" PNEU	6"+1.5" EXT	22" FULL/PNEU	18.5"	2	4	LARGE
8" PNEU	6"	24" LOW	16"	1	2	MED
8" PNEU	6"	24" FULL/PNEU	16"	1	1	MED
8" PNEU	6"	24" LOW	17"	2	3	LARGE
8" PNEU	6"	24" FULL/PNEU	17"	2	2	LARGE
8" PNEU	6"+1.5" EXT	24" LOW	17.5"	1	3	LARGE
8" PNEU	6"+1.5" EXT	24" FULL/PNEU	17.5"	1	3	LARGE
8" PNEU	6"+1.5" EXT	24" LOW	18.5"	2	4	LARGE
8" PNEU	6"+1.5" EXT	24" FULL/PNEU	18.5"	2	3	LARGE