

# JAY<sup>®</sup>

## JAY<sup>®</sup> J2<sup>®</sup> Back

Owner's Manual

### IMPORTANT CONSUMER INFORMATION

**NOTICE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:** This manual must be given to the user of this product.

**USER:** Before using this product, read this entire manual and save for future reference.

## Respaldo JAY<sup>®</sup> J2<sup>®</sup>

### INFORMACIÓN IMPORTANTE PARA EL CONSUMIDOR

**AVISO:** Este manual contiene instrucciones importantes que deben ser entregadas al usuario de este producto. Por favor no retire este manual antes de la entrega al usuario.

**DISTRIBUIDOR:** Este manual debe ser entregado al usuario de este producto.

**USUARIO:** Antes de usar este producto, lea este manual en su totalidad y guárdelo para futura referencia.

Cada una de las sillas se envía con un manual de instrucciones en inglés. El manual en español o francés está disponible en formato PDF en nuestra página en Internet: [www.SunriseMedical.com](http://www.SunriseMedical.com). Ingrese a la página del producto específico para descargar el manual, o comuníquese con el proveedor autorizado de Sunrise Medical.

## Dossier JAY<sup>®</sup> J2<sup>®</sup> Back

### INFORMATIONS IMPORTANTES DESTINÉES AU CONSOMMATEUR

**AVIS:** Ce manuel contient des instructions importantes qui doivent être communiquées à l'utilisateur de ce produit. Prière de ne pas le retirer avant la livraison à l'utilisateur.

**FOURNISSEUR:** Ce manuel doit être remis à l'utilisateur de ce produit.

**UTILISATEUR :** Avant d'utiliser ce produit, lisez entièrement ce manuel. Conservez-le pour référence ultérieure.

Chaque fauteuil est livré avec un manuel d'instructions en anglais. Les versions en espagnol et en français sont à votre disposition en format PDF sur le site: [www.SunriseMedical.com](http://www.SunriseMedical.com). Veuillez vous rendre à la page de votre produit pour télécharger le manuel dans la langue souhaitée, ou contactez un fournisseur agréé Sunrise Medical.

Manual de  
instrucciones

Manuel  
d'instructions

P/N XT2505 Rev. L

Owner's Manual

# JAY<sup>®</sup> J2<sup>®</sup> Back

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a J2® Back is appropriate. Backs should only be installed by an authorized Sunrise Medical supplier.

Maximum user weight: 250 lbs. (113 kg).

**CAUTION**

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

**EASY CLEANING AND REASSEMBLY FOR JAY® J2® BACK -2500 SERIES MODELS**

**To Clean Back Cover**

Pull the foam insert away from back shell. Unzip cover. Remove foam and turn cover inside out. Zip cover closed to avoid snagging. If applicable, note where lumbar shims are placed to ensure proper re-assembly. Machine wash in warm water. Dry in dryer on medium heat.

**NOTE-** Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Replace foam in cover and zip closed. The "Top" label on the cover should be at the top, rear of the foam. Ensure that the cover fits properly by aligning the seams with the edge of the foam.

Replace lumbar shims if applicable. Place foam insert on shell. Ensure that "Top" label on cover is at the top of the Back shell. Press into the Back shell, matching the Velcro® strips on the cover with the Velcro strips on the shell.

**To Clean Back Shell**

Shell may be wiped clean with soap and water. No other maintenance should be required.

**MAINTENANCE AND TORQUE SPECIFICATIONS**

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to 2.3 N-m (20 in-lbs). Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

**DETERMINING COMPATIBILITY**

**Wheelchair Types**

The J2 Back is designed to be compatible with most wheelchairs with the following exceptions:

Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

**⚠ WARNING**

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

The J2 Back has not been tested for use as seating in a motor vehicle. As a result, the performance of the back in combination with wheelchairs that are labeled for transit use, when used as a seat in a motor vehicle, is unknown. Sunrise Medical recommends that users not occupy a wheelchair in a motor vehicle when using the J2 Back.

**⚠ WARNING**

Using a J2 Back as seating in a motor vehicle may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

**Wheelchair Dimensions**

The J2 Back is designed to replace the wheelchair's sling upholstery. The mounting hardware system is designed for tubular back canes. Prior to installation determine if the wheelchair used has compatible back canes. Start by checking the table below to see if the back canes are a correct width.

Back Model Number	Width of back canes from outside to outside of the tubes (Fig 1 - A)			
	Minimum Width		Maximum Width	
	inch	cm	inch	cm
2512K	10.3	32.2	11.5	35.4
2513K	11.3	33.2	12.5	36.4
2514K, 2514TK	12.3	34.2	13.5	37.4
2515K, 2515TK	13.3	35.2	14.5	38.4
2516K, 2516TK	14.3	36.2	15.5	39.4
2517K, 2517TK	15.3	37.2	16.5	40.4
2518K, 2518TK	16.3	38.2	17.5	41.4
2519K, 2519TK	17.3	39.2	18.5	42.4
2520K, 2520TK	18.3	40.2	19.5	43.4


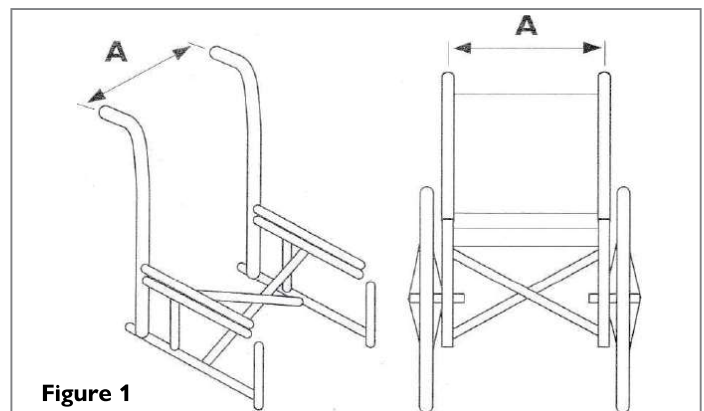


Figure 1

Next check that the back canes are a compatible diameter. The strap style back mounting hardware may be used on back canes from 0.62 to 1.1 inches (16mm to 28mm) in diameter. (Fig. 2)

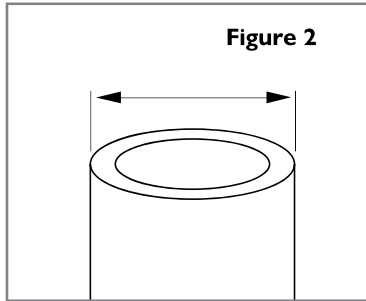


If you cannot determine that the back canes are compatible please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

**FITTING THE J2 BACK**


The J2 Back is designed to replace the wheelchair’s siing upholstery. The initial installation should be done without the foam insert in the back shell. Tighten all fasteners to 2.3 Nm [20 in-lbs] using a Phillips No. 2 screwdriver.

1. **Remove the upholstery from the back of the wheelchair.**
2. **Attach one bottom post bracket to each upright post of the wheelchair.**
  - a. Check the diagram for the proper holes for different wheelchair post diameters. (The bracket comes assembled to attach to 7/8”-1” (2.2cm -2.5cm) posts. To attach 5/8”-3/4” (1.6cm-1.9cm) posts remove caps and insert screws into both inside holes.) Wrap the band around the wheelchair post. Screw ‘A’ may have to be loosened to align strap hole with barrel nut. Place a washer on the screw and insert through the proper hole.
  - b. Insert screw “B” into the side of the bracket and tighten firmly to 2.3 Nm [20 in-lbs]. Ensure that the screw engages the insert inside the bracket.
  - c. Place caps over both screw heads.
  - d. To adjust the height of the bracket, loosen screw ‘A’ until bracket slides freely on wheelchair post. Once proper height of the back is determined, tighten screw ‘A’ to 2.3 Nm [20 in-lbs]. All future adjustments of the back height are done with this screw.

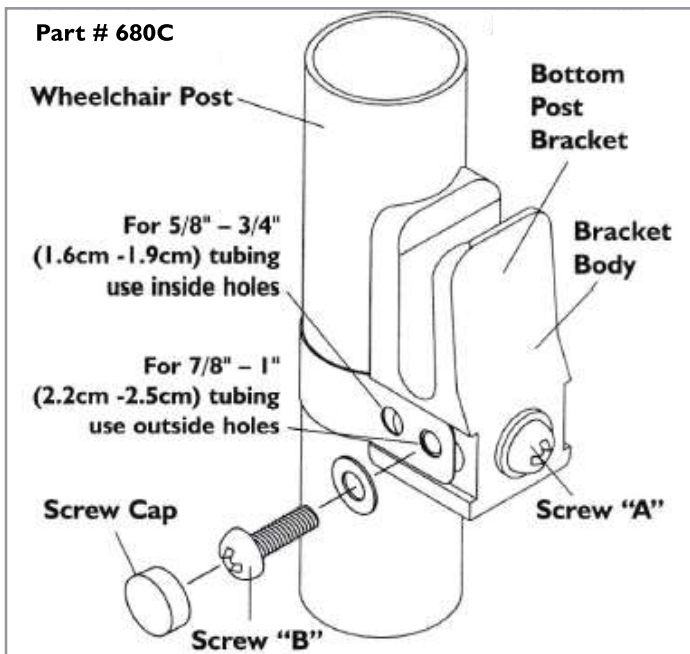


**3. Install the J2 Back**

Place the J2 Back in the wheelchair so that the bottom pins on the back shell slide into the bottom post brackets and the top of the back rests against the uprights. The top hooks (on the upper portion of the back shell) should rest flush against the posts.

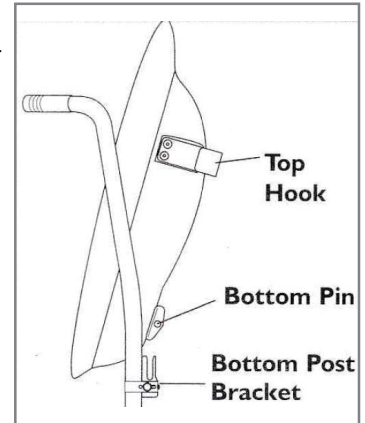


**4. Bottom width adjustment.**

Remove the J2 Back from the wheelchair. Lift tab on bottom brackets with a screwdriver tip and slide the pins to adjust width to match wheelchair post width. Check to ensure the pins extend to the full width of the bottom post brackets. Count the number of holes exposed to ensure the back is adjusted evenly on both sides.

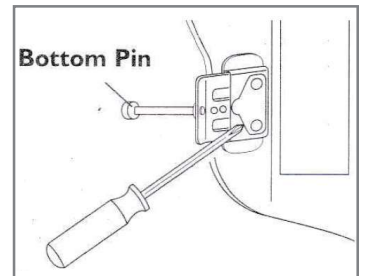


**5. Top width adjustment.**

Depress the silver pin on top brackets with a screwdriver tip and slide top bracket to adjust width to match wheelchair post width. Ensure that the back is adjusted evenly on both sides. Replace back in wheelchair.

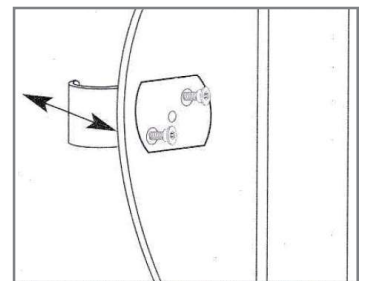


**6. Assess and adjust back height.**

Replace the foam insert. Assess client for back height. Position the user’s hips to the back of the wheelchair. The center of the bottom pin on the J2 Back should be placed at approximately the level of the PSIS or SI-S2 spinal level of the individual. (The PSIS is the top, rear of the pelvis.) This will help maintain a stable pelvis when the back angle is adjusted. To adjust the height of the back, loosen screw “A” and slide post brackets up or down. Once proper height is determined, tighten brackets firmly to 2.3 Nm [20 in-lbs].

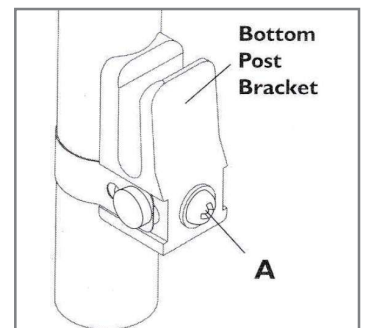


**7. Assess and adjust back angle.**

Assess client for back angle. To recline, turn screw “C” on the back of the top bracket at the rear of the back. Turning the screw clockwise will make the back more vertical. Turning the screw counter clockwise will recline the back. Check the hash marks on both sides of top brackets to ensure that the back is evenly adjusted.

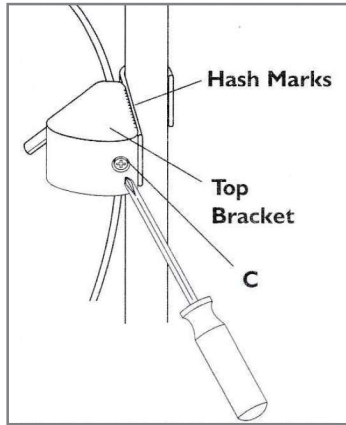

**8. Add lumbar/pelvic shim(s) as required.**

Remove foam insert and unzip the back cover. Insert the lumbar/pelvic shim(s) inside the cover behind the large piece of foam where needed. Zip the cover closed and place the foam insert back in the shell.

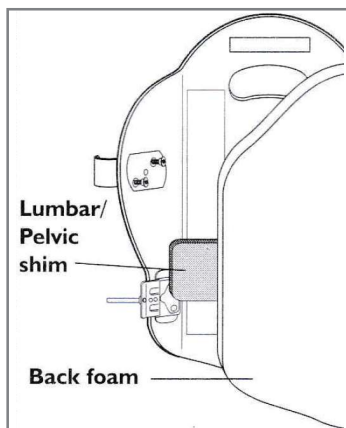



**INSTALLING THE TOP QUICK-RELEASE QUARTER TURN BRACKET HARDWARE**

1. Attach one top quick-release quarter turn bracket to each upright post of the wheelchair.
  - a. The top quick-release quarter turn bracket is designed to fit on 5/8", 3/4", 7/8" and 1" (1.6, 1.9, 2.2 and 2.5 cm) diameter back post tubing. (The bracket comes assembled to attach to 7/8" or 1" (2.2 or 2.5cm) tubing) Check diagram 1 to determine which strap hole is appropriate for the installation. Wrap the strap around the wheelchair post. Screw “A” may have to be loosened to align the strap hole with the barrel nut located inside the bracket body. Place a washer on screw “B” and insert the screw through the appropriate hole in the strap.
  - b. Tighten screw “B” (to 2.3 Nm [20 in-lbs]) into the barrel nut through the side of the bracket body. Tighten the opposite screw if necessary.
  - c. Snap the screw caps over both screw heads.
  - d. To adjust the height of the bracket, loosen screw “A” until the bracket slides freely on the back post. Once the proper height of the back is determined, tighten the screw firmly to 2.3 Nm [20 in-lbs]. All future adjustments of the back height are done with screw “A”.
  - e. Always turn the quick-release quarter turn bracket to the “DOWN” position (diagram 2) to secure the top hook in place.

**CAUTION-** The top quick-release quarter turn brackets must be in their downward position when the back is in use. Improper use may cause the back to unexpectedly detach from the wheelchair.

**CAUTION-** Do not use J2 Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

**NOTE-** The back is properly secured when the bottom pins are resting in the lower post brackets and the upper quick-release quarter turn brackets are installed above the back support's top hook brackets with the quick release mechanism pointing down.

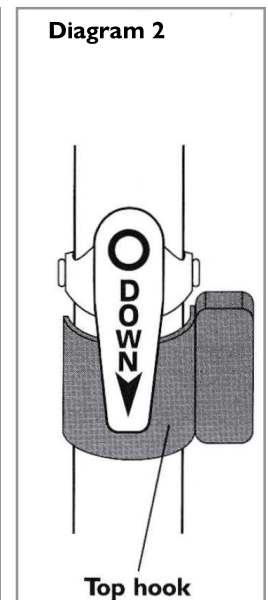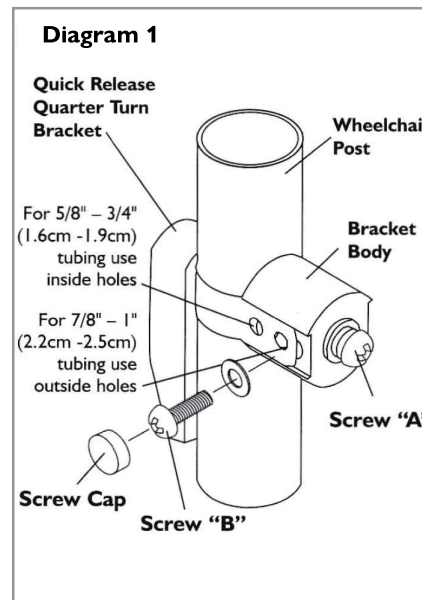


**REMOVAL AND REPLACEMENT**


Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. Simply rotate the top quick-release quarter turn brackets away from the top hooks and lift the back up and out of the wheelchair. To replace, lower the bottom pins into the lower post brackets and rock the back backward until it rests against the upright posts. Rotate the quick-release brackets into the “Down” position. When installed correctly, the quick-release bracket should cover the top hook. The Back angle will not change when removed and re-installed on the wheelchair.

**Proper reattachment after removal**

Check to make sure the pin attached to the backrest is resting completely at the bottom of the slotted receptacle attached to the wheelchair back cane.

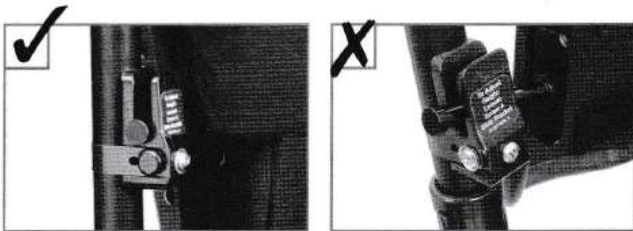

Check to make sure that the quarter fastener attached to the wheelchair back cane is turned downward in its locked position.

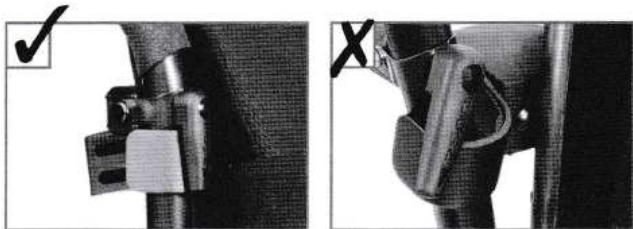

**24 MONTH LIMITED WARRANTY**

Each Jay J2 Back is carefully inspected and tested to provide peak performance. Every Jay J2 Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back’s removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.





Sunrise Medical S.r.l.  
Via Riva, 20 – Montale  
29122 Piacenza  
Italia  
Tel.: +39 0523 573111  
www.SunriseMedical.it

Sunrise Medical AG  
Erlenauweg 17  
CH-3110 Münsingen  
Schweiz/Suisse/Svizzera  
Tel +41 (0)31 958 3838  
www.SunriseMedical.ch

Sunrise Medical AS  
Delitoppen 3  
1540 Vestby  
Norge  
Telefon: +47 66 96 38 00  
post@sunrisemedical.no  
www.SunriseMedical.no

Sunrise Medical AB  
Neogatan 5  
431 53 Mölndal  
Sweden  
Tel.: +46 (0)31 748 37 00  
post@sunrisemedical.se  
www.SunriseMedical.se

MEDICCO s.r.o.  
H – Park, Heršpická 1013/11d,  
639 00 Brno  
Czech Republic  
Tel.: (+420) 547 250 955  
Fax: (+420) 547 250 956  
www.medicco.cz  
info@medicco.cz  
Bezplatná linka 800 900 809

Sunrise Medical Aps  
Mårkærvej 5-9  
2630 Taastrup  
Denmark  
+45 70 22 43 49  
info@sunrisemedical.dk  
www.SunriseMedical.dk

Sunrise Medical Australia  
11 Daniel Street  
Wetherill Park NSW 2164  
Australia  
Ph: +61 2 9678 6600  
Email: enquiries@sunrisemedical.com.au  
www.SunriseMedical.com.au

Sunrise Medical (US) LLC  
North American Headquarters  
12002 Volunteer Blvd.  
Mount Juliet, TN 37122, USA  
(800) 333-4000  
(800) 300-7502  
www.SunriseMedical.com



© 2025 Sunrise Medical (US) LLC  
XT2505 Rev. L

Sunrise Medical GmbH  
Kahlbachring 2-4  
D-69254 Malsch  
Deutschland  
Tel.: +49 (0) 7253/980-0  
Fax: +49 (0) 7253/980-222  
www.SunriseMedical.de

Sunrise Medical Ltd.  
Thorns Road  
Brierley Hill  
West Midlands  
DY5 2LD  
England  
Phone: 0845 605 66 88  
Fax: 0845 605 66 89  
www.SunriseMedical.co.uk

Sunrise Medical S.L.  
Polígono Bakiola, 41  
48498 Arrankudiaga – Vizcaya  
España  
Tel.: +34 (0) 902142434  
Fax: +34 (0) 946481575  
www.SunriseMedical.es

Sunrise Medical Poland  
Sp. z o.o.  
ul. Elektronowa 6,  
94-103 Łódź  
Polska  
Telefon: + 48 42 275 83 38  
Fax: + 48 42 209 35 23  
E-mail: pl@sunrisemedical.de  
www.Sunrise-Medical.pl

Sunrise Medical B.V.  
Defensiedok 20  
3433 NE Nieuwegein  
The Netherlands  
T: +31 (0)30 – 60 82 100  
F: +31 (0)30 – 60 55 880  
E: info@sunrisemedical.nl  
www.SunriseMedical.nl

Sunrise Medical HCM B.V.  
Vossenbeemd 104  
5705 CL Helmond  
The Netherlands  
T: +31 (0)492 593 888  
E: customerservice@sunrisemedical.nl  
www.SunriseMedical.nl  
www.SunriseMedical.eu (International)

Sunrise Medical S.A.S  
ZAC de la Vrillonnerie  
17 Rue Mickaël Faraday  
37170 Chambray-Lès-Tours  
Tel : + 33 (0) 2 47 55 44 00  
Email: info@sunrisemedical.fr  
www.SunriseMedical.fr

Sunrise Medical Canada Inc.  
1000 Creditstone Rd., Unit #2  
Concord, ON, L4K 4P8  
Canada  
Phone: 1-800-263-3390  
Fax: 1-800-561-5834  
www.SunriseMedical.ca